

Instruction Manual

Osaki - Vista



Before operating this unit, please read these instructions completely.

Contents

Before Use

Safety Precautions	1-4
• FCC Radiation Exposure Statement	4
Chair Components	5-7
• Bluetooth Speaker	6
Preparation before Use	8-9
Moving the massage chair	10

How to Use

Remote Controller	11-15
-------------------------	-------

Q&A / Important

Cleaning and maintenance	16
Troubleshooting	17
Grounding Instructions.....	18
Chair Specifications.....	19

Thank you for purchasing **this** Massage Chair.
Before using your massage chair, please read the contents of this
instruction manual and refer to it when needed.

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following:

Read all instructions before using this appliance.

DANGER To reduce the risk of electric shock:

- Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
- Never use pins or other metallic fasteners with this appliance.
- Carefully examine the covering before each use. Discard the appliance if the covering shows any sign of deterioration, such as checking, blistering, or cracking.

WARNING To reduce the risks of burns, fire, electric shock or injury to persons:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
- Close supervision is necessary when this appliance is used by, on, or near children, or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments unless recommended by the manufacturer.
- Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair.
- Do not carry this appliance by the supply cord or use cord as a handle.
- Keep the cord away from heated surfaces.
- Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- Never drop or insert any object into any opening.
- Do not use outdoors.
- Do not operate where aerosol (spray) products are being used.
- To disconnect, turn all controls to the off position, then remove plug from outlet.
- Keep children away from extended foot support/rest.
- Connect this appliance to a properly grounded outlet only. See Grounding Instructions.
- Use the heated surfaces carefully. May cause serious burns. Do not use over sensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or persons may be dangerous.
- Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.

SAVE THESE INSTRUCTIONS

Safety Precautions

When using the massage chair, basic precautions should always be followed to reduce the risk of electric shock, burns, fire or injury to persons. Please read the following precautions before using the massage chair.

- Make sure the power plug is securely inserted into a proper electrical outlet to reduce the risk of short-circuit and fire.
- It is highly recommended that a surge protector is used to prevent from damage.
- DO NOT use the massage chair in areas where the humidity is high, such as sauna steam rooms, pools, bathrooms, AC or heating vents, etc.
- DO NOT use the massage chair outdoors and avoid areas with direct sunlight.
- If the chair is not being used, turn the power switch to the OFF position and unplug the massage chair from the electrical outlet to prevent any damages to the chair when there are power outages/surges that may cause serious damage.
- Never plug/unplug the power cord plug socket with wet hands.
- Close and continuous supervision is necessary when the massage chair is used by, on or near children or disabled persons.
- No more than one person should use the massage chair at the same time.
- The massage chair is intended for use as described in the instruction manual. Do not use any attachments other than those recommended by the manufacturer.
- DO NOT attempt to stand or sit on the backrest, armrest or footrest in order to prevent accident, injury or damage to the chair.
- Avoid falling asleep while using the massage chair.
- Always keep the massage chair clean, and never insert any objects into or around the gaps of the massage chair.
- DO NOT move the massage chair by pulling on the power supply cord.
- Make sure there are no children, pets or other obstacles behind or underneath the massage chair.
- If you find any rips or tears in the fabric of the massage chair, contact the service department and stop using the chair immediately.
- This massage chair is designed for household use and should not be used in non-residential settings.

Safety Precautions

- Please consult your doctor if you are in doubt as to whether or not you should use this massage chair.
- Please consult a doctor before using the massage chair if you are pregnant, undergoing any medical treatment, using a medical electronic device such as a pacemaker, or are experiencing any of the following: malignant tumor, heart disease, back pain, abnormal or curved back bone, osteoporosis or acute illness.
- If you experience pain while using the massage chair, stop using the chair immediately and consult your doctor.
- Never use a massage function on swollen or inflamed parts of the body.
- DO NOT use the massage chair for more than 30 minutes at a time.
- DO NOT use the massage chair where aerosol (spray) products are being used or where oxygen is being administered.
- This massage chair is a non-professional product designed to provide a comfortable massage and should not be substituted for appropriate medical treatment.
- Never operate the appliance with the air hoses that inflate while they are blocked. Keep the air openings free of lint, hair, and any other obstructions.
- Check the skin which is in contact with the heated area of the appliance frequently to reduce the risk of injury. If the heating pad malfunctions and extreme heat is produced, it may cause serious burns.
- Burns may result from improper use. If this is the case, immediately shut off the chair and unplug from the outlet.
- Remove all sharp objects from the body and pockets, such as keys and pens.
- Keep children away from extended foot support (or other similar parts.)
- DO NOT WET-DO NOT USE PINS.
- NEVER REMOVE COVER.

Caution

The user's weight is usually limited to 264 lbs.

For those who are weighing 264 lbs or more, the product operations may make more noise and the fabric cover may wear out faster.

FCC Radiation Exposure Statement

FCC Radiation Exposure Statement:

This equipment complies with FCC radiation exposure requirement.
The device can be used in portable exposure condition without RF striction.

FCC STATEMENT


1. This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:
 - (1) This device may not cause harmful interference, and
 - (2) This device must accept any interference received, including interference that may cause undesired operation.
2. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/ TV technician for help.

Chair Components

Front

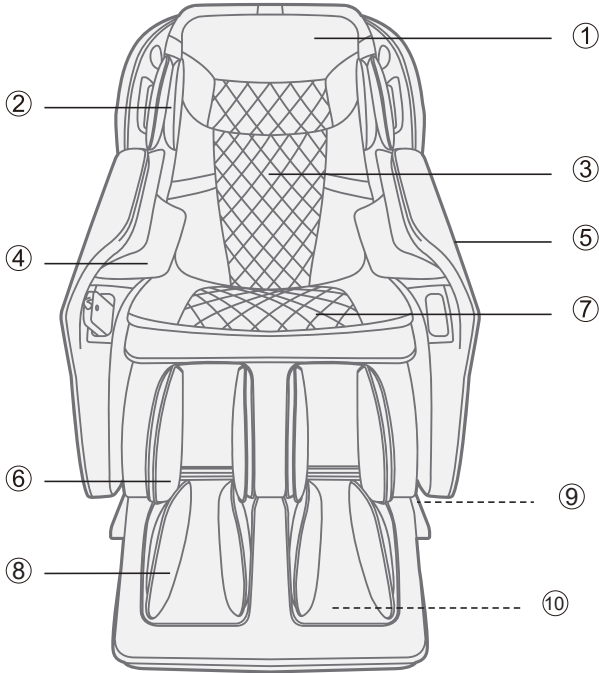

① Pillow Pad

③ Backrest Pad

⑤ Side Panel

⑦ Seat Pad

⑨ Wheel

② Shoulder Airbag

④ Arm Airbag


⑥ Leg Airbag

⑧ Foot Airbag

⑩ Foot Roller

Chair Components


Bluetooth Speaker

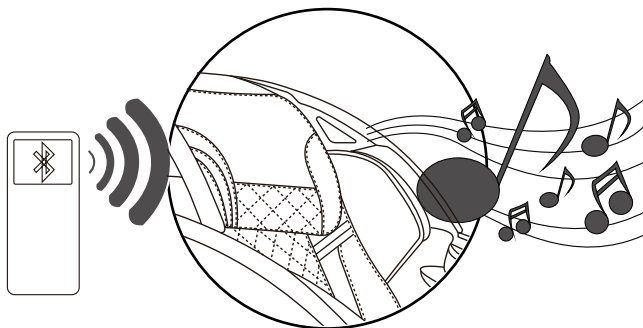

NOTE:

The bluetooth speaker can only connects with one bluetooth device at a time. Please turn off your bluetooth device after using so that other users can enjoy it as well.


Chair Components

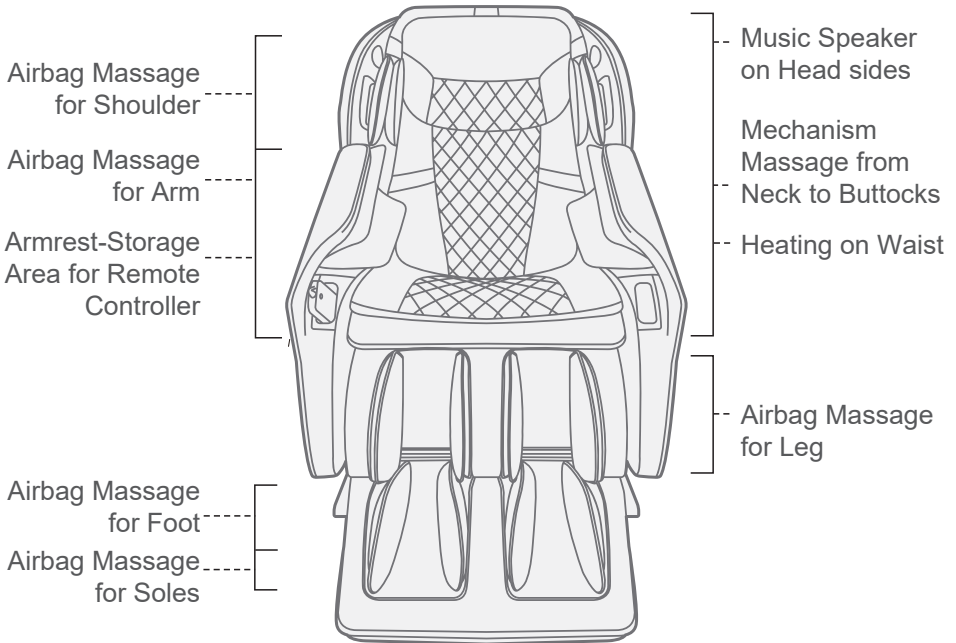

Preparation before Use


1. Chair placement

- When placing the massage chair, be sure to leave 2.4 in. of space between the back of the chair and wall or any other objects to allow it recline safely.
- Please make sure to use a mat or rug under chair if you placed on top of a hard surface like hardwood and ceramic floors.
- Do not expose the product to direct sunlight or a high-temperature place (e.g., in front of heating equipment or by a window where sunlight can cause the synthetic leather fade, melt or crack.)

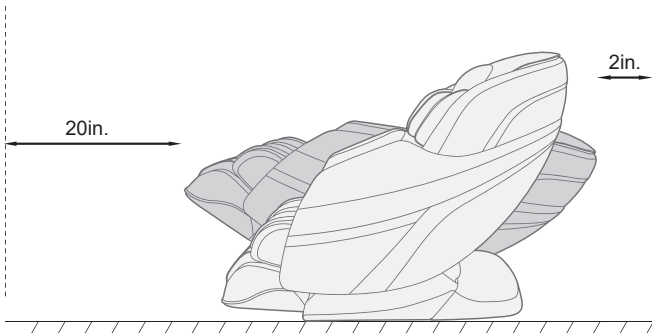

2. Check the space around the product.


Make sure there is no person, pet, or other object close to the product.

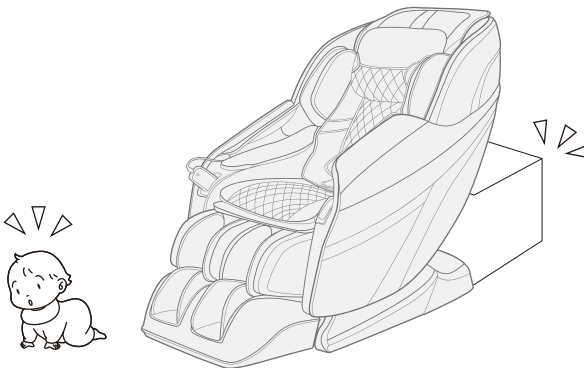

Preparation before Use




3. Make sure there is no foreign matter in the seams of the product.

- Before sitting on the product, make sure there is no foreign matter between any two parts of it.
- Do not put hands, arms or head between the backrest and the back cover.
- Do not put pets on the product (such as its back cover, backrest and armrests).

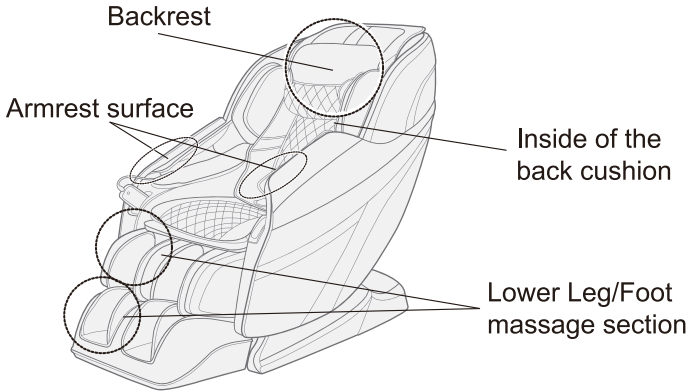


4. Power on the product

- ① Push the power connector into the appliance inlet till the end.
- ② Insert the power plug into the power outlet till the end.
- ③ Turn the switch to "I" position

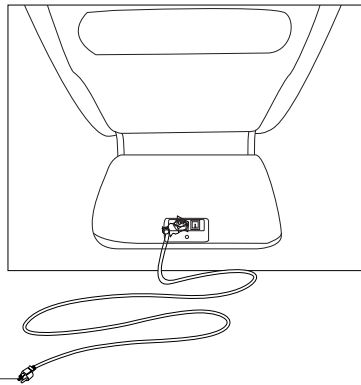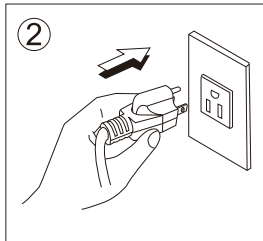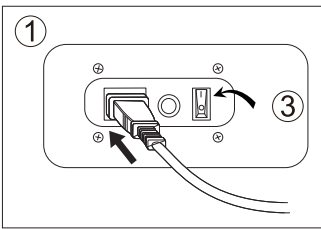

Moving the massage chair

- Before moving the chair, please switch off the power and remove the power cord from socket.
- Lower the chair by the backrest until the base is inclined at a 45° angle.
- Move the chair carefully by the caster.

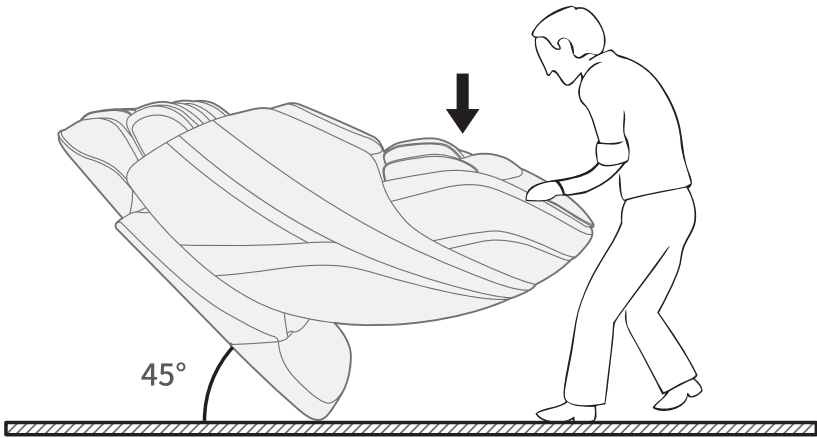

How to use

(1) Remote Controller



(2) Operation Guide

Connect to the power supply, and turn the switch to “I” position. When in idle mode, the red light keeps flashing, and there are only four functions: ON/OFF, Recline, Sit Up, Pause that can be operated.

How to use

Control Buttons for the Massage Chair

1.ON/OFF

The Start/Stop Control Button

Press this button, and the backlight of the remote controller will blink to indicate that the massage chair is in idle mode.

Then press any function button to activate the massage chair operations. The timer will begin to calculate time.

Press this button again to stop all massage functions and to return the massage rollers, backrest and legrest to their original positions.

2.Timer

Time Interval Control Button

When a massage function is activated, the time is automatically set for 15 minutes.

Every time this button is pressed, the time interval will change in the following sequence: 15>20>25>30>5>10>15 minutes.

3.Pause

The Pause Control Button

Press this button once to stop all massage functions and pause the chair to idle mode. And then the orange light would be flashing, along with stopping the timing function.

Press this button again to resume the original massage function, and continue timer function.

4.Full Body Auto Program

If one of the six auto programs is activated, the massage chair will automatically do a height scan for the back. At the same time, the user can manually adjust the massage height to the preferred level.

The followings are the auto programs available:

- Rejuvenate
- Neck/Shoulder
- Lower Back
- Relax
- Stretch
- Sleeping

How to use

5. ▲ / ▼

Manual Backrest Up/Down Control Buttons

When the spot or partial massage functions are activated, these buttons move the massage rollers up or down. Long press the buttons to manually adjust the massage rollers, and release the button when the desired position is achieved.

Note: when the auto massage program is activated, the massage chair will automatically do a height scan for the back. Press these two buttons in 10 seconds after scanning, in order to adjust the shoulder height slightly if needed.

6. ◀ / ▶

Width Adjustment Buttons for Tapping, Shiatsu, Clapping, Rolling Modes (3 Widths)

After activating the Tapping, Shiatsu, Clapping, or Rolling function, press the left button to narrow the width, and press the right button to widen the width.

7.

The Spot/ Partial Back Massage

When a manual massage function is activated, press this button to start the functions in the cycle.

Press this button once to start the spot back massage.

Press this button twice to start the partial back massage, which takes the original position as the centre to perform partial massage with the width of approx. 3.2 in.

Press the button again to resume full back massage function.

8.

Massage Manipulation Control Button

After activating the massage manipulation control function on the full back, every time the button is pressed, six massage manipulations: Kneading, Swedish, Tapping, Shiatsu, Clapping, Rolling, that can be adjusted in a cycle.

Press the button again to stop the massage manipulation control function.

How to use



Message Mode Control Button

After activating one message mode control function, press once to start the Mode#1 message, press again to start the Mode#2 message.

For example, when activating Kneading Mode, press once to start Kneading#1 message, press again to start the Kneading#2 message.



Air Squeeze Control Button

After activating the air massage function on the full body, calves, arms, and shoulders, every time the button is pressed, the massage parts can be adjusted in the cycle.

Press the button again to stop the air massage function.



The Adjustment Button for Kneading Strength (5 Levels)

After activating the kneading function, every time the button is pressed, the strength can be adjusted by 5 levels in a cycle.



The Speed Adjustment Button for Swedish, Tapping, Shiatsu and Clapping Modes (5 Levels)

After activating the Swedish, Tapping, Shiatsu or Clapping function, every time the button is pressed, the speed can be adjusted by 5 levels in a cycle.



Air Pressure Strength Control Button (5 Levels)

After activating the auto massage function or manual air massage function, every time the button is pressed, the air pressure strength can be adjusted by 5 levels from weak to strong in a cycle.

How to use



Backrest Angle Adjustment Control Button

Press this button once to adjust the backrest sit up to a comfortable position. Press again to confirm the position.



Backrest Angle Adjustment Control Button

Press this button once to adjust the backrest reclined to a comfortable position. Press again to confirm the position



Waist Heating Control Button


Press this button once to activate the heating function for lumbar area. Press this button again to stop the heating function.



Foot Roller Function Start/Stop Control Button

Press this button once to activate the foot rolling function under the feet, and the foot roller icon will be lighted.

Press this button again to stop the foot rolling function, and the foot roller icon will be darkened.

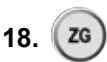

Press this button once to activate the zero gravity function #1, press twice to activate the zero gravity function #2, and press again to stop the zero gravity function.

Cleaning and maintenance

1. Cleaning of the synthetic leather

• Wipe the synthetic leather with a soft dry cloth.

- Do not clean the product with chemicals such as thinners, gasoline and alcohol.
- When using a leather maintenance product (cleaning cloth) sold on the market, follow its directions for use.
- If the synthetic leather is very dirty, wipe it with the method below.

- ① Put a piece of soft cloth into water or 3% to 5% neutral detergent contained hot water and wring it out.
- ② Wipe surface of the synthetic leather with the soft cloth.
- ③ Wipe the detergent with a piece of cloth once soaked in clean water and wrung out.
- ④ Wipe with a piece of wrung soft cloth.
- ⑤ Have the synthetic leather take natural air drying.

• If it is difficult to remove the stains on the synthetic leather, soak a piece of melamine foam bought from the market into a neutral detergent and use it wipe the product.

- Do not dry the surface by a hair drier.
- Surface of the synthetic leather may be dyed by fabric colors; therefore, be careful when wiping the product with denim or colored cloth.
- Synthetic leather may discolor, so the position in contact with the hair dye used should be covered by a towel.



2. Cleaning of the plastic parts

1) Put a piece of soft cloth into water or neutral detergent contained hot water, wring it out and wipe the plastic parts.

- Do not clean the product with chemicals such as thinners, gasoline and alcohol.

2) Wipe with a piece of cloth once soaked in clean water and wring out.

- Wring the cloth out before using it to wipe the remote controller.

3) Have the plastic parts air dried



Troubleshooting

This page summarizes the most common problems you could encounter with the product. If you are unable to solve the problem with the information below, contact service center immediately.

If the problem encountered is not listed, please try the following suggestions:

1. Turn off the product and unplug it.
2. Wait for 60 minutes.
3. Plug the product back to the recommended outlet and restart.
4. If the problem persists, contact the authorized service center.


Problem	Possible Cause	Suggestion
The product does not work	The product is not plugged in properly.	Ensure the voltage on the product corresponds to the local mains voltage and then plug in again. Make sure the outlet is properly working.
The product stopped working	The product has just finished its auto timer session and has automatically switched to standby mode.	If you want another session, press the Power button again.
The product stopped working	You may have used the product continuously for 45 minutes where the over heat protection has been activated.	Unplug the product and let it cool down for 60 minutes before you use it again.
The product does not work	An excessive load has been applied on the product.	Reduce the excessive load and start again.
The product is abnormally warm	The product is likely to be overused.	Unplug the product and let it cool down before using it again.
Noise can be heard from the product	Noise is made by the mechanical parts of the product.	This is normal and does not mean that the product is faulty.

Grounding Instructions

This product must be grounded. If it should malfunction or break down, grounding provides a path of least resistance for the electric current to reduce the risk of electric shock. This product is equipped with a cord with an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

Danger-Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or service technician if the product will not fit the outlet, have a proper outlet installed by a qualified electrician.

This product is for use on a nominal 120V circuit and has grounding plug that looks like that plug illustrated below. Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.

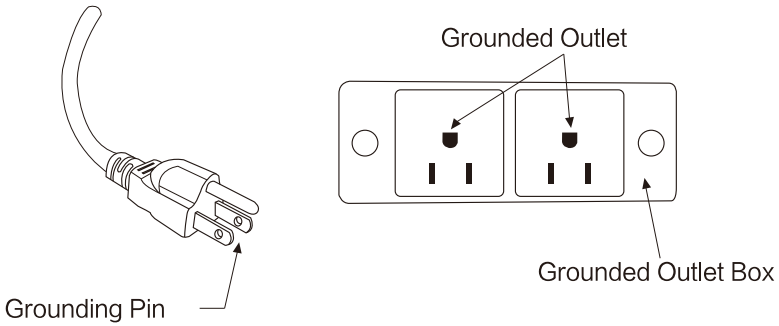

Chair Specifications

Description	Specifications
Model No.	Osaki Vista
Rated Voltage	110-120V~
Rated Frequency	60Hz
Rated Power Consumption	80W
Timer	5/10/15/20/25/30 minutes
Rated Time	15 minutes
Dimensions(L x W x H)	Upright: 58.7 x 29.5x 43.7 in
	Reclined: 70.5 x 29.5x 31.9 in
Weight	Gross Weight: 201.9 lbs
	Net Weight: 154.3 lbs
Length of Wire	Controller wire: 32"
	Power supply wire: 70.9"
Usage Condition	Environment temperature: 32°F~95°F
	Contrasting humidity: 20-80RH
Storage Condition	Storage temperature: 32°F~95°F
	Storage humidity: 20-80RH
Safety Feature	Equipped with overheated and power
Maximum body weight of user	264 lbs

This equipment contains a certified radio transmission module:

FCC ID : 2A006-WLT3266

