


MAKING POWER CONNECTIONS IN YOUR NEW LIFT CHAIR


Model PR505-M26

1. Install two 9 volt alkaline batteries in the inline battery holder as shown. The battery holder is zip tied (white) to the rear of the metal frame. Cut the white zip tie and extend the cable under and out behind the chair.

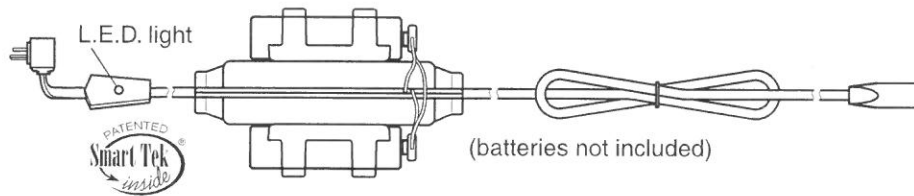

2. Remove the transformer from the transformer box follow steps 1, 2 and 3 on page 7 to connect the transformer to the chair and AC outlet.
3. Pull the hand control out of the magazine pocket and secure the cord into the quick release cable clamp located above the pocket on the arm of the chair.
4. Place the chair in the desired position in the room, leaving the chair a safe distance from the wall.
See chair location on page 7 for the proper distance from the wall.
5. Adjust the leg levelers on the bottom of the chair if necessary. Some high pile carpets can obstruct the footrest operation by rubbing against it. Adjust the levelers to a height above the pile.


AutoDrive 3.0
CUSTOMIZED COMFORT

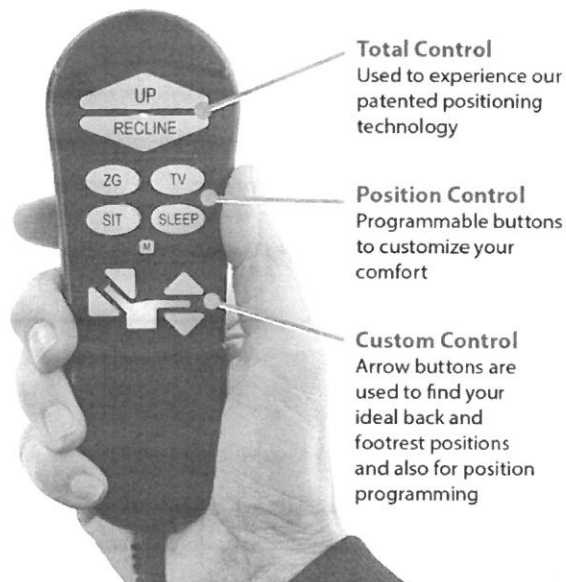

AUTODRIVE HAND CONTROL OPERATION

1. **Getting into your lift chair.** While standing next to your chair, press button (S7) and raise the chair to a height you can sit against. This height will vary from person to person depending on the individual's needs. Raise the chair high enough so you do not have to flop down into the chair. **Flopping down into the chair will damage it and could cause personal injury.** Press button (S5). As the chair continues downward, adjust yourself into a position that is comfortable to you. Continue holding button until chair is in the seated position (chair firmly on the floor), then release the button. You can also press buttons (S6, S8, S9, S10) and the chair will move to those positions.
2. **Getting out of your chair.** Press button (S7) and raise the chair to a height that enables you to easily stand. Release the button. Return the chair to the seated position (S5) (chair firmly on the floor) after getting out of the chair, so children, pets or obstructions do not get under the chair. Repeat no. 1 when you want to sit down.


Micro Adjustments

1. **To recline the back of the chair,** press button (S1) and hold until the back reaches the desired position. Then release the button.
2. **To bring the back of the chair to an upright position,** press button (S2) and hold until the back reaches the desired position. Then release the button.
3. **To raise the foot rest and seat,** press button (S3) and hold until the foot rest reaches the desired position. Then release the button.
4. **To lower the foot rest and seat,** press button (S4) and hold until the footrest closes. Then release the button.

Note: The motors are set to shut off when they reach their fully extended or fully retracted position if the selected button is not released. If you should go past your desired position, select the button to bring you back to your desired position. You can repeat the process as often as you like.

Programming the Memory Buttons S5, S6, S8 and S10

Press one of the preprogrammed buttons (S5, S6, S8, S10) and hold until the chair stops. Press buttons (S1 thru S4) to micro adjust those positions to suit your needs. Use the following procedure to reprogram (S5, S6, S8, S10) buttons after you have the chair in your desired position.

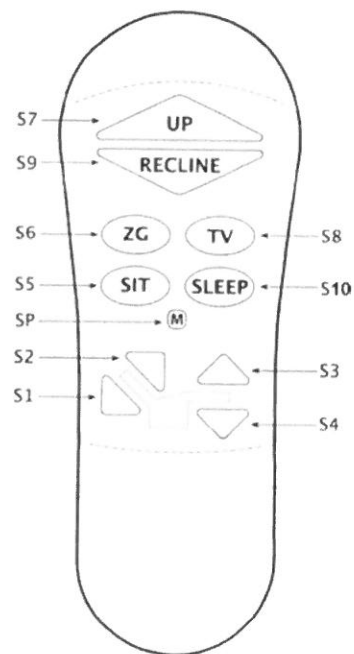

1. Press and hold program button (SP). The back light goes off while pressed.
2. In about 3 seconds the backlight will start flashing (one flash)
3. Within 5 seconds, press one of the memory buttons (S5, S6, S8, S10) to save the position to the button pressed. The back light will return to non-flashing position indicating that the new position is stored.

If a programmable memory button is not pressed within 5 seconds, or a non-programmable button is pressed, then the programming sequence is aborted and you will need to repeat steps 1 to 3 above. You may reprogram the memory buttons as often as you like.

Factory Reset

Resets all the programmed buttons (S5, S6, S8 and S10) back to the factory settings.

1. Press and hold program button (SP). The backlight goes off while button is pressed.
2. In about 4 seconds the backlight will start flashing.
3. Within 5 seconds, press button (S1). The backlight will change to non-flashing.
4. Within another 5 seconds, press button (S4). Factory reset is done.

If button (S1) is not pressed within 5 seconds, or a different button is pressed, then the programming sequence is aborted and you'll need to repeat steps 1 to 4 above.

Hand control disable

Hand control disable, disables all functions on the hand control

1. To disable, press and hold the program button (SP) for 4 seconds until the backlight starts flashing. Then press button (S7).
2. To enable, press and hold the program button (SP) for 4 seconds until the backlight starts flashing. Then press button (S7).

Function Lock

Should you ever want to limit the button function of your hand control, you can use the function lock feature to disable all button function except for buttons (S7 and S9).

1. To lock function, press and hold the program button (SP) for 4 seconds until the backlight starts flashing.
2. Press button (S1), press button (S2), press button (S3), press button (S6).
3. To unlock function, press and hold the program button (SP) for 4 seconds until the backlight starts flashing.
4. Press button (S1), press button (S2), press button (S3), press button (S6).