How-To:

Remove Ventura DLX Shroud



1. First step is to remove the seat. Pull up on the lever and lift to remove.



2. Place the removed seat aside. Now the shroud is exposed.



3. Now locate the seam that separates the two parts of the shroud.



4. Place your hands 2 inches away from the seam. One hand on the front section, the other on the rear.



5. To remove, place pressure inward on the rear section, and pull outward and away on the front. This will cause the latch to disengage. (See step 8 for picture of latch)



6. Once one side latch disengages, the shroud will come off.



7. Please see below for a picture of the latch that holds the shroud together.



8. Note that the batteries will appear to be hooked up. You may need to plug in the blue cable on the left and white cable on the right.



9. The batteries will need to be connected by these two plugs to get the unit running.



10. Connect those two cables together.



11. You will now have power to the unit, and can put the unit back together.

