

INVACARE®

Matrix® Seating Series



Yes, you can.®

INVACARE®

Matrix® Seating Series

With the number of new seating and positioning products available today, it is only natural to choose the products that you know best. Invacare Corporation and Motion Concepts are joining forces and combining product portfolios to offer you the Invacare Matrix Seating Series- simple and yet highly effective solutions to meet the widest range of user needs. This catalog is your source for the Invacare Matrix Seating Series.

Invacare is committed to providing you with products that exceed your expectations. As the landscape of seating changes, Invacare's dedication to innovation, quality and, most of all, your comfort will stay the same.

INVACARE® MATRX® BACK SUPPORTS

Invacare® MatrX® Elite Back	E2620	4
Invacare® MatrX® Elite Deep Back	E2620	5
Invacare® MatrX® PB Back	E2615/E2616	6
Invacare® MatrX® PB Deep Back	E2615/E2616	7
Invacare® MatrX® Easy Set Hardware		8-9
Invacare® MatrX® PCS Back	E2620/E2621	10
Invacare® MatrX® Personal Back™ 10 Back	E2615/E2616	11
Invacare® MatrX® Personal Back™ 10 Plus Back	E2615/E2616	11
Invacare® MatrX® Back Support Specifications		12-13

INVACARE® MATRX® SEAT CUSHIONS

Invacare® MatrX® Vi Cushion	E2607/E2608	16
Invacare® MatrX® Kid*ab*ra Vi Cushion	E2607	16
Invacare® MatrX® Flovair™ Cushion	E2607/E2608	17
Invacare® MatrX® PS Cushion	E2605	17
Invacare® MatrX® Stabilite™ Cushion	E2605/E2606	18
Invacare® MatrX® Flo-tech™ Cushion	E2622/E2623	19
Invacare® MatrX® AirFlo™ Lo Contour Cushion	E2622/E2623	20
Invacare® MatrX® AirFlo™ Gentle/Max Cushion	E2624/E2625	21
Invacare® MatrX® Seat Cushion Specifications		22

INVACARE® MATRX® GENERAL USE BACK SUPPORTS AND SEAT CUSHIONS

Invacare® Versair™ Back	E2611/E2612(Pending)	23
Invacare® MatrX® Genera Back	E2611	23
Invacare® Propel™ Back	E2611	23
Invacare® Absolute™ Cushion	E2601	24
Invacare® Single Density Cushion	E2601/ E2602	24
Invacare® Comfort-Mate Extra Cushion	E2603/ E2604	24
Invacare® MatrX® Accessories		25-26



 **matrx**[®]
SEATING SERIES

**INVACARE**
Yes, you can.[®]

INVACARE®

Matrix® Elite Back

The Invacare® Matrix® Elite Back is designed to provide precise orientation within the wheelchair for optimal postural support. The attractive, lightweight design appeals to users with active lifestyles.

HCPCS Code
E2620

Invacare® Matrix® Elite Back

Model no. PBE

Features and Functional Benefits

Outer cover:

- Oversized cover/foam for extra comfort and protection
- Moisture resistant and breathable
- Includes removable lumbar positioning pad
- Optional privacy flap available

Hardware:

- "Easy Set" hardware installed and adjusted in minutes – adjustments can be made while user is seated in the wheelchair
- Refer to hardware page

Back shell:

- Stylish and breathable back shell
- Lightweight - most models under 3 lb.

Sacral/lumbar support:

- Includes removable positioning pad



STANDARD

Width	12"– 20"
Height	10"
	12"– 20"
	12"– 20"
	14"– 14"
	12"– 20"
	12"– 20"
	14"– 20"
20" Full Back*	16"– 20"
Weight capacity	300 lb.

* Optional Full back style - no scapular cut-outs



FIXED
2.5" of vertical adjustment on back pan
Mount inside back shell without interfering with depth/angle adjustment
Available in standard 5" h x 7.5" l and pediatric 3" h x 5.5" l sizes



OFFSET FIXED
Provides an additional 3" of vertical position (higher or lower) relative to mounting slots
Available 5" h x 7.5" l



SWING-AWAY
2.5" of vertical adjustment on back pan
Mount inside back shell without interfering w/depth/angle adjustment
Easy push button release 4 pad sizes



OFFSET SWING-AWAY
Provides an additional 3" of vertical position (higher or lower)
Mount inside back shell without interfering w/depth/angle adjustment
Easy push button release, 4 pad sizes

Swing-Away and Offset Swing-Away sizes:
XS: 3.25" h x 4.25" l, **S:** 3.5" h x 5.25" l, **M:** 3.75" h x 5.75" l, **L:** 4.25" h x 6.5" l

INVACARE®

Matrix® Elite Deep Back

The Invacare® Matrix® Elite Deep Back is designed with lateral supports to provide optional trunk support while allowing clearance for the shoulders and hips. The attractive, lightweight design appeals to users with active lifestyle.

HCPCS Code
E2620

NEW

Invacare® Matrix® Elite Deep Back

Model no. EDB

Features and Functional Benefits

Outer cover:

- Oversized cover/foam for extra comfort and protection
- Moisture resistant and breathable
- Includes removable lumbar positioning pad
- Optional privacy flap available

Hardware:


- "Easy Set" hardware installed and adjusted in minutes – adjustments can be made while user is seated in the wheelchair
- Refer to hardware page

Back shell:

- Stylish and breathable back shell
- Lightweight - most models under 4 lb.

Sacral/lumbar support:

- Includes removable positioning pad

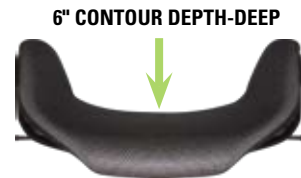

**SACRAL/
LUMBAR
SUPPORT**

DEEP

Width	14" – 20"
Height	12", 16", 18", 20"
Weight capacity	300 lb.

LATERAL PADS



Lateral Pad sizes: (for Elite Back)

XS: 3.25" high x 4.25" long

S: 3.5" high x 5.25" long

M: 3.75" high x 5.75" long

L: 4.25" high x 6.5" long

HEADREST ADAPTER



Allows the headrest to be mounted directly onto Invacare Matrix Elite backs

Available in 3 sizes

PRIVACY FLAP



INVACARE®

Matrix® PB Back


The Invacare® Matrix® PB Posture Back is contoured to provide midline support and positioning. The extensive range of sizes provides exceptional support and comfort for a variety of customers.

HCPCS Code
E2615/E2616



BACK SHELL

5" CONTOUR DEPTH-STANDARD

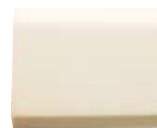

**SACRAL/
LUMBAR
SUPPORT**

Invacare® Matrix® PB Back

Model no. MPB

Features and Functional Benefits

Foam:

- Molded foam wraps around shell for maximum comfort and protection

Outer cover:

- Moisture resistant and breathable
- Available in black or Kid*ab*ra print
- Optional privacy flap is available

Sacral/lumbar support:

- Includes removable positioning pad

Back shell:

- Contoured shell provides exceptional midline support and positioning
- Durable, lightweight ABS back shell includes built-in plate for headrest attachment

Hardware:

- "Easy Set" hardware is easily installed and adjusted in minutes – adjustments can be made while user is seated in the wheelchair
- Refer to Easy Set Hardware page

STANDARD

Width 14", 16", 17", 18", 20"

Height

12" 14"–20"

16" 14"–20"

20" 16"–20"

Weight capacity 300 lb.



Kid*ab*ra fabric optional

PRIVACY FLAP



FOAM-IN-PLACE KIT



Individually molded insert produced in minutes with user seated in back
Optional high stretch cover available

VISCO FOAM UPGRADE



Laminate of .5" high resilient foam and .625" Visco Foam
Available on Invacare Matrix PB and Elite Backs
Molded foam is not available on Invacare Matrix PB Back with this option

HEAVY-DUTY

Width 20", 22", 24"

Height

16" 20", 22", 24"

20" 20", 22", 24"

Weight capacity 500 lb.

EXTRA WIDE

Width 24"–30"

Height

16" 24"–30"

20" 24"–30"

Weight capacity 500 lb.

INVACARE®


Matrix® PB Deep Back

The Invacare® Matrix® PB Deep back is contoured to provide increased midline support and positioning. The extensive range of sizes provides exceptional comfort and support for a wide range of user needs.

HCPCS Code
E2615/E2616



BACK SHELL

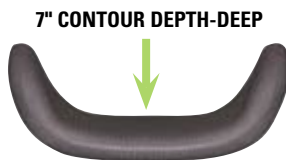

7" CONTOUR DEPTH-DEEP



**SACRAL/
LUMBAR
SUPPORT**

Invacare® Matrix® PB Deep Back

Model no. PBD (Deep)

Features and Functional Benefits

Foam:

- Molded foam wraps around shell for maximum comfort and protection

Outer cover:

- Moisture resistant and breathable
- Available in black or Kid*ab*ra print
- Optional privacy flap is available

Sacral/lumbar support:

- Includes removable positioning pad

Back shell:

- Contoured shell (standard/deep) provides exceptional midline support and positioning
- Durable, lightweight ABS back shell includes built-in plate for headrest attachment

Hardware:

- "Easy Set" hardware is easily installed and adjusted in minutes – adjustments can be made while user is seated in the wheelchair
- Refer to Easy Set Hardware page

DEEP

Width	14", 16", 17", 18", 20"
Height	
12"	14" – 20"
16"	14" – 20"
20"	16" – 20"

Weight capacity 300 lb.



Kid*ab*ra fabric optional

PRIVACY FLAP



FOAM-IN-PLACE KIT



Individually molded insert produced in minutes with user seated in back
Optional high stretch cover available

VISCO FOAM UPGRADE



Laminate of .5" high resilient foam and .625" Visco Foam
Available on Invacare Matrix PB and Elite Backs
Molded foam is not available on Invacare Matrix PB Back with this option

HEAVY-DUTY

Width	20", 22", 24"
Height	
16"	20", 22", 24"
20"	20", 22", 24"

Weight capacity 500 lb.

EXTRA WIDE

Width	24" – 30"
Height	
16"	24" – 30"
20"	24" – 30"

Weight capacity 500 lb.

INVACARE® Matrix® Hardware

The Invacare® Matrix® Easy Set hardware is easily installed in minutes, and all adjustments can be made with the user seated in the wheelchair.



Invacare® Matrix® Hardware

Features and Functional Benefits

- Quick release for easy portability
- Infinite adjustment of depth (4"), angle (40°) and rotation
- Fits 3/4", 7/8" and 1" diameter canes


Styles

- Invacare Matrix backs: Elite, Elite Deep, PB and PB Deep are interchangeable on the Easy Set Hardware




Heights

- Several back heights available to provide support from low to upper thoracic

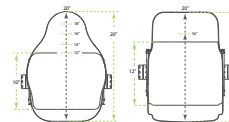

Widths

- Backs widths from 12" - 30" are available to provide exact fit for each user

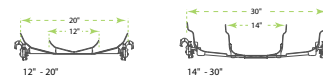


Contour Depth

- Choice of 4 contour depths for appropriate midline support requirements




STANDARD MOUNTING PINS

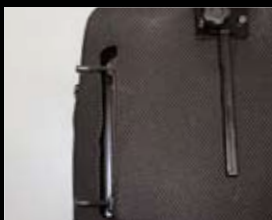


Standard mounting pins (included with all Invacare Matrix PB/Elite backs) allow the Invacare Matrix PB back to fit chairs 1" narrower or same model width
Allows Elite Back to fit chairs the same width

LONGER MOUNTING PINS

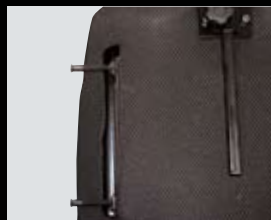

Longer mounting pins (included with all Invacare Matrix PB/Elite backs) allow the Invacare Matrix PB and Elite Backs to fit chairs 1" wider

EXTENDED MOUNTING PINS

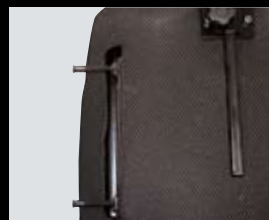

Extended mounting pins available for Invacare Matrix PB & Elite Backs (Not available on Heavy Duty PB backs)
Allows the back to fit chairs up to 2" wider

EXTENDED RANGE MOUNTING HARDWARE UPGRADE



Provides 1 3/4" depth adjustment in front or behind canes
Available on all models of Posture Backs and Elite Backs – Easy Set, Heavy-Duty or Fixed Space Saver hardware

INVACARE®
Matrix®
 Hardware

EASY-SET HARDWARE ADJUSTMENTS



DEPTH
 Adjusts 2" behind back canes



DEPTH
 Adjusts 2" in front of back canes



ANGLE
 20° posterior range of back angle
 adjustment in front of back canes



ANGLE
 20° anterior range of back angle
 adjustment behind back canes



ROTATION
 Back may be oriented
 to accommodate pelvic/
 trunk rotation

**HEAVY DUTY "EASY SET"
 MOUNTING HARDWARE**



Standard with Heavy-Duty
 Posture Backs
 Recommended for users over
 300 lb., high tone or heavy use

**FIXED SPACE SAVER
 MOUNTING HARDWARE**



Requires tools for removal
 Saves space for tighter width
 applications
 Fully adjustable

**SPLIT MOUNTING
 HARDWARE**



Provides clearance of mid back
 cane rigidizer bars
 Not recommended for Heavy-Duty
 Posture backs
 Available as a no charge option

**EXTENDED SHEAR REDUCTION
 MOUNTING HARDWARE**



Allows all Invacare Matrix backs
 to be mounted directly to Motion
 Concepts TRx seating with Power
 Recline and ESR

INVACARE®

Matrix® PCS Back

The Invacare® Matrix® PCS positioning back has applications for both manual and power chair users, for upright seating systems and for tilt and/or recline systems. The back was designed with features that synergize to provide the exceptional Positioning, Comfort and Stability required by wheelchair users every day.

HCPCS Code
E2620/E2621



Spacer fabric:
Regulation of
heat and moisture

Invacare® Matrix® PCS Back

Model no. ITPCSR, ITPCST

Features and Functional Benefits

Outer cover-Spacer Fabric mesh:

- Improved airflow between user and back
- Optimal management of heat and moisture

Foam:

- 1" soft foam provides comfort with pressure redistribution

Back Shell:

- Lightweight aluminum shell with 3" of contour depth
- Scapular cutouts to facilitate upper extremity function
- Sacral support facilitates stability for function
- Pre-drilled headrest holes allow mounting of standard headrest mounting hardware

Hardware:

- Unique hardware that securely mounts between canes without any unwanted play
- Adjustable in height and angle up to 30°, depth up to 1.5"
- Quick release latch for easy removal and remounting of back during transport
- Fits 7/8" and 1" diameter back canes
- One tool for all installations and adjustments



Pre-drilled headrest holes


Aluminum Shell

SPECIFICATIONS

Width	14" – 24"
Height	16" (Regular); 20" (Tall)
Depth adjustment	1 1/2" to the back
Angle adjust	up to 30°
Product weight	4.7 lb.
Weight capacity	
14" – 21"	300 lb.
22" – 24"	500 lb.



Height and width adjustment for
optional thoracic lateral supports
and pelvic stabilizers

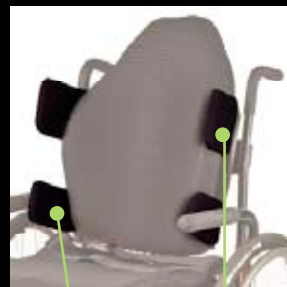

Pelvic
stabilizers

Thoracic
lateral
supports

Optional Thoracic Lateral Supports

- Provide improved trunk control and stability
- Adjustable in height and width for accommodating fixed postural asymmetries, correcting flexible postural deformities or adjusting over time to support incremental postural changes

Optional Pelvic Stabilizers

- Promote pelvic stability
- Adjustable in height and width for accommodating fixed postural asymmetries, correcting flexible postural deformities or adjusting over time to support incremental postural changes

INVACARE®

Matrix® Personal Back™ 10 Back

Personal Back™ 10 Plus Back

The Invacare® Matrix® Personal Back™10 and Personal Back™10 Plus Backs combine simplicity of design with effectiveness of application. The highly popular Invacare Personal Back line continues to provide the comfort, postural support and spinal alignment that you have come to expect.

HCPCS Code
E2615/E2616



Personal Back10 Back



Personal Back10 Plus Back

Invacare® Matrix® Personal Back™ 10 and Invacare® Matrix® Personal Back™ 10 Plus Backs

Model no. PBR, PBT

Model no. PPR, PPT

Features and Functional Benefits

Outer cover:

- Stretchable and breathable cover to allow additional foam inserts

Soft foam:

- Foam lining held in place with hook and loop fasteners provides comfort and support
- Allows greater flexibility for addition of foam
- Visco foam upgrade available

Back shell:

- Regular and tall heights to accommodate different user postural requirements
- Lightweight aluminum shell that mounts quickly and easily to the wheelchair
- Pre-contoured shell provides excellent contact, improved lateral stability and support
- Deep lateral contour provides increased midline support without interfering with upper extremity range of motion
- Pre-drilled headrest pattern allows mounting of standard headrest mounting hardware

Sacral/lumbar support

- Includes sacral support that is easily adjustable to meet individual needs



SACRAL/
LUMBAR
SUPPORT

Hardware

- Invacare® Quick Set™ fits canes 7/8 or 1" in diameter
- Can be easily mounted in front of or behind the canes
- Quick release levers for easy removal
- Allows increased adjustability in height, depth (2" in front or behind the canes) and back angle (up to 24°)

STANDARD

Width	Regular	Tall
13" – 14"	14"	16"
15" – 26"	16"	19"
Weight capacity		
13" – 20"	250 lb.	
22" – 26"	500 lb.	

PLUS

Weight capacity		
13" – 20"	250 lb.	
22" – 26"	500 lb.	



FOAM-IN-PLACE KIT

Individually molded insert produced in minutes with user seated in back
Optional high stretch cover available

INVACARE®

Matrix® Specifications

Backs



Invacare® Matrix® Elite/Deep Back

Invacare® Matrix® Elite Back Support

Back height	10", 12", 14" (available in 12", 13" and 14" widths), 16", 18", 20"								
Width	12"	13"	14"	15"	16"	17"	18"	19"	20"
A. Outer Width	9.75"	10.25"	12.5"	13.5"	14.5"	15.5"	16.5"	17.5"	18.5"
B. Inner Width	8.5"	9.5"	11.5"	12.5"	13.5"	14.5"	15.5"	16.5"	17.5"
C. Contour Width	3"	3"	3"	3"	3"	3"	3"	3"	3"

Invacare® Matrix® Elite Deep Back

Back height	12", 16", 18", 20"								
Width	14"	15"	16"	17"	18"	19"	20"		
A. Outer Width	13"	14"	15"	16"	17"	18"	19"		
B. Inner Width	11.5"	12.5"	13.5"	14.5"	15.5"	16.5"	17.5"		
C. Contour Width	6"	6"	6"	6"	6"	6"	6"		
Depth adjustment	4" (2" in front or behind)								
Angle adjustment	40°								
Product weight	3 lb. Standard/4 lb. Deep								
Weight capacity	300 lb.								
Limited warranty	2 years								

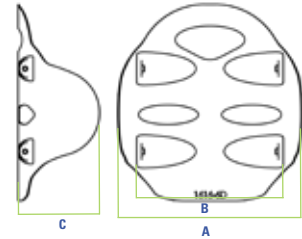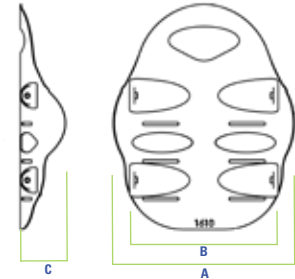



Invacare® Matrix® PB/Deep Back

Invacare® Matrix® PB Back

Back height	12", 16", 20"				
Width	14"	16"	17"	18"	20"
A. Outer Width	11.75"	13.75"	14.75"	15.75"	17.75"
B. Inner Width	10.5"	12.5"	13.5"	14.5"	16.5"
C. Contour Width	5"	5"	5"	5"	5"


Invacare® Matrix® PB Deep Back

Back height	12", 16", 20"				
Width	13"-15"	15"-17"	16"-18"	17"-19"	19", 20"
A. Outer Width	12"	14"	15"	17"	19"
B. Inner Width	12"	14"	15"	16"	18"
C. Contour Width	7"	7" 7"	7"	7"	
Depth adjustment	4" (2" in front or behind)				
Angle adjustment	40°				
Product weight	4 lb. Standard/6 lb. Deep				
Weight capacity	300 lb. Standard/500 lb. Deep				
Limited warranty	2 years				

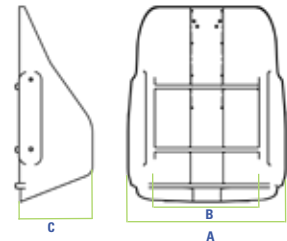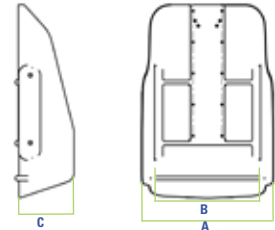


Invacare® Matrix® PB Heavy-Duty Back

Back height	16", 20"						
Width	20", 21"	22", 23"	24", 25"	26", 27"	28", 29"	30"	
A. Outer Width	18.5"	20.5"	22.5"	24.5"	26.5"	28.5"	
B. Inner Width	17.5"	19.5"	21.5"	23.5"	25.5"	27.5"	
C. Contour Width	5"	5"	5"	5"	5"	5"	
Depth adjustment	4" (2" in front or behind)						
Angle adjustment	40°						
Product weight	6 lb. Standard/7 lb. Deep						
Weight capacity	500 lb. Standard/Deep						
Limited warranty	2 years						

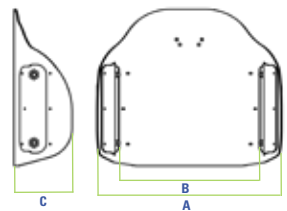


Invacare® Matr[®] PB Deep Heavy-Duty Back

Back height	16", 20"					
Width	20", 21"	22", 23"	24", 25"	26", 27"	28", 29"	30"
A. Outer Width	19"	21"	23"	25"	27"	29"
B. Inner Width	18.5"	20.5"	22.5"	24.5"	26.5"	28.5"
C. Contour Width	7"	7"	7"	7"	7"	7"
Depth adjust	4" (2" in front or behind)					
Angle adjust	40°					
Product weight	6 lb. Standard/7 lb. Deep					
Weight capacity	500 lb. Standard/ Deep					
Limited warranty	2 years					

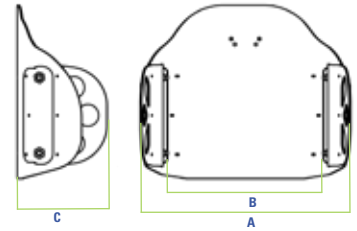

Invacare® Matr[®] PCS Back


Back height	16" (Regular); 20" (Tall)										
Widths	14"	15"	16"	17"	18"	19"	20"	21"	22"	23"	24"
A. Outer Width	12.5"	13.5"	14.5"	15.5"	16.5"	17.5"	18.5"	19.5"	20.5"	21.5"	22.5"
B. Inner Width	7.55"	8.55"	9.55"	10.55"	11.55"	12.55"	13.55"	14.55"	15.55"	16.55"	17.55"
C. Contour Width	3"	3"	3"	3"	3"	3"	3"	3"	3"	3"	3"
Depth adjustment	1.5" to the back										
Angle adjust	up to 30°										
Product weight	4.7 lb.										
User weight limits											
14" – 21" widths	300 lb.										
22" – 24" widths	500 lb.										
Limited warranty	2 years										

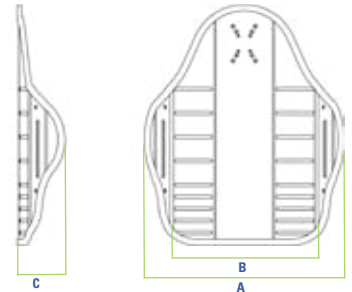

Invacare® Matr[®] Personal Back™10/Personal Back™10 Plus Back


Invacare® Matr[®] Personal Back™10 Back

Back height											
13" – 14" widths	14", 16"										
15" – 26" widths	16", 19"										
Width	13"	14"	15"	16"	17"	18"	19"	20"	22"	24"	26"
A. Outer	12.13"	13.13"	14.13"	15.13"	16.13"	17.13"	18.13"	19.13"	21.13"	23.13"	25.13"
B. Inner	9.5"	10.5"	11.5"	12.5"	13.5"	14.5"	15.5"	16.50"	18.43"	20.43"	22.43"
C. Contour	5.14"	5.28"	5.41"	5.54"	5.68"	5.81"	5.94"	6.08"	6.60"	6.86"	7.13"
Product weight											
13" – 20" widths	5 lb.										
22" – 26" widths	10 lb.										
Weight capacity											
13" – 20" widths	250 lb.										
22" – 26" widths	500 lb.										
Limited warranty	2 years										

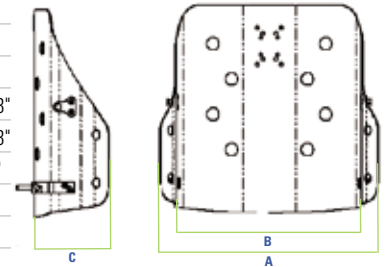

Invacare® Matr[®] Personal Back™10 Plus Back

Back height											
13" – 14" widths	14", 16"										
15" – 26" widths	16", 19"										
Width	13"	14"	15"	16"	17"	18"	19"	20"	22"	24"	26"
A. Outer	13"	14"	15"	16"	17"	18"	19"	20"	22"	24"	26"
B. Inner	9.5"	10.5"	11.5"	12.5"	13.5"	14.5"	15.5"	16.5"	18.43"	20.43"	22.43"
C. Contour	6.85"	6.99"	7.12"	7.25"	7.39"	7.52"	7.65"	7.79"	8.27"	8.54"	8.81"
Product weight											
13" – 20" widths	6 lb.										
22" – 26" widths	11lb.										
Weight capacity											
13" – 20" widths	250 lb.										
22" – 26" widths	500 lb.										
Limited warranty	2 years										

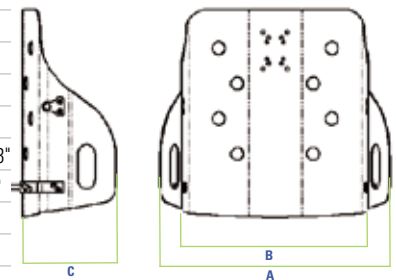

* Height, width and depth measurements are approximate





 **matrix**[®]
SEATING SERIES


Yes, you can.[®]

INVACARE®

Matrx® Vi Cushion

The Invacare® Matrx® Vi Cushion is designed to provide skin protection and positioning for users at moderate to high risk of skin breakdown. The unique three part foam construction targets skin protection to where it is needed most, and features improved stability, support and comfort.

HCPCS Code
E2607/E2608



Invacare® Matrx® Vi Cushion

Model no. MA(VVDD)-Vi

Features and Functional Benefits

Outer cover:

- Moisture resistant and breathable
- Includes slip resistant base, hook and loop fasteners, lifting strap
- Available in black and Kid*ab*ra print

Inner cover:

- Moisture resistant
- Easily wipes clean
- Zippered cover allows access for cushion modifications

Top layer:

- Soft (ILD) High Resilient (HR) foam for optimal comfort and support
- Pre-ischial contour provides stability to pelvis
- Abductor/adductor contour promotes midline leg positioning
- Made with Ultra-Fresh™ for antibacterial and odor protection

Visco foam insert:

- Molded visco-elastic foam ischial relief insert
- Anatomically targeted to ischial/sacral area
- Provides immersion to sensitive bony prominences

Bottom layer:

- Firm (ILD) High Resilient (HR) foam for support and durability
- Trochanteric shelf provides stability to hips
- Waterfall front edge
- Rear radius for sacral support



STANDARD

Width	14", 16", 18", 20", 22"
Depth	15" – 20"
A. Height at adductor	3.75"
B. Height at abductor	3.25"
C. Height at leg trough	2.5"
Weight capacity	300 lb.

HEAVY-DUTY

Width	18" – 30"
Depth	18" – 26"
A. Height at adductor	3.75"
B. Height at abductor	3.25"
C. Height at leg trough	2.5"
Weight capacity	600 lb.



Kid*ab*ra fabric standard on pediatric sizes

PEDIATRIC

Width	12", 14"
Depth	12", 14"
A. Height at adductor	2.875"
B. Height at abductor	2.75"
C. Height at leg trough	2"

Kid*ab*ra fabric standard
black fabric optional

DROP BASE



Includes built-in drop base, drop hooks, DuraClip fasteners

Allows adjustment of seat angle

PELVIC OBLIQUITY KIT



Easily installed on all Invacare Matrx Vi/ Kid*ab*ra-Vi cushions/bases

Accommodates .5" or 1" obliquities

CUSTOM MODIFICATIONS



Leg length discrepancies, width/depth reductions

Foam, inner and outer covers modified to your specifications

INVACARE®

Matrix® Flovair™ Cushion

The Invacare® Matrix® Flovair™ Cushion is designed to provide pelvic stability, leg positioning and pressure management for individuals with postural asymmetry and skin integrity concerns. The Invacare Matrix Flovair Cushion begins with the base for comfort, stability and function. This hybrid cushion captures the stability of foam, the pressure redistribution of air and the shear management of fluid.

HCPCS Code
E2607/E2608
**(Gentle Contour/
Max Contour)**



Invacare® Matrix® Flovair™ Cushion

Model no. ITFG, ITFM

Features and Functional Benefits

Outer cover:

- Moisture resistant and breathable
- Includes non-slip base, hook and loop fasteners and a lifting strap

Inner cover:

- Four way stretch to allow immersion
- Vapor permeable to allow air circulation while preventing fluid penetration

Fluid overlay:

- Works in tandem with the Invacare® ThinAir™ bladder to maximize surface area for optimal pressure redistribution
- Shear management as the two layers work in harmony without weight penalty
- Compensates for different body types as fluid/air layers envelop body contours

ThinAir bladder:

- Comfort layer assists in redistributing peak pressures away from bony prominences
- Helps manage shear forces

Positioning foam base:

- Soft foam allows immersion of pelvis for comfort and stability
- Medial and lateral thigh supports facilitate positioning and alignment of lower extremities
- Trochanter cutouts facilitate proper pressure redistribution and comfort. Firm foam inserts under the cutouts add support
- Posterior pelvic support provides pelvic control that is required for stability and function
- Sacral cutout provides additional comfort and support for those sitting in a posterior pelvic tilt
- Gentle or max contour choices based on need for gentle or high level of posture control

GENTLE CONTOUR

Width	15" – 24"
Depth	15" – 22"
A. Height at adductor	3.5"
B. Height at abductor	3.5"
C. Height at leg trough	2.87"
Weight capacity	
15" – 20"	300 lb.
21" – 24"	500 lb.

MAX CONTOUR

Width	15" – 20"
Depth	15" – 20"
A. Height at adductor	4.38"
B. Height at abductor	4.38"
C. Height at leg trough	2.88"
Weight capacity	
15" – 20"	300 lb.
21" – 24"	500 lb.


REMOVABLE LEG WEDGES



Allow lower seat-to-floor height for foot propellers

Accommodate limited hip flexion by opening up seat-to-back angle

PELVIC OBLIQUITY KIT

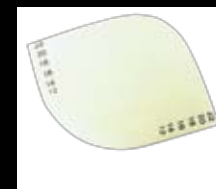

Easily installed on Invacare Matrix Flovair cushions
Accommodates 0.5" or 1" obliquities

Modifications

- Leg length cutouts
- 1" or 2": cutouts to accommodate leg length discrepancies

INVACARE®

Matrix® PS Cushion

The Invacare® Matrix® PS Cushion is designed to provide superior positioning, stability and comfort. The design features an anatomically contoured shape with a unique waffled ischial relief recess and an innovative reversible outer cover.

HCPCS Code
E2605/(E2606 pending)



Invacare® Matrix® PS Cushion

Model no. PS

Features and Functional Benefits

Reversible outer cover:

- Innovative reversible Dartex™ outer cover may be used smooth or fabric side out
- Dartex™ fabric is moisture resistant and breathable
- Hook and loop fastener strips

Inner cover:

- Moisture resistant
- Easily wiped clean

Cushion:

- Durable high resilient foam prevents "bottoming out"
- Anatomically molded shape for optimal positioning & comfort
- Rear radius for sacral support
- Waterfall front edge
- Made with Ultra-Fresh™ for antibacterial and odor protection

Ischial relief area:

- Waffled recess provides immersion of bony prominences
- Helps maintain pelvic position and prevent sliding

PEDIATRIC

Width	10", 12", 14"
Depth	12", 14", 16"
A Height at adductor	3.75"
B Height at abductor	3.5"
C Height at leg trough	2.5"
Weight capacity	150 lb.

STANDARD

Width	16", 18", 20"
Depth	16", 18", 20"
A Height at adductor	4"
B Height at abductor	3.5"
C Height at leg trough	2.5"
Weight capacity	300 lb.

HEAVY-DUTY

Width	21" – 30"
Depth	21" – 30"
A Height at adductor	4"
B Height at abductor	3.5"
C Height at leg trough	2.5"
Weight capacity	600 lb.

CUSTOM MODIFICATIONS



Leg length discrepancies, width/depth reductions
Foam, inner and outer covers modified to your specifications

CUSHION RIGIDIZER



Lightweight, slim profile solid seat insert
Contoured to eliminate "hammock effect" of sling upholstery

PELVIC OBLIQUITY KIT



Easily installed on Invacare Matrix PS cushions
Accommodates .5" or 1" obliquities

INVACARE®

Matrix® Stabilite™ Cushion

The Invacare® Matrix® Stabilite™ Cushion is developed to provide pelvic stability and leg positioning for patients with postural asymmetry. The Invacare Matrix Stabilite Cushion comes fitted with a patent-pending Invacare® ThinAir™ bladder, a unique comfort layer that assists in shear management. The Invacare Matrix Stabilite Cushion begins with the base for comfort, stability and function.

HCPCS Code
E2605/E2606
(Pending)



Invacare® Matrix® Stabilite™ Cushion

Model no. ITSG

Features and Functional Benefits

Outer cover:

- Moisture resistant and breathable
- Includes non-slip base, hook and loop fasteners and a lifting strap

Inner cover:

- Four way stretch to allow immersion
- Vapor permeable to allow air circulation while preventing fluid penetration

Invacare® ThinAir™ bladder:

- Comfort layer assists in redistributing peak pressures away from bony prominences
- Helps manage shear forces

Positioning foam base:

- Soft foam allows immersion of pelvis for comfort and stability
- Medial and lateral thigh supports facilitate positioning and alignment of lower extremities
- Trochanter cutouts facilitate proper pressure redistribution and comfort. Firm foam inserts under the cutouts add support
- Posterior pelvic support provides pelvic control that is required for stability and function
- Sacral cutout provides additional comfort and support for those sitting in a posterior pelvic tilt

SPECIFICATIONS

Width	14" – 24"
Depth	14" – 22"
A. Height at adductor	3.75"
B. Height at abductor	3.75"
C. Height at leg trough	2.87"
Weight capacity	
15" – 20"	300 lb.
21" – 24"	500 lb.

Modifications

- Leg length cutouts
- 1" or 2" cutouts to accommodate leg length discrepancies

CUSHION RIGIDIZER



Lightweight, slim profile solid seat insert

Contoured to eliminate "hammock effect" of sling upholstery

REMOVABLE LEG WEDGES



Allow lower seat-to-floor height for foot propellers

Accommodate limited hip flexion opens up seat-to-back angle

PELVIC OBLIQUITY KIT



Easily installed on Invacare Matrix Stabilite cushions

Accommodates 0.5" or 1" obliquities

INVACARE®

Matrix® Flo-tech™ Cushion

The Invacare® Matrix® Flo-tech™ Cushion uses a combination of molded foam and dual layer fluid sacs to provide the highest level of skin protection for users with high risk of skin breakdown. The Fit Kit components are available to address individual postural requirements.

HCPCS Code
E2622/E2623



Invacare® Matrix® Flo-tech Cushion

Model no. FT

Features and Functional Benefits

Outer cover:

- Moisture resistant and breathable
- Includes non-slip base, hook and loop fastener strips and a lifting strap
- Expandable cover facilitates use with Fit Kit components

Platilon liner:

- An additional layer of moisture protection

Fluid sac:

- Unique dual-layer fluid in ischial/sacral area to help prevent bottoming out
- Non-temperature sensitive fluid resists freezing
- No kneading or maintenance required

Foam base:

- High quality, comfortable, resilient foam base
- Molded in hook and loop fastener strips for secure attachment of Fit Kit components
- Pre-ischial contour, trochanteric support and sacral support

STANDARD

Width	14" – 20"
Depth	14" – 21"
A Height at adductor	3.375
B Height at abductor	3.375
C Height at leg trough	2.75
Weight Capacity	300 lb.

HEAVY-DUTY

Width	20" – 24"
Depth	18", 20", 22"
Weight capacity	500 lb.

PELVIC OBLIQUITY BUILD-UP

Accommodates obliquities up to 1". For correctable obliquities use under low side of pelvis and for fixed obliquities use under high side of pelvis

ADDUCTORS

FRONT - Provides increased adductor contour to thighs
REAR - Provides increased adductor contour to hips

SEAT WEDGES

Increases degree of pre-ischial contour angle to help prevent sliding

HEMI WEDGE

May be used to provide increased support to one lower extremity – useful for clients who use foot propulsion or with hip flexion limitations

ABDUCTOR

SMALL - Provides .75" increased height of abductor contour
LARGE - Provides 2" increased height of abductor contour


TAPERED WEDGES

Accommodate mild pelvic rotation, mild hip flexion discrepancies or increase adductor contour

FLUID SUPPLEMENT

May be placed to provide pelvic obliquity accommodation or increased sacral/ischial immersion

FIT KIT COMPONENTS:

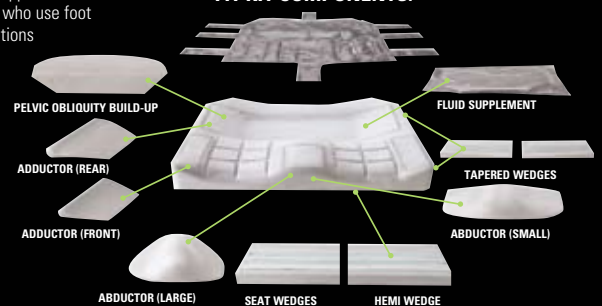

INVACARE®

Matrix® AirFlo™ Cushion

The Invacare® Matrix® AirFlo™ Cushion is designed to provide rehab solutions for comfort, stability and pressure redistribution, as well as addressing positioning needs. The Invacare Matrix AirFlo Cushion is constructed using a foam base and air flotation cells and is an excellent choice for individuals who have a history or are considered to be at very high risk of skin breakdown.

HCPCS Code

E2624/E2625 (Gentle/Max Contour)

E2622/E2623 (LoContour)



Invacare® Matrix® AirFlo™ Cushion

Model no. ILCAIR, IGCAIR, IMCAIR

Features and Functional Benefits

Outer cover:

- 4-way stretch to allow immersion
- Water resistant and breathable to dissipate heat and moisture
- Includes a slip resistant base, hook and loop fasteners, lifting strap and convenient front pouch for storage

Inner incontinence cover:

- Moisture proof and water resistant
- 4-way stretch to allow immersion
- Easily wipes clean
- Zippered cover allows for cushion modifications

Air flotation cells:

- 2" cells sit on top of a contoured foam base to provide optimal immersion and suspension of bony prominences similar to 4" cells
- Single valve adjustable air flotation cells
- Sits on top of the foam base to provide additional protection against bottoming out
- Inflation bulb easily operates with one hand

Foam base:

- Provides adequate postural support
- Contours provide comfort and positioning to enhance pelvic stability
- Available in Lo, Gentle or Max Contour



LO CONTOUR

Excellent choice for individuals who:

- Have good postural control
- Need to accommodate postural asymmetry, for example, windswept hips
- Need low profile for lateral transfer
- Foot propel

LO CONTOUR

Width	13" – 22"
Depth	15" – 20"
A. Height at adductor	2.75"
B. Height at abductor	2.25"
C. Height at leg trough	2.25"
Weight capacity	250 lb.

GENTLE CONTOUR

Excellent choice for individuals who:

- Have moderate postural instability
- Need minimal control of posture
- Need moderate control of abduction/adduction tendency

	GENTLE	MAX
Width	13" – 22"	13" – 22"
Depth	15" – 20"	15" – 20"
A. Height at adductor	3.38"	4.25"
B. Height at abductor	3.38"	4.40"
C. Height at leg trough	2.25"	2.66"
Weight capacity	250 lb.	250 lb.

MAX CONTOUR

Excellent choice for individuals who:

- Require high level of postural control
- Require optimal pressure reduction and redistribution
- Have moderate postural instability
- Need moderate control of posture
- Need moderate control of abduction/adduction tendency

Modifications


- Leg length cutouts
 - 1" or 2": cutouts to accommodate leg length discrepancies
- Removable leg wedges
 - Allow lower seat-to-floor height for foot propellers
 - Accommodate limited hip flexion opens up seat-to-back angle

Matrix® Specifications

Cushions

Invacare® Matrix® Vi™ Cushion

Seat width	14", 16", 18", 20", 22" Standard 18" – 30" Heavy-Duty
Seat depth	15" – 20" Standard 18" – 26" Heavy-Duty
Product weight	3 lb. Standard 4 lb. Heavy-Duty
Weight capacity	300 lb. Standard 600 lb. Heavy-Duty
Limited warranty	2 years

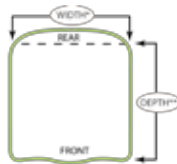


Invacare® Matrix® Stabilite™ Cushion

Seat width	14" – 24"
Seat depth	14" – 22"
Product weight	3 lb.
Weight capacity	14" – 20" 300 lb. 21" – 24" 500 lb.
Limited warranty	2 years


Invacare® Matrix® Flo-tech™ Cushion

Seat width	14" – 20" Standard 20" – 24" Heavy-Duty
Seat depth	14", 16" – 21" Standard 18", 20", 22" Heavy-Duty
Product weight	7.2 lb. Standard 9.5 lb. Heavy-Duty
Weight capacity	300 lb. Standard 500 lb. Heavy-Duty
Limited warranty	3 years

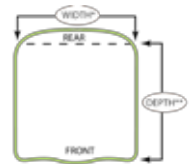

Invacare® Matrix® Kid*ab*ra Vi™ Cushion

Seat width	12" – 14"
Seat depth	12" – 14"
Product weight	1.5 lb.
Weight capacity	150 lb.
Limited warranty	2 years

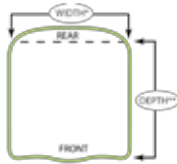

Invacare® Matrix® Flovair™ Cushion (Gentle, Max)


Seat width	15" – 24"
Seat depth	15" – 22"
Product weight	4.20 lb.
Weight capacity	15" – 20" 300 lb. 21" – 24" 500 lb.
Limited warranty	2 years

Invacare® Matrix® AirFlo™ Cushion (Lo, Gentle, Max)

Seat width	13" – 22"
Seat depth	15" – 20"
Product weight	3 lb.
Weight capacity	250 lb.
Limited warranty	2 years

Invacare® Matrix® PS Cushion

Seat width	10", 12", 14" Pediatric 16", 18", 20" Standard 21" – 30" Heavy-Duty
Seat depth	12", 14", 16" Pediatric 16", 18", 20" Standard 21" – 30" Heavy-Duty
Product weight	1 lb. Pediatric 2 lb. Standard 7 lb. Heavy-Duty
Weight capacity	150 lb. Pediatric 300 lb. Standard 600 lb. Heavy-Duty
Limited warranty	2 years

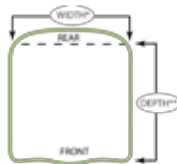

* Actual cushion width is ½" narrower than stated size

** Actual cushion depth is 1½" longer than stated size due to rear cushion radius

General Use Backs

The new adjustable tension **Invacare® Versair™ Back** is designed for positioning and comfort, offering versatile and clinically beneficial solutions when standard sling upholstery does not meet an individual's need. Wide tension straps are easily adjustable to accommodate positioning requirements with maximum support. They can be adjusted taut in the lower back for optimal sacral support or relaxed in the upper back to accommodate a thoracic kyphosis. The padded back upholstery uses mesh fabric for added comfort and improved airflow between the user and the back.

HCPCS Code
E2611/E2612 (Pending)



Mesh fabric effectively dissipates heat and moisture.

Specifications

Width	16"– 26"
Height	
Regular	15"– 17"
Tall	17"– 19"
Weight limit	
16"–21"	300 lb.
21"–26"	500 lb.
Limited warranty	2 years

NEW Invacare® Versair™ Back Support

Model No. VA1621R, VA1621T, VA2126R, VA2126T

Features and Functional Benefits

- **Lightweight**
- **Comfortable padded upholstery:** A slim profile that does not sacrifice seat depth
- **Mesh upholstery:** To wick away heat and moisture
- **Secure and easy fit around canes:** Using hook and loop fasteners
- **Alternating 2" tension straps:** Easy to adjust as needed to provide optimal postural support and pelvic support/stability
- **Easily folds:** With the chair for transport

The **Invacare® Matrx® Genera™ Back** provides support and comfort at a price so affordable that no wheelchair user needs to make do with sling upholstery.

HCPCS Code
E2611



Hardware fits 7/8" and 1" back canes

Invacare® Matrx® Genera™ Back Support

Model no. GB

Features and Functional Benefits

Outer cover:

- Attractive, water resistant and breathable cover, vinyl backed

Back shell:

- Sturdy contoured shell with 3" of contour depth, durable plated steel hardware
- Built-in T-nuts for headrest attachment

Hardware:

- Hardware adjusts to fit chairs 1" wider than back model width
- Provides 1" depth and 20° recline adjustability
- Quick release hardware for easy portability

Specifications

Width	16" 18" 20"
Height	
16"	16"– 20"
20"	16"– 20"
Weight capacity	300 lb.
Limited warranty	1 year

The **Invacare® Propel™ Back** is so lightweight and comfortable consumers will forget it's there. Hardware installation is no longer a hassle: just grab it, set it and go. The Invacare Propel Back provides a stable surface for propulsion and completion of daily living activities.

HCPCS Code
E2611



Adjust depth/angle while client is seated

Invacare® Propel™ Back Support

Model no. ITPR

Features and Functional Benefits

- **Four way stretch:** Allows immersion
- **Moisture resistant**
- **Solid, lightweight back:** Provides support for an upright posture and improved sitting tolerance with less fatigue
- **Scapular cutouts:** Improved upper extremity function and efficient wheelchair propulsion
- **Redundant tissue cutouts:** Provides increased lower back comfort
- **Contoured foam cushion:** Provides gentle support and optimal comfort
- **Adjustable in height/width/depth/angle:** Accommodates user's positioning requirements. Adjustments can be made while user is seated in the chair
- **One handed quick release:** Allows easy removal and remounting for transport
- **One tool assembly:** 7/8" wrench required for all installations and adjustments
- **Two point mountings:** Fits 7/8" and 1" canes

Specifications

Width	16"– 21"
(in 1" increments)	
Height	16.5"
Depth adjustment	1.75"
Angle adjustment	up to 27°
Weight limit	300 lb.
Limited warranty	2 years

General Use Cushions

For individuals with mild to moderate seating needs, the **Invacare® Absolute™ Cushion** offers simple, but highly functional solutions. This cushion is designed to provide basic comfort using highly resilient foam and is intended for individuals who are able to do own weight shift for pressure relief.

HCPCS Code
E2601



Specifications

Width:	16" – 20"
Depth:	16" – 18"
A. Height at adductor	3.25"
B. Height at abductor	2.25"
C. Height at rear	2"
Cushion weight approximately	2 lb.
Limited warranty	1 year

Invacare® Absolute™ Cushion

Model no. EC

Features and Functional Benefits

- **Lightweight Molded Polyurethane foam:** Provides comfort and durability
- **Gentle Contour:** For comfort and centering in the seat
- **Shaped cushion bottom:** Allows use of cushion on taut sling upholstery
- **Outer cover:** Breathable water repellent cover to protect foam and keep the consumer dry
- **Non-slip bottom:** Keeps cushion secure and in place

For individuals with mild to moderate seating needs, the **Invacare® Single Density Cushion** offers simple, but highly functional solutions. This cushion is designed to provide basic comfort using highly resilient foam for pediatrics to geriatrics who are able to do own weight shift for pressure relief.

HCPCS Code
E2601/E2602



Invacare® Single Density Cushion

Model no. SD

Features and Functional Benefits

- **Lightweight contoured foam with deformation cut:** Provides comfort and stability to the individual
- **Outer cover:** Durable water repellent cover to protect foam and keep the consumer dry

Specifications

Width	10" – 24"
Depth	7" – 20"
A. Height at rear	2"
B. Height at rear (center)	1.5"
C. Height at front	2.5"
Cushion weight approximately	2 lb.
Limited warranty	1 year

For individuals with mild to moderate seating needs, the **Invacare® Comfort-Mate Extra Cushion** goes one step further with a comfortable, molded design and a durable moisture-resistant coating. Ideal for individuals who are able to do own weight shifts for pressure relief, who need comfort, stability, gentle positioning and some protection.

HCPCS Code
E2603/E2604



Specifications

Width	14" – 22"
Depth	14" – 20"
A. Height at adductor	2.625"
B. Height at abductor	3.0"
C. Height at leg trough	2.25"
Cushion weight approximately	2.3 lb.
Limited warranty	1 year

Invacare® Comfort-Mate Extra Cushion

Model no. CMEX


Features and Functional Benefits

- **Low profile, lightweight cushion:** Designed for long-term sitting comfort and easy lateral transfers
- **Molded dual-firmness polyurethane foam:** Soft top layer provides exceptional comfort and firmer bottom layer stabilizes and supports the pelvis
- **Contoured seating surface:** Promotes neutral leg positioning while providing gentle positioning
- **Beveled sides:** Allows use with a drop seat
- **Moisture resistant coating:** Seals and protects foam from moisture; easily wipes clean
- **Outer cover:** Breathable water repellent cover to keep the user cool and dry

Matrix® Accessories

ARMPAD OPTIONS

MULTI-POSITION ARM PADS

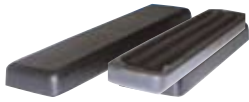

- Soft, gel-like feel provides extra comfort and protection for arms
- Highly durable skinned polyurethane resists cracking and peeling
- Triple track extrusion allows adjustability of mounting position and attachment location for other accessories
- 3" wide; desk length 10" or full length 12"

ERGONOMIC ARM TROUGH/HAND PAD KIT




- Molded, skinned polyurethane fits the natural arm curvature
- Unique cut-out at elbow provides users with flexibility of various arm positions
- Triple track extrusion allows a variety of mounting positions
- Complete with flat hand pad or interfaces with other manufacturer's hand pieces

GEL ARM PADS




- Fabric covered polymer gel arm pads
- Provides extra protection for highly sensitive arms
- Available in 3 models:
Std. Narrow: 2" wide x 12" long
Std. Wide: 3.5" wide x 12" long
Long Wide: 3.5" wide x 14" long

MULTI-AXIS ARMREST MOUNT

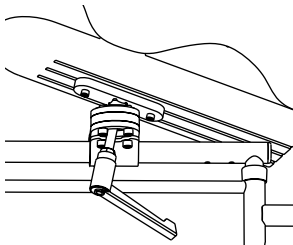

- Allows infinite adjustment of angle and rotation of arm pad
- Interfaces with multi-position arm pad

JOYSTICK MOUNTING BAR FOR MULTI-AXIS ARMREST MOUNT

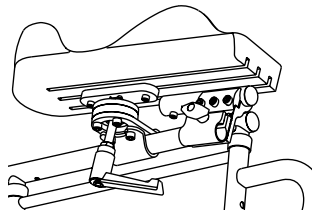

- Available for 3/4" or 7/8" joystick bar size

POSITIONING STRAP OPTIONS

AIR MESH CALF STRAP



- Available in 4 sizes: M, L, MD, LG
- For 15" – 18" wide chairs
M: 3.5" high
MD: 9" high
- For 18" – 22" wide chairs
L: 3.5" high
LG: 9" high

PADDED LAP BELTS

2 POINT



4 POINT



- 2" wide belts, comfortable padding
- Center pull tightening, push release buckle
- 2 or 4 point attachment

HEAD PAD OPTIONS

OTTO BOCK™



- Easy to clean self skinning polyurethane foam
- Anatomically shaped to provide lateral flexion control
- 7" wide x 5" high or 9" wide x 6" high

STANDARD



- Easy to clean self skinning polyurethane foam
- Gentle lateral flexion contouring
- 6.5" wide x 7" high


STEALTH COMFORT PLUS



- Soft, comfortable layered foam on coated metal shell
- Removable, washable cover
- Four pad sizes: 8", 10" or 14" wide x 6" high and 6" wide x 3.75" high

HEAD SUPPORT KIT


ONYX AUTO STYLE KIT

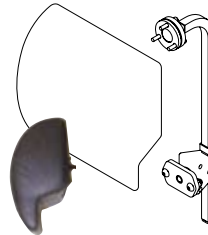

- Onyx leatherette cover
- Auto-style contouring
- 8" wide x 8" high x 4" thick
- Dual pivot ball for pad orientation
- Limited adjustability – recommended for Contour or High backs
- Height adjustable
- Includes fixed mount clamp


HEAD SUPPORT HARDWARE OPTIONS

STANDARD HARDWARE

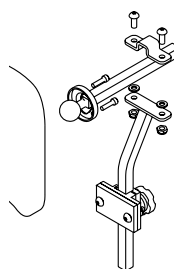


- Comes standard with Otto Bock™, Standard & Stealth Comfort Plus Pads

MULTI-AXIS

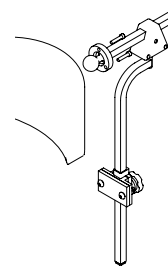


- Upgrade option for Otto Bock, Standard & Stealth Comfort Plus pads
- Dual pivot balls for pad orientation
- Depth adjustable through horizontal slide bar
- Height adjustable
- Includes mounting clamp for attachment to back with knob style release for headrest removal

STEALTH "WORLD'S BEST"

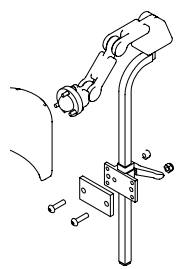

- Upgrade option for Otto Bock, Standard & Stealth Comfort Plus Pads
- Dual pivot balls for pad orientation
- Depth adjustable through dual link system
- Height adjustable
- Includes mounting clamp for attachment to back with latch style release for headrest removal

Matrix® Accessories

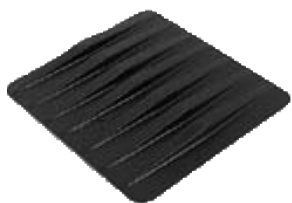

CUSHION RIGIDIZER

- Lightweight plastic seat insert
- Securely attaches to bottom of cushion inside cover
- Fits between seat rails for level support
- Eliminates hammock effect of seat upholstery



BELTS AND STRAPS

- Three sizes of shoulder supports fit child, junior and adult
- Three sizes of padded lap belts fit child, junior and adult; pinch release buckle
- Push-button style release provides secure attachment
- Three sizes of four-point lap belt to fit child, junior and adult; push button release buckle



BASIC HEADREST

- Padded to provide comfortable occipital support
- Horizontal and vertical adjustable hardware
- Outer cover easily removes for laundering



ADJUSTABLE HEADRESTS

- Contoured head and neck supports available in four styles
- Dual-ball swivel hardware for multi-plane adjustment
- Durable steel reinforced polyurethane pads - easily wipe clean



DROP BASE

- Provides stable support base for the cushion
- Lowers seat-to-floor height which aids in foot propulsion, opens up seat to back angle, allows easy lateral transfers and improves basic functionality when in a seated position
- Width adjustable: Three sizes fit wheelchairs from 14" to 22" wide
- Height and angle adjustable: up to 2" of drop provided
- Portable quick-release hardware allows easy removal of base from wheelchair



FULL TRAY - HOOK AND LOOP MOUNT

- Lightweight and easy to handle
- Break-resistant clear polycarbonate tray surface
- Easily attaches to chair armrest with hook and loop straps
- Removable trim allows thorough cleaning



FULL TRAY - SLIDE-ON MOUNT

- Break-resistant clear polyester resin tray surface
- Width adjustable mounting hardware securely fastens tray to chair arm pads
- Molded-in rim helps keep items on tray



FOAM-IN-PLACE KIT






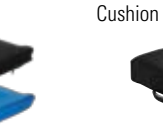









- Easy-to-use Foam-in-Place kit provides custom foam mold insert in minutes
- Provides additional support and positioning for users who need a more custom fit
- Custom-molded insert can be easily removed for modification or remolded to meet the changing needs of the user
- Can be used with Invacare® Matrix® Personal Back™10, Personal Back™ 10 Plus, PB and PB Deep Backs



HALF TRAY - SWINGAWAY

- Break-resistant clear polycarbonate tray surface
- Tray securely bolts to wheelchair armrest and is durable
- Tray easily swings away when not in use
- Molded-in rim helps keep items on tray

Seating & Positioning Product Guide*

	E2601	E2602	E2603	E2604	E2605	E2606	
HCPCS Codes Description	General Use Wheelchair Seat Cushion, Width Less Than 22", Any Depth	General Use Wheelchair Seat Cushion, Width 22" or Greater, Any Depth	Skin Protection Wheelchair Seat Cushion, Width Less Than 22", Any Depth	Skin Protection Wheelchair Seat Cushion, Width 22" or Greater, Any Depth	Positioning Wheelchair Seat Cushion, Width Less Than 22", Any Depth	Positioning Wheelchair Seat Cushion, Width 22" or Greater, Any Depth	
Qualifying Diagnosis	Not Specified	Not Specified	Spinal Cord Injury (paraplegia, quadriplegia), Other Spinal Cord Disease, Multiple Sclerosis, Cerebral Palsy, Amyotrophic Lateral Sclerosis, Post Polio, Traumatic Brain Injury, Spina Bifida, Childhood Cerebral Degeneration, Alzheimer's Disease, Parkinson's Disease, Hemiplegia, Huntington's chorea, Idiopathic torsion dystonia, Athetoid Cerebral Palsy, Transverse myelitis	Spinal Cord Injury (paraplegia, quadriplegia), Other Spinal Cord Disease, Multiple Sclerosis, Cerebral Palsy, Amyotrophic Lateral Sclerosis, Post Polio, Traumatic Brain Injury, Spina Bifida, Childhood Cerebral Degeneration, Alzheimer's Disease, Parkinson's Disease, Hemiplegia, Huntington's chorea, Idiopathic torsion dystonia, Athetoid Cerebral Palsy, Transverse myelitis	Spinal Cord Injury (paraplegia, quadriplegia), Cerebral Palsy, Paralysis, Post Polio, Amyotrophic Lateral Sclerosis, Monoplegia of Lower Limb, Hemiplegia, Muscular Dystrophy, Alzheimer's Disease, Spina Bifida, Multiple Sclerosis, Traumatic amputation of leg(s), Osteogenesis imperfecta, Transverse myelitis	Spinal Cord Injury (paraplegia, quadriplegia), Cerebral Palsy, Post Polio, Amyotrophic Lateral Sclerosis, Monoplegia of Lower Limb, Hemiplegia, Muscular Dystrophy, Alzheimer's Disease, Spina Bifida, Multiple Sclerosis, Traumatic amputation of leg(s), Osteogenesis imperfecta, Transverse myelitis	
Available Products	Invacare® Absolute™ Cushion  Invacare® Single Density Cushion 	Invacare® Single Density Cushion 	Invacare® Comfort-Mate Extra Cushion 	Invacare® Comfort-Mate Extra Cushion 	Invacare® Matrx® Stabilite™ Cushion  Invacare® Matrx® PS Cushion 	Invacare® Matrx® Stabilite™ Cushion  Invacare® Matrx® PS Cushion 	
	E2611		E2612		E2615		
HCPCS Codes Description	General Use Wheelchair Back Cushion, Width Less Than 22", Any Height, Including Any Type Mounting Hardware		General Use Wheelchair Back Cushion, Width 22" or Greater, Any Height, Including Any Type Mounting Hardware		Positioning Wheelchair Back Cushion, Posterior-Lateral, Width Less Than 22", Any Height, Including Any Type Mounting Hardware		
Qualifying Diagnosis	Not Specified		Not Specified		Spinal Cord Injury (paraplegia, quadriplegia), Cerebral Palsy, Paralysis, Post Polio, Amyotrophic Lateral Sclerosis, Monoplegia of Lower Limb, Hemiplegia, Muscular Dystrophy, Alzheimer's Disease, Spina Bifida, Multiple Sclerosis, Traumatic amputation of legs(s) (complete/partial), Osteogenesis imperfecta, Transverse myelitis		
Available Products	Invacare® Matrx® Propel™ Back 	Invacare® Matrx® Genera™ Back 	Invacare® Versair™ Back 	Invacare® Versair™ Back 	Invacare® Matrx® Personal Back™10/ Personal Back™10 Plus Backs 	Invacare® Matrx® PB/ PB Deep Back 	

* Providers are responsible for determining the appropriate billing codes when submitting claims to the Medicare Program and should consult an attorney or other advisor to discuss specific situations in their practice.
 † Qualifying diagnosis code requires additional provision for decubitus ulcer (707.03, 707.04, 707.05)
 © 2011 Invacare Corporation. All rights reserved. Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted. Specifications are subject to change without notice.

06 E2607 E2608 E2622 E2623 E2624 E2625

Wheelchair
Width 22"
Depth

Skin Protection & Positioning Wheelchair Seat Cushion, Width Less Than 22", Any Depth

Skin Protection & Positioning Wheelchair Seat Cushion, Width 22" or Greater, Any Depth

Adjustable Skin Protection Wheelchair Seat Cushion, Width Less Than 22", Any Depth

Adjustable Skin Protection Wheelchair Seat Cushion, Width 22" or Greater, Any Depth

Adjustable Skin Protection & Positioning Wheelchair Seat Cushion, Width Less Than 22", Any Depth

Adjustable Skin Protection & Positioning Wheelchair Seat Cushion, Width 22" or Greater, Any Depth

Spinal Cord Injury (paraplegia, quadriplegia), Other Spinal Cord Disease, Multiple Sclerosis, Cerebral Palsy, Post Polio, Traumatic Brain Injury, Spina Bifida, Amyotrophic Lateral Sclerosis, Alzheimer's Disease, Parkinson's Disease, 'Monoplegia of Lower Limb, 'Hemiplegia, 'Muscular Dystrophy

Spinal Cord Injury (paraplegia, quadriplegia), Other Spinal Cord Disease, Multiple Sclerosis, Cerebral Palsy, Post Polio, Traumatic Brain Injury, Spina Bifida, Amyotrophic Lateral Sclerosis, Alzheimer's Disease, Parkinson's Disease, 'Monoplegia of Lower Limb, 'Hemiplegia, 'Muscular Dystrophy

Spinal Cord Injury (paraplegia, quadriplegia), Other Spinal Cord Disease, Multiple Sclerosis, Cerebral Palsy, Amyotrophic Lateral Sclerosis, Post Polio, Traumatic Brain Injury, Spina Bifida, Childhood Cerebral Degeneration, Alzheimer's Disease, Parkinson's Disease, Hemiplegia, Huntington's chorea, Idiopathic torsion dystonia, Athetoid Cerebral Palsy, Transverse myelitis

Spinal Cord Injury (paraplegia, quadriplegia), Other Spinal Cord Disease, Multiple Sclerosis, Cerebral Palsy, Amyotrophic Lateral Sclerosis, Post Polio, Traumatic Brain Injury, Spina Bifida, Childhood Cerebral Degeneration, Alzheimer's Disease, Parkinson's Disease, Hemiplegia, Huntington's chorea, Idiopathic torsion dystonia, Athetoid Cerebral Palsy, Transverse myelitis

Spinal Cord Injury (paraplegia, quadriplegia), Other Spinal Cord Disease, Multiple Sclerosis, Cerebral Palsy, Post Polio, Traumatic Brain Injury, Spina Bifida, Amyotrophic Lateral Sclerosis, Alzheimer's Disease, Parkinson's Disease, 'Monoplegia of Lower Limb, 'Hemiplegia, 'Muscular Dystrophy

Spinal Cord Injury (paraplegia, quadriplegia), Other Spinal Cord Disease, Multiple Sclerosis, Cerebral Palsy, Post Polio, Traumatic Brain Injury, Spina Bifida, Amyotrophic Lateral Sclerosis, Alzheimer's Disease, Parkinson's Disease, 'Monoplegia of Lower Limb, 'Hemiplegia, 'Muscular Dystrophy



E2616 E2620 E2621

Positioning Wheelchair Back Cushion, Posterior-Lateral, Width 22" or Greater, Any Height, Including Any Type Mounting Hardware

Positioning Wheelchair Back Cushion, Planar with Lateral Supports, Width Less Than 22", Any Height, Including Any Type Mounting Hardware

Positioning Wheelchair Back Cushion, Planar with Lateral Supports, Width 22" or Greater, Any Height, Including Any Type Mounting Hardware

Spinal Cord Injury (paraplegia, quadriplegia), Cerebral Palsy, Paralysis, Post Polio, Amyotrophic Lateral Sclerosis, Monoplegia of Lower Limb, Hemiplegia, Muscular Dystrophy, Alzheimer's Disease, Spina Bifida, Multiple Sclerosis, Traumatic amputation of legs(s) (complete/partial), Osteogenesis imperfecta, Transverse myelitis

Spinal Cord Injury (paraplegia, quadriplegia), Cerebral Palsy, Paralysis, Post Polio, Amyotrophic Lateral Sclerosis, Monoplegia of Lower Limb, Hemiplegia, Muscular Dystrophy, Alzheimer's Disease, Spina Bifida, Multiple Sclerosis, Traumatic amputation of legs(s) (complete/partial), Osteogenesis imperfecta, Transverse myelitis

Spinal Cord Injury (paraplegia, quadriplegia), Cerebral Palsy, Paralysis, Post Polio, Amyotrophic Lateral Sclerosis, Monoplegia of Lower Limb, Hemiplegia, Muscular Dystrophy, Alzheimer's Disease, Spina Bifida, Multiple Sclerosis, Traumatic amputation of legs(s) (complete/partial), Osteogenesis imperfecta, Transverse myelitis



Further detail. This information is subject to change without notice.

Specifications are subject to change without notification.

Form # 10-106 Rev. 02/11 100911





Meiko (left) and Casey (right) are high school students and best friends who enjoy playing basketball, racing and listening to music. When it comes to their futures, both have high ambitions. Inspired by Paul Schulte, Invacare Top End Design Engineer and renowned athlete, Meiko aspires to be a design engineer himself. Casey was the first student in the state of Ohio to be included on his school's track team using a racing wheelchair. His immediate goal is to qualify for the Paralympics.



For more information on Invacare® Matrix® Seating
Series Products visit
www.invacare.com/Matrix

Invacare Corporation
www.invacare.com

USA
One Invacare Way
Elyria, Ohio
44035-4190
(800) 333-6900

Canada
570 Matheson Blvd. E.
Unit 8
Mississauga, Ontario
L4Z 4G4 Canada
(800) 668-5324

Motion Concepts
84 Citation Dr., Unit 1-7
Concord, ON L4K 3C1
(866) 748-7943
Tel: 905-695-0134
Fax: 905-695-0138

© 2011
Invacare Corporation.

All rights reserved.
Trademarks are
identified by the symbols
™ and ®.

All trademarks are
owned by or licensed
to Invacare Corporation
unless otherwise
noted. Otto Bock is a
trademark of Otto Bock
Orthopaedische Industrie
GmbH & Co. Matrix is a
registered trademark of
Motion Concepts. Dartex
is a registered trademark
of Darlington Fabrics
Corporation. Ultra-
Fresh is a trademark
of Thomson Research
Associates.

Specifications are
subject to change
without notification.

Form No. 10-106
Rev. 02/11
100911



Yes, you can.®