PULSE

Quickie® Pulse

6 BC, 6 SC, 6 CC, 5 BC, 5CC

SUPPLIER: This manual must be given to the rider of this wheelchair.

RIDER: Before using this wheelchair read this entire manual and save for future reference.

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Mode d'emploi

I. INTRODUCTION**SUNRISE LISTENS**

Thank you for choosing a Quickie wheelchair. We want to hear your questions or comments about this manual, the safety and reliability of your chair, and the service you receive from your Sunrise supplier. Please feel free to write or call us at the address and telephone number below:

SUNRISE MEDICAL

Customer Service Department

2842 Business Park Ave

Fresno, CA 93727

(800) 333-4000 or (303) 218-4500

FOR ANSWERS TO YOUR QUESTIONS

Your authorized supplier knows your wheelchair best and can answer most of your questions about chair safety, use and maintenance. For future reference, fill in the following:

Supplier: _____

Address: _____

Telephone: _____

Serial #: _____ Date/Purchased: _____

USER WARNING**WARNING**

Attention wheelchair users, do not operate this wheelchair without first reading the owner's manual. If you do not understand the instructions and warnings of this owner's manual please contact your dealer or qualified technician before operating this Quickie wheelchair. Failure to do so may result in damage and/or injury.

DEALER/TECHNICIAN WARNING**WARNING**

Attention dealers and qualified technicians, do not operate or service this wheelchair without first reading this owners manual. If you do not understand the instructions and warnings of this owners manual please contact the Quickie Technical Service Department before operating and/or servicing this Quickie wheelchair. Failure to do so may result in damage and/or injury.

WARNING

In addition, power wheelchairs may contain printed circuit boards. Careful attention should be given to the Printed Circuit Board Handling Techniques insert supplied with the chair.

Related Documents

Listed below are additional documents which are referenced in this owners manual.

- RNET Owners Manual
- RNET OBP Owners Manual
- RNET Omni Owners Manual
- Pulse Service Manual

I. INTRODUCTION.....	2
III. YOUR CHAIR AND ITS PARTS	4
IV. NOTICE- READ BEFORE USE.....	5
V. EMI (ELECTROMAGNETIC INTERFERENCE).....	5
A. WHAT IS EMI?	5
B. WHAT EFFECT CAN EMI HAVE?	5
C. SOURCES OF EMI	5
D. DISTANCE FROM THE SOURCE	6
E. IMMUNITY LEVEL	6
F. REPORT ALL SUSPECTED EMI INCIDENTS	6
VI. GENERAL WARNINGS.....	7
A. NOTICE TO RIDER	7
B. NOTICE TO ATTENDANTS	7
C. WEIGHT LIMIT	7
D. CONTROLLER SETTINGS.....	7
E. EMI	7
F. SAFETY CHECK-LIST	7
G. CHANGES & ADJUSTMENTS	7
H. WHEN SEATED IN A PARKED WHEELCHAIR	8
I. ENVIRONMENTAL CONDITIONS	8
J. TERRAIN	8
K. STREET USE	8
L. MOTOR VEHICLE SAFETY	8
M. CENTER OF BALANCE.....	8
N. TRANSFERS.....	8
O. REACHING OR LEANING	9
P. DRESSING OR CHANGING CLOTHES.....	9
Q. OBSTACLES.....	9
R. DRIVING IN REVERSE	9
S. RAMPS, SLOPES & SIDEHILLS	9
T. TO REDUCE THE RISK OF A FALL, TIP-OVER OR LOSS OF CONTROL	10
U. RAMPS AT HOME & WORK	10
V. WHEELCHAIR LIFTS.....	10
W. CURBS & SINGLE STEPS	10
X. STAIRS.....	10
Y. ESCALATORS	10
VII. WARNINGS: COMPONENTS & OPTIONS.....	11
A. ARMRESTS.....	11
B. BATTERIES.....	11
C. CUSHIONS.....	11
D. FASTENERS	11
E. FOOTPLATE & FOOTRESTS	11
F. MOTOR LOCKS	11
G. ON/OFF SWITCH.....	11
H. POSITIONING BELTS	11
I. SEATING SYSTEMS	11
J. UPHOLSTERY FABRIC.....	12
K. SUSPENSION.....	12
L. WIRING	12
M. POWER SEATING	12

N.S.P.O.T. (SINGLE POWER OPTION TILT)	
POWER ACCESSORY	12
O. TRANSIT OPTION	12
VIII. SET-UP, ADJUSTMENT & USE.....	16
TOOLS YOU WILL NEED	16
A. BATTERY REMOVAL.....	17
B. CENTER MOUNT HANGER ADJUSTMENT	17
C. SWING-AWAY FOOTRESTS.....	18
D. ELEVATING LEGREST	18
E. FOLDING BACKREST	18
F. DUAL-POST HEIGHT-ADJUSTABLE ARMRESTS FIXED HEIGHT STANDARD.....	19
G. SINGLE POST HEIGHT-ADJUSTABLE ARMRESTS	19
H. ASAP SEAT DEPTH 14" - 16", 20" - 22" ONLY	19
I. CAPTAIN'S SEAT	20
J. MANUAL RECLINE OPERATION	22
K. JOYSTICK	22
L. JOYSTICK RETRACTABLE MOUNT	23
M. ASAP II SEATING ADJUSTMENT	23
N. SEAT HEIGHT ADJUSTMENT	23
O. SEAT REMOVAL	23
P. SHROUD REMOVAL	24
Q. CONTROLLER POSITION	24
R. CHECK-OUT	24
IX. OPERATING GUIDE.....	25
A. PERFORMANCE CONTROL SETTINGS.....	25
B. THERMAL ROLL-BACK	25
C. JOYSTICK STATIONARY	25
D. VR2 JOYSTICK ASSEMBLY	26
E. LED JOYSTICK.....	27
F. MONOCHROME LCD SCREEN	28
G. COLOR LCD SCREEN.....	29
H. BRAKE LEVER	30
X. BATTERIES.....	31
A. INTRODUCTION	31
B. BATTERY CHARGER	31
C. ACID BURNS (UNSEALED WET CELL BATTERIES).....	31
D. CONNECTING BATTERIES IN BATTERY WELL	31
E. CHARGING BATTERIES	32
F. DISPOSING OF BATTERIES	32
XI. MAINTENANCE.....	33
A. NOTES	33
B. CLEANING	33
C. STORAGE TIPS	33
D. TO REPAIR OR REPLACE A TIRE	33
E. MOTOR BRUSHES	33
F. ORDERING PARTS	33
G. MAINTENANCE CHART	33
H. WIRING DIAGRAM.....	34
XII. SUNRISE LIMITED WARRANTY.....	35

ESPAÑOL36

FRANÇAIS.....70

III. YOUR CHAIR AND ITS PARTS

1. Flip-back, height-adjustable armrest
2. VR2 controller joystick (RNET optional)
3. Captain's Seat
4. Front caster 7"
5. Rear stabilizing caster 6"
6. Single plate foot platform (shown);

Optional:

Swing-away 65° Hangers;
Swing-away ELR hangers
Swing-away 70° Hangers;
(tapered) Swing-away 75° Hangers;
Swing-away 90° Hangers;

7. Drive wheels
8. Brake release
9. Caster Fork (front)
10. Caster Fork (rear)
11. Front Shroud

Models

Pulse 6 BC
Pulse 6 SC
Pulse 6 CC (shown)
Pulse 5 BC
Pulse 5 CC

QUICKIE PULSE

Weight

	SC	BC	CC
complete chair:	260 lbs	235 lbs	250
w/o batteries:	182 lbs	157 lbs	172

Drive Wheels

13" Mag: Std. airless insert

Joystick

Standard –VR2 (right-hand or left-hand mount)
Option - VR2 w/actuator control (SC only)
Option - RNET–LED/Monochrome/Color

Batteries

(2 deep cycle batteries required to operate chair)
Standard - Group 22

Battery Charger

Off board 8 AMP

Shroud Colors

Red, Black, Blue

Seats, Upholstery and Style

Standard- BC & SC: A.S.A.P. II Seat Frame
CC: Captain's Seat
Sizes: 12" - 22" wide, 12" - 22" deep

Footrest:

CC- Center Mount
SC & BC: Standard- SA Swing away footrests with heel loops
Option- Foot platform, Manual elevating legrests, Power Articulating Legrest (ALR), Power Center Mount, Center Mount

Backrest

SC & BC: Standard Canes Angle Adjustable
Option- 8° Bend, Semi Recline, Folding
CC: Standard- Captain's Seat Back

Casters (front)

Standard- 7" solid

Casters (rear)

Standard- 6" solid

Armrest

SC & BC: Standard- 14" Cantilever, height adjustable 14" arm pad
Option- Locking, two-point arm, Dual post Arm, Single Post arm
CC: Standard- Flip-up

Suspension

Standard- all wheel

All features may not be available with some chair setups or in conjunction with another chair feature. Please consult your supplier for more information. Your authorized supplier can also provide you with more information on accessories.

IV. NOTICE— READ BEFORE USE

A. CHOOSE THE RIGHT CHAIR & SAFETY OPTIONS

Sunrise provides a choice of many power wheelchair styles, sizes and adjustments to meet the needs of the rider. However, final selection of a wheelchair rests solely with you and your health care professional. Choosing the best chair for you depends on such things as:

1. Your size, disability, strength, balance and coordination.
2. Your intended use, and your level of activity.
3. The types of hazards you must overcome in daily use (in areas where you are likely to use your chair).
4. The need for options for your safety and comfort (such as positioning belts or special seat systems).

B. ADJUST CHAIR TO YOUR ABILITY

You need to work with your doctor, nurse or therapist, and your supplier, to fit this chair and adjust the controller settings for your level of function and ability.

C. REVIEW THIS MANUAL OFTEN

Before using this chair you, and each person who may assist you, should read this entire Manual and make sure to follow all instructions. Review the warnings often, until they are second nature to you.

D. WARNINGS

The word “WARNING” refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons. The “Warnings” are in four main sections, as follows:

1. V — EMI

Here you will learn about electromagnetic interference and how it can affect your chair.

2. VI — GENERAL WARNINGS

Here you will find a safety checklist and a summary of risks you need to be aware of before you ride this chair.

3. VII — WARNINGS — COMPONENTS & OPTIONS

Here you will learn about your chair. Consult your supplier and your health care professional to help you choose the best set-up and options for your safety.

4. X — BATTERIES

Here you will learn about battery and charger safety, and how to avoid injury.

NOTE— Where they apply, you will also find “Warnings” in other sections of this Manual.

V. EMI (ELECTROMAGNETIC INTERFERENCE)

V. EMI (ELECTROMAGNETIC INTERFERENCE)

WARNING

Heed all warnings to reduce the risk of unintended brake release or chair movement:

1. **Beware of the danger from hand-held transceivers. (A “transceiver” is a device that both sends and receives radio wave signals). Never turn on or use a hand-held transceiver while power to your chair is on. Use extra care if you believe that such a device may be in use near your chair.**
2. **Be aware of nearby radio or TV stations, and avoid coming close to them.**
3. **If unintended movement occurs, turn your chair off as soon as it is safe to do so.**

A. WHAT IS EMI?

WARNING

1. EMI means: electromagnetic (EM) interference (I). EMI comes from radio wave sources such as radio transmitters and transceivers.
2. There are a number of sources of intense EMI in your daily environment. Some of these are obvious and easy to avoid. Others are not, and you may not be able to avoid them.
3. Powered wheelchairs may be susceptible to electromagnetic interference (EMI) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two way radios, and cellular phones.
4. EMI can also be produced by conducted sources or electrostatic discharge (ESD).

B. WHAT EFFECT CAN EMI HAVE?

WARNING

1. EMI can cause your chair, without warning, to:
 - Release its brakes
 - Move by itself
 - Move in unintended directions

If any of these occurs, it could result in severe injury to you or others.

2. EMI can damage the control system of your chair. This could create a safety hazard, and lead to costly repairs.

C. SOURCES OF EMI

WARNING

The sources of EMI fall into three broad types:

1. Hand-Held Transceivers:

The antenna is usually mounted directly on the unit. These include:

- Citizens band (CB) radios
- “Walkie-talkies”
- Security, fire and police radios
- Cellular phones
- Lap-top computers with phone or fax
- Other personal communication devices

NOTE— These devices can transmit signals while they are on, even if not in use.

2. Medium-Range Mobile Transceivers:

These include two-way radios used in police cars, fire trucks, ambulances and taxi cabs. The antenna is usually mounted on the outside of the vehicle.

3. Long-Range Transceivers:

These include commercial radio and TV broadcast antenna towers and amateur (HAM) radios. Note: The following are not likely to cause EMI problems: Lap-top computers (without phone or fax), Cordless phones, TV sets or AM/FM radios, CD or tape players.

D. DISTANCE FROM THE SOURCE**WARNING**

EM energy rapidly becomes more intense as you get closer to the source. For this reason, EMI from hand-held devices is of special concern. (See C.1 above) A person using one of these devices can bring high levels of EM energy very close to your chair without you knowing it.

E. IMMUNITY LEVEL**WARNING**

1. The level of EM energy is measured in volts per meter (V/m). Every power wheelchair can resist EMI up to a certain level. This is called its "immunity level".
2. The higher the immunity level, the less the risk of EMI. It is believed that a 20 V/m immunity level will protect the power wheelchair user from the more common sources of radio waves.
3. The Quickie Pulse with the following configuration was tested and found to be immune to at least 20 V/m:
Quickie Pulse power wheelchair with a right-hand mounted RNET controller, A.S.A.P. seat, flip-back armrests, solid flip-up footplate and Group 22 batteries.
4. The following specialty input devices have an unknown effect on the immunity level because they have not been tested with the Quickie Pulse and RNET control system:

WARNING

Individuals with physical limitations requiring the use of a specialty control input device known not to be immune to 20V/m, or not known, should exercise extra care around known sources of EMI.

WARNING

There is no way to know the effect on EMI if you add accessories or modify this chair. Any change to your chair may increase the risk of EMI. Parts from other suppliers have unknown EMI properties.

F. REPORT ALL SUSPECTED EMI INCIDENTS**WARNING**

You should promptly report any unintended movement or brake release. Be sure to indicate whether there was a radio wave source near your chair at the time. Contact: Sunrise Medical Customer Service Department at (800) 333-4000.

VI. GENERAL WARNINGS**WARNING**

Heed all warnings in this section. If you fail to do so a fall, tip-over or loss of control may occur and cause severe injury to you or others.

A. NOTICE TO RIDER**WARNING**

1. Before using this chair, you should be trained in its safe use by your health care professional.
2. Every wheelchair is different. Take the time to learn the feel of this chair before you begin riding.
3. Be aware that you must develop your own methods for the safe use of this chair that are best suited to your level of function and ability.
4. Have someone help you practice bending, reaching and transferring until you learn how to do them safely.
5. Never try a new maneuver on your own unless you are sure it is safe.
6. Get to know the areas where you plan to use your chair. Look for hazards and learn how to avoid them.
7. Do not allow additional riders on the chair, armrests or chassis. Doing so may result in injury or damage.

B. NOTICE TO ATTENDANTS**WARNING**

Make sure you heed all warnings and follow all instructions in each section of this manual. (Be aware that warnings that apply to the rider also apply to you).

Notes:

1. You need to work with the rider and the rider's doctor, nurse or therapist, to develop safe methods best suited to your abilities and those of the rider.
2. To manually push the chair you must release the motor locks.
Make sure you have full control over the chair when you release the motor locks. When you do so the chair will not have brakes.
3. Propel this chair by the armrest supports only. If using a seat frame, propel the chair by the push handles on the top of the backrest. They provide secure points for you to hold the rear of the chair to prevent a fall or tip-over.

C. WEIGHT LIMIT**WARNING**

1. Never exceed a total weight of 300 pounds (136.3 kilograms) for rider plus items carried.
2. Never use this chair for weight training if the total weight (rider plus additional weights) exceeds 300 pounds (136.3 kilograms).
3. Exceeding the weight limit is likely to damage the seat, frame, or fasteners and may cause severe injury to you or others from chair failure.
4. Exceeding the weight limit will void the warranty.

D. CONTROLLER SETTINGS**WARNING**

Be aware that you may need to adjust the controller settings of your chair to reduce the risk of a collision, fall or tip-over.

1. Check and adjust the settings every six to twelve months (or more often, if needed).
2. Consult your supplier to adjust the control settings immediately if you notice any change in your ability to:
 - Control the joystick.
 - Hold your torso erect.
 - Avoid running into objects.

E. EMI**WARNING**

Read Section V to learn about EMI. To reduce the risk of unintended brake release or chair movement:

1. Never turn on or use a hand-held transceiver while power to your chair is on. Use extra care if you believe that such a device may be in use near your chair.
2. Be aware of nearby radio or TV stations, and avoid coming close to them.
3. If unintended movement or brake release occurs, turn your chair off as soon as it is safe.

F. SAFETY CHECK-LIST**WARNING**

Before each use of this chair:

1. Make sure the chair operates smoothly. Check for noise, vibration, or a change in ease of use. (They may indicate low tire pressure, loose fasteners, or damage to your chair). If you detect a problem, make sure to repair or adjust the chair. Deferring repair or adjustment could increase the risk for injury. Your supplier can help you find and correct the problem.
2. Make sure batteries are charged. Green lights on the controller charge indicator will light up when charge is full. Yellow lights indicate battery charge level is getting low. Red lights indicate batteries are in immediate need of charging.

G. CHANGES & ADJUSTMENTS**WARNING**

Never use non-Quickie parts or make changes to your chair unless authorized by Sunrise. (Doing so will void the Warranty and may create a safety hazard).

1. If you modify or adjust this chair it may increase the risk of a fall or tip-over.
2. Modifications unauthorized by Sunrise constitutes remanufacturing of the wheelchair. This voids the warranty. The rider then assumes all future liability for the wheelchair.

H. WHEN SEATED IN A PARKED WHEELCHAIR

WARNING

1. Always turn off all power to your chair when you are parked, even for a moment. This will prevent:
 - Accidental movement from contact with the joystick by you or others.
 - Unintended brake release or movement from EMI sources. (See Section V)
2. Make sure that persons who help you (for example, store clerks) are aware of the joystick and do not touch it. If they do, your chair may move suddenly when you do not expect it.

I. ENVIRONMENTAL CONDITIONS

WARNING

Your chair is not designed for use in a heavy rain storm, or in snowy or icy conditions.

1. Contact with water or excessive moisture can cause an electrical malfunction. The frame, motors and other chair parts are not watertight and may rust or corrode from the inside. To avoid a chair failure:
 - Minimize exposure of your chair to a rain storm or very wet conditions.
 - Never take your chair into a shower, tub, pool or sauna.
 - Do not use your chair in fresh or salt water (such as at the edge of a stream, lake, or ocean).
 - Make sure shroud cover and deck lid are secure.
 - Replace joystick boot if it becomes torn or cracked.
 - Make sure all electrical connections are secure.
 - Dry the chair as soon as you can if it gets wet, or if you use water to clean it.
2. Proceed slowly and use extra care if you must operate your chair on a wet or slick surface.
 - Do so only if you are sure it is safe.
 - Stop if one or both main wheels lose traction. If this occurs, you may lose control of your chair or fall.
 - Never operate your chair on a slope or ramp if there is snow, ice, water or oil film present.
 - When in doubt, have someone help you.
3. When not in use, keep your chair in a clean, dry place.

J. TERRAIN

WARNING

1. This chair is designed for use on firm, even surfaces such as concrete, asphalt and indoor flooring.
2. Do not operate your chair in sand, loose soil or over rough terrain. Doing so may damage wheels, bearings, axles or motors, or loosen fasteners.

K. STREET USE

WARNING

In most states, power chairs are not legal for use on public roads. Be alert to the danger of motor vehicles on roads or in parking lots.

1. At night, or when it is hard to see, use reflective tape on your chair and clothing.
2. It may be hard for drivers to see you. Make eye contact with drivers before you proceed. When in doubt, yield until you are sure it is safe.

L. MOTOR VEHICLE SAFETY

WARNING

To date, the U.S. Department of Transportation has not approved any tie down system for transporting this wheelchair in a motor vehicle.

1. Never sit in this chair while in a moving vehicle. In an accident or sudden stop you may be thrown from the chair.
 - **Wheelchair belts are designed to position the rider only and will not protect you in an accident; further injury may result from the belts.**
2. Always move to an approved vehicle seat. You must be secured with proper motor vehicle restraints.
3. Never transport this chair in the front seat of a vehicle. It may shift and interfere with the driver.
4. Always secure this chair so that it cannot roll or shift.

M. CENTER OF BALANCE

WARNING

The point where this chair will tip forward, back or to the side depends on its center of balance and stability.

The Center Of Balance Is Affected By:

1. The seat height and seat angle.
2. A change in your body position, posture or weight distribution.
3. Using this chair on a ramp or slope.
4. The use of a back pack or other options, and the amount of added weight.

To Reduce The Risk Of A Fall Or Tip-Over:

1. Consult your supplier for information on modifications authorized by Sunrise before you modify or adjust this chair.

NOTE— You may need to make additional changes to correct the center of balance.

2. Use extreme care until you know the balance points of this chair and how to avoid a fall or tip-over.

N. TRANSFERS

WARNING

It is dangerous to transfer on your own. It requires good balance and agility. Be aware that there is a point during every transfer when the wheelchair seat is not below you. To avoid a fall:

1. Always turn off power before you transfer to or from your chair. If you fail to do so you may touch the joystick and cause your chair to move when you do not expect it.

2. Make sure motor locks are engaged. This keeps the chair from moving when you transfer.
3. Work with your health care professional to learn safe methods.
 - Learn how to position your body and how to support yourself during a transfer.
 - Have someone help you until you are sure you can do a safe transfer on your own.
4. Move your chair as close as you can to the seat you are transferring to. If possible, use a transfer board.
5. Rotate the front casters until they are as far forward as possible.
6. Be careful of the foot platform. If you can, remove or swing it out of the way.
 - Never stand on footrests when you transfer. Doing so may damage them or cause your chair to tip.
 - Make sure your feet do not “hang up” or get caught in the space between the footrests.
7. Make sure armrests do not interfere.
8. Transfer as far back onto the seat surface as you can. This will reduce the risk that you will miss the seat or fall.

O. REACHING OR LEANING

WARNING

Reaching or leaning affects the center of balance of your chair. If done improperly, a fall or tip-over is likely. When in doubt, ask for help or use a device to extend your reach.

To Reduce the Risk of injury and/or Damage to the Chair:

1. Never reach or lean if you must shift your weight sideways or rise up off the seat.
2. Never reach or lean if you must move forward in your seat to do so. Always keep your buttocks in contact with the backrest.
3. Never reach with both hands (you may not be able to catch yourself to prevent a fall if you lose your balance).
4. Never try to pick up an object from the floor by reaching down between your knees.
5. Never put pressure on the foot platform while reaching. This may cause the chair to tip if you lean too far.
6. Never reach or lean over the top of the seat back. This may damage the backrest and cause you to fall.

IF YOU MUST REACH OR LEAN, DO SO AT YOUR OWN RISK.

Remember to:

1. Move your chair as close as you can to the object you wish to reach.
2. Rotate the front casters until they are as far forward as possible. This makes the chair more stable.

Note— To do this: Move your chair past the object you want to reach, then back up alongside it. Backing up will rotate the casters forward.

3. Turn off all power to your chair. If you fail to do so, you may touch the joystick and cause your chair to move when you do not expect it.
4. Firmly grasp an armrest with one hand. This will help to prevent a fall if the chair tips.

P. DRESSING OR CHANGING CLOTHES

WARNING

Be aware that your weight will shift if you dress or change clothes while seated in this chair. To make the chair more stable, rotate the front casters until they are forward.

Q. OBSTACLES

WARNING

Obstacles you may have to overcome in daily use include door thresholds, lifts, ramps and hazards such as potholes and broken pavement. These can damage your chair and may cause a fall, tip-over or loss of control.

1. Be aware that thresholds are very dangerous. (Even a small change in height may stop a caster wheel and cause your chair to tip). You may need to:
 - Remove or cover threshold strips between rooms.
 - Install a ramp at entry or exit doors.
2. Keep your eyes moving when you ride; scan the area well ahead of your chair.
3. Make sure the floor areas where you use this chair are level and free of obstacles.
4. To help correct your center of balance:
 - Lean your upper body forward slightly as you go up over an obstacle.
 - Press your upper body backward as you go down from a higher to a lower level.

R. DRIVING IN REVERSE

WARNING

Use extra care when you drive your chair in reverse. You may lose control or fall if one of the rear wheels hits an object.

1. Operate your chair slowly and at an even speed.
2. Stop often and check to make sure your path is clear of obstacles.

S. RAMPS, SLOPES & SIDEHILLS

WARNING

The center of balance of your chair changes when you are on a slope.

NOTE— “Slope” includes a ramp or sidehill. Your chair is less stable when it is at an angle. Never use this chair on a slope unless you are sure it is safe. When in doubt, have someone help you.

Beware Of:

1. Steep slopes. Do Not use this chair on a slope steeper than 10%. (A 10% slope means: One foot in elevation for every ten feet of slope length).
2. Wet or slippery surfaces (such as when ice, snow, water or oil film is present). A loss of traction may cause a fall or tip-over.
3. A change in grade on a slope (or a lip, bump or depression). These may cause a fall or tip-over.
4. A drop-off at the bottom of a slope. (A drop-off of as small as 3/4 inch can stop a front caster and cause the chair to tip forward).

T. TO REDUCE THE RISK OF A FALL, TIP-OVER OR LOSS OF CONTROL

WARNING

1. Never use your chair on a slope unless you are sure you can do so without losing traction.
2. Always go as straight up and as straight down as you can.
 - Do not “cut the corner” on a slope or ramp.
 - Do not turn or change direction on a slope.
3. Always stay in the center of the ramp. Make sure ramp is wide enough that you are not at risk that a wheel may roll off the side.
4. Lean or press your body uphill. This will help adjust for a change in the center of balance caused by the slope.
5. Keep your chair moving at a slow, steady speed. Keep control over the chair at all times.
 - On a descent, do not let your chair accelerate beyond its normal speed.
 - If the chair picks up speed, center the joystick to slow down or stop.

NOTE— *The solid state controller of your chair has a logic system that will help control your speed when driving on a slope or uphill.*

- If you stop, re-start slowly.

U. RAMPS AT HOME & WORK

WARNING

Make sure ramps meet all Building Codes for your area.

1. For your safety, have a licensed contractor build or remodel ramp to meet all standards.
2. **NOTE—** The proper design will vary, depending on such things as: the length and height of the ramp; the need for an intermediate platform; landing size; doors and the direction of swing; and whether the ramp includes a turn or angle.

At A Minimum:

1. Open sides of ramp must have side rails to prevent your chair from going over the edge.
2. Slope must not be steeper than one inch in height for every one foot of slope length.
3. Ramp surface must be even, and have a non-skid surface.
4. You may need to add a section at the top or bottom to avoid a lip or drop-off.
5. Ramp must be sturdy. Add bracing if needed, so ramp does not “bow” when you ride on it.

V. WHEELCHAIR LIFTS

WARNING

Wheelchair lifts are used in vans, buses, and buildings to help you move from one level to another.

1. Always turn off all power to your chair when you are on a lift. If you fail to do so, you may touch the joystick by accident and cause your chair to drive off the platform. (Be aware that a “roll-stop” at the end of the platform may not prevent this).
2. Make sure there is not a lip or drop-off at the top or bottom of the platform. These may cause a fall or tip-over. When in doubt, have someone help you.
3. Always secure the rider with a positioning belt to help prevent falls during transfers.
4. Avoid moving forward if a wheel is “hung up” on the lip of the ramp. Backup, reposition the caster for a more direct approach and slowly try again.

W. CURBS & SINGLE STEPS

WARNING

1. Your chair is not designed to drive up or down a curb or step more than two and a half (2.5) inches high. Doing so may:
 - Result in a fall or tip-over.
 - Damage the frame, wheels, axles or other chair parts, or loosen fasteners.
2. To prevent a fall or tip-over, use wheelchair access ramps or have someone help you.
3. If you must climb or descend a curb or step alone do so at your own risk and use extreme care.
 - Go as straight up or straight down as you can. Never turn or climb or descend at an angle as a fall or tip-over is likely.
 - Proceed slowly, at a steady speed.
4. Avoid driving along curb edges and drop-offs. A minimum distance from the edge of 18” is recommended.

X. STAIRS

WARNING

Never use this chair to go up or down stairs, even with an attendant. Doing so is likely to cause a fall or tip-over.

Y. ESCALATORS

WARNING

Never take this chair on an escalator, even with an attendant. Doing so is likely to cause a fall or tip-over.

VII. WARNINGS: COMPONENTS & OPTIONS**WARNING**

If you use parts or make changes not authorized by Sunrise it may create a safety hazard and will void the Warranty.

A. ARMRESTS**WARNING**

Armrests cannot be used to lift chair or the seat.

1. Never lift this chair by its armrests.
2. Lift this chair only by non-detachable parts of the main frame.

B. BATTERIES**WARNING**

1. Only deep cycle sealed case construction batteries should be used in this device.
2. To prevent an acid spill, always keep batteries upright. (wet cell batteries only)
3. Never smoke or hold an open flame near batteries. They are a known explosion hazard.
4. Always wear rubber gloves and safety glasses when you handle batteries.
5. Read all of section X Batteries before attempting to change or charge batteries.

C. CUSHIONS**WARNING**

1. Quickie Highback seats are designed for comfort, not specifically for the relief of pressure.
2. If you suffer from pressure sores, or if you are at risk that they will occur, you may need a special seat system such as can be provided by the Seat Frame, or a device to control your posture.
 - Consult your doctor, nurse or therapist to find out if you need such a device for your well-being.

D. FASTENERS**WARNING**

Many of the screws, bolts and nuts on this chair are special high-strength fasteners. Use of improper fasteners may cause your chair to fail.

1. Only use fasteners recommended by Sunrise.
2. If fasteners become loose, tighten them as soon as you can.
3. Over- or under-tightened fasteners may fail or cause damage to chair parts.
 - See Section VIII, "Set-Up, Adjustment & Use", for proper torque settings.

E. FOOTPLATE & FOOTRESTS**WARNING**

1. At the lowest point, the foot platform and footrests should be at least 2 1/2" off the ground. If set too LOW, it may "hang up" on obstacles you can expect to find in normal use. This may cause the chair to stop suddenly and tip forward.

2. To avoid a trip or fall when you transfer:
 - Make sure your feet do not "hang up" or get caught in the space surrounding or in between the footrests.
 - Avoid putting weight on the footplate or footrests, as the chair may tip forward.
 - Flip the footplate back or swing the footrests out of the way, if possible.
3. Never lift this chair by the footplate or footrests. Footrests fold or detach and will not bear the weight of this chair. Lift this chair only by non-detachable parts of the main frame.

F. MOTOR LOCKS**WARNING**

1. Do not engage or disengage motor locks unless power to the chair is off.
2. Be aware that the chair will not have brakes when motor locks are in the free-wheel position. (See Section F of Operating Guide.)
3. Make sure that the person pushing the chair has full control when motor locks are disengaged.

G. ON/OFF SWITCH**WARNING**

1. Never use the ON/OFF switch to stop the chair except in an emergency. This will result in an abrupt stop and may cause you to fall.
2. To slow your chair to a stop, return the joystick to neutral, before turning off the chair.

H. POSITIONING BELTS**WARNING**

Use a positioning belt only to help support your posture. Improper use of such belts may cause severe injury or death.

1. Make sure you are not at risk to slide down in the wheel-chair seat. If this occurs, you may suffer chest compression or suffocate due to pressure from the belt.
2. A pelvic wedge or a similar device can help you from sliding down in the seat. Consult your health care professional to find out if you need such a device.
3. The belt must be snug, but must not be so tight that it interferes with breathing. You should be able to slide your open hand, flat, between the belt and your stomach.
4. Make sure you can easily remove the belt in an emergency.
5. Never use a positioning belt:
 - In place of a motor vehicle seat belt. In an accident or sudden stop you may be thrown from the chair. A positioning belt will not prevent this, and further injury may result from the belt.
 - As a restraint. A restraint requires a doctor's order.
 - On a rider who is comatose or agitated.

I. SEATING SYSTEMS**WARNING**

1. Use of a seating system not approved by Sunrise may alter the center of balance of this chair. This may cause a fall or tip-over.
2. Never change the seating system of your chair unless you consult your supplier first.

J. UPHOLSTERY FABRIC**WARNING**

1. Replace worn or torn fabric of the seat as soon as you can. If you fail to do so, the seat may fail and cause you to fall. Worn fabrics may increase the potential for a fire hazard.
2. Fabric will deteriorate with age and use. Look for fraying, thin spots, or stretching of fabric at rivet holes. Replace fabric as required.
3. Be aware that washing may reduce flame retardation of the fabric.

K. SUSPENSION**WARNING**

The suspension system does contain Dry Natural Rubber. The suspension, or any other component on the chair, does not contain any latex based product. The suspension components are enclosed and inaccessible to the user.

L. WIRING**WARNING**

Never pull on cables directly. This can result in wires breaking inside the connector or harness. To remove a plug or connector, always grasp the plug or connector itself. (See section XI for wiring diagram.)

M. POWER SEATING (SC only)**WARNING**

1. **Operation**
 - While driving this wheelchair in the tilted position use caution.
 - Do Not attempt to operate any power seating option while positioned on an incline.
 - Do Not attempt to operate any power seating option while this wheelchair is in motion.
 - Do not attempt to operate the power seating options around children.
2. **Drive Inhibit**

This power chair is designed to automatically reduce driving speed when the power seating is tilted more than 16°.
3. **Pinch points**

Pinch points may occur when operating the powered seating options on this chair. Ensure all hands and body parts are clear of all power seating components that may result in pinch points before operating them.
4. **Tilting Power Base with Push Handles**

Do not attempt to tilt power wheelchairs by pulling down on the push handles. Power wheelchairs have heavy bases. As such, attempting to tilt a power wheelchair to traverse obstacles can damage components in the seating system and/or modular actuators.

5. Remain Clear During Power Accessory Operation

Please stay clear of any powered actuator while the component is in motion. Sunrise power mechanisms move over large ranges of travel. Users should be aware of their surroundings while components are in motion.

6. Maximum User Weight

Do not exceed the weight capacity for your specific chair/seating system configuration. Doing so could result in premature failure or injury.

N. S.P.O.T. (Single Power Option Tilt) POWER ACCESSORY (SC only)**WARNING**

The powered S.P.O.T. accessory for this chair has a maximum user weight capacity of up to 300 lbs. Never exceed this capacity while operating the powered actuator.

WARNING

The powered S.P.O.T. should not be operated for more than three minutes of continuous use. Doing so may shorten the life of the powered S.P.O.T..

O. TRANSIT OPTION**WARNING**

Contact Sunrise Medical Customer Service (800-333-4000) with any questions about using this power wheelchair for seating in a motor vehicle or if a Pulse 6 is a candidate for the transit option

Whenever possible, wheelchair occupants should transfer into the vehicle seat and use the OEM (Original Equipment Manufacturer) vehicle-installed restraint system.

This wheelchair has been dynamically tested in a forward-facing mode with a specified crash test dummy restrained by both pelvic and upper-torso belt(s) (shoulder belts). It has been determined that both the pelvic and upper-torso belt(s) should be used to reduce the possibility of head and chest impacts with vehicle components. When travelling in a motor vehicle, use only Wheelchair Tie-down and Occupant Restraint Systems (WTORS) which meet the requirements of the SAE (Society of Automotive Engineers) J2249 Recommended Practice.

This wheelchair has been tested for seating in a motor vehicle with the factory-installed seating system only. This wheelchair must be in a forward facing position during travel in a motor vehicle.

This wheelchair is equipped with wheelchair-anchored pelvic belts. It has been dynamically tested to rely on wheelchair-anchored pelvic belts. If desired, vehicle-anchored pelvic belts may also be used.

It is strongly recommended that the combination of both pelvic and upper-torso belt(s) be used to reduce the risk of injury if a sudden stop and/or collision were to occur.

To reduce the potential of injury to vehicle occupants, wheelchair-mounted accessories, including but not limited to IV poles, trays, respiratory equipment, backpacks, and other personal items should be removed from the wheelchair and secured separately.

Postural supports, positioning devices, and/or strap(s) should not be relied on for occupant restraint. These items may be used in addition to the wheelchair anchored, or vehicle-anchored belts.

Wheelchairs with adjustable seat angles must be set to 10°. Do not alter or substitute wheelchair frame parts, components, or seating systems.

A sudden stop and/or collision may structurally damage your wheelchair. Wheelchairs involved in such incidents should be replaced.

Spill proof batteries, such as “gel cells”, should be installed on wheelchairs to be used during travel in a motor vehicle.

WARNING

NOTE: Only use the transit brackets included with Quickie Pulse 6 for the purposes described in this manual.

ABOUT TRANSPORT READY PACKAGES

The Quickie Pulse 6 Transit System includes four factory-installed transport brackets and an optional wheelchair anchored pelvic belt. The Quickie Pulse 6 has been crash-tested in accordance with ANSI/RESNA WC Vol I Section 19 Frontal Impact Test requirements for wheelchairs with a 170 lbs. crash dummy.

As of this date, the Department of Transportation has not approved any tie-down systems for transportation of a user while in a wheelchair, in a moving vehicle of any type. It is Sunrise Medicals position that users of wheelchairs should be transferred into appropriate seating in vehicles for transportation and use be made of the restraints made available by the auto industry. Sunrise Medical cannot and does not recommend any wheelchair transportation systems.

COMPLIANCE INFORMATION

This wheelchair conforms to the requirements of the ANSI/RESNA WC/Vol. I - Section 19.

NOTE: ANSI = American National Standards Institute, RESNA = Rehabilitation Engineering and Assistive Technology Society of North America.

This wheelchair has been dynamically tested in a forward-facing mode with a specified crash test dummy, restrained by both pelvic and upper-torso belts in accordance with ANSI/RESNA WC Vol I Section 19. Both pelvic and upper-torso belts should be used to reduce the possibility of head and chest impacts with vehicle components in the event of a sudden stop and/or collision

SECURING THE WHEELCHAIR TO THE VEHICLE

(See Figures 1,2 below)

WARNING



This wheelchair must be in a forward facing position during travel in a motor vehicle. The recommended clear zones for wheelchair seated occupants restrained by both pelvic and upper-torso belt(s) and only by a pelvic belt are shown in the diagrams and described below. Frontal Clear Zones (FCZ) need to be larger when upper-torso belt(s) are not used.

The rear clear zone of 16-inches is measured from the rearmost point on an occupant's head.

The frontal clear zone is measured from the front most point on an occupant's head and is 26-inches with pelvic and upper-torso belt(s) and 37-inches with only a pelvic belt.

The frontal clear zone may not be achievable for wheelchair-seated drivers.

The estimated seated height (HHT) from the ground or floor to the top of the wheelchair-seated occupant's head ranges from approximately 47-inches for a small adult female to about 61-inches for a tall adult male.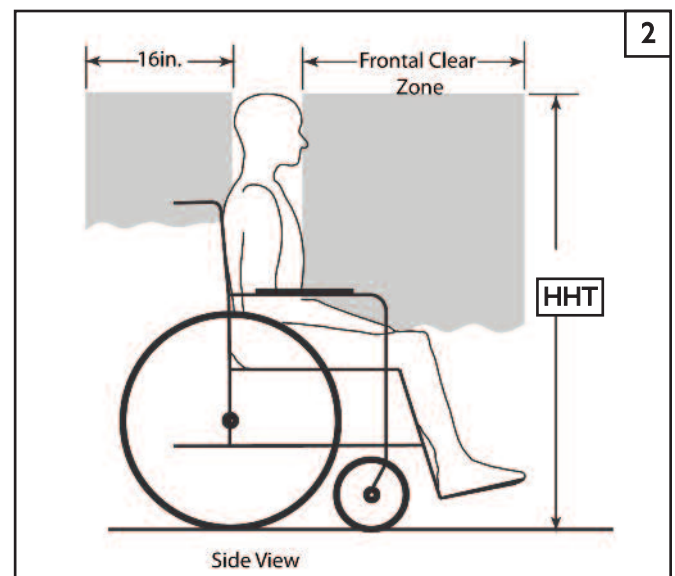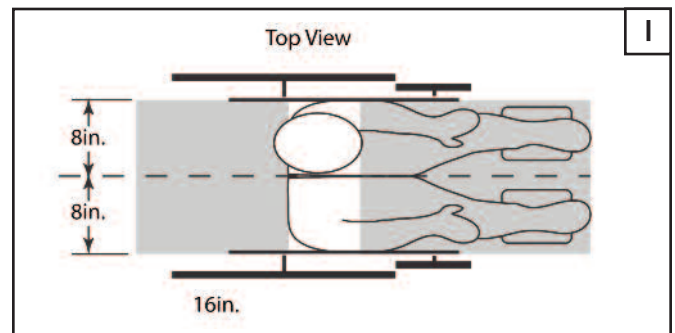

SECUREMENT POINTS

Rear Securement points(A), Front Securement points(B), Pelvic Belt Securement point(C)

SECURING THE WHEELCHAIR

This wheelchair is to be used only with Wheelchair Tie-down and Occupant Restraint Systems (WTORS) that have been installed in accordance with the manufacturer's instructions and SAE J2249.

NOTE: A copy of SAE J2249 Wheelchair Tie-down and Occupant Restraint Systems (WTORS) for use in Motor Vehicles can be obtained from:

SAE International,
400 Commonwealth Drive,
Warrendale, PA 15096-0001
(877) 606-7232 or (724) 776-4970

Attach WTORS to the tie-down brackets in accordance with the manufacturers instructions and SAE J2249.

SECURING THE OCCUPANT

WHEELCHAIR-ANCHORED BELTS (Figure 3,4)

WARNING

Only use a pelvic belt system that meets the requirements of ANSI/RESNA W/C 19 and SAE J2249 and has been specified to work in conjunction with the Quickie Pulse 6 Transit System.

1. Install the pelvic belt (A) onto the pelvic belt securement point anchored to the seating frame of the Quickie Pulse 5 or 6. Using a 10mm Allen head wrench and torque to 144 in-lbs securing the pelvic belt to the mounting anchor (B).
2. Repeat Step 1 for the opposite side.

VEHICLE-ANCHORED BELTS (Figure 5)




This wheelchair has an overall rating of "A" with regard to accommodating the use and fit of vehicle-anchored belts. This rating is scored as follows:

- A = Excellent
- B = Good
- C = Fair
- D = Poor

The test for Lateral Stability Displacement for Point (P) is shown in the figure below. The average test result for point (P) is:

- Quickie Pulse 6 - 0.547-inches (13.9 mm).
- Quickie Pulse 5 - 0.547-inches (13.9 mm).

NOTE: Figure 5 shows Rear view of the wheelchair and human surrogate secured on test platform and tilted to 45°.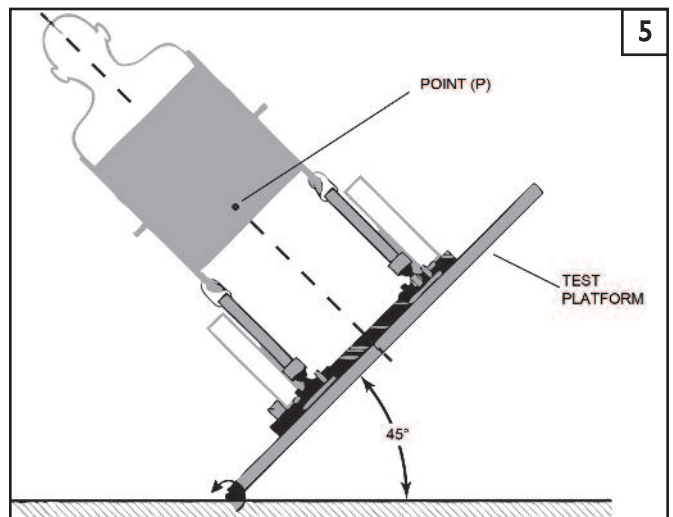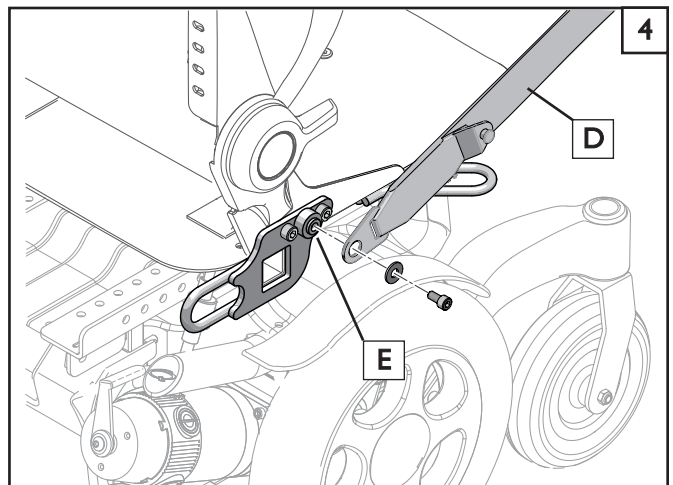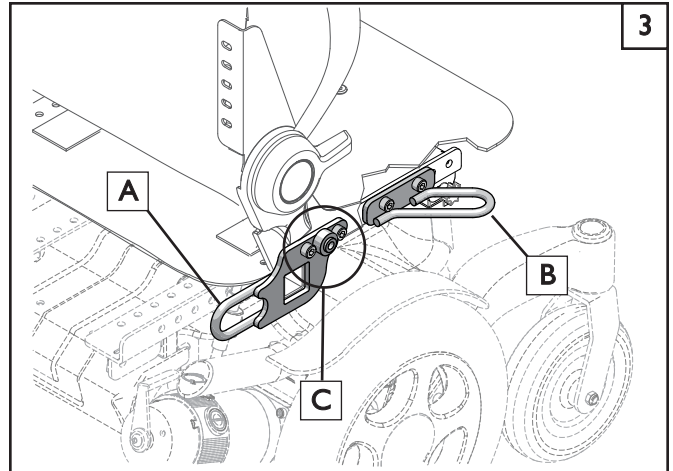

SEATING SYSTEM

WARNING

This wheelchair has been tested for seating in a motor vehicle with the factory installed seating system only.

Ensure that the factory installed seating system is secured to the wheelchair frame before operation. Refer to the seating system owner's manual

CAPTAINS SEAT TRANSIT OPTION

LOCKING PIN ACTIVATION (Figure 6,7)

WARNING

Captains seat must be used in conjunction with WC-19 Tie-down brackets.



1. Configuration for Standard use (Driving the Chair)

- Locking Pin Lever (A) should be Disengaged, (Figure 6)
- Backrest can be reclined or folded as needed.

2. Transit Configuration (Captains seat used in a vehicle)

- Locking Pin Lever (B) should be engaged, (Figure 7)
- Backrest is now in locked position (99°).

MAKE SURE TO ENGAGE LOCKING PIN FOR TRANSIT USE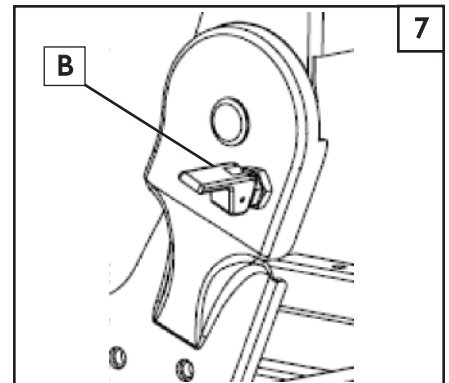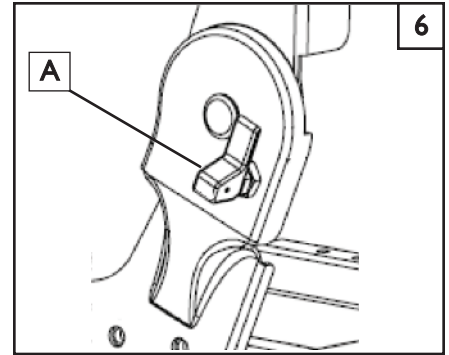

POSITIONING BELTS (Figure 8,9)



WARNING

The angle of the pelvic belt should be within the preferred zone of 45 to 75 degrees to the horizontal or within the optional zone of 30 to 45 degrees to the horizontal.

Steeper side-view pelvic belt angles are especially important if the pelvic belt is intended to be used for postural support in addition to occupant restraint in a frontal crash. Steeper angles will reduce the tendency for a vertical gap to develop between the user and the belt due to compliance of seat cushions and belt movement, thereby reducing the tendency for the user to slip under the belt and for the belt to ride up on the soft abdomen during normal use. Steeper belt angles also reduce the tendency for upper-torso belts to pull the pelvic belt onto the abdomen during frontal impact loading.

NOTE: For this procedure, refer to figure 9

1. The pelvic belt should be worn low across the front of the pelvis.
2. Position the upper-torso belt(s) over the shoulders.
3. The belt(s) should not be held away from the body by wheelchair components or parts, including but not limited to wheelchair armrests or wheels.
4. Ensure the belt(s) are not twisted.
5. Adjust belts as firmly as possible, being mindful of user comfort.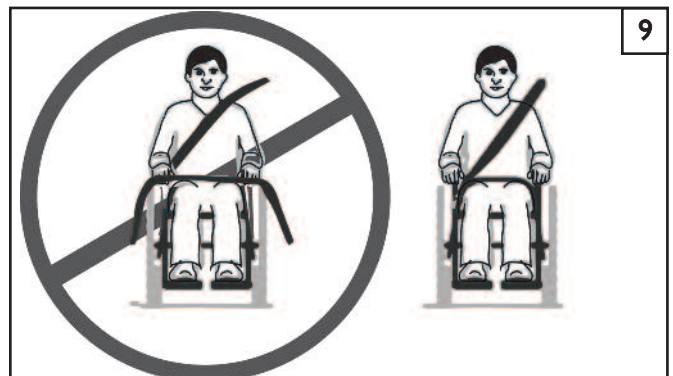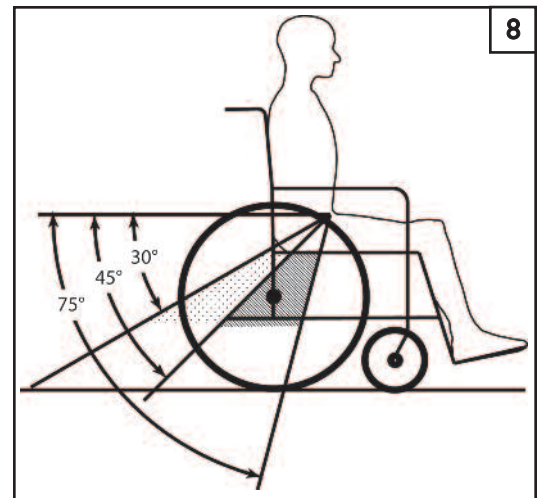

VIII. SET-UP, ADJUSTMENT & USE

NOTES:

1. **Work Surface For Set-Up:**

Use a flat surface, such as a table, to assemble, adjust and check your chair. This makes the steps easier and helps ensure a correct set-up.

2. **Fasteners:**

- Many of the screws and bolts on this chair are special high-strength fasteners and may have special coatings.
- Many nuts are of the Nylock type. They have a plastic insert to help prevent loosening.

WARNING

1. Use of improper fasteners may cause the chair to fail.
 2. Over- or under-tightened fasteners may fail or cause damage to chair parts.
 3. If bolts or screws become loose, tighten them as soon as you can. Loose bolts or screws can cause damage to other chair parts causing them to fail.
3. **Washers & Spacers:**
- Note the position of washers and spacers before disassembly.
 - To avoid damage to the frame, replace all washers and spacers when you reassemble parts.
4. **Torque Settings:**
- A torque setting is the optimal tightening for a particular fastener. Use a torque wrench that measures 120 inch-pounds to secure screws, nuts and bolts on this chair.

NOTE— Unless otherwise noted, use a torque setting of 120 inch-pounds for all fasteners.

TOOLS YOU WILL NEED

- | | |
|---------------------------|---------------------------|
| • 5mm Allen Wrench | • 17mm deep socket wrench |
| • 6mm Allen Wrench | • 19mm socket wrench |
| • 8mm Allen Wrench | • Phillips screwdriver #2 |
| • 3/8" combination wrench | • Flat-head screwdriver |
| • 10mm combination wrench | • Wire cutters |
| • 13mm combination wrench | • Needle nose pliers |
| • 18mm combination wrench | |

1. **Basic Tool Kit:**

To set-up, adjust and maintain your chair you will need the following tools:

2. **Torque Wrench:**

If you plan to adjust and maintain this chair yourself, Sunrise recommends that you use a torque wrench.

NOTE— The wrench must measure inch-pounds. You can buy a torque wrench and proper sockets from a hardware store.

CHECK-OUT

Be sure this chair performs to your chosen operational settings. If it does not, turn the chair off immediately and have your supplier reprogram your wheelchair as needed.

NOTE— Repeat this procedure until the chair performs to your specifications.

WARNING

When properly set up, this chair will operate smoothly. Check to see that all components work properly. If you detect a problem, be sure to correct it before use.

A. BATTERY REMOVAL (Figures 10, 11)

To remove the batteries for transporting or servicing:

1. Remove Batteries



- Carefully remove seat (see Section O. Seat Removal).
- Remove the shroud by first removing the six fasteners (Figure 10). Then lift up on the shroud.
- Unplug main power connect (A) (Figure 11).
- Lift batteries out of the frame using the battery strap as a lifting handle

2. Battery Installation

See Section X. Batteries, D for installation.

WARNING

Batteries may weigh up to 39 lbs. Care must be taken to avoid injury when lifting up on battery straps.

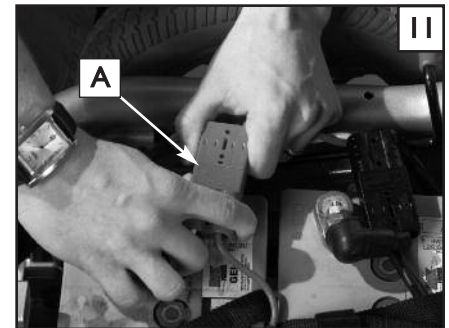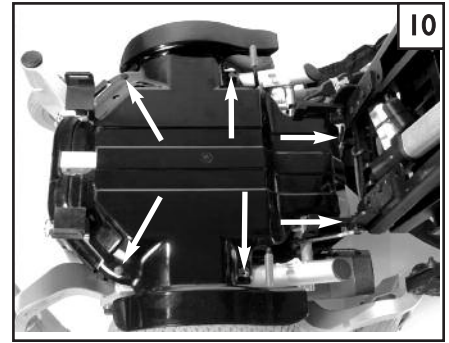**B. CENTER MOUNT HANGER ADJUSTMENT****1. Removal (Figure 12)**




- Remove the four bolts (B) and slide the footrest out.

2. Height Adjustment (Figure 13)

- Remove the six securing bolts (C).
- Move footplate to desired height.
- Re-install bolts (C) through desired holes.

3. Calf Pad Adjustment (Figure 14)

- Remove the two securing bolts (D).
- Move calf pads to desired height.
- Re-install bolts (D) through desired holes.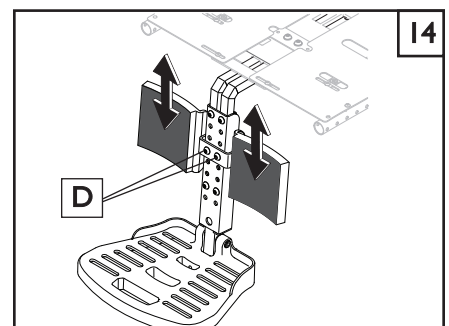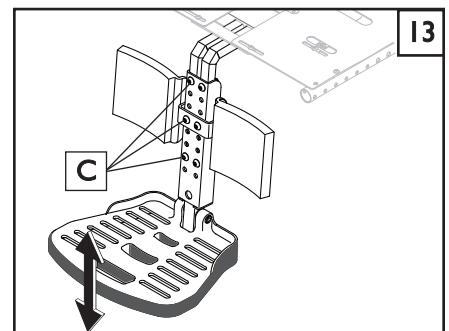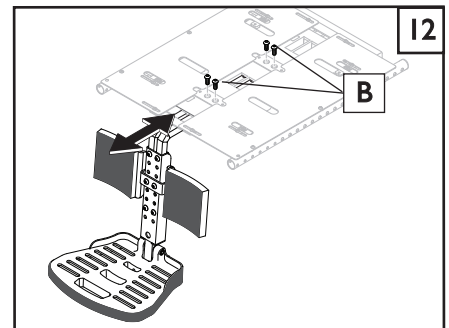

C. SWING-AWAY FOOTRESTS

1. Installation (Figure 15)



- Place swing-away pivot pin (E) into the locating hole (F) on top of the hanger mount with the footrest facing outward from frame.
- Rotate the footrest inward until latch plate locks (G) into place on locking bolt.

2. Removal (Figure 16)

- To remove footrest, press quick-release lever (H).
- Rotate footrest outward and lift

3. Height Adjustment (Figure 16)

- Remove both bolts (I) in the hanger frame tube.
- Reposition footrest height.
- Re-tighten all bolts (I).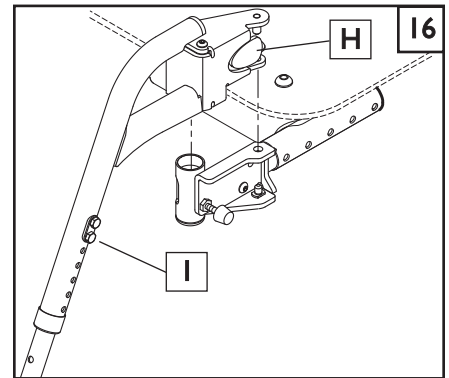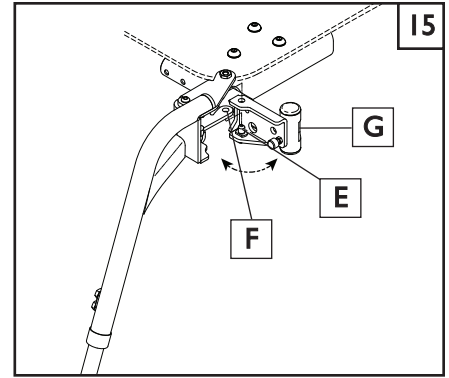


D. ELEVATING LEGREST (Optional)

1. Installation

To install or remove Elevating Legrest (ELR) see instructions for Swing-away Footrest installation.

2. Adjustment (Figure 17)

- To raise legrest, lift to desired position. Legrest will automatically lock in place.
- To lower legrest, while seated in chair, first lift slightly then press release lever (J) down and lower legrest to desired position.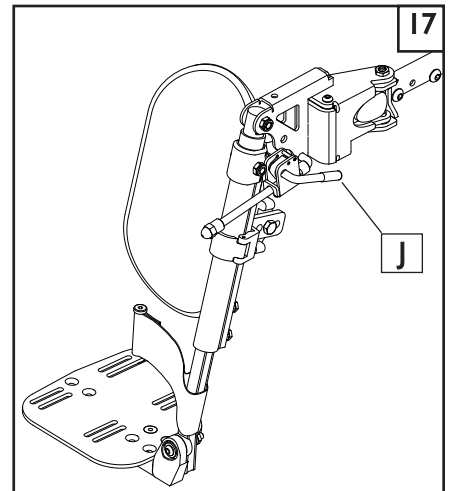

E. FOLDING BACKREST (optional) (Figure 18)

1. Upright Positioning


- Lift backrest to upright position so that latch handles (K) lock into place.
- Backrest is ready for use.

2. Folding Backrest

- Lift and turn backrest latch handles (K) inward or outward.
- Fold backrest forward.

WARNING


Backrest latches must be locked into place before operating wheelchair.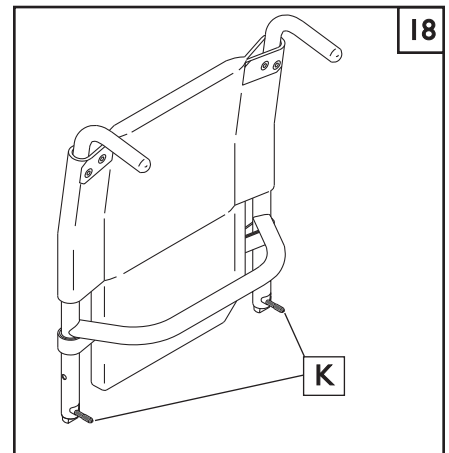

F. DUAL-POST HEIGHT-ADJUSTABLE ARMRESTS (OPTION), FIXED HEIGHT STANDARD (Figure 19)

The dual-post flip-back armrest can be used as either a flip-back armrest or a removable armrest.

1. **Installing Armrest**
 - a. Insert front and rear posts into armrest receivers.
 - b. Engage levers (A and B) to secure armrest.
2. **Set-up for Flip-back Operation**
 - a. Disengage lever (A) so front post is able to come free.
 - b. Armrest now has the ability to be flipped back without removal.
3. **Set-up for Removal**
 - a. Disengage levers (A and B) so front and rear posts are able to come free.
 - b. Armrest can now be easily removed for transferring.
4. **Height-Adjustment (option)**
 - a. Release the upper securing lever (C).
 - b. Set at desired height
 - c. Return securing lever to locked position.
 - d. Move armrest up or down to allow armrest to snap into place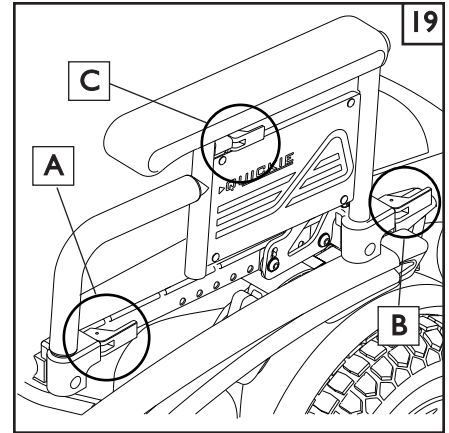


Height-Adjustable Armrest Key

1. Inner Arm Post
2. Receiver
3. Release Lever, Locked Position
4. Armrest Pad
5. Transfer Bar
6. Side Panel
7. Tension Adjustment Set Screw
8. Outer Arm Post
9. Receiver Adjustment Bolts


G. SINGLE POST HEIGHT-ADJUSTABLE ARMRESTS (optional) (Figure 20)

1. **Installation**
 - a. Slide the outer armpost into the receiver mounted to the wheel chair frame.
 - b. The armrest will automatically lock into place.
2. **Height Adjustment**
 - a. Rotate release lever to stop.
 - b. Slide armrest pad up or down to desired height.
 - c. Return lever to locked position.
 - d. Push arm pad until upper armpost locks firmly into place.
3. **Removing Armrest**
 - a. Rotate release lever and remove the armrest.
4. **Replacing Armrest**
 - a. Slide armrest back into receiver.
 - b. Return release lever to locked position against armpost.
5. **Adjusting Armrest Receiver Fit (Tighten or loosen fit)**
 - a. Loosen the four bolts on the sides of the receiver.
 - b. With the armrest in the receiver, squeeze the receiver to achieve the desired fit.
 - c. Tighten the four bolts.
6. **Adjusting Inner Armpost Fit**
 - a. Two set screws are installed in the outer armpost.
 - b. Turn the set screws in or out until the desired fit is achieved.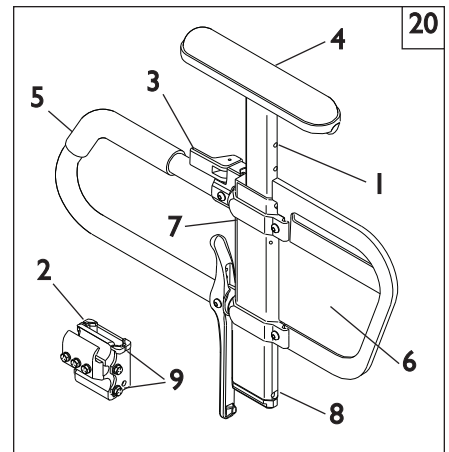

H. ASAP SEAT DEPTH 14" - 16", 20" - 22" only (Figure 21)

1. **Seat Frame Depth Adjustment**
 - a. To adjust, remove the two bolts (A & B) from each side of the backrest pivot plate.
 - b. Reposition the backrest to the desired position.
 - c. Replace and retighten bolts on each side of the backrest pivot plate.
2. **Back Angle Adjustment on Seat Frame**
 - a. Remove the front securing bolt (D) on the side of the backrest hinge plate.
 - b. Loosen the lower rear bolt (E)
 - c. Set at desired angle. There are five holes (in 4° increments) to choose from.
 - d. Reinstall the front bolt and tighten both bolts securely.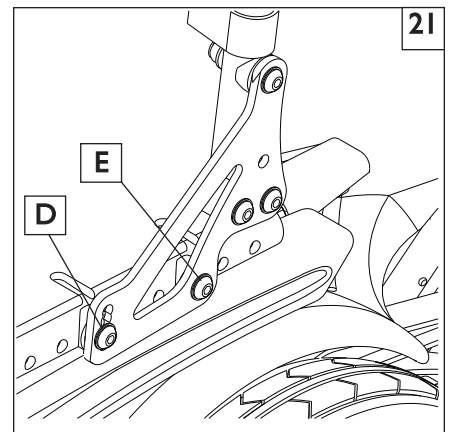


I. CAPTAIN'S SEAT

Manual Recline/Folding Seatback adjustment

To adjust the recline angle(Figure 22,23)


1. Reclining Position

- To adjust, pull up on the seat back release lever (A) located on the right side of the backrest.
- Lean forward or backwards to your desired position..
- Release the lever (A) Wiggle the backrest to make sure it locks in place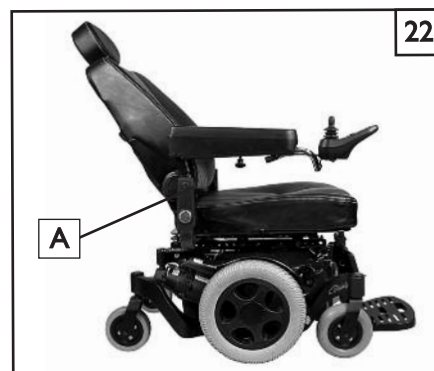

2. To Fold Seat back (unoccupied)


- Pull up on the seat/back release lever (A) located on the right side of the backrest.
- Backrest should spring forward by itself into the folded position, flat against the seat cushion.
- Release the lever (A).
- To unfold the back, pull up on the seat/back release lever (A) and adjust the back to an upright position.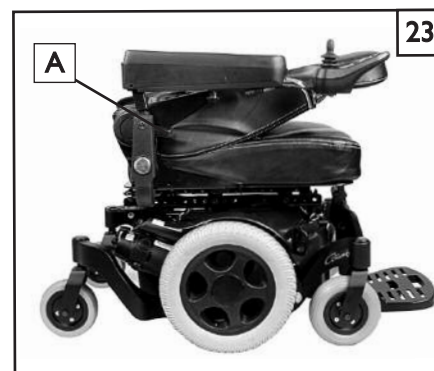

Seat Depth Adjustment (Figure 24)

1. Adjustment with 1" increments

- Using a 5mm Hex Key, remove the M8 flat head screws (B) (3) on each side
- Using a 5mm Hex Key remove the M8 Flat Head screws (C) (1) on each side
- Using a 5mm Hex Key loosen the M8 Flat Head screws (D) (1) on each side
- Holding the backrest on each side, slide it front or back to achieve desired position.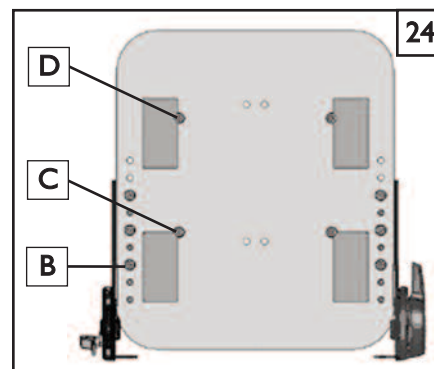


Seat Depth Mounting Positions(Figure 24,25 and Table)

Using the Seat Depth Mounting Positions table, and the hole number callouts (Figure 25) use the correct hole positions to achieve depth and width.

Table Seat Depth Mounting Positions

	16" WIDE	18" WIDE	20" WIDE
14" DEEP	-	-	-
15" DEEP	2,4,6	-	-
16" DEEP	3,5,7	1,3,5	-
17" DEEP	4,6,8	2,4,6	-
18" DEEP	5,7,9	3,5,7	1,3,5
19" DEEP	-	4,6,8	2,4,6
20" DEEP	-	5,7,9	3,5,7
21" DEEP	-	-	4,6,8

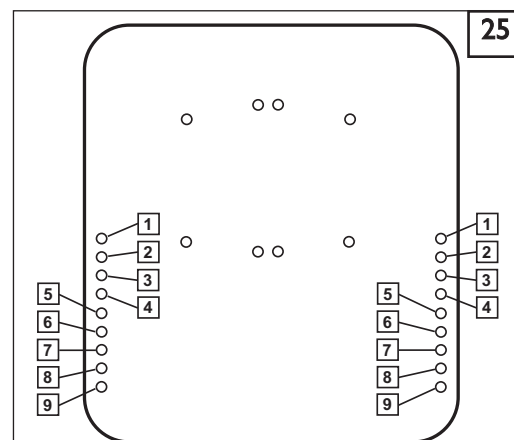
Back Height Adjustment

I. There are 3 positions of Adjustment for the back with 1" increments.

Position 2 (+2) (A), Position 1 (+1) (B), and Position 0 (default) (C)

- Using a 5mm Hex Key, remove the M8 flat head screws (3) on each side
- Position the backrest at the desired height. Top screw should be mounted as shown (Figure 26).
- Holding the backrest in place, tighten the screws.
- Repeat steps with other side, and ensure that the left and right sides are positioned at the same height.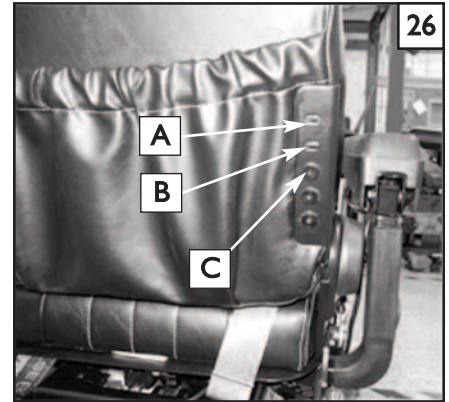


Armrest Width Adjustment

NOTE— You can change the armrest widths independantly, remember that this adjustment will increase the overall width of the Wheelchair.

I. Changing the armrest Width


- Locate the 2 armrest fasteners (D) on each side of the armrest receiver bracket (figure 27)
- Loosen the fasteners
- Slide the armrests in or out to the desired width.
- Re-tighten the fasteners.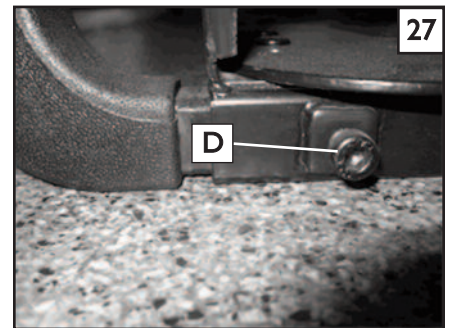

Armrest Height Adjustment

I. To change the armrest Height.


- Using a 4mm Hex Key, Loosen the two M5 setscrews (E) located on the armrest receiver (figure 28)
- Raise or lower the armrest to the desired height
- tighten the setscrews to secure the armrest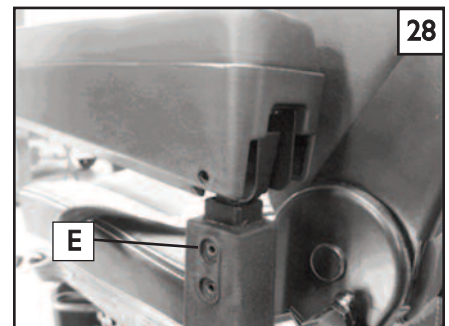

Armrest Angle Adjustment

I. To change the armrest angle (Figure 29)

- Lift the armrest straight up so that it is perpendicular to the floor
- Loosen the jam nut (G)
- Turn the adjusting screw (F) to raise or lower the front of the armrest
- Tighten the Jam nut (G) to lock the adjusting screw into place.
- Make sure that armrest knob (H) is tight when finished.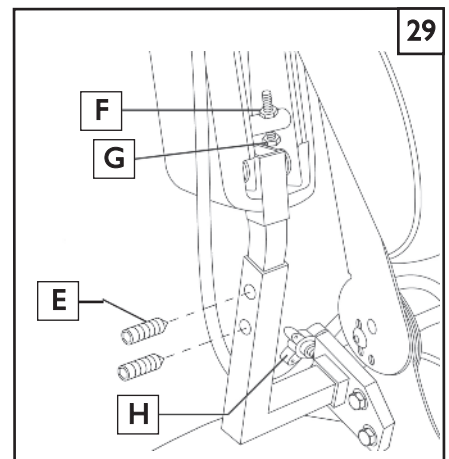

J. MANUAL RECLINE OPERATION

The ASAP II seating system has two manual recline options. You can use either a 16mm socket wrench to turn the Hex Rod (A) on the side of the chair (Figure 30). Or use a knob that fits on the end of the Hex Rod. To recline the back, rotate the Rod counter-clockwise and turn it clockwise to return the back to a more upright position.

K. JOYSTICK

Controls:

- 3 types of Joysticks are available: LED, Monochrome, and a Joystick with color display (Figure 31)
- The Joystick assembly has 3 components, (Figure 32) The Joystick, Power Module, and the communication cable

Omni Input/Display (Figure 33)

The Omni is a universal specialty controls interface that accepts signals from many different types of SIDs (Specialty Input Device) and translates them into commands compatible with the PG Drives Technology RNET control system.

Mating Connectors (Figure 34)

To connect the communication cables: (Figure 34)






Holding the connector housing, firmly push the connector into its mate until you can no longer see the yellow plastic. The connectors are secured using a friction system.

To disconnect the communication cables:

Holding the connector housing firmly, pull the connectors apart.

NOTE— Do not hold or pull on the cable. Always grip the connector when connecting and disconnecting.

When the control system is first switched on after a connection, or system component change, the timer will be displayed while the system checks itself. Then the re-start icon will be displayed. Switch the control system off and on again to operate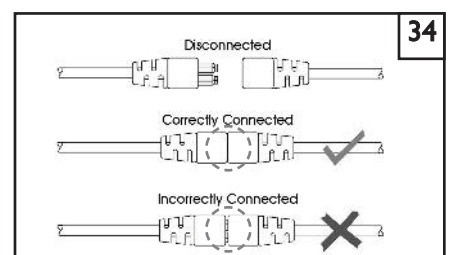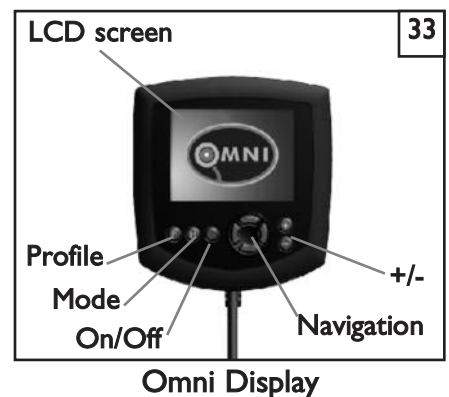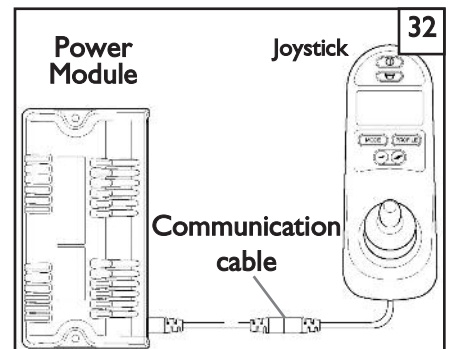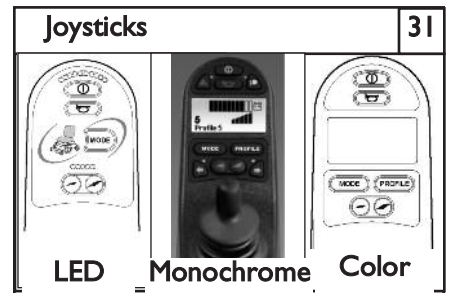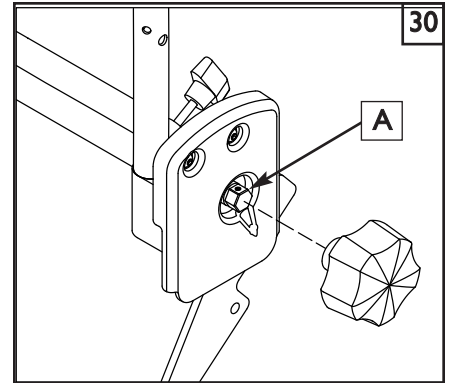


L JOYSTICK RETRACTABLE MOUNT (Optional) (Fig. 35)

The joystick is mounted with a mechanism which allows the control to be locked in a forward or retracted position using magnets.

Adjustment

- To retract the joystick, push outward on the inside of joystick.
- Push the joystick away from front of armrest until it locks into retracted position.
- To return to forward position, push the joystick forward until it locks into place.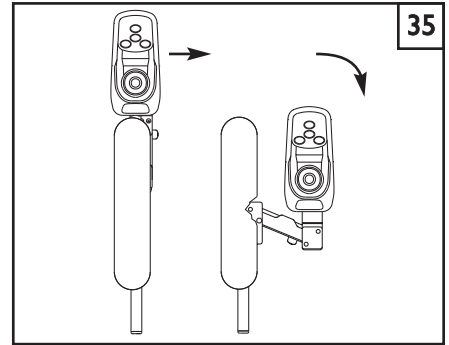


M ASAP II SEATING ADJUSTMENT

Please refer to the Service Manual for details on how to adjust the seat depth, seat width, armrest height, and backrest angle.

N. SEAT HEIGHT ADJUSTMENT (Figure 36,37)

- Remove seat (See Section O. Seat Removal).
- Remove the hardware and lockout pin (A) from all four seat mount posts.
- Select the desired seat height.
- To allow for 3° pretilt, rear seat mount should never be more than one position lower than the front.
- Reinsert the hardware.

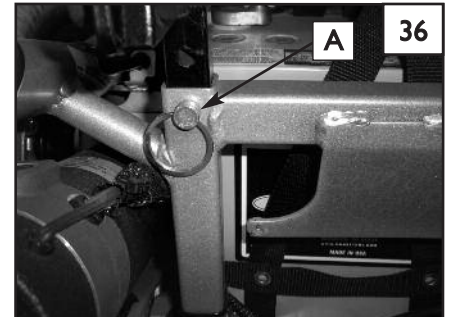WARNING

Ensure both front seat posts are positioned to the same height. Rear seat posts should also be set to the same height.

O. SEAT REMOVAL

Remove Seat

- Disconnect the joystick power cable in the rear of the chair. (Figure 37)
- Remove the hardware that attaches the seat to the front seat posts. (Figure 38)
- Tilt the seat back to allow access to the battery compartment (Figure 39).

WARNING

When replacing the seat into the mounts, ensure you do not pinch your fingers.

WARNING

Always replace the safety lock-out pin (Figure 35) to prevent inadvertent release of the seat.

P. SHROUD REMOVAL (Figure 40 & 41)

The plastic protective shroud can be removed for adjustment or servicing.

1. Remove Top Shroud


- a. Remove the seat (see Section O. Seat Removal).
- b. Remove the six fasteners (Fig. 40).
- c. Remove shroud by lifting up.

2. Remove Rear Shroud

- a. Remove two rear shroud fasteners.
- b. Pull downward at the top of the rear shroud (Fig. 41) then lift up.

3. Replace Shroud

Replace shroud by reversing above instructions.

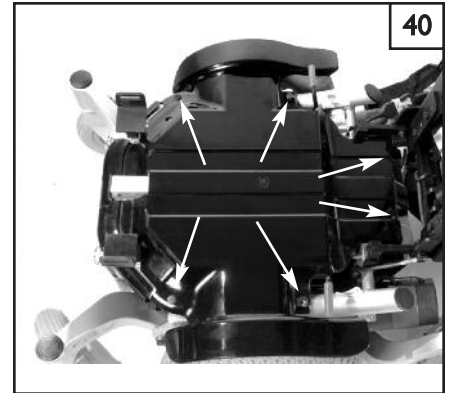**Q. CONTROLLER POSITION** (Figure 42)

You can move the controller in or out in relation to the armrest. You can also change the position of the controller for either left-hand or right-hand use.

WARNING

When controller is moved, the harness and cord can be at risk of pinching from the Seat Frame, and/or Power base frame. Be sure to secure the cord after the controller has been moved, to avoid cord pinching which can disable the unit.

1. To extend the controller:

- a. Flip up the armrest so that it is perpendicular to the floor.
- b. Loosen the thumbscrew on the controller bracket (Figure. 41).
- c. Slide the controller in or out of the armrest to the desired position.
- d. Tighten the thumbscrew to secure the controller.

2. To change the controller from one side to the other:

- a. Turn off the power to the controller.
- b. Unplug the controller connectors from the power base
- c. Remove any wireties securing the controller harness to the armrest.
- d. Flip up the armrest so that it is perpendicular to the floor.
- e. Loosen the thumbscrew on the controller bracket, and slide the controller out.
- f. Re-tighten the thumbscrew (so it won't be lost accidentally)
- g. Loosen the thumbscrew in the other armrest.
- h. Slide the controller into the other armrest, and tighten the thumbscrew
- i. Route and secure the controller harness to the back of the power base and plug in the controller. Make sure to avoid any pinch points.

**R. CHECK-OUT**


Once the wheelchair is assembled and adjusted, it should roll smoothly and easily. All accessories should also perform smoothly.

After the wheelchair has been set up be sure the chair performs to your specified operational settings (see Operating Guide). If the chair does not perform to specifications, turn the wheelchair OFF immediately and contact your authorized supplier to reprogram operational specifications using the RNET Operating Guide.

Repeat this procedure until the wheelchair performs to specifications before attempting active use of the wheelchair.

If you have any problems, follow these procedures:

1. Review the Set-up, Adjustment & Use section and Operating Guide to make sure chair was properly prepared.
2. If your problem persists, contact your authorized supplier. If you still have a problem after contacting your authorized supplier, contact Sunrise customer service. See the introduction page for details on how to contact your authorized supplier or Sunrise customer service.

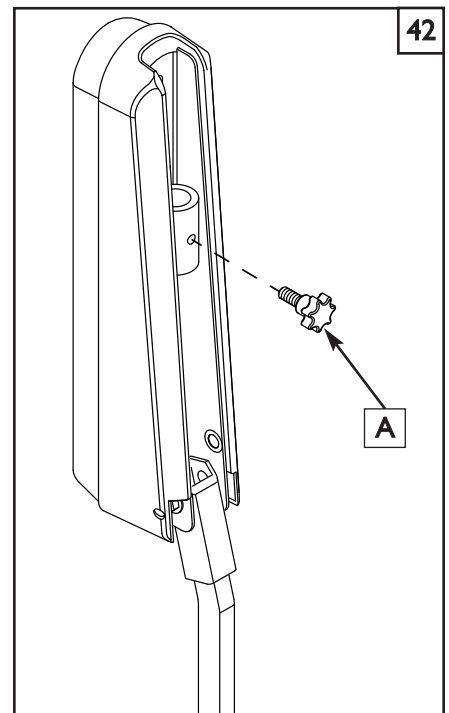IX. OPERATING GUIDE

A. PERFORMANCE CONTROL SETTINGS

NOTE– Seek advice regarding checking and adjusting settings.

1. It is vital to match control settings to your level of function and ability.
2. Consult your health care professional and your supplier to select the best control settings for you.
3. Check and adjust the settings every six to twelve months.

WARNING

Adjust the control settings immediately if you notice any change in your ability to:

- Control the joystick.
 - Avoid running into objects.
4. Control Settings can be adjusted by your dealer.

B. THERMAL ROLL-BACK

Your chair has a thermal roll back circuit. This protects the controller from damage due to overheating. In extreme conditions (such as repetitive hill climbing) the circuit will decrease the power to your motors. This allows the chair to operate at a reduced speed. When the controller cools, the chair will return to normal speed.

C. JOYSTICK STATIONARY

Your chair has an available programming feature called “Joystick Stationary” which can be set by your dealer. This feature senses when the joystick is displaced from neutral, yet left in a steady position for a predetermined period of time. The controller will automatically turn off if the set time has been reached. This feature is designed to protect the user, batteries and electronics if the joystick is inadvertently displaced. There are two parameters that can be adjusted:

1. Joystick Stationary Time:
This is the time the joystick has to remain within the stationary range before the unit will come to a stop. It can be programmed from 0 to 60 minutes in steps of one minute. Select 0 to disable the function. Default setting is 0 (Disabled).
2. Joystick Stationary Range:
This determines the size of a region around the present joystick position. If the joystick remains within this region for the stationary time the controller will stop. The range can be programmed from 1 to 50, in steps of 1. Default setting is 1.

WARNING

Before turning on or adjusting these parameters, carefully read and understand the implications for use of these features, Joystick Stationary Time and Joystick Stationary Range. When the “Joystick Stationary Time” is reached within the “Joystick Stationary Range”, the drive motors on the chair will shut off. As an example, this will happen while driving in a straight line for extended periods of time with no joystick movement. Once the drive motors turn off, the user will need to turn the chair off and back on to drive.

D. VR2 JOYSTICK ASSEMBLY

The standard VR2 joystick controls the chair's performance. It consists of the following parts:

1. Speed Control or Drive Mode Selection Buttons (A and B)

To decrease the speed level depress button (A) Or, when 1- 5 Drive modes are programmed, depress button (A) to activate lower drive mode(s).

To increase speed level, depress button (B). Or, when 1- 5 Drive modes are programmed, depress button (B) to activate higher drive mode(s).

Speed level active, or Drive Mode selected, is indicated on display (C) by a series of five lights. One light indicates slowest level/mode currently active, while five lights indicates highest level/mode currently active.

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- Green lights indicate that batteries are fully charged.
- Yellow lights indicate that batteries need recharging.
- When the lights stay in the red band, the batteries must be recharged.

3. On/Off Button (E)

Turning the chair on or off is accomplished by depressing button E.

4. Horn (F)

5. Joystick (G)

The joystick controls the direction and speed of your chair. Turn the chair on and move the joystick in the direction you want to go.

- Moving the joystick from the neutral (center) position disengages the motor locks, allowing the chair to move.
- The chair will move faster the more you move the joystick away from neutral.

Note— If your speed becomes hard to manage, release the Joystick and the chair will come to a complete stop.

- When you release the joystick it will return to neutral; the chair will slow to a stop and the motor locks will reengage.
- We recommend that you switch the chair off if you stop for any length of time. This will conserve battery power.


Note— Once the chair stops, switching the chair off will not affect the motor locks.

6. Joystick Boot (H)

Make sure the boot is not torn or cracked (this could allow debris, water or moisture to enter). If the boot is torn or cracked, replace it as soon as you can.

7. Tilt Control (SC only) (I)

You can depress either button (I), to enter Tilt Mode, this action will allow you to control the up and down movement of the tilt. Once in Tilt Mode, push the joystick either forward to tilt back, or pull the joystick back to tilt forward. Pressing button (I) a second time will exit Tilt Mode and allow regular driving operations to resume.

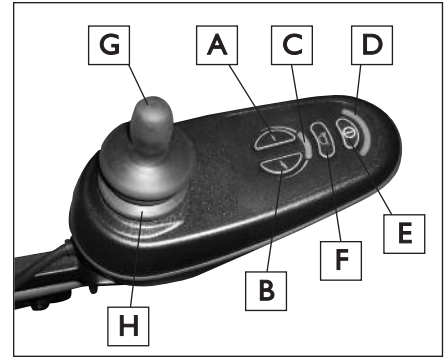VR2 Joystick

VR2 Joystick
with Actuator Control

E. LED JOYSTICK

1. On/Off Button

The on/off button applies power to the control system electronics, which in turn supply power to the wheelchair's motors. Do not use the on/off button to stop the wheelchair unless there is an emergency. (If you do, you may shorten the life off the wheelchair drive components.)

2. Horn Button

The horn will sound while this button is depressed.

3. Battery Gauge

The battery gauge show you that the wheelchair is switched on. It also indicates the operating status of the wheelchair.

- If the battery gauge shows red, yellow and green, the batteries are charged (bars 1 - 10).
- If the battery gauges show just red and yellow, then you should charge the batteries as soon as you can (bars 1 - 7).
- If the battery gauge shows just red, either steady or flashing slowly, then you should charge the batteries immediately (bars 1 - 3).

4. Maximum Speed Indicator

This is a gauge that shows the maximum speed setting of the wheelchair. There are five speed settings— step 1 is the lowest speed and step 5 is the highest speed.

5. Speed Decrease Button

This button decreases the maximum speed setting or, if the control system is programmed for drive profile operation, selects a lower drive profile.

6. Speed Increase Button

This button increases the maximum speed setting or, if the control system is programmed for drive profile operation, selects a higher drive profile.

7. Mode Button

The Mode button allows the user to navigate through the available operating modes for the control system. The available modes are dependent on programming and the range of auxiliary output devices connected to the control system.


8. Actuator Indicator

This LED set displays which actuator channel is currently being controlled when the control system is in Actuator Mode. Actuator selection and operation is achieved using the joystick.


- Motions to the LEFT or RIGHT select different actuator channels.
- Motions FORWARD and BACKWARD move the actuator(s) selected.

9. Speed Indication

The number of LED's illuminated shows the maximum speed setting. For example, if the setting is speed level 4, then the four left hand LED's will be illuminated.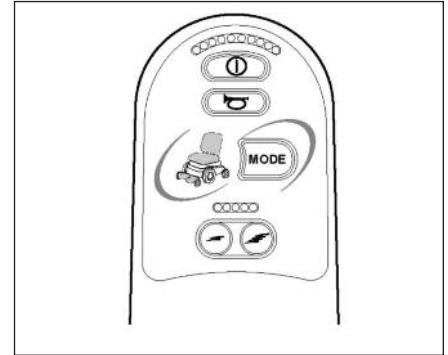

LED Joystick

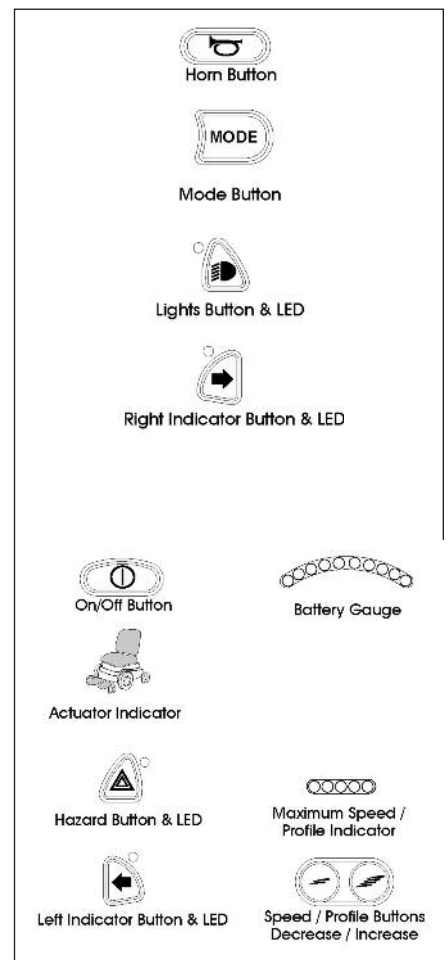F. MONOCHROME LCD SCREEN


The status of the control system can be understood by observing the LCD screen.

Screen Symbols

The Drive screen for the R-net has common components, which will always appear, and components that will only appear under certain conditions. Below is a view of a typical Drive screen in Profile 1.

Battery Indicator

This displays the charge available in the battery and can be used to alert the user to the status of the battery.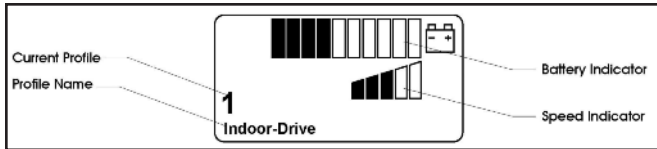

Steady: This indicates that all is well.

Flashing Slowly: The control system is functioning correctly, but you should charge the battery as soon as possible.

Stepping Up: The wheelchair batteries are being charged. You will not be able to drive the wheelchair until the charger is disconnected and you have switched the control system off and on again.

Speed Indicator

This displays the current speed setting. The speed setting is adjusted using the Speed Buttons.

Current Profile

The Profile Number describes which Profile the control system is currently operating in.

The Profile Text is the name or description of the Profile the control system is currently operating in.


In Focus

When the control system contains more than one method of direct control, such as a secondary Joystick Module or a Dual Attendant Module, then the Module that has control of the wheelchair will display the In Focus symbol.

Speed Limit

If the speed of the wheelchair is being limited, for example by a tilted seat, then this symbol will be displayed.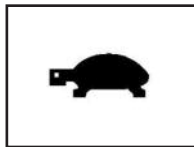

If the wheelchair is being inhibited from driving, then the symbol will flash.

Latched

When the control system is operating in a latched condition this symbol will be displayed.


Restart

When the control system requires a reboot; for example, after a module re-configuration, this symbol will be flashed.


Fault

The control system can detect a wide variety of errors. When the system has detected an error that is not severe enough to cause the system to trip, then this symbol will be displayed.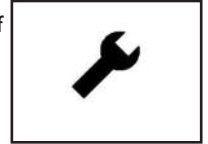

Motor Temperature

This symbol is displayed when the control system has intentionally reduced the power to the motors, in order to protect them against heat damage.

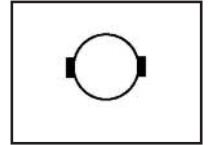Control System Temperature

This symbol is displayed when the control system has intentionally reduced its own power, in order to protect itself against heat damage.

Timer

This symbol is displayed when the control system is changing between different states. An example would be entering into Programming Mode. The symbol is animated to show the sands falling.


E-Stop

If the control system is programmed for latched drive or actuator operation, then it is normal for an Emergency Stop Switch to be connected into the External Profile Switch Jack. If the Emergency Stop Switch is operated or disconnected, this symbol will be displayed.


Bluetooth

When Bluetooth Mode is entered the screen will display the following icon.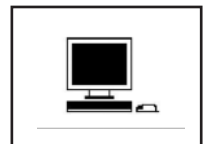

Actuator Selection Screen

To adjust the seat position the actuator screen must be visible. Depress the Mode Button to scroll through the Mode screens until you reach the actuator screen, displayed below.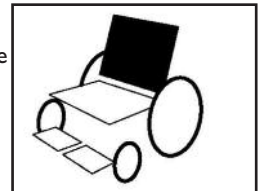

Actuator adjustment is achieved as follows.

- Move the Joystick sideways to select the desired axis. (This is indicated by the section of the wheelchair that is highlighted)
- Move the joystick forwards and backwards to move the actuator.
- Repeat these steps for each actuator that requires adjustment.
- To drive again depress the Mode button until the Drive screen is reached.

G. COLOR LCD SCREEN

This section covers those joystick modules that are fitted with a color LCD screen. The color LCD screen is split into three areas of information. The Top Bar, the Base Bar and the Main Screen Area.

1. Top Bar (Fig. 1 & 2)

Battery Indicator (Fig. 2): Displays the charge available in the battery and can be used to alert the user to the status of the battery.

Steady: Indicates that all is well.

Flashing Slowly: The control system is functioning correctly, but you should charge the battery as soon as possible.

Stepping Up: The wheelchair batteries are being charged. You will not be able to drive the wheelchair until the charger is disconnected and you have switched the control system off and on again.

2. Base Bar (Fig. 3)

Current Profile: The currently selected profile is shown in numeric form.

3. Main Screen Area

Drive Screen

Profile Name (Fig. 4): This is a text string that displays the name of the currently selected profile.

Clock (Fig. 5): This displays the current time in a numeric format.

Speed Display (Fig. 6): This gives a proportional display of the wheelchair's speed.

Maximum Speed Indicator (Fig. 7): This displays the current maximum speed setting.

Digital Speed Display (Fig. 8): This displays the actual speed of the wheelchair derived from the motors.

Inhibit (Fig. 9): If the speed of the wheelchair is being limited; for example, by a tilted seat, then this orange symbol will be displayed.

Mode Screens

Actuator Mode (Fig. 10): Displays the sections of the chair currently selected for movement, the name given to the selection and a direction arrow showing what sort of movement is available.

Speed Adjust (Fig. 11): When Speed Adjust Mode is entered, the following screen shown in Figure 11 will be displayed.





Message Window (Fig. 12): The RNET displays warning icons and informational messages, in a dedicated message window.

Restart (Fig. 13): When the control system requires a reboot; for example, after a module re-configuration, this symbol will be flashed.

Timer (Fig. 14): This symbol is displayed when the control system is changing between different states. An example would be entering into Programming Mode. The symbol is animated to show the sands falling.

Diagnostic Screen (Fig. 15)

When the control system safety circuits have operated and the control system has been prevented from moving the wheelchair, a diagnostics screen will be displayed. This indicates a system trip, i.e. the RNET has detected a problem somewhere in the wheelchair's electrical system.

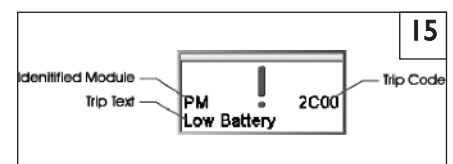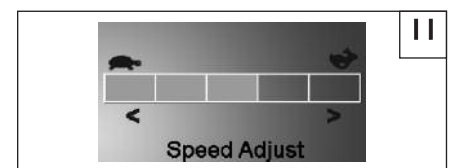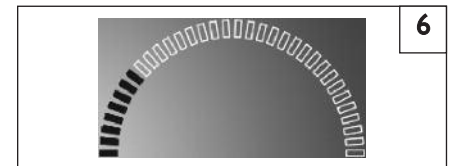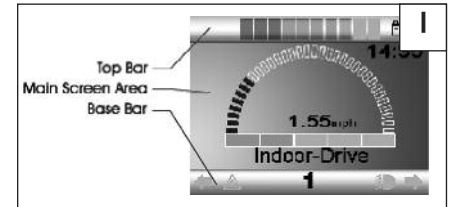H. BRAKE LEVER

Disengage the brake lever when you need to manually push the chair.
(For example, in an emergency, or if batteries fail).

WARNING

1. Do not disengage brake lever unless power to the chair is off.
 2. With either one or both brakes released, the chair will not operate and the battery charge indicator will flash 9 bars rapidly (if power to the chair is on.)
 3. Be aware that the chair will not have brakes in the free-wheel position.
 4. Make sure that the person pushing the chair has full control when brake lever is disengaged.
-
1. **Release brake lever to push the chair manually**
 - a. The brake lever (A) is attached to the rear section of the motors located towards the back of the chair.
 - b. Push the brake lever down.
 2. **Engage brake lever to drive the chair**
 - a. The brake lever (A) is attached to the upper rear section of the motors located at the back of the chair.
 - b. Pull the brake lever up.

X. BATTERIES

A. INTRODUCTION

1. Notes:

- Batteries supply the power for your chair. They contain a finite amount of energy and have limits on how long they can store and supply energy.
- You can charge batteries only a certain number of times before they will fail and no longer hold a charge.
- For answers to questions about batteries, consult your supplier.

2. Use Proper Batteries:

Your chair operates on two 12 volt batteries.

- They should be Group 22 batteries.
- When you buy a replacement, insist on a deep cycle sealed case. Do not use a car starter battery.

3. Breaking In:

- A battery requires "breaking-in" for the first 6 to 12 charges. It will not accept a full charge for this period.
- It is best to limit the length of your trips until you break the batteries in and you know the range of your chair.

4. Discharged Batteries:

- Never** allow a battery to completely discharge. If you operate your wheelchair until it has almost stopped, you will greatly reduce the life of your batteries.
- Never** let a battery sit in a discharged condition. Give unused or stored batteries a full charge once per month.
- Always** fully charge the batteries. Avoid "topping off" with frequent charges.

WARNING

Never connect a life support or auxiliary device to a wheelchair battery. The electrical system may fail and result in severe injury to or death of rider.

B. BATTERY CHARGER

A battery charger produces a direct current (DC). When applied to a discharged battery, this reverses the chemical reaction that led to its discharge.

1. Charge Rate. How fast a battery will charge depends on:

- Its electrical capacity, state of charge, electrolyte temperature, and internal condition.
- The DC output of the charger. (The charge rate will vary if the alternating current (AC) supply is higher or lower than 110 volts).

2. CAUTION- To Avoid Damage to the Charger

- Never** expose charger to rain or snow.
- Never** unplug charger by pulling on the electrical cord. This will damage the cord.
- Never** open a charger or attempt to repair it yourself. Return charger to Sunrise or have repairs made only by a qualified person.
- Keep cord out of the way, where it will not be stepped on, tripped over, or damaged.

3. CAUTION- To Avoid Damage to the Battery

- Make sure you use the correct setting for sealed (gel) batteries (located on off-board chargers).

- Never** charge a frozen battery. A fully charged battery will rarely freeze, but the fluid in a discharged battery can freeze at 16 degrees Fahrenheit (minus 9 degrees Centigrade). If you suspect a battery is frozen, thaw it before charging.

WARNING

Lead acid batteries generate explosive gas while charging. Completely read and follow all warnings about the batteries in this user instruction manual and any labels applied to the product. Failure to do so could result in fire, explosion, injury and/or death.

C. ACID BURNS (unsealed wet cell batteries)

Acid in batteries is corrosive. If you choose to use wet cell batteries, they can cause serious burns to the eyes and skin and can damage floors, furniture, clothing and your wheelchair.

WARNING


- Use extreme care not to spill acid when you handle wet cell batteries. Keep batteries upright.
- Avoid contact of acid with bare skin or clothing.
- Always wear rubber gloves and safety glasses when you handle batteries.
- If acid contacts your skin or clothing, wash immediately with soap and water.
- If acid contacts your eyes, immediately flood eyes with cold running water for at least 15 minutes. Seek medical attention immediately.

Sunrise Medical does not recommend the use of wet cell batteries on this chair.

D. CONNECTING BATTERIES IN BATTERY WELL

WARNING

- Each battery weighs up to 39 lbs. Take care to avoid injury when lifting.
- Keep batteries upright. Take care not to spill acid (wet cell batteries).
- Always wear rubber gloves and safety glasses when you handle batteries.
- Before working around batteries, remove all metal personal effects, such as necklaces, rings, watches, pins, and other metal jewelry that might contact battery terminal and cause a short.
- Attach battery harness to battery post. Connect red to positive terminal and black to negative terminal.
- Tighten all terminal fasteners using two wrenches.
- Cover battery posts with insulating caps on harness.
- Lift batteries into battery well.

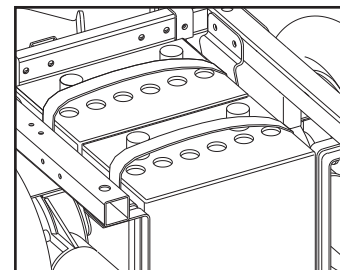Refer to wiring diagram for battery connections

WARNING

Batteries weigh up to 39 pounds. Lifting batteries may cause back strain. If you fail to heed these warnings severe injury may occur to you.

5. Plug battery harnesses into chair harness.
6. Place shroud into position.

WARNING

Never make direct contact across both battery terminals, as an explosion may occur. A spark may result in an explosion and/or fire resulting in severe injury or death.

E. CHARGING BATTERIES**WARNING**

To avoid the risk of severe injury or death from electrical shock, fire or explosion while charging:

1. **Never** connect charger to a 240 volt source. The main input voltage is 110 volts.
2. **Never** touch the charger after it is connected to an electrical outlet. This may cause electrical shock.
 - **Never** connect or disconnect charger from battery while there is power to the charger.
3. **Never** use an extension cord. Use of an improper cord could damage the charger or cause a fire or electrical shock.
4. A battery emits explosive hydrogen gas during charging. To reduce the risk of fire or explosion:
 - Make sure area is well vented. Never charge battery in a closed-in area.
 - **Never** smoke or allow a spark, flame or high heat near battery during charging.
 - **Never** allow metal tools or chair parts to make direct contact across both battery terminals.
5. Never look directly into cells when charging battery (wet cell batteries).

ALWAYS:

1. Use the charger that comes with your wheelchair. Read and follow all instructions and warnings.
2. Make sure room is well ventilated.
3. Turn off all power to your chair.
4. Connect and disconnect battery cables with caution.
5. Make sure to allow enough time to fully charge batteries.

NOTE—Batteries should never be left for long periods in the discharged state. Unused or stored batteries should be given a charge once per month.

6. Charge batteries by plugging in the battery charger cord into the chair and an outlet.
7. Or, use an off-board charger by plugging it into the joystick.

F. DISPOSING OF BATTERIES**WARNING**

1. All batteries once they have reached the end of their useful life are deemed to be hazardous waste.
2. For further information on handling and recycling contact your local recycling authority.
3. Always dispose of product through a recognized agent.

Off-board charger port (located under joystick)

XI. MAINTENANCE

A. NOTES

1. Proper maintenance will improve performance and extend the useful life of your chair.
2. Clean your chair regularly. This will help you find loose or worn parts and will make your chair easier to use.
3. To protect your investment, have all major service and repair work done by your supplier.

WARNING

1. Your chair needs regular maintenance for peak performance and to avoid injury from chair failure, damage or premature wear.
2. Inspect and maintain this chair strictly per the "Maintenance Chart".
3. If you detect a problem, make sure to service or repair the chair before use.
4. At least once a year, have a complete safety check and service of your chair performed by a supplier.

B. CLEANING

1. Plastic Shroud Cover:

- Clean the plastic finish with mild soapy water monthly.

2. Motors:

- Clean around motor area weekly with a slightly damp (not wet) cloth.
- Wipe off or blow away any fluff, dust, or dirt on or around the motors.
- Note: You do not need to grease or oil the chair.

3. Upholstery:

- Hand-wash only as needed. Machine washing may damage fabric (rehab seat only).
- Drip-dry only. Heat from a dryer may damage fabric.

NOTE— Washing the fabric may decrease fire retardant properties.

C. STORAGE TIPS

1. Store your chair in a clean, dry area. If you fail to do so, parts may rust or corrode.
2. Before using your chair, make sure it is in proper working order. Inspect and service all items on the "Maintenance Chart".
3. If you store this chair for more than three months, have it inspected by a supplier before use.

D. TO REPAIR OR REPLACE A TIRE

WARNING

Residual air pressure in tires can cause severe injury. Make sure you release all air in tire before attempting to service or repair tires.

Drive Wheels:

- a. Elevate and securely support chair so that wheel is off the ground. Release all air from tire (solid aluminum rim only).
- b. Remove center cap.
- c. Remove four Allen bolts.
- d. Remove wheel from chair.

- e. Remove all bolts holding two halves of rim together (not applicable on solid aluminum rim), and remove tire and solid insert.
- f. Repair or replace.

NOTE— Do not use pneumatic inner tube in tires with two-piece rims.

- g. Reassemble the wheel.
Use a torque setting of 420 in.-lbs. to re-tighten bolts.
- h. Install wheel on chair by replacing center lock bolt. Use a torque setting of 180 in.-lbs. to retighten bolts.
- i. Inflate tire to proper pressure level (listed on tire sidewall).

E. MOTOR BRUSHES

Check the motor brushes every four (4) months for wear. The brushes should be clean and shiny. Replace worn or blackened brushes.

F. ORDERING PARTS

When you order parts, provide the following:


1. Model of chair
2. Serial number of chair
3. Left hand or right hand control
4. Part number, description and quantity of parts you need.
5. State reason for replacement.

G. MAINTENANCE CHART

You should check the items on this chart at the indicated intervals. If any of the items are loose, worn, bent or distorted, immediately have them checked and/or repaired by your authorized Sunrise supplier. Frequent maintenance and servicing will improve performance and extend wheelchair life, and help prevent injuries.

	Daily	Weekly	Monthly	Quarterly	Annually
CHECK...					
Charge batteries	✓				
Check tires for proper inflation level		✓			
Check batteries for proper electrolyte level (wet cell only)		✓			
Check plugs and connectors for proper connections			✓		
Check all moving parts for wear				✓	
Inspect all nuts, bolts and fasteners for looseness or wear				✓	
Inspect upholstery for wear				✓	
Remove and inspect motor brushes				✓	
Service by authorized supplier					✓

H. WIRING DIAGRAM

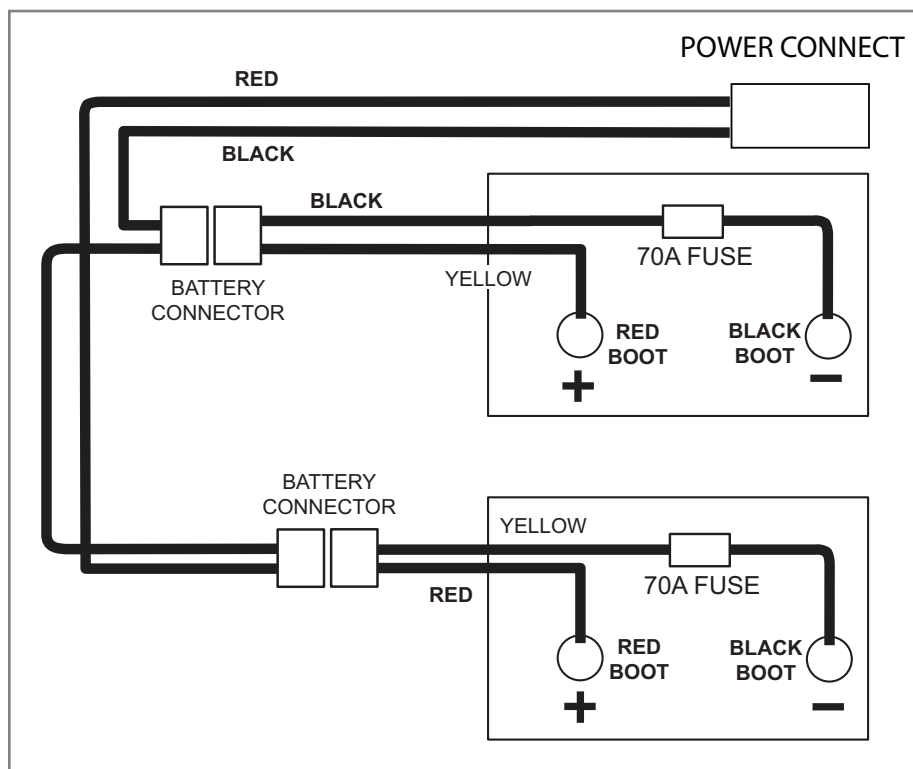XII. SUNRISE LIMITED WARRANTY**A. FOR FIVE (5) YEARS**

Sunrise warrants the frame of this wheelchair against defects in materials and workmanship for five years from date of first consumer purchase.

B. FOR ONE (1) YEAR

We warrant all Sunrise-made parts and components of this wheelchair against defects in materials and workmanship for one year from the date of first consumer purchase.

C. FOR ONE (1) YEAR

We warrant all motors, gearboxes, batteries, and electronic components of this wheelchair against defects for one year from the date of first consumer purchase.

D. LIMITATIONS

1. We do not warrant:
 - a. Tires and tubes, upholstery, pads and push-handle grips.
 - b. Damage from neglect, accident, misuse, or from improper installation or repair.
 - c. Products modified without Sunrise Medicals express written consent.
 - d. Damage from exceeding the weight limit.
2. This warranty is VOID if the original chair serial number tag is removed or altered.
3. This warranty applies in the USA only. Check with your supplier to find out if international warranties apply.
4. This warranty is not transferable and only applies to the first consumer purchase of this wheelchair through an authorized Sunrise Medical dealer.

E. WHAT WE WILL DO

Our sole liability is to repair or replace covered parts. This is the exclusive remedy for consequential damages.

F. WHAT YOU MUST DO

1. Obtain from us, while this warranty is in effect, prior approval for return or repair of covered parts.
2. Return the wheelchair or part(s), freight pre-paid, to Sunrise Mobility Products Division at: 2842 Business Park Avenue, Fresno, CA 93727-1328.
3. Pay the cost of labor to repair, remove or install parts.

G. NOTICE TO CONSUMER

1. If allowed by law, this warranty is in place of any other warranty (written or oral, express or implied, including a warranty of merchantability, or fitness for a particular purpose).
2. This warranty gives you certain legal rights. You may also have other rights that vary from state to state.

Record your serial number here for future reference:

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©2012 Sunrise Medical (US) LLC