


Special Safety Features



The **tilt feature** eliminates the need for ramps. With the pull of a handle, the carrier tilts easily toward the passenger side.

The **deck extender** pulls out from underneath the scooter, reducing the angle of incline. It also reaches the curb for loading and unloading. This feature is especially helpful when a handicap curb ramp is not available.

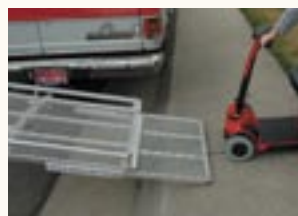

There is a hinging device located just behind the hitch, which has three positions. The first position keeps the carrier locked in the horizontal position. The second position allows the loaded Scooter Carrier to lift and float over dips and curbs. The third position allows the empty carrier to be folded into a vertical position for storage. **Class II, 1 1/4" hitch bars** and an optional **Adjustable-Height Hitch Bar** (class II & III) are also available for low and smaller vehicles.

Tilt-a-Rack® Scooter Carriers are warranted to be free from defects in material and workmanship within one year from the date purchased by the consumer.

Mobility Rebate Program

Numerous vehicle manufacturers now have mobility programs that provide rebates for the purchase of eligible vehicles with adaptive equipment. Check with your local dealer for details.

Covers, Liners, Tie-Downs, and Locks




Customize our carrier to your needs with our entire line of accessories. Pick up a custom-fit cover for your cycle, sport scooter, ATV or mobility scooter. Made especially for our carriers by **Diestco Manufacturing**, these covers fit perfectly and protect your valuable assets on the road. **Class II, 1 1/4" Hitch Bars** and an optional **Adjustable-Height Hitch Bar** (class II and III) are also available for low and smaller vehicles.

We also offer a complete line of liners, tie-downs and locks.
To learn more, visit us online at www.tiltarackmobility.com. Or call 1-866-777-9727.

Bogie Wheel

Hitching and unhitching a rack has never been this easy. Our optional **Bogie Wheel** attaches to the front of the rack, allowing a single person to maneuver the entire carrier with ease. When you're ready to go, the **Bogie Wheel** detaches quickly and easily.

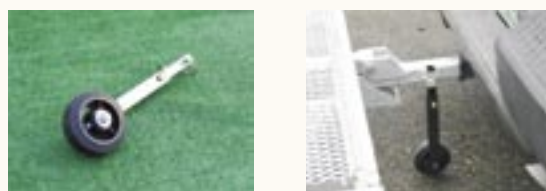

MADE IN THE USA!

4700 West 60th Avenue, Unit 4
Arvada, Colorado 80003
Phone: 303-450-2881
Fax: 303-450-2880
Toll-Free: 1-866-777-9727
www.tiltarackmobility.com



Now Our
Carriers are
Aluminum!

Tilt-a-Rack is federally registered by ATV Solutions, Inc.



TILT-A-RACK MOBILITY

OUR TILT-A-RACK® CARRIERS FOR MOBILITY
SCOOTERS ARE THE SAFE, EASY, CONVENIENT, AND
AFFORDABLE WAY TO TRANSPORT YOUR SCOOTER.



Now Our Carriers

are Aluminum!



Tilt-a-Rack is federally registered by ATV Solutions, Inc.




MADE IN THE USA!



The **Tilt-A-Rack®** Scooter and Wheelchair Carrier makes transporting motorized wheelchairs and scooters effortless. The **Tilt-A-Rack®** Scooter and Wheelchair Carrier is easy to assemble, install and use. The basket design also permits general purpose hauling (suitcases and trunks, etc.) during long trips.

Tilt-A-Rack® Scooter and Wheelchair Carriers do not have to be connected to your car battery or wired to the vehicle in any way, so you don't have to worry about recharging your battery.

Our carriers can be customized to perfectly fit your needs. We offer a complete line of accessories, including custom tie downs, weather-resistant covers and specially designed silent locking hitch pins. See the back of this brochure for details.

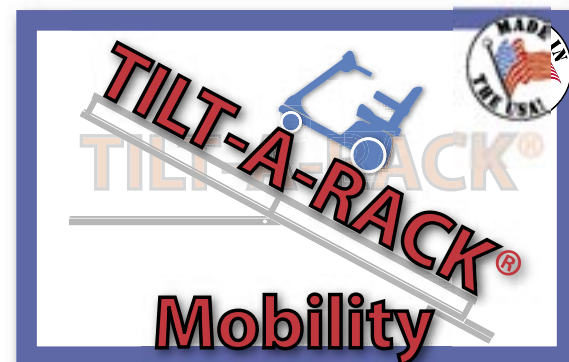

Now you can have the freedom to go anywhere you want, safely and easily, with our Aluminum Mobility Scooter Carriers. Simply tilt, load, and lock and your Mobility Scooter goes anywhere you go!

**Your "one stop shop"
for all your Mobility
Scooter Carrier needs!**




No More:

**Cramped trunk space
Dealing with dangerous ramps
Disassembly of chair
Cramped garage space, tilts up
when not in use**

**Licensing
Insurance
Extra lights
Failing electrical components
Problems with non-handicap vehicles**

- 1 Pull up and remove end bar.**
- 2 Pull out the Deck Extender.**
- 3 Pull safety pin from Spring Handle.**
- 4 Pull out and rotate Spring Handle.**
- 5 Tilt carrier toward passenger side.**
- 6 Drive scooter up onto Carrier and the Carrier automatically levels!**
- 7 Push in Deck Extender.**
- 8 Replace the End Bar and pin.**
- 9 Replace safety pin on Spring Handle**
- 10 Tie down your scooter and you're ready to go!**

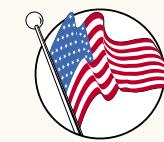

MADE IN THE USA!

Model #500ARV

Designed for full-sized scooters and power chairs.

- All Aluminum construction
- Holds up to 500 pounds
- Weight is 79 pounds
- Usable platform area is 32" x 60"
- Deck extends and tilts for loading
- 19" of maneuverable area from receiver to edge of carrier
- Requires a 2" class III or class II hitch

Model #350ARV

Designed for most mid-sized scooters and power chairs.

- All Aluminum construction
- Holds up to 350 pounds
- Weight is 58 pounds
- Usable platform area is 26" x 60"
- Deck extends and tilts for loading
- 19" of maneuverable area from receiver to edge of carrier
- Requires a 2" class III or class II hitch

Model #350ARV Mini

Designed for smaller travel scooters, power chairs and wheelchairs.

- All Aluminum construction
- Holds up to 350 pounds
- Weight is 48 pounds
- Usable platform area is 24" x 44"
- Deck **extends a total of 40"** and tilts for loading
- 19" of maneuverable area from receiver to edge of carrier
- Requires a 2" class III or class II hitch
- May require an optional Class-II Height Adjuster

VISIT OUR WEBSITE AT WWW.TILTARACKMOBILITY.COM FOR MORE INFORMATION ON BECOMING AN APPROVED VENDOR.