

# arc lite rollator

item #s:

102EXL-BL

102EXL-RD

**drive**<sup>TM</sup>

[www.drivemedical.com](http://www.drivemedical.com)

## parts list



Item # 102EXL-BL



Item # 102EXL-RD

- |                 |                           |
|-----------------|---------------------------|
| 1. Hand Grips   | 5. Height Adjustment Knob |
| 2. Back Support | 6. Folding Basket         |
| 3. Seat         | 7. Front wheel            |
| 4. Brake Cables | 8. Rear wheel             |

**WEIGHT CAPACITY: 300 LBS (136 KG)**

# installation

**IMPORTANT:** Do NOT use as a transportation device!  
This should be used as a walking aid only!

## STEP 1

---

1. Upon receiving your rollator, please check the contents carefully. Report any missing items to your authorized Drive dealer immediately.
2. Contents:
  - a. Folded rollator with main frame, seat and wheels installed.
  - b. 2 Handles attached by brake cables
  - c. Soft folding basket
  - d. 2 Handle Knobs, 2 Knobs Bolts
  - e. Back Support
  - f. Soft seat cushion
  - g. Owner's manual
3. Adjust handle height. Put Knob Bolts through from inside. Tighten until it is snug.

## STEP 2

---


Checking the Brakes and Wheels:

1. The braking system provides safety and ease of operation. Hold (squeeze) to brake and press down on the brake handle for parking.
2. Check braking system. The wheels should stay locked while parking brake is activated.
3. Do not activate parking brake while walker is moving. This may cause permanent damage to the wheels.
4. Always activate parking brake before using the seat.

## STEP 3

### Maintenance & Care:

1. Adjust brake cable from time to time to ensure proper braking. To tighten the brake, loosen brake adjustment A & B. Test brakes for tightness. When desired tightness is reached, tighten A. (figure 1)
2. Check tightness of nut and bolts regularly.
3. Check frame and tire integrity at least once a month.

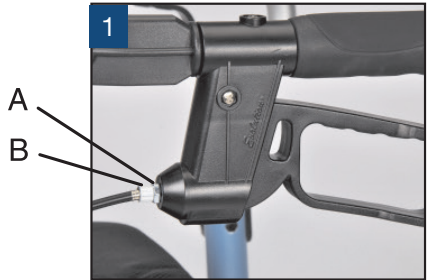

## fold & transport

To fold your Arc Lite rollator, pull up on the seat center handle and the rollator will remain standing up.

To unfold your Arc Lite Rollator, simply push down on the middle of the seat until the rollator is in a fully open position.

To carry your Arc Lite Rollator while folded, simply pick it up by the center handle and lift the rollator, carrying it at your side like a suitcase.

## brakes

Apply the brakes by pulling both brake handles upwards. Do not walk with brakes on.

## parking brakes

Press down on both brake handles until you hear a distinct click sound. Always make sure that both brakes are engaged. Pull upwards to release the parking brakes. When using the seat, both brakes MUST be in the locked position for your safety.


## brake adjustment

From time to time, you might need to adjust the tension on the brake cable system. To increase the tension on the brake cable simply loosen brake adjusting nut “A”, unscrew the knurled end piece “B” and turn it counter-clockwise about 4 complete turns. “A” and “B” are found at the back of the brake handle. When desired tension is reached, retighten the nut. No tools are necessary to perform this adjustment.

## handle adjustment

To adjust the handle knobs and knob bolts to a suitable height, stand behind the rollator with your shoulders level. The handles should be at the same height or slightly higher than the user’s wrist. A therapist might suggest a different handle height to accommodate a specific need for the user.

Referring to Figure 2, remove the knob and pull the hex bolt out to adjust the handle to the prescribed or desired handle height. Then, insert the hex bolt ensuring that the hexagonal part of the bolt fits properly on the hexagonal cut out on the frame of the Arc Lite rollator. Lastly, tighten the knob snugly.

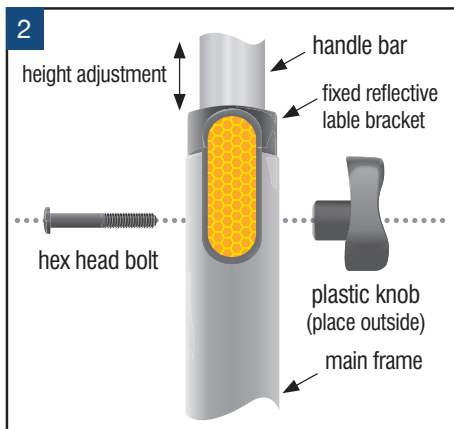

**CAUTION:** Do NOT over-tighten!

## basket installment

To install the folding soft basket, fold the Arc Lite rollator part way by pulling on the center seat handle, and insert the black basket brackets on the rails on both inside frames. Push on the middle of the seat to open the rollator.

**CAUTION:** Make sure you hear the “Click” sound for proper positioning of the basket.

## cleaning / maintenance

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

## safety instructions

- Use only as a walking aid!
- Maximum user weight: 300 lbs.
- Regularly check the screws and fasteners on the Nitro and retighten if necessary.
- Use bag only for transport of objects and do not overload.
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace tires when significantly worn or damaged.
- **DO NOT MOVE ROLLATOR WHILE SEATED.** This is a walking aid only and is not to be used as a transportation device.

# lifetime limited warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

Warning: The items and packaging in this shipment contain chemicals known to the state of California to cause cancer, birth defects or other reproductive harm.

\*This product meets or exceeds all federal safety and environmental regulations.



© 2015 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

**drive**<sup>TM</sup>