

FAA  
Approved  
for air travel!



iGo Portable  
Oxygen System  
Model 306DS-A



## iGo® Portable Oxygen System

### Reliable Portable Oxygen On The Go

The **iGo** is a **lightweight, portable 3 liter oxygen concentrator** that is capable of operating from three different power sources: AC Power, DC Power, or Rechargeable Battery.

The DeVilbiss iGo has been put through **aggressive durability testing to ensure consistent, reliable performance**. The iGo is the perfect solution to keep pace with an active lifestyle. This small, portable oxygen system provides **two modes of operation – continuous flow or PulseDose® delivery**.

The reliability and safety of **PulseDose® oxygen delivery has been proven effective in clinical testing as well as through independent tests** performed by physicians and respiratory therapists. Many users find **PulseDose oxygen delivery more comfortable** than continuous flow delivery systems. The **short “pulse” of oxygen delivered during inhalation is almost undetectable**.

### Performance Features

- Settings 1 to 6 in PulseDose Mode or 1 to 3 lpm in Continuous Flow Mode
- Integrated PulseDose oxygen conserving technology extends the use time of the iGo Portable Oxygen System offering increased mobility with improved comfort and increased efficiency.
- Built in oxygen sensing device to ensure accurate oxygen delivery and reduced periodic maintenance schedule
- Increased Battery Capabilities – iGo is powered by one battery, which can last as long as 5.4 hours when operating on setting 1 in PulseDose Mode
- Audible alerts for Power Failure, Low Battery, Low Oxygen Output, High Flow/Low Flow, No Breath Detected in PulseDose Mode, High Temperature, Unit Malfunction
- Can be used with 50 foot tubing/cannula in continuous flow mode and 35 foot tubing/cannula in PulseDose mode
- Easy to select settings of 1.0, 1.5, 2.0, 2.5, 3.0, 4.0, 5.0, and 6.0

## Design Features

- All of the high-performance capabilities are conveniently placed in a small compact design weighing less than 20 lbs with battery
- Control Panel was designed to be easily viewed whether sitting on the ground or under an airplane seat
- Audible alerts for Power Failure, Low Battery, Low Oxygen Output, High Flow/Low Flow, No Breath Detected in PulseDose Mode, High Temperature, Unit Malfunction
- Easy to read Battery Status Gauge indicates battery life
- Three handles molded into the body of the concentrator
- Easy to access filter compartment
- Rolling carry case features easy access openings for cannula and external power supply
- Fast battery charge time. The typical time to recharge the battery is 2.0 hours to 4.5 hours dependent upon the flow setting. Battery will automatically recharge when connected to an AC power source.
- If the unit is operating on AC power and the power is interrupted, the iGo will automatically switch to battery operation
- 3-year limited warranty for the iGo Portable Oxygen System
- 1-year limited warranty on battery

## Technical Specifications

Designed to meet HCPCS Code E1392 and E1390

**Weight**

19 pounds (8.6 kg) with Battery; 15.5 pounds (7.0 kg) without Battery

**Dimensions (H x W x D)**

15" x 11" x 8" (38cm x 28cm x 20cm)

**Flow Rate Settings**

Settings 1 to 6 in PulseDose Mode; 1 to 3 lpm in Continuous Flow Mode

**Max. Recommended Continuous Flow**

3 lpm

**Oxygen Concentration**

91% ± 3% for all flow settings

**Operating Temperature**

41°F to 104°F (5°C to 40°C)

**Transportation and Storage Temperature**

-4°F to +140°F (-20° to 60°C)

**Altitude**

0 -13,123 feet (0 - 4,000 meters) tested @ ~933 hPa

**Sound Level (3.0 PulseDose Mode)**

40 dBA

## Parts and Accessories

### DEVILBISS PART #

### PRODUCT DESCRIPTION

306DS-627	Humidifier Kit
HUM16	Humidifier (200 ml Capacity) - 50/case (Continuous Flow mode only)
444-507	Elbow Humidifier Adapter Kit
CAN00	Cannula (disposable) - 50/case
CAN70	Cannula w/ 2.1 m (7 ft) Oxygen Tubing (disposable) - 50/case
OST07	Oxygen Tubing (disposable)
OST15	2.1 m (7 ft) - 50/case
OST25	4.2 m (14 ft) - 50/case
306D-413	7.5 m (25 ft) - 25/case
	DeVilbiss Rechargeable Battery

### DEVILBISS PART #

### PRODUCT DESCRIPTION

306DS-625	Rolling Carry Case
306DS-626	Detachable Wheeled Cart
306DS-611	Air Filter
306DS-652	DC Adapter
306DS-651	AC Adapter (AC Power Cords) *
306DS-601	USA Power Cord
306DS-602	Continental Europe Power Cord *
306DS-603	UK Power Cord
306DS-604	Australia Power Cord
306DS-605	China Power Cord

\*Refer to Instruction Guide for More Information



DeVilbiss Healthcare | 100 DeVilbiss Drive | Somerset, PA 15501 USA  
800.338.1988 | 814.443.4881 | [www.DeVilbissHealthcare.com](http://www.DeVilbissHealthcare.com)