MOSAIC™

CUSHION

The logo features the word 'MOSAIC' in a large, stylized, black-outlined font. Below it, the word 'CUSHION' is written in a white, sans-serif font inside a black rectangular box. To the right of the text is a graphic of a cushion with a grid of squares, some of which are filled with a pixelated pattern.

OPERATIONS MANUAL


MOSAIC Pude • Het MOSAIC kussen • Coussin MOSAIC
MOSAIC-pehmike • MOSAIC-Kissen • ΜΩΣΑΙΚΟ μαξιλάρι
Cuscino MOSAIC • MOSAIC-pute • Almofada MOSAIC
Cojín MOSAIC • MOSAIC-dyna

The **MOSAIC® cushion** offers you the ability to adjust the system to fit your individual needs. The objective is to "immerse" the user into the cushion as deeply as possible without bottoming out.

To get the maximum benefit from your new **MOSAIC cushion**, follow these easy steps.


- 1. Turn cushion valve counter-clockwise to open.**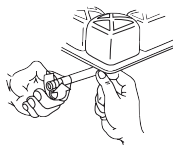

- 2. Place hand pump hose over valve and inflate until cushion arches slightly.**

- 3. Remove pump hose from valve and turn clockwise to close.**
- 4. With cushion in this over-inflated condition, place the individual on the cushion in his/her normal sitting position with arm and footrests properly placed.**

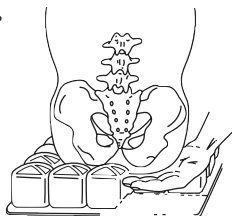5. **Place your hand between the cushion and the individual's deepest bony prominence.**
6. **With your hand in place, slowly turn the valve counter-clockwise to release some air to immerse the individual into the cushion. When you can barely move your finger tips (approx. 1-1 1/2") between the individual's deepest bony prominence and the cushion base, turn the valve clockwise to close.**

Cleaning: Mild soap and warm water may be used to clean the **MOSAIC cushion**. Rinse thoroughly and allow to dry. (Make sure valves are closed prior to immersing in water.) Do not use bleach when cleaning this product.

DO NOT AUTOCLAVE

DO NOT USE COMPRESSOR TO INFLATE.

A cover is included with every **MOSAIC cushion** to help protect the cushion surface. Machine wash cold. Air dry.

Repairs: Two pressure adhesive patches are included with the MOSAIC cushion. If leak occurs, locate leak and clean surrounding area. Let surface dry. Peel backing from the patch provided and center over hole. Press firmly until a good seal is achieved. This product is now ready for use. Follow adjustment instructions above.

Returns: All returns require authorization from The ROHO Group and are subject to a restocking charge. Proof of purchase is required with warranty return.

The following are registered trademarks and trademarks of The ROHO Group: SOFFLEX®, PRODIGY Mattress Overlay®, EVOLUTION™ and MOSAIC®.

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Medical Device Safety Service GmbH
100 N. Florida Ave. • Belleville, IL 62221 U.S.A.