




CAUTION:

Proper installation is critical to the performance of your handrims. Failure to follow these instructions will void the product warranty.

1

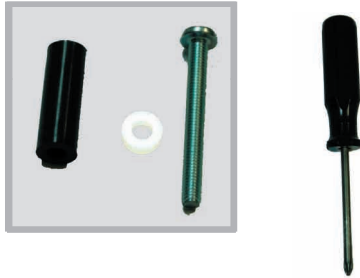

Hardware and Tools

Hardware Needed:

- Screws (use existing screws for X-Core wheels)
- Spacers

Tool Needed:

- Phillips head screwdriver

2



Old Handrim

Remove the Old Handrim

- Using a phillips screwdriver, disassemble and remove the screws that mount the handrim to the wheel.

1.



2.



3.



Set-up the Natural-Fit Assembly

1. Place the new oval component on a flat surface with the rivet-nuts (threaded holes) facing up.
2. Place the new concave shaped thumb guard on top of the oval component (the wider edge faces up to make contact with your wheel).

NOTE: IF YOUR MAG WHEEL HAS STAND-OFFS, BE CAREFUL WHEN PLACING THE THUMB GUARD AROUND THE STAND-OFFS. TO AVOID DAMAGING YOUR THUMB GUARD AND TO ALLOW IT TO FIT PROPERLY, YOU MAY NEED TO BEND THE STAND-OFFS IN OR REMOVE A SMALL AMOUNT OF PLASTIC FROM THE OUTSIDE OF THE STAND-OFF OF YOUR WHEEL.

3. Place your wheel (recessed hole for quick release facing down) on top of the thumb guard and oval component. Make sure the mounting holes on the wheel match up with the rivet-nut holes for the oval component.

3

Natural-Fit Assembly





4

Attach and Center

NOTE: IF YOU FORGET OR DO NOT USE THE SPACERS AND SCREWS PROVIDED (ONLY SPACERS ARE PROVIDED FOR X-CORES), YOU MAY DAMAGE THE THUMB GUARD AND VOID THE WARRANTY.

For Spoked Wheels Only

- Use the 6 screws and spacers provided.
- Insert the new screws through the wheel, new spacer and into the holes in the oval component.

NOTE: DO NOT USE THE OLD SCREWS AND SPACERS. SCREWS AND SPACERS ARE PROVIDED.

For Mag Wheels Only

- For **6-spoke** mag wheels, use the 6 screws and spacers provided.
- For **7-spoke** mag wheels, use the 7 screws and spacers provided
- Insert the new screws through the wheel, new spacer and into the holes in the oval component.

NOTE: DO NOT USE THE OLD SCREWS AND SPACERS. SCREWS AND SPACERS ARE PROVIDED.

For X-Core Wheels Only

- Use the 6 spacers provided.
- Insert your old screws through the wheel, new spacer and into the holes in the oval component.

NOTE: NEW SPACERS ARE PROVIDED.



5

Finish up



Last Step: Tighten the Screws

- Secure the Natural-Fit Assembly by tightening the screws in a criss/cross pattern.

NOTE: THE THUMB GUARD SHOULD NOT SPIN WHEN YOU PUSH ON IT AND SHOULD BE CENTERED ON THE WHEEL. IF THE THUMB GUARD BECOMES TILTED TO ONE SIDE, LOOSEN THE SCREWS AND READJUST THE THUMB GUARD.

Congratulations!

Your new Natural-Fit Handrim is ready to use!