
User manual


Clean

etac[®]
Creating Possibilities


78644 - 2025-02-13 - Version 20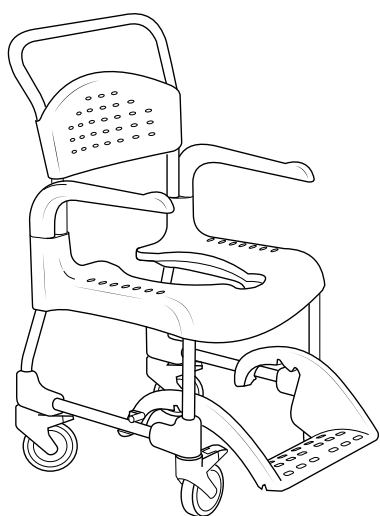

Clean

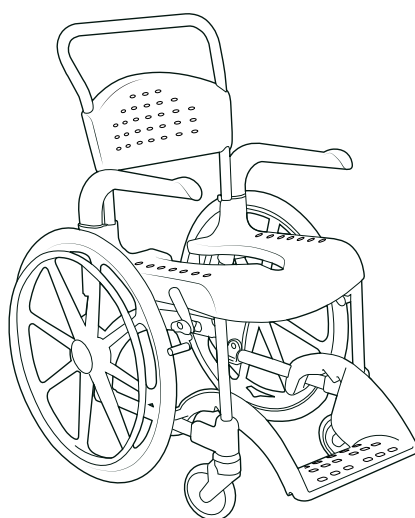Clean 24"

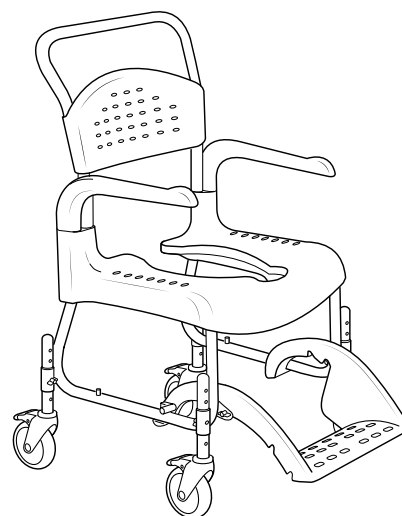**Clean
Height adjustable**

sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar

www.etac.com

MD

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General

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at

www.etac.com. You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

In the manual, the user is the person sitting in the chair. The carer is the person helping the user.

Item list: ● = Applies to, ○ = Accessories

See the item list on the right which displays the information applicable to each product.

The images which are referred to on each page are found at the start of the manual (pages 3 to 9). See Figures A to F for the corresponding text.

Device description

The Clean family is a range of mobile shower commode chairs whereof Clean and Clean Height adjustable are assistant propelled and Clean 24" can be propelled by the user.

Intended purpose

Clean, Clean 24" and Clean Height adjustable (hereafter also referred to as "the device" or "the product"), are medical devices intended for alleviation of or compensation for a functional impairment due to an injury or disability. The device is designed to be used when performing hygiene tasks in the shower, at a sink, or over the toilet, as well as for transfers to and from the hygiene room.

Intended user group

The user group for the device is based on the individual's functional ability and not on a specific diagnosis, health condition or age. It is intended for individuals with a height of 146 cm or more, or a mass of 40 kg or more.

Secondary users of the device are caregivers providing assistance and clinicians/technicians who set up the device.

Intended environment

The device is intended for indoor use at home environments or institutions and is suitable for use in bathrooms, but not swimming pools or similar corrosive environments.

Intended application

The device is intended for short-term and long-term use and can be applied several times per day.

The device is intended for use with intact skin. The device is intended for refurbishment and reuse.

Expected service life

The expected service life is 10 years. For complete information regarding the service life of the device, see www.etac.com.

Indications

Indication for use is disability, including, but not limited to, an individual with physical disabilities who needs backrest and/or armrests to maintain a sitting position and assistance with transfers to/from the chair.

Contra indications:

There are no known contraindications.

Warnings

Warnings describing a risk element for a specific action or setting of the device can be found in the relevant section.

Check user sensitiveness for pressure ulcer before use.

A defective device must not be used. Risk of injury.

A heated device can cause burns.

Body parts may get pinched while applying or removing the pan.

Ignoring the instructions for use may result in injury.

Declaration of conformity

The device meets the requirements of the Medical Device Regulation (EU) 2017/745.

The device is tested and meets the requirements of the EN ISO 10993-1 and ISO 17966:2016.

Pre-purchase information

Pre-purchase information can be found at www.etac.com.

Adverse event

In case of an adverse event occurred in relation to use of the device, it must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Customization

is everything that goes beyond the manual's instructions and settings. Device specially adapted by the customer must not retain Etac's CE marking. Etac's warranty expires.

Device description

See figure A

A1. Back support **A2.** Arm support **A3.** Seat **A4.** Foot support **A5.** Castor **A6.** Product label (warnings, CE-marking)
A7. Item number label (ser.no., production date)

A8. Rear wheel **A9.** User brake

A10. Height adjustment

Symbols, labels

See figure B

Symbols in the manual and on the device:

B1. Warning, precaution or limitation. **B2.** Useful advice and tips. **B3.** Material for recycling. **B4.** The part/function is locked and can not be moved/adjusted. **B5.** Pinch point! **B6.** Tool box-end spanner **B7** Max user weight (see technical data). **B8.** Video **B9-B10.** Cleaning (see service and maintenance). **B11.** Do not stand on the footrest, risk of tipping. **B12.** Seating position affects the stability. **B13.** Product weight **B14.** Refer to user manual (blue and white).

Information on item no label:

B15. Product name. **B16.** Product description. **B17.** Serial number. **B18.** Item number. **B19.** CE marked **B20.** Medical device **B21.** Production date. **B22.** Bar code acc. to GS1-128 GTIN-14 and serial no.*

*The device's date of manufacture can be read from the barcode on the device. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

Technical data

See figure C

Handling

See figure D

Installation upon delivery

Transfer over a threshold:

Chair only, push down on the wheel attachment.

With user, pull the chair backwards.

In stationary position:

Point the castors forward and activate the brakes to reduce the risk of tipping when the chair is used stationary and when transferring. Risk of injury if the device tips over.

D5

Do not sit on the front edge of the seat. Risk of injury if the device tips over.

Always lock as many castors as possible when the chair is to be used in a stationary position.

D6

Transferring:

Point the castors forward and activate the brakes to reduce the risk of tipping when the chair is used stationary and when transferring. Risk of injury if the device tips over.

Assess the risks and take notes. You as a carer are responsible for the safety of the user. Aids should be used if the

D1-D2 transfer is deemed as risky. Visit www.etac.com for manual transfer aids.

D3

D4

Make sure the castors are locked when the user sits down or gets up from the chair. Risk of injury.

Remove the armrest on the transfer side of the chair.

Carry out the transfer.

D5-D10

Arm supports:

The arm supports are swing away and detachable.

D11

The arm supports are lockable.

D12

Foot supports:

The carer can easily push in the foot support with their foot.

D13

The foot support is detachable.

D14

Do not stand on the foot plate. Risk of injury if the device tips over.

Settings

See figure E

Height adjustment:

The height can be adjusted with ease by placing the screw in the required position.

Check that all four castors are similarly adjusted, otherwise stability will be reduced.

E1-E2

Ensure that the wing nuts are tightened. Risk of injury.

Foot supports:

The foot support has 4 different angle and height settings.

E3

The foot support has 8 different angle and height settings.

E4

Accessories		See figure F		
		Clean	Clean 24"	Clean H. adj.
F1.	Pan	☐	☐	☐
F2.	Pan and lid with handle	☐	☐	☐
F3.	Lid with handle	☐	☐	☐
F4.	Disposable pan	☐	☐	☐
F5.	Pan holder	☐	☐	☐
F6.	Bucket holder	☐	☐	☐
F7.	Disposable pan holder	☐	☐	☐
F8.	Attachment for low mounting of pan/bucket	☐	☐	☐
F9.	Soft seat	☐	☐	☐
F10.	Comfort seat	☐	☐	☐
F11.	Comfort cushion	☐	☐	☐
F12.	Seat lid	☐	☐	☐
F13.	Soft comfort seat*	☐	☐	☐
F14.	Comfort padding	☐	☐	☐
F15.	Armrest cushion, pair	☐	☐	☐
F16.	Soft back	☐	☐	☐

Accessories		See figure F		
		Clean	Clean 24"	Clean H. adj.
F17.	Trunk support	☐	☐	☐
F18.	Heightening spacers	☐	☐	☐
F19.	Two 75 mm lockable castors	☐	☐	☐
F20.	Rear wheel adaptor	☐	☒	☐
F21.	Soap dish	☐	☐	☐
F22.	Crossbar	☐	☐	☐
F23.	Foot support, foldable and detachable	☐	☐	☐
F24.	Backrest upholstery	☐	☐	☐
F25.	Safety belt	☐	☐	☐
F26.	Positioning belt	☐	☐	☐
F27.	Chest harness belt	☐	☐	☐
F28.	Chest and hip positioning belts	☐	☐	☐
F29.	Splash guard	☐	☐	☐

*Only suitable for Clean models with a seat opening at the front