
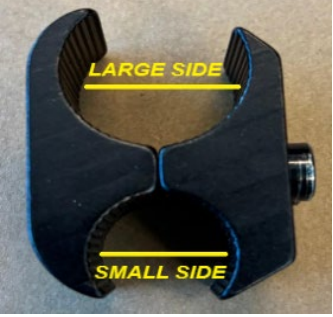
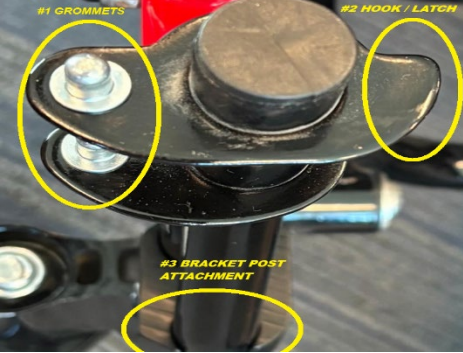


ADJUSTABLE LEG INSTALLATION INSTRUCTIONS

Works with IQ9000 and IQ8000 powerchairs only

	<p>PARTS INCLUDED:</p> <ul style="list-style-type: none">✓ BRACKETS (2)✓ VERTICLE POSTS (2)✓ LEG RESTS (2)
	<p>LEG REST BRACKET:</p> <ul style="list-style-type: none">✓ LARGE SIDE CONNECTS TO CHAIR FRAME✓ SMALL SIDE CONNECTS TO VERTICLE POST
	<p>BRACKET ATTACHMENT:</p> <ul style="list-style-type: none">✓ THE LARGE SIDE CONNECTS TO THE CHAIR FRAME RIGHT ABOVE THE WHEEL✓ THE SMALL SIDE WILL CONNECT TO THE VERTICLE POST
	<p>VERTICLE LEG REST POSTS:</p> <ul style="list-style-type: none">✓ NOTE THESE ARE LEFT AND RIGHT SIDE SPECIFIC
	<p>VERTICLE LEG REST POSTS:</p> <ul style="list-style-type: none">✓ INSERT POST INTO SMALL SIDE OPENING ON BRACKET FIG (#3)✓ THE SILVER GROMMETS ARE ON THE OUTSIDE FIG (#2)✓ THE HOOK IS ON THE INSIDE FIG (#1)

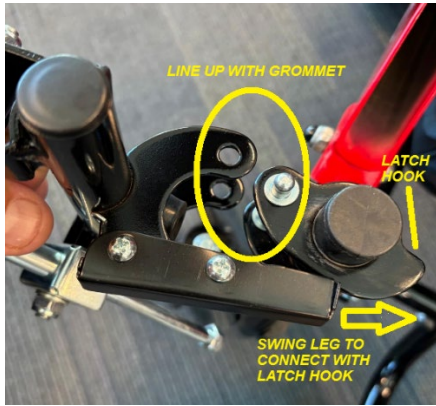

LEG REST ATTACHMENT:

- ✓ LINE UP HOLES ON LEG REST WITH THE GROMMET ON POST
- ✓ ONCE CONNECTED SWING THE LEGS TO THE FRONT TO LOCK WITH THE LATCH HOOK



COMPLETED ASSEMBLY