



## AT1000 Scooter Trailer



### Contents

1. Introduction
2. Parts
3. Assembly
4. Specifications
5. Warnings

# 1. Introduction

Your **AT1000 Scooter Trailer** provides an easy and convenient way to carry items with you when using your **8fjj Y Medical** scooter. It's perfect for carrying whatever you need for your favorite outdoor activities. The AT1000 attaches easily to the rear of your Drive scooters and can be folded quickly for storage or transport.

## 2. Parts



1. Connection Bar

2. Basket Cover

3. Frame

4. Wheels (quick release)

5. Front Legs (available w/ wheels)

6. Fabric basket

### 3. Assembly

1. Place the fabric basket over the top of the trailer frame so that the top flap of the fabric cover overlaps the top of the frame. Use the snaps in the fabric basket's top flap to fasten the basket into place.



#### Note

The fabric basket features an attached cover that can be snapped into place to protect items placed in the trailer. The cover can be rolled back out of the way when not needed.



2. After attaching the fabric basket to the trailer frame, locate your scooter's accessory connection point. This is a square metal tube located at the back of your scooter's seat.



Insert the trailer's connection bar into the accessory connection point and then tighten the screw to lock the connection bar in place.



### Note

Due to the difference in seat height on Drive scooters the connection bar must be turned the proper way to correctly attach the trailer to your scooter. See the images below for the proper connection bar positioning:



**For HD with higher seat  
to floor height**



**For transport to midsize  
with lower seat to floor  
height**

3. The trailer's wheels are removable. To remove the wheels, loosen the screw located on the inside of each wheel, the lift up on the hinged piece on the frame to release the wheel.



To reattach the wheels, insert the wheel's metal pin into the appropriate hole in the frame while holding up the hinged piece. Once the wheel is inserted, release the hinged piece and tighten the screw to lock the wheel back in place.



4. Your trailer can be folded for easy storage and transport. Simply push the front legs back toward the wheels.



**4. Specifications**

Overall Dimensions	25" (L) x 25" (W) x 22" (H)
Tires	12" PU tire
Total Weight	21.25 lbs.
Weight capacity	55 lbs.
Compatible Scooters	All Drive brand scooters
Color	Gray

## 5. Warnings

1. When using the trailer is attached to your scooter, **DO NOT** swivel the scooter's seat.
2. When using the trailer with your scooter, the maximum safe incline to drive on is 6 degrees.
3. When driving your scooter in reverse, or turning, when the trailer is attached, please pay careful attention to whether any people or obstacles are nearby. When traveling in reverse, the trailer will angle in the direction **opposite** of your scooter's direction.
4. For safety, **DO NOT** carry people or animals in your trailer.

# Limited 1 Year Warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for 1 year from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 1 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2015 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

REV1.01.28.15