




4-1 Align the headboard and footboard with the standard slots, insert them fully, and ensure they are installed steadily without any tilting.

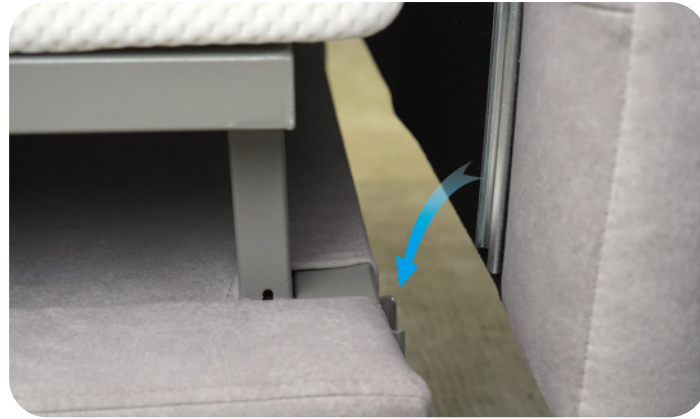

4-2 Align the left and right bed sideboards with the standard slots, gently insert them, and ensure they are laid flat for use. To disassemble, press and hold the semicircular metal lock button, then gently remove the sideboard in the direction indicated.

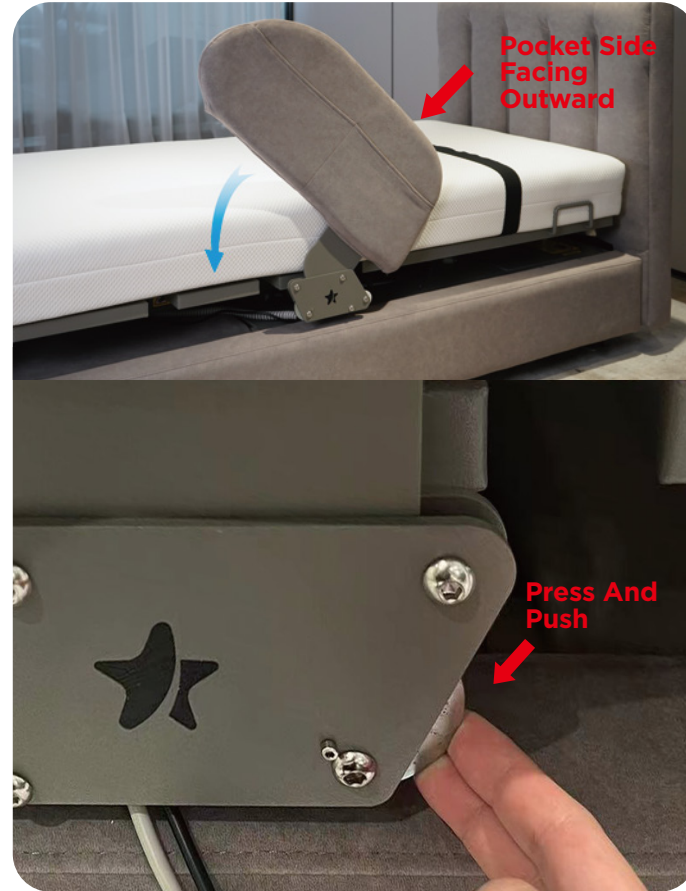


5-1 After raising the backrest using the hand controller, you can remove the shock-absorbing sponge manually.

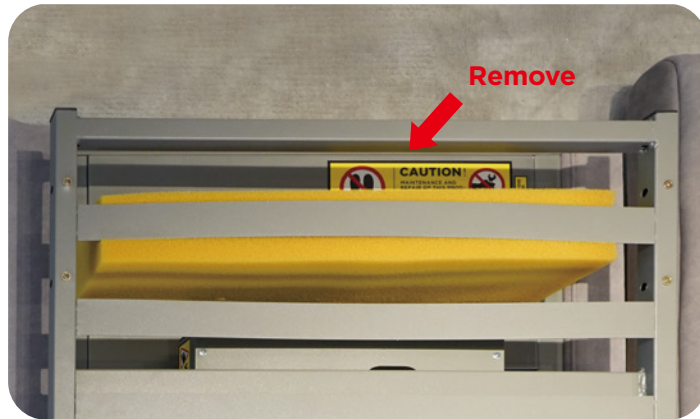

2 Bed Models for Selection

Left Rotation | Right Rotation

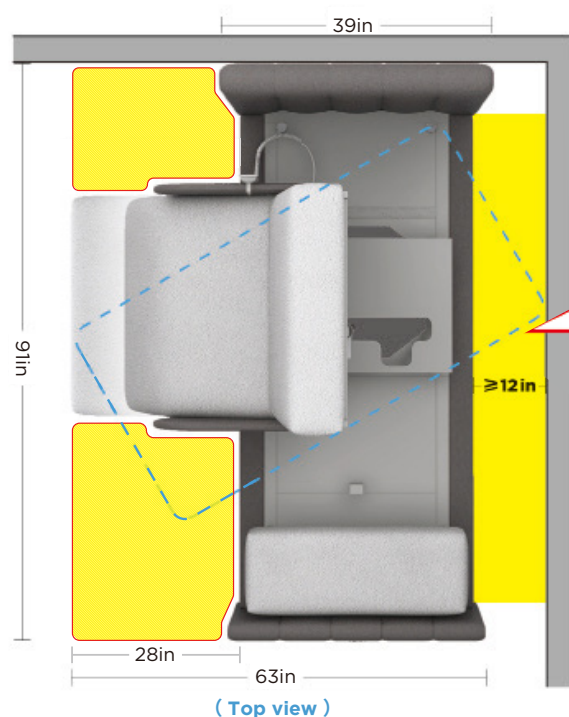

Distance from wall
≥12in

The Arlo bed should be placed at least 12 inches away from the wall and should face away from the exit. Do not place any items in the yellow zone.

Please Kindly Register and Activate Your Warranty:

Visit our website, please go to www.starsleepus.com

Create an Account or Log In:

If you do not already have an account, you can easily create one. If you have an existing account, simply log in.

Upload Your Invoice:

To complete the registration process, kindly upload a copy or screenshot of your purchase invoice.

The Arlo Bed should be placed at least 12inch away from the wall, opposite to the direction of exit.



Before you begin... A minimum of two people are required to assemble the Arlo Bed.



1-1 Place the two cardboard boxes flat in the designated position, check if the boxes are intact and free from damage.



1-2 Carefully use tools to open the outer and inner packaging, gather and store the removed packaging materials for disposal.



1-3 Place all unpacked accessories in a centralized location, check the quantity and completeness of all product accessories, and prepare for installation.



① Arlo Bed Frame x1

② Headboard x1

③ Footboard x1

④ Side Panels x2

⑤ Arlo User Manual x1

⑥ Side Retainer x2 & Screws x4

⑦ Mattress-Body Section x1

⑧ Mattress-Leg Section x1




⑨ Safety Belt x1

⑩ Remote Control x1

⑪ Shock Absorption Sponge x1

⑫ Feet & Feet Pad X4

⑬ Feet Screw x4

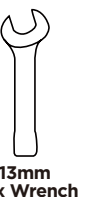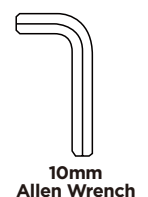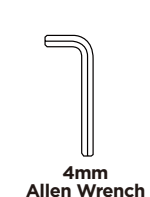

Please prepare the following installation tools:
4mm/10mm Allen Wrenches, 13mm Hex Wrench

2-1 Flip the bed over and use a 10mm Allen Wrench to screw the foot 13 into the pre-drilled mounting holes on each of the 4 legs then attach the foot pads.




3-1 Place the bed frame flat in the designated position, ensure the bed frame is stable, and cut the three black cable ties.




3-2 Begin installing the mattress use 4mm allen wrench to install the left&right side retainer with screws.



3-3 Remove the outer non-woven fabric protection, with the mattress bottom facing up, use the 13mm Allen wrench to remove all nuts and store them properly.

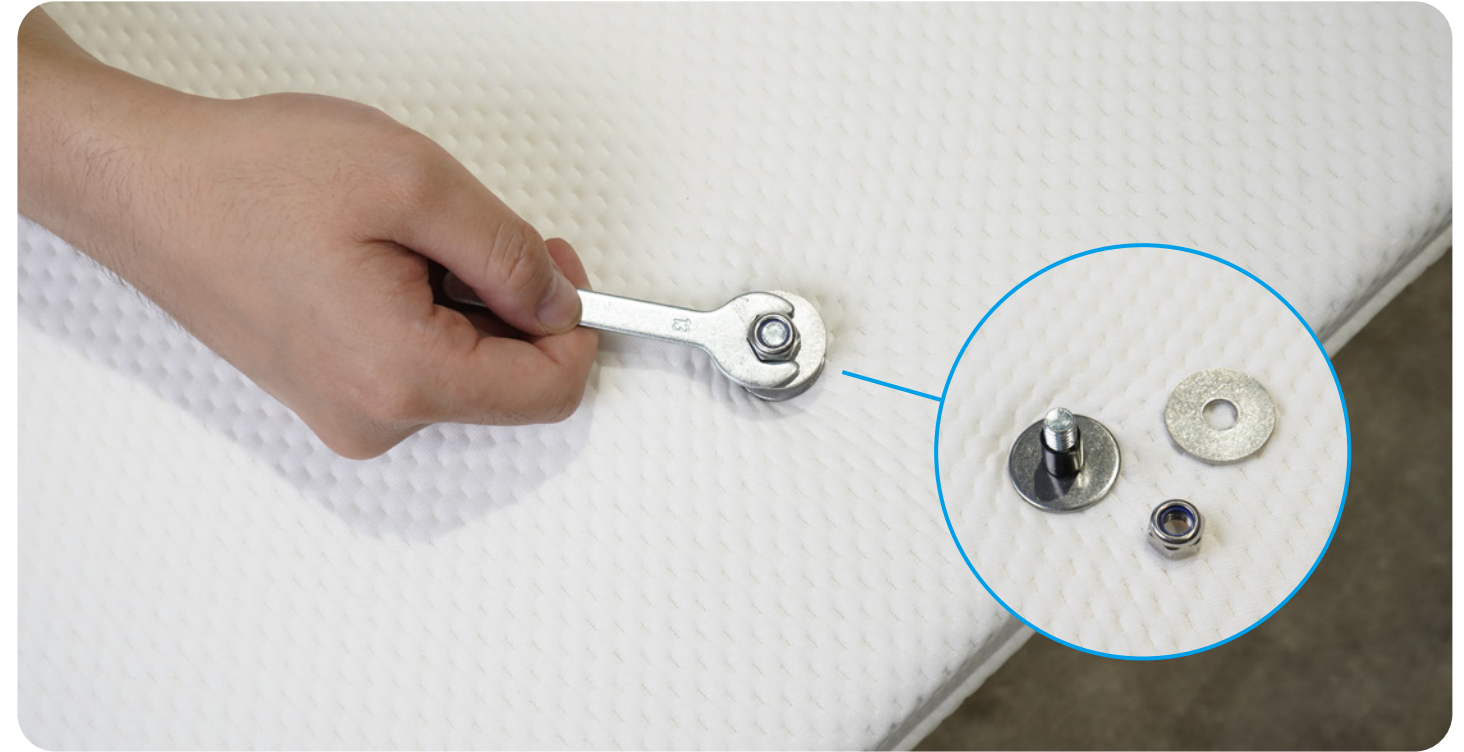

3-4 Align the screws with the holes and insert them downward into the designated holes. There are a total of 7 holes for the two mattresses combined.

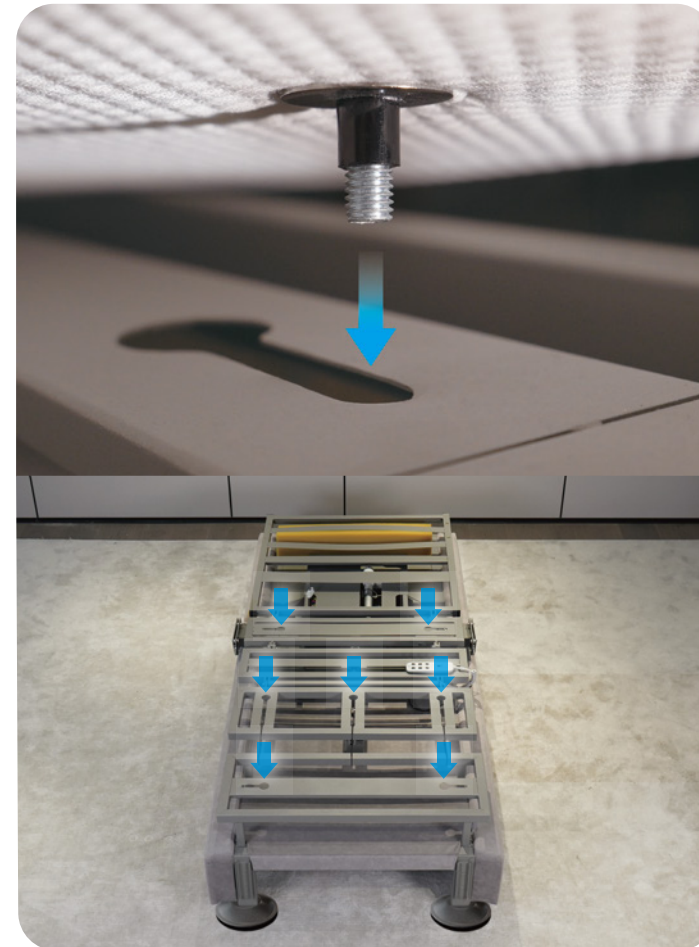

3-5 Use your hands or an external hexagonal wrench, or professional power tools, to tighten the washers and nuts upwards (not too tightly).

