




GOLDEN
AutoDrive™
 CUSTOMIZED COMFORT

Introducing a Revolution
 in Power Lift and Recline Chair Comfort...

AutoDrive™!

The unique AutoDrive hand control drives your chair into the perfect position for you!

This technologically advanced system offers pre-programmed positions to give you an even more comfortable, customized recline experience! The AutoDrive system has pre-programmed buttons for the TV Watching, Zero-Gravity and Lift positions. It includes two additional memory buttons that you can program to be sure your favorite positions won't be lost!

The keypad is backlit to provide easy nighttime use. The new AutoDrive hand control fits the same quick disconnect receiver as the current MaxiComfort hand control, and has the same hand control electronics warranty!

Available on select models.





Let AutoDrive drive you to any position smoothly and quietly!



TV Watching

AutoDrive's TV button smoothly moves you to the TV Watching position from any other position so you can comfortably watch TV, read or use your laptop computer with ease! The TV Watching position provides ergonomic upper body positioning in an effort to minimize neck strain and discomfort.



Zero Gravity

AutoDrive's ZG button glides you into the pressure-free comfort of the Zero-Gravity position, where the spine, hip and knee joints are positioned on the balanced midpoint of the muscles. This results in natural relaxation and minimal muscle tension.




Chair Button

AutoDrive's Chair button operates the same way as the buttons on a two-button lift chair. When you're experiencing AutoDrive for the first time, use the chair button's up and down arrows to recline back and stand up. It's that easy!



Lift Button

AutoDrive's Lift button raises you to the standing position no matter how your relaxing in your lift chair. The Lift button's unique positioning keeps the backrest more open so that you won't feel as though you are being pushed out of the chair, but instead being gently helped to the standing position.

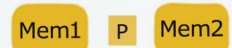

Mem1, Mem2 and P Buttons

AutoDrive's unique Memory buttons can be programmed to move the chair to your favorite two positions in a matter of seconds! To program either Memory button, move the chair to a comfortable position that you would like the chair to "remember." Hold down the P (programming) button until the button lights flash. Hold down the Memory button you want to use for the position. Hold the button until the lights are on and not flashing. That Memory button is now set for your favorite position! Even better, the Memory buttons can be reprogrammed again and again as you find even more comfortable positions!

