MACLAREN GENERAL INFORMATION

Job No:	Edited by:
File Name:	Round No.: 9
Designer:	Date: 10.13.25
Fonts: MyriadPro, EurekaSans, CircleFrame, ArialUnicodeMS	
Print Substrate: 80 GSM Wood free paper (Goodbaby)	
Trim Size:	

PMS 199U

Black

Laminate  
(type?)  
 Y  N

Varnish  
(type?)  
 Y  N

**IMPORTANT: KEEP FOR FUTURE REFERENCE**  
IMPORTANT : CONSERVER POUR TOUTE RÉFÉRENCE ULTÉRIEURE.  
IMPORTANTE: CONSEVAR PARA FUTURAS REFERENCIAS.  
WICHTIG: BITTE SICHER AUFBEWAHREN.  
IMPORTANTE: CONSERVARE PER CONSULTAZIONI SUCCESSIVE.  
BELANGRIJK: BEWAAR VOOR NASLAG.  
WAŻNE: PROSZĘ ZACHOWAĆ.  
ВАЖНО: СОХРАНИТЕ В КАЧЕСТВЕ СПРАВОЧНОЙ ИНФОРМАЦИИ.

**Coche Mobility Solutions Inc. is the official global licensee of Maclaren trademarks**

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## IMPORTANT SAFETY REMINDER: PLEASE READ

If you have read and fully understood the instructions provided in the Owner's Manual and accompanying safety warnings you should now remove the red safety strap on your Major Elite Transport Chair.

If you have not fully understood any aspect of the Owner's Manual and accompanying safety warnings, which have to be adhered to, you must not use this transport chair until you have sought clarification on any issues by either:

1. Contacting customer service by visiting [www.maclarenmajorelite.com/customer-service](http://www.maclarenmajorelite.com/customer-service);
2. Emailing a representative at [info@maclarenmajorelite.com](mailto:info@maclarenmajorelite.com);
3. Logging on to [maclarenmajorelite.com](http://maclarenmajorelite.com) in order to download the most up-to-date Owner's Manual.

**CAUTION: This pushchair is not suitable for use as a seat in a motor vehicle**

**IMPORTANT:** Please ensure that this Owner's Manual is kept for future reference and is passed on to anyone who is not familiar with this product before they use this transport chair. Failure to do so may affect the occupants safety. Do not place the occupant's safety at risk - always follow the Owner's Manual and accompanying safety warnings.

**IMPORTANT: IF YOU ARE UNABLE TO UNDERSTAND OR USE THIS TRANSPORT CHAIR IN ACCORDANCE WITH THE SAFETY OPERATION INSTRUCTIONS IN THE OWNER'S MANUAL PLEASE REFRAIN FROM USING THIS TRANSPORT CHAIR AND RETURN TO THE PLACE OF PURCHASE.**

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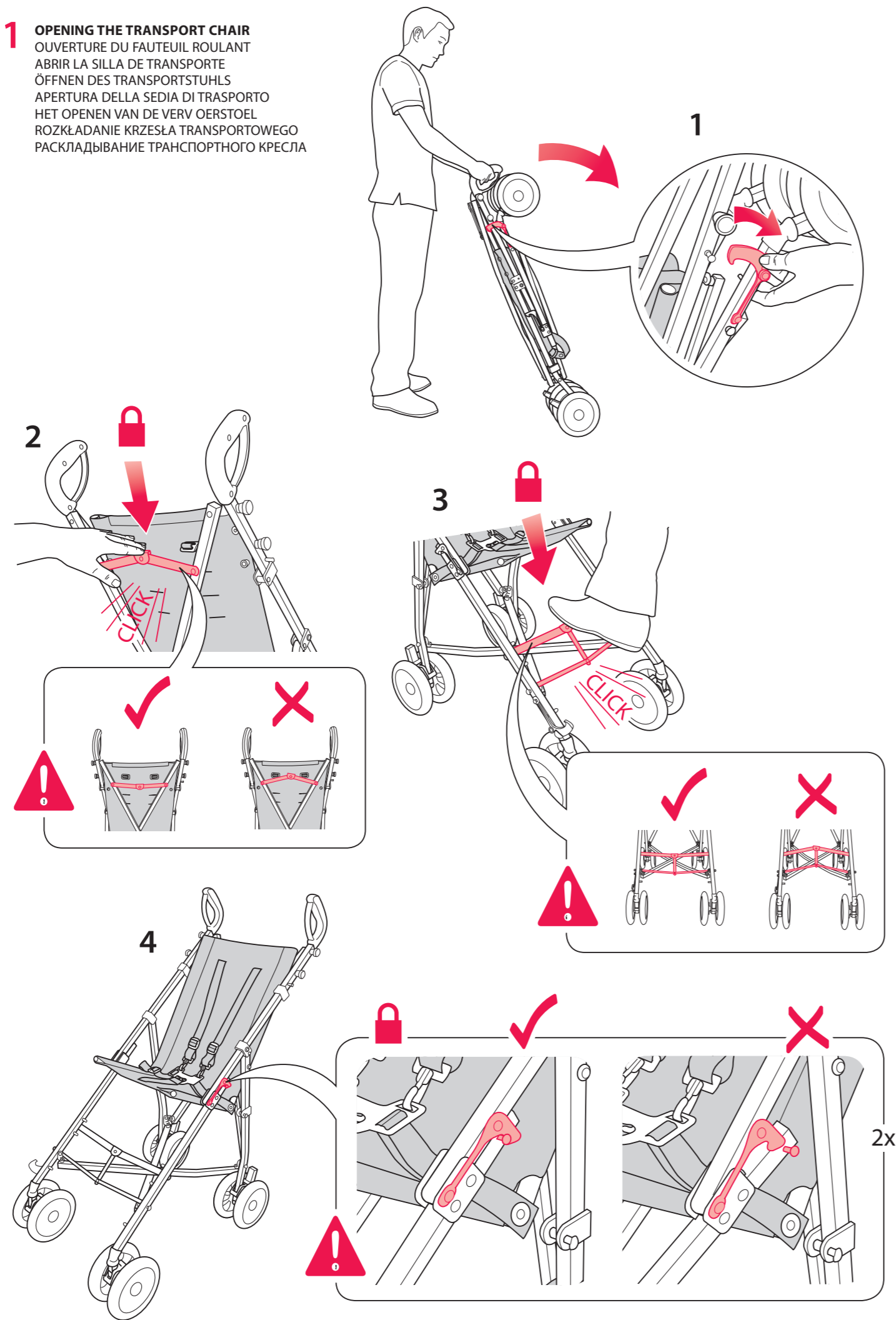
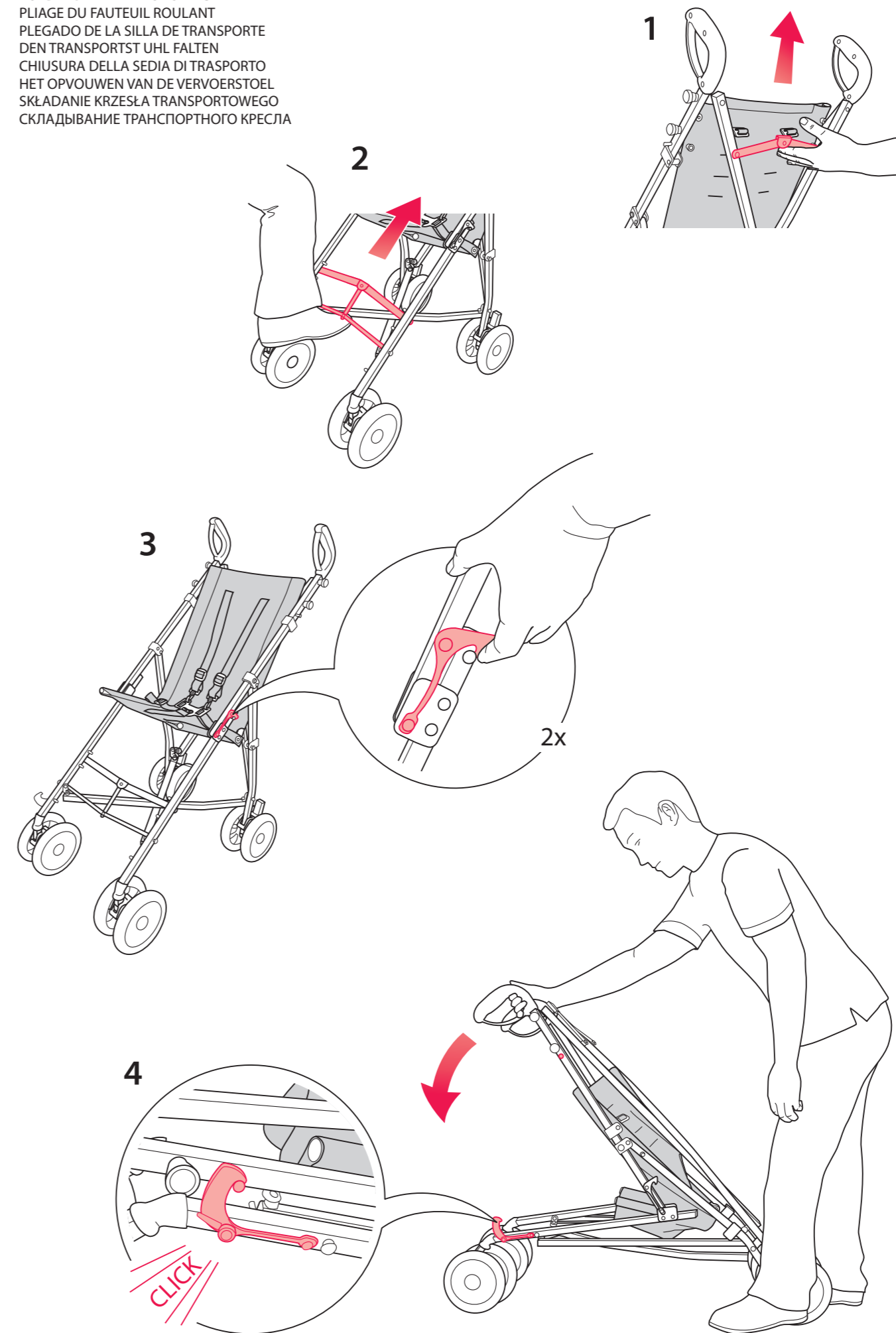
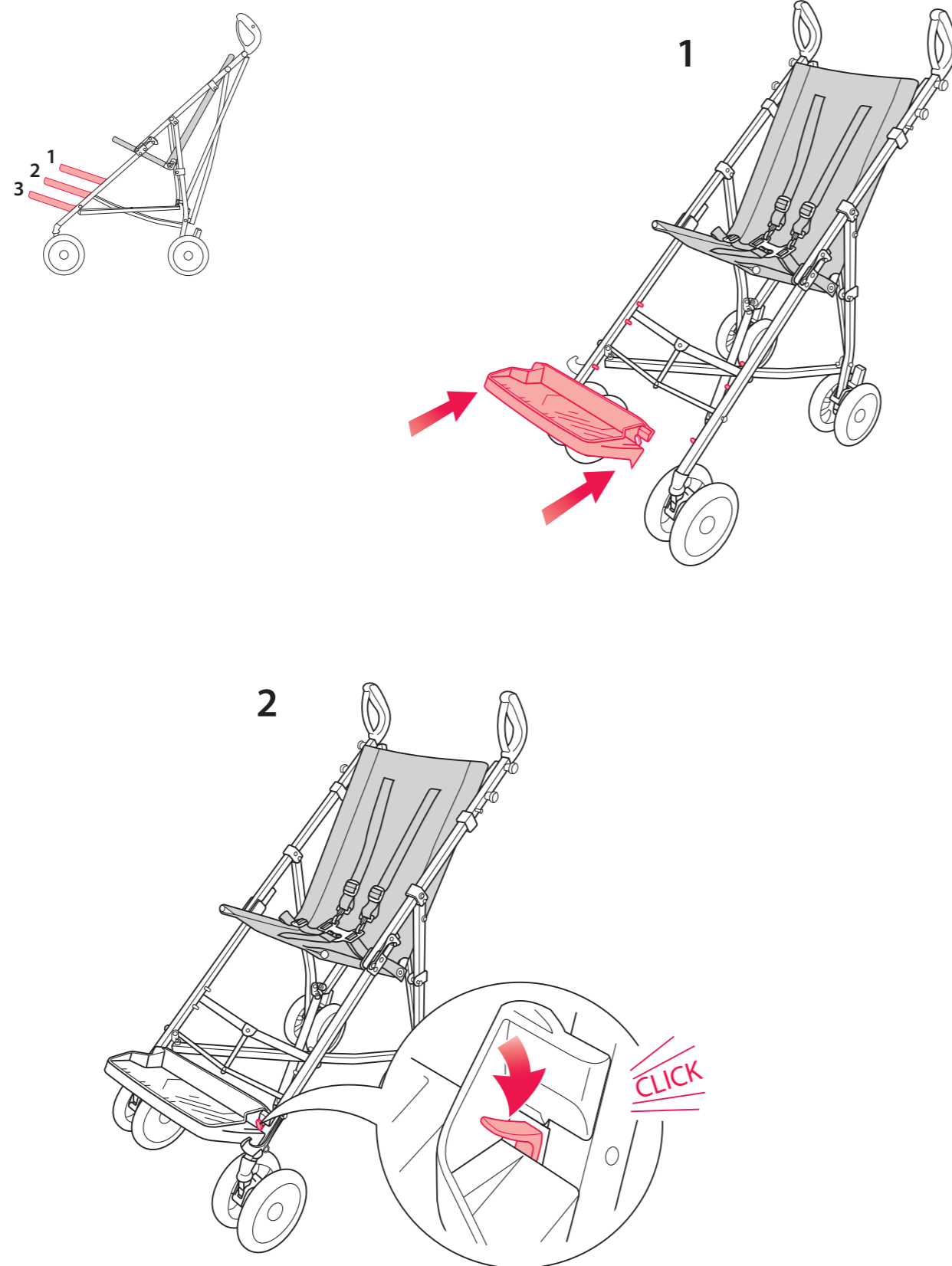
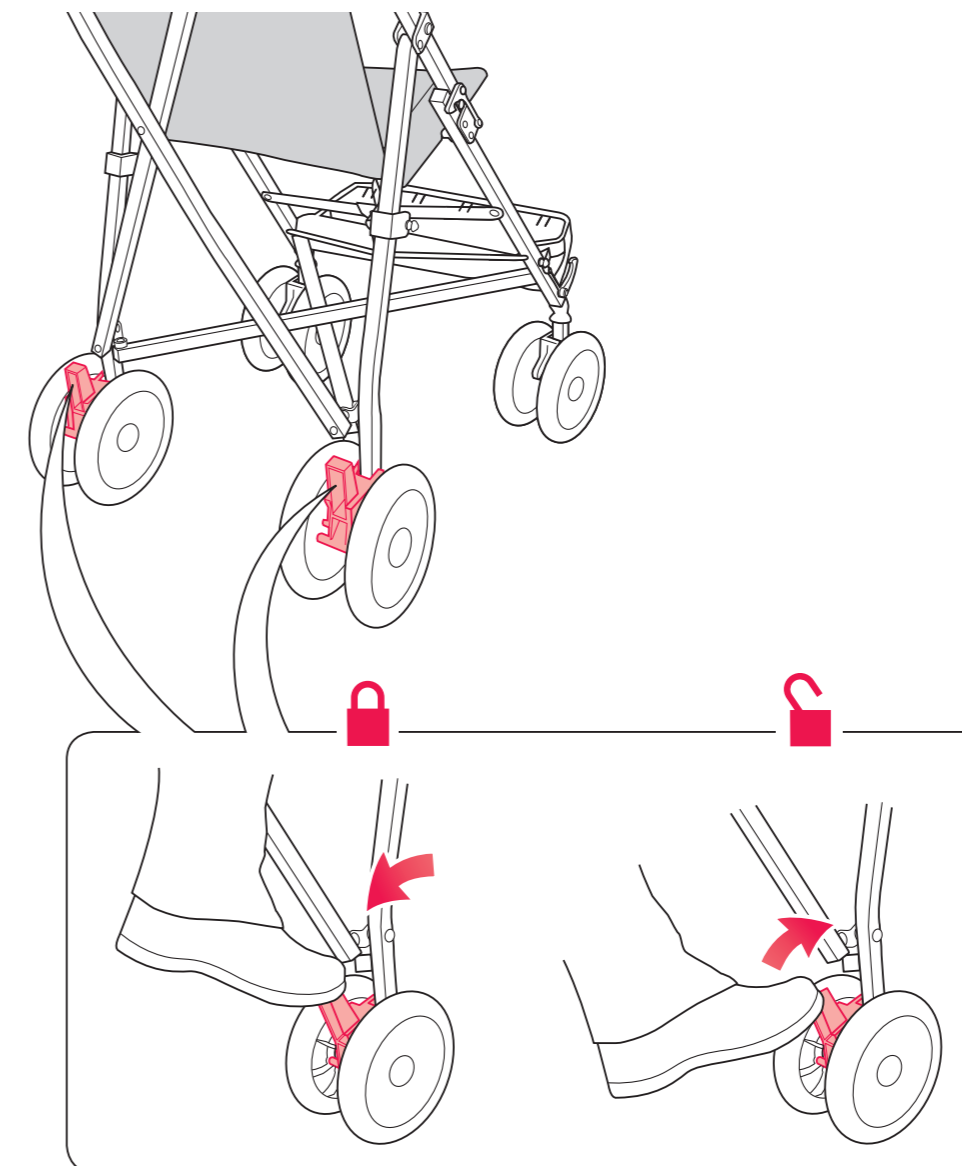
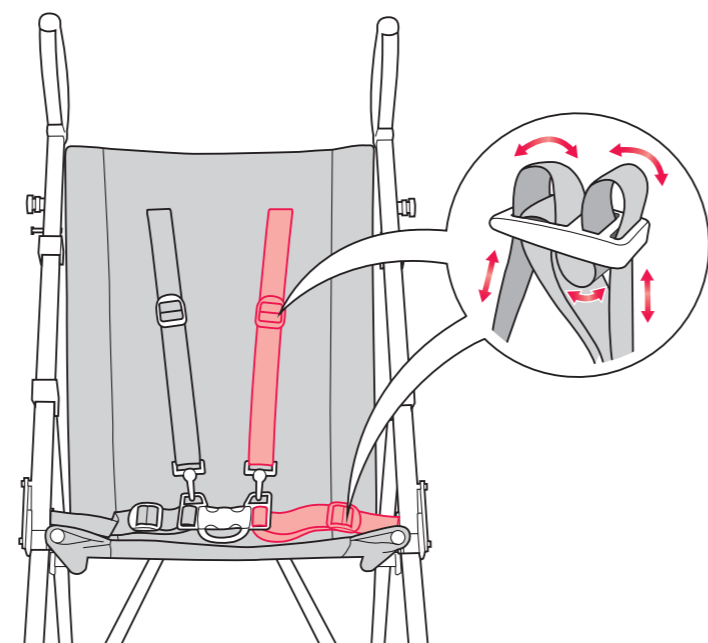
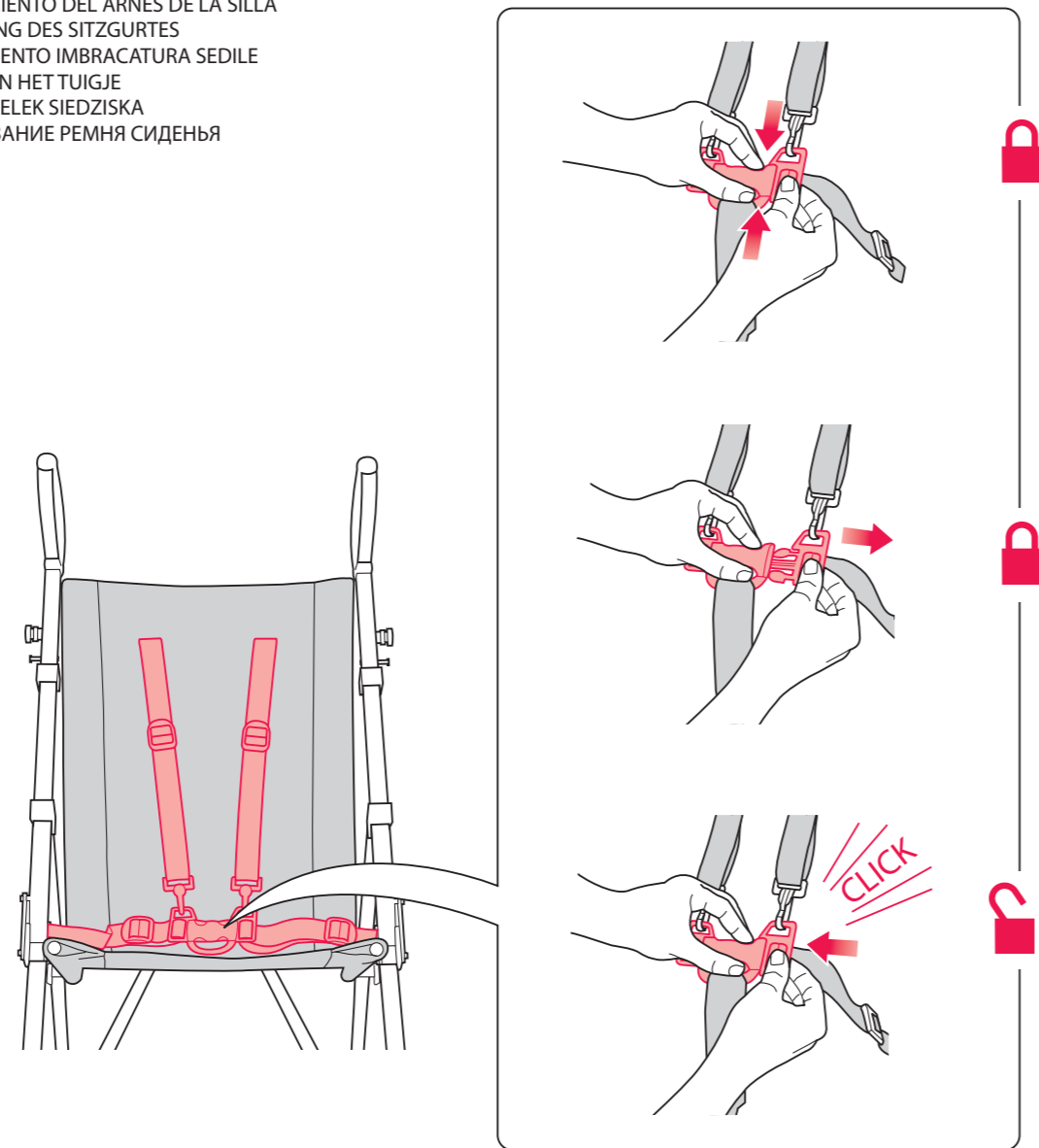
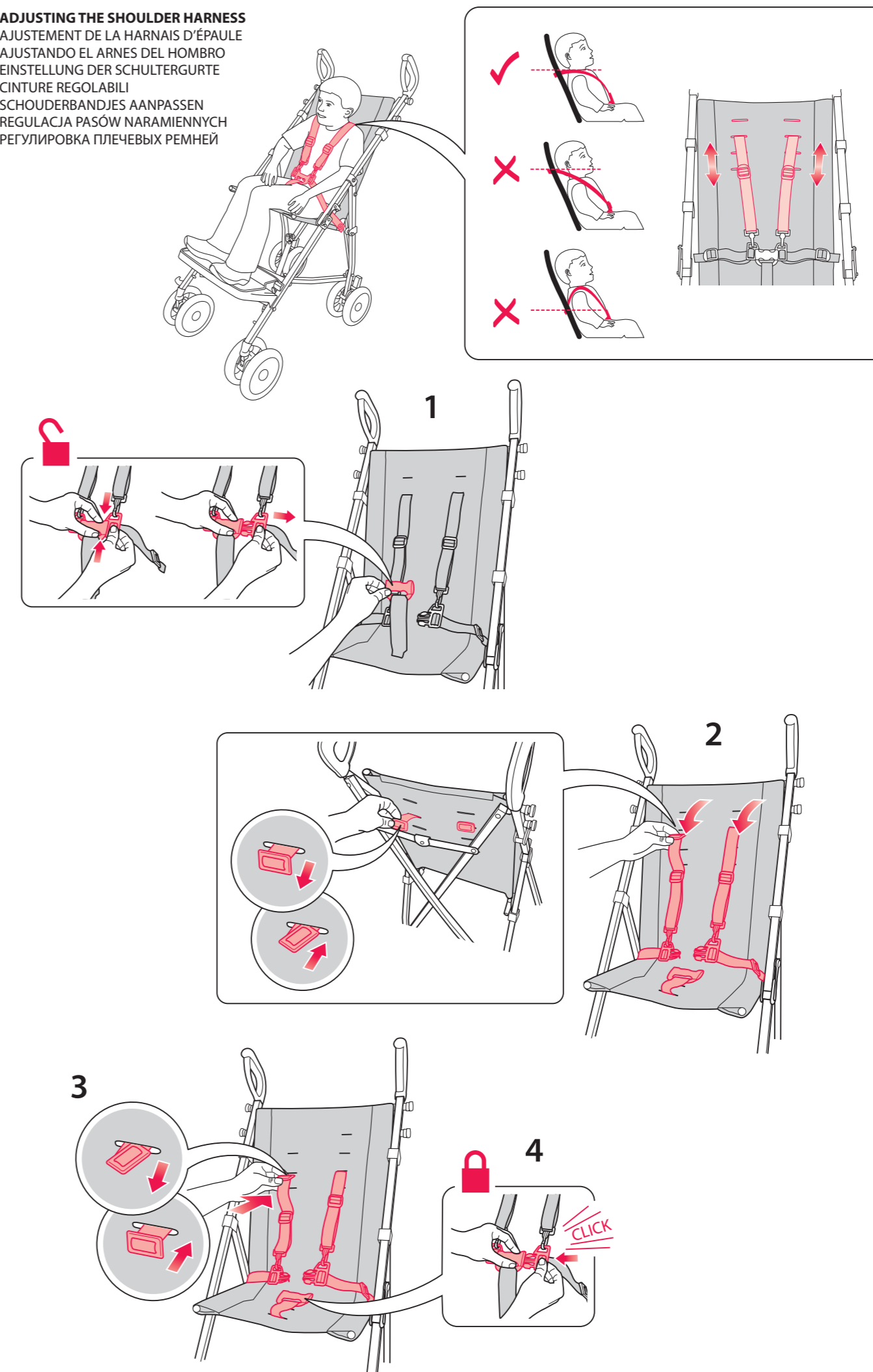
**IMPORTANT: KEEP THESE INSTRUCTIONS FOR FUTURE REFERENCE****⚠ SAFETY INFORMATION**

The Major Elite is a lightweight folding Transport Chair developed for journeys of short duration for occupants up to 110 lbs who are without significant postural instability and need minimal support.

- **ALWAYS seek expert medical advice regarding the suitability of this product before use.**
- To adapt the seat for a smaller occupant, a pair of Lateral Inserts is available.
- The product must be operated by a responsible caregiver at all times.
- Ensure that all the locking devices are engaged before use.
- Never leave a child unattended in the product.
- This product is intended for use by one occupant only.
- This seat unit is not suitable for children under six (6) months.
- Always use the restraint system.
- This product is not suitable for running or skating.
- Always apply the parking devices when the product is stationary.
- Do not stand on the footrest.
- Do not let children play with or hang on the product.
- Do not use the product in a manner for which it is not designed.
- **IMPORTANT-DANGER:** It is important that great care is taken to ensure that children are kept well clear of the product when folding or unfolding. Finger traps and scissoring actions are unavoidable during these operations.
- It is important for the occupants safety that the harness be correctly fitted and adjusted (especially after replacement for washing purposes). If not adjusted correctly in accordance with the instructions the safety of your child and stability of the product will be impaired.

- Any load attached to the handle and/or on the back of the backrest and/or on the sides of the unit will affect the stability of the product.
- The product may be folded with the shopping basket still attached but all contents must be removed. Failure to do this could damage product.
- Accessories which are not approved by the manufacturer shall not be used. Only replacement parts supplied or recommended by the manufacturer/distributor shall be used. Any damage caused by the use of accessories not recommended will not be covered by the terms of our warranty.
- The harness and seat belt are not a substitute for proper adult supervision.
- Follow all instructions carefully. Incorrect use will cause damage to product.
- Do not leave the product exposed to heat e.g. by an open fire, exposed flame, radiator or in direct sunlight.
- Never leave occupant in the product when ascending or descending escalators or when travelling on other forms of transport.
- This is designed for use on flat or gently sloping surfaces and may be unstable on higher sloping and uneven surfaces.
- This product must NOT be used as a seat while travelling in a motor vehicle or public transport.
- Hold on to your transport chair when close to moving road vehicles or trains. Even with the parking devices on, the draught from the vehicle may move the transport chair.

**GETTING INTO THE MAJOR ELITE**

- Ensure both parking devices are ON.
- To make access easier, the footrest can be removed.
- If possible, place the back of the product squarely against a wall or secure object.

- Alternatively, another helper may hold it steady.
- If the occupant is not being lifted into the product, align the front wheels with the back wheels. Place the occupant's feet on the ground between the front wheels, one foot just forward of the other and lower gently into the seat. The occupant may assist by holding the product's front legs.
- Fasten the harness and adjust the strap lengths. Belt and shoulder harness should be snug. Replace the footrest at the appropriate height.
- Take note of how the occupant wishes to move or be moved. Cooperation will reduce the risk of injury occurring during lifting and manoeuvring operations.
- Do not allow the occupant to climb unassisted into the seat.

**GETTING OUT OF THE MAJOR ELITE**

- Ensure both parking devices are ON and the footrest is removed.
- If possible, place the back of the product squarely against a wall or secure object. Alternatively, another helper may hold it steady.
- Align the front swivel wheels with the rear wheels to allow room for the occupant's feet.
- Unfasten the harness clasp and if necessary unclip the shoulder straps.
- If the occupant is not being lifted out of the product, they should grasp each front leg and bending slightly forward, place both feet firmly on the ground, one forward of the other. The occupant should then push upward to a standing position with assistance if necessary.

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## CARE OF YOUR TRANSPORT CHAIR

- Your transport chair has been designed to a high standard and with correct use and maintenance will give many years of trouble free service.
- Regularly inspect the locking devices, parking devices, harness assembly, catches, footrest, joints and fixings to ensure they are secure and in full working order.
- Only Maclaren replacement parts and accessories should be used on this transport chair.

- If wheels squeak, lubricate sparingly with silicone coating. Do NOT use oil or grease based products, as this will attract dirt, which will clog the movement.
- Plastic and painted parts may be sponged clean with warm water and a mild detergent.
- The seat fabric can be sponged or scrubbed clean using a mild detergent and/or mild disinfectant.
- Never clean with abrasive, ammonia based, bleach based or spirit type cleaners.
- After exposure to damp conditions, dry off with a soft cloth and leave fully open in a warm environment.
- Do not fold up the vehicle or accessories whilst wet, or store in a damp place. Either of these will encourage mildew to form. Always dry all items before storing.
- Store your transport chair in a dry, safe place.

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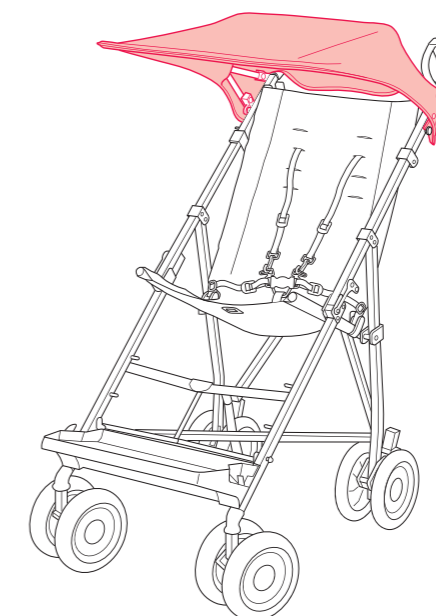
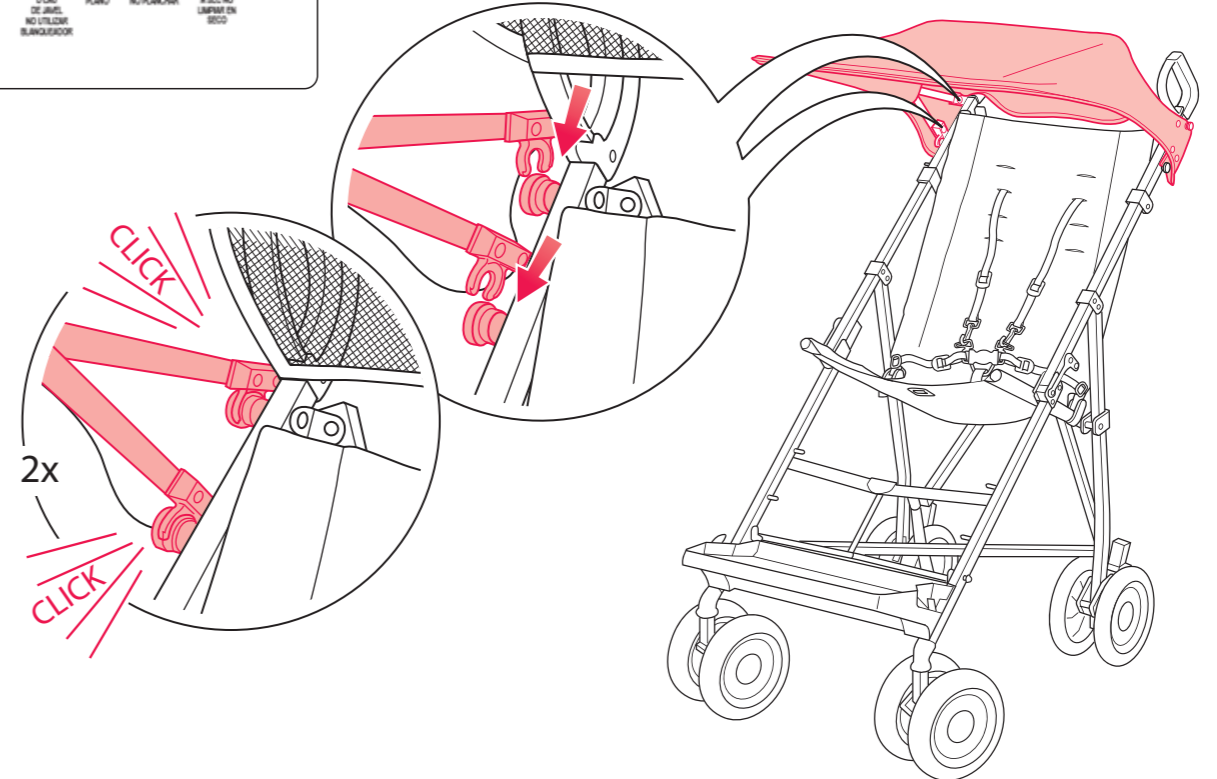
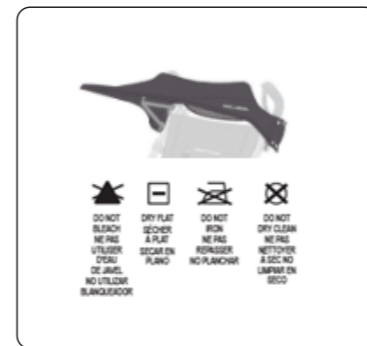
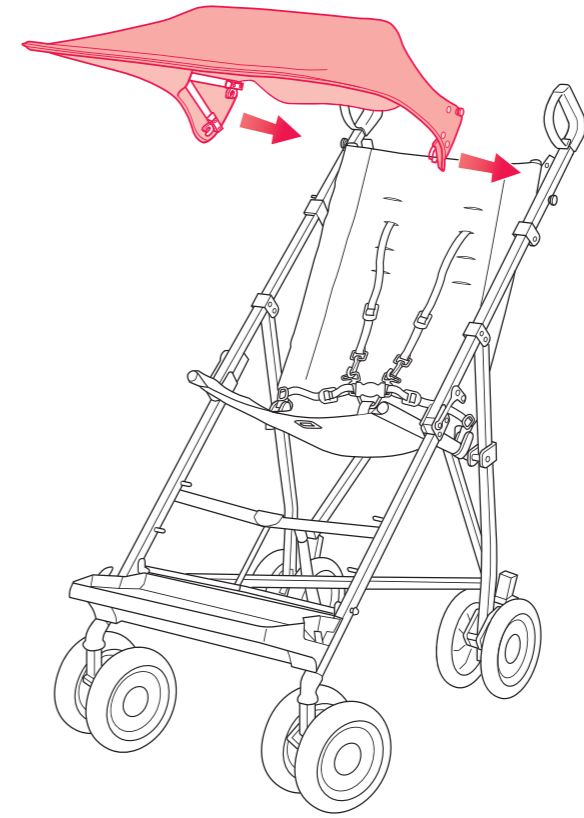
**Hood**  
Capote  
Capota  
Haube  
Cappotta  
Kap  
Тент  
Budka

**EN** **⚠ WARNING**

- NEVER leave the occupant unattended
- ALWAYS use the restraint system – This product is not suitable replacement for the restraint system
- Do not allow any part of this product to cover a child's head/nose/mouth
- Parking devices shall be engaged when placing or removing occupants or when making adjustments
- This product is not intended for prolonged periods of sleeping
- To avoid the risk of hyperthermia, monitor the temperature of the area where the occupant is placed

*Read these instructions carefully before use and keep them for future reference*

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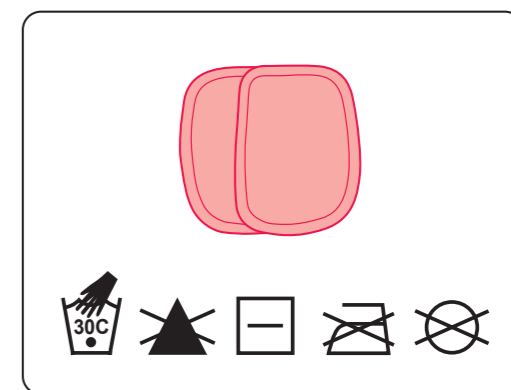
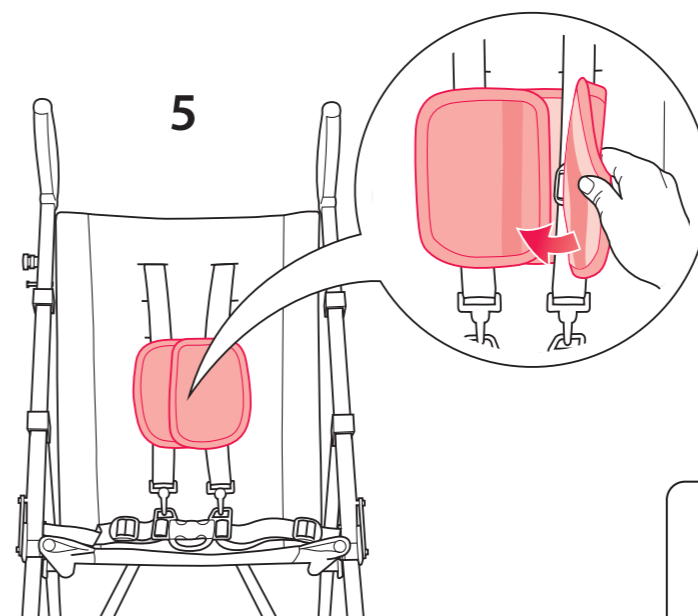
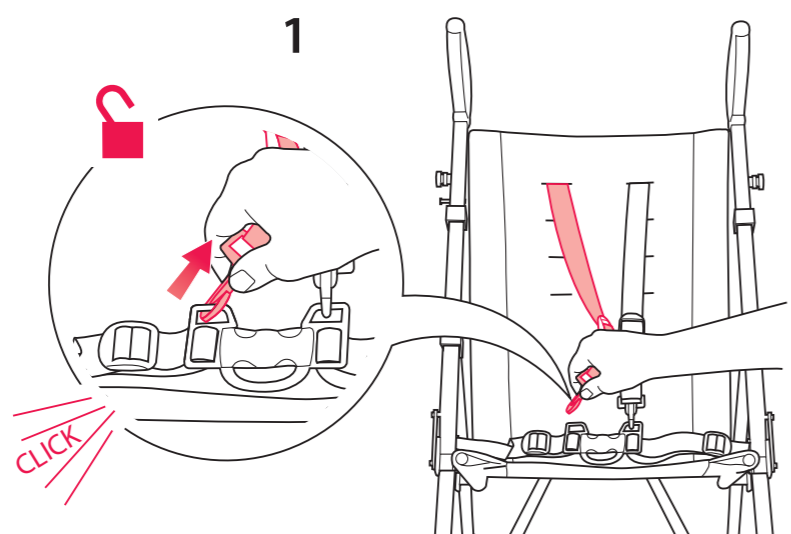
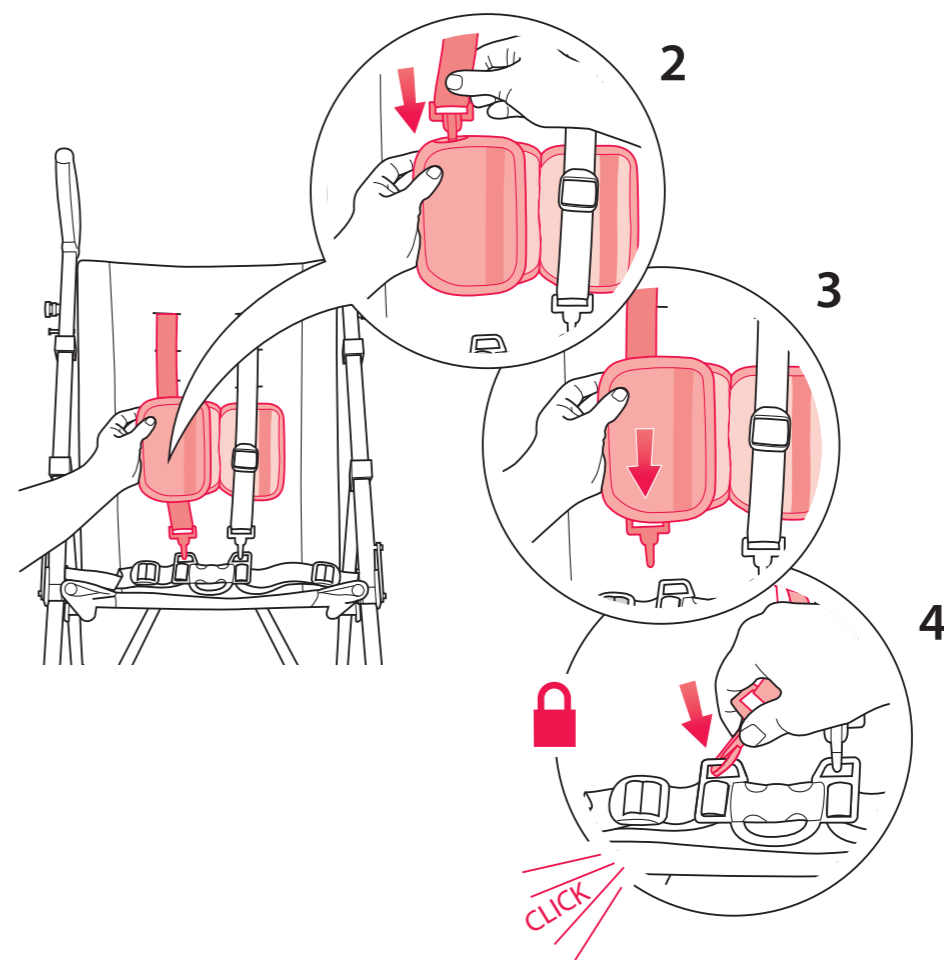
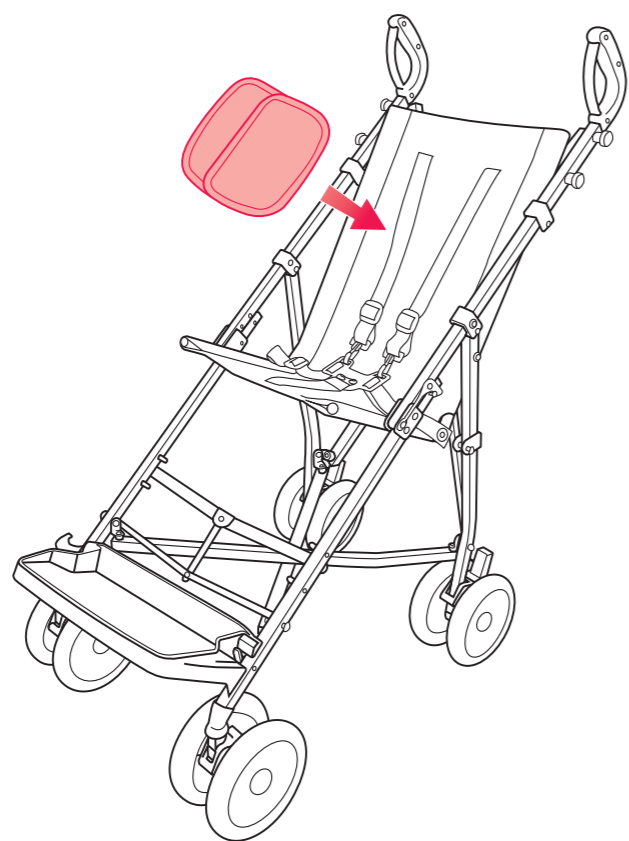
**EN** **⚠ WARNING**

- NEVER leave the occupant unattended
- ALWAYS use seat harness system
- Adjust the harness to fit the occupants size
- Parking device shall be engaged when placing or removing occupants or when making adjustments

**⚠ IMPORTANT**

- Seek expert medical advice regarding the suitability of this product before use.
- The lateral inserts **MUST BE REMOVED** from the transport chair before folding

Read these instructions carefully before use and keep them for future reference

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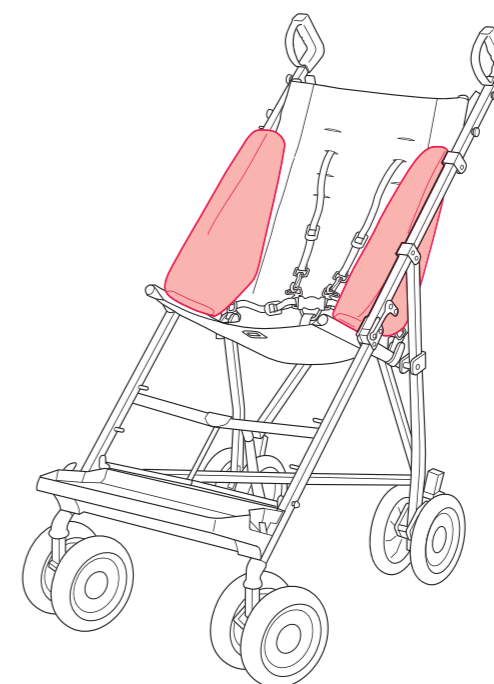
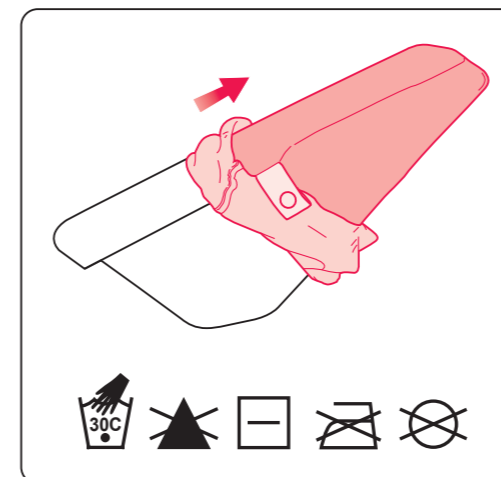
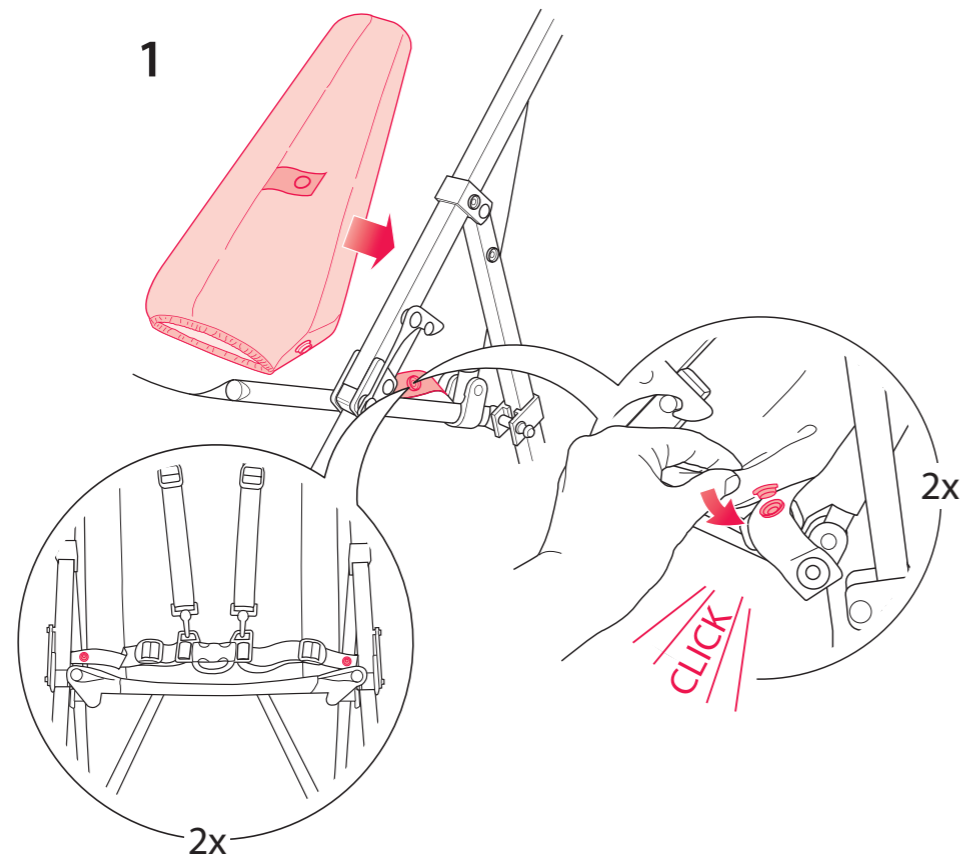
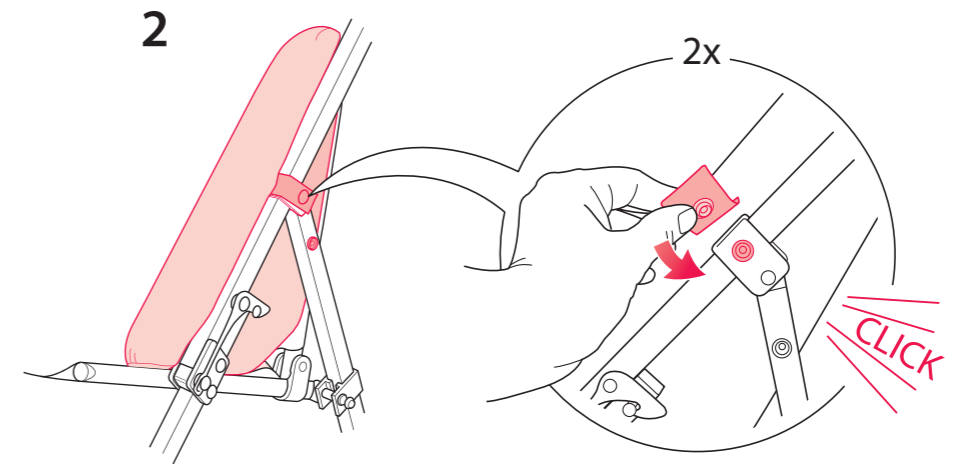
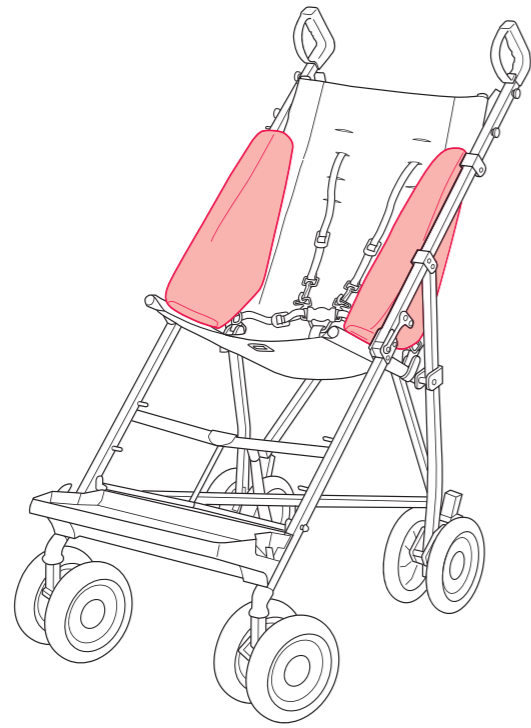
**EN** **⚠ WARNING**

- The stability of the transport chair may be affected in high winds with the raincover in place and should be steadied by an adult at all times
  - NEVER leave the occupant unattended
  - To be used under adult's supervision
  - Raincover shall not be used on the product without any hood for support
  - Raincover must be removed when indoors and in hot weather to prevent overheating and possible risk of suffocation
  - Parking device shall be engaged when placing or removing occupants or when making adjustments
- Read these instructions carefully before use and keep them for future reference*

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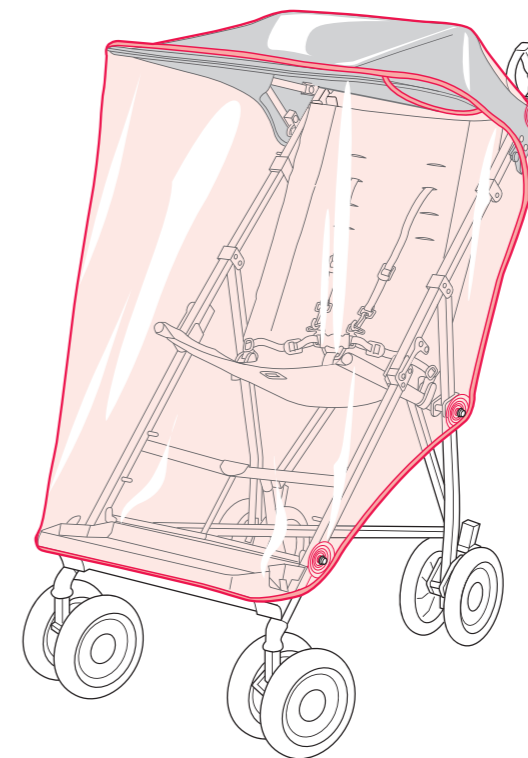
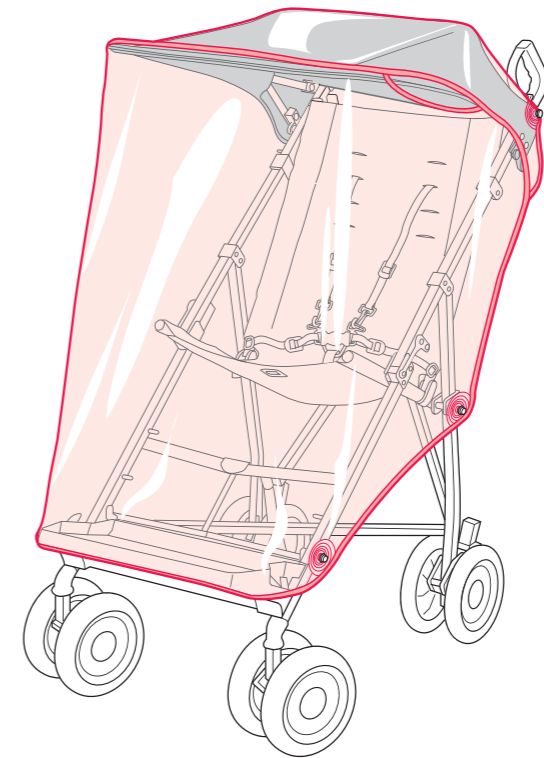
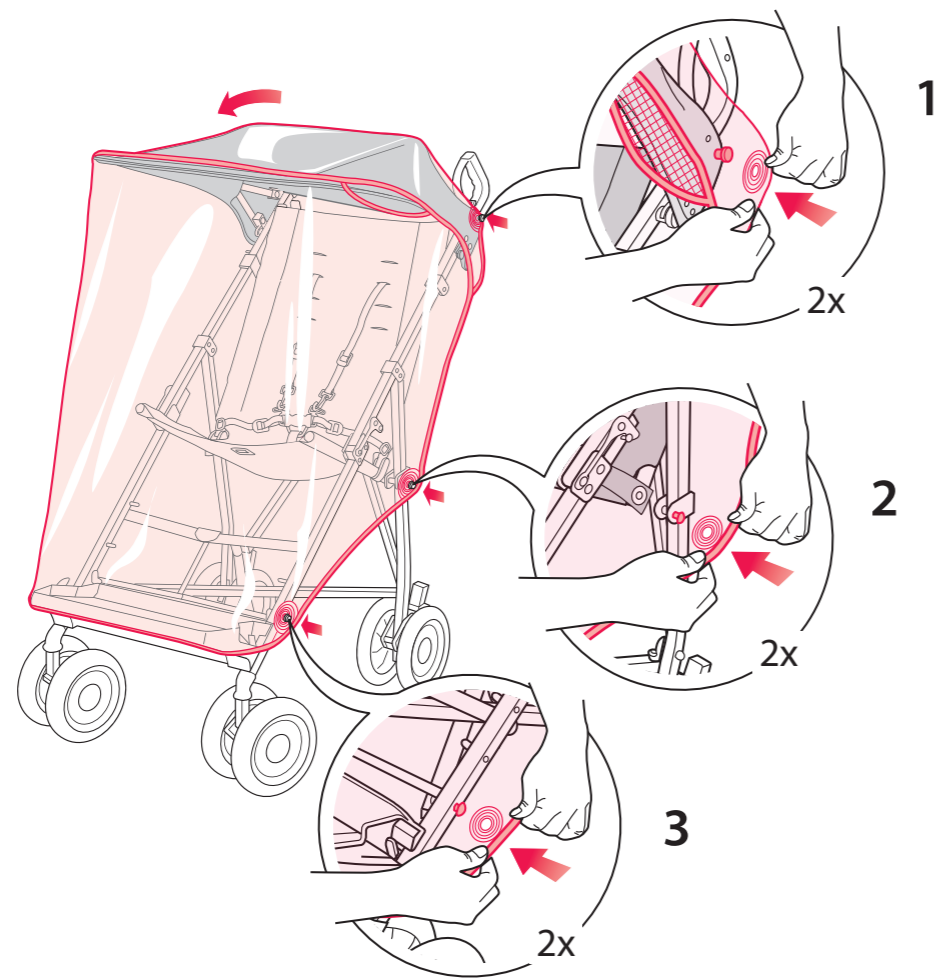
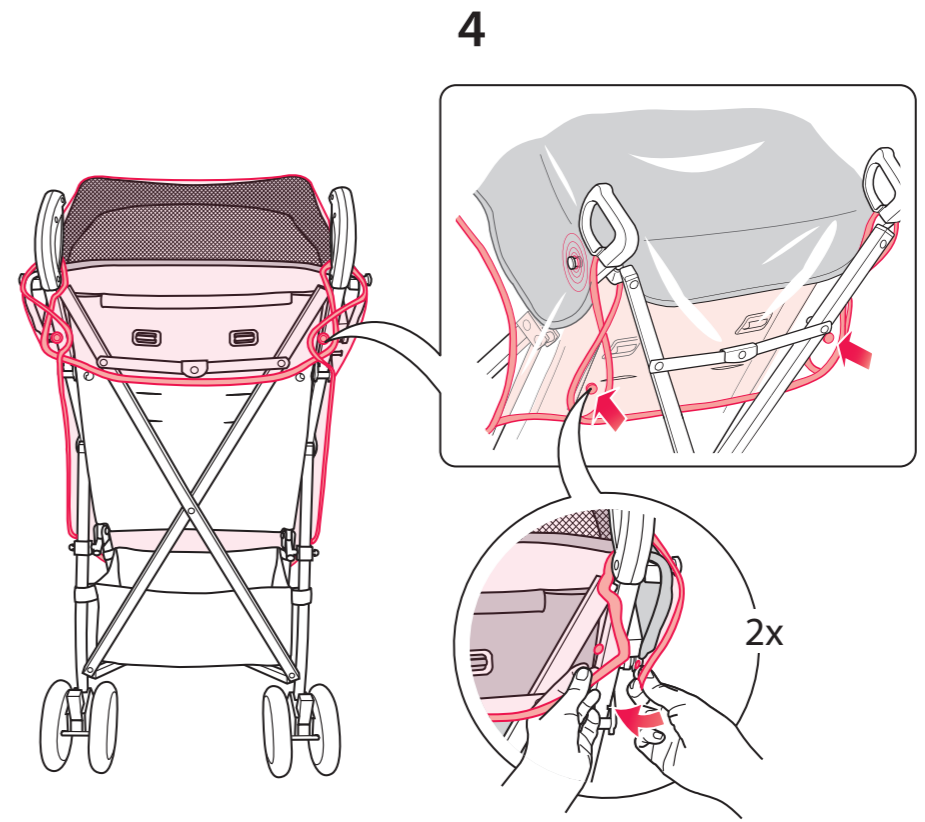
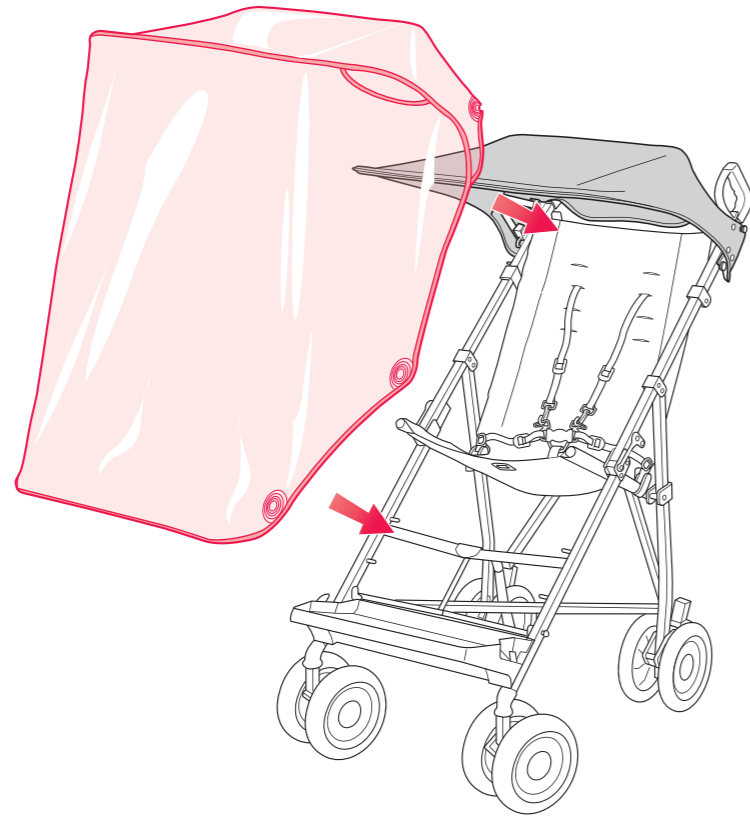
**EN** **⚠ WARNING**

- NEVER leave the occupant unattended
- ALWAYS use the restraint system – It is important for the occupant's safety that the restraint system is refitted after the liner is installed or removed and the restraint system is readjusted to fit the occupant
- This product is not intended for prolonged periods of sleeping
- Parking devices shall be engaged when placing or removing occupants or when making adjustments

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EN

### Major Elite Organizer

Fits all single buggies, with easy to attach adjustable straps  
Holds baby's essentials, including two bottle pockets  
Pocket for keys, phone or valuables  
Expandable mesh pouch with zipper closure  
Folds with the buggy

#### ▲ WARNING

Maximum load 0.5 kg (1.1 lb) evenly distributed  
*Read these instructions carefully before use and keep them for future reference*

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# MACLAREN

## Two(2) Year Manufacturer's Warranty

The warranty is built upon the expectation that when you purchase this item you receive a guarantee of quality, engineered to withstand years of use. Our guarantee: If you find any defect in material or workmanship, or the product should fail to function with proper use and care, we will gladly repair or replace it.\*

\* Visit [maclarenmajorelite.com](http://maclarenmajorelite.com) for details, conditions, and eligibility

## GENERAL PROVISIONS

This warranty covers manufacturing defects in materials and workmanship for applicable products used in normal conditions, in compliance with the operating instructions, and conjunction with accessories.

This warranty applies only to qualified products purchased from an authorized Major Elite retailer. This warranty is offered only to the original owner of the product, and it shall remain in force only as long as the original owner retains ownership of the product. This warranty is not transferable. This warranty does not apply to normal wear and tear, nor to defects, malfunctions or failures that result from the abuse, neglect, improper maintenance, alteration, modification, accident, or misuse. Spot clean with mild soap and water and wipe plastic components with a damp cloth and mild soap.

For complete and most up to date Terms and Conditions of this warranty visit [maclarenmajorelite.com](http://maclarenmajorelite.com)

## THIS WARRANTY DOES NOT COVER

- Normal wear and tear
- Improper assembly
- Improper follow-up maintenance
- Damages caused by ignoring weight requirements
- Damages caused by installation of non Maclaren components, parts, or accessories not originally intended for or compatible with the product as sold (e.g. riding boards)
- Damage or failure due to accident, misuse, abuse, or neglect including but not limited to airline, elevator, escalator, or damages resulting from car accidents
- Damage to moving components due to extreme environmental conditions, including sand, gravel, snow, mud or off-road driving
- Corrosion or rust on the wheels and chassis due to extreme environmental conditions, including high humidity, salt spray, ice or snow
- Modification or alteration of product in any way
- Products purchased from an unauthorized retailer or source
- Repairs carried out by an unauthorized third party
- Shipping costs

Wear items are defined as: wheels, tires, grips, cables and housing, fabric worn down by regular use and the natural breakdown of colors and materials over extended time and use. All of these items are subject to varying wear life depending on use, load, weather, and environmental conditions. Premature wear or "defect" will be taken into account in any warranty claim involving a wear item. A defect in material or workmanship is defined as: "a flaw, failure or cosmetic blemish caused by the manufacturing process or material that was not consistent with the original design". This warranty implies no strength or fatigue value beyond regional government standards.

This Warranty is not meant to suggest or imply that the product is indestructible or will last forever. Every product has a useful life cycle. The useful lifecycle as defined in this Warranty will vary depending on the use, environmental conditions and care the product receives. Severe conditions or climates, operating with heavy loads, or any other non-standard use can substantially shorten the useful product lifecycle. Any one or a combination of these conditions may result in an unpredictable failure that is not covered by this warranty

This warranty is expressly limited to the repair or replacement of a defective item, and said repair or replacement is the sole remedy of the warranty. Repair services are made possible based on availability of spares. Coche Mobility Solutions is not responsible for incidental or consequential damages.

THIS WARRANTY DOES NOT AFFECT THE STATUTORY RIGHTS OF THE CONSUMER. WHERE APPLICABLE, LOCAL LAWS TAKE PRECEDENT OVER THIS WARRANTY CONTRACT. THIS IS THE ONLY WARRANTY MADE BY COCHE MOBILITY SOLUTIONS. THERE ARE NO OTHER WARRANTIES. ANY WARRANTY THAT MIGHT OTHERWISE BE IMPLIED BY LAW INCLUDING, BUT NOT LIMITED TO ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ARE LIMITED STRICTLY TO THE APPLICABLE LENGTH OF THIS WARRANTY. COCHE MOBILITY SOLUTIONS RESERVES THE EXPRESS RIGHT TO VARY ALL WARRANTY TERMS AND CONDITIONS AT ANY TIME. ANY VARIATIONS ARE MADE AT COCHE MOBILITY SOLUTIONS SOLE DISCRETION. PLEASE CHECK OUR WEBSITE [maclarenmajorelite.com](http://maclarenmajorelite.com) FOR THE MOST UP TO DATE WARRANTY TERMS AND CONDITIONS

This Warranty applies to 2026 and newer Maclaren Major Elite qualifying products. Please consult your owner's manual or contact us at [maclarenmajorelite.com](http://maclarenmajorelite.com) for applicable Warranty Terms and Conditions.

Maclaren Major Elite is offered by Coche Mobility Solutions, Inc January 2026

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**MACLAREN®** is a registered trademark of L'Inglesina Baby S.p.A.