# Swifty

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## 2. Preface / Manufacturer

### Dear Customer,

Thank you for choosing our product and for the trust you have placed in us. In order for you and the user to enjoy your new product safely, conveniently, and comfortably, we kindly ask that you read the Instructions for Use. Doing this will also help avoid damage, injury, and incorrect operation and handling of the product. Please ensure that a copy of the Instructions for Use remains with the product so that every user has access to it.

If you have any questions, please contact your authorised distributor or us directly.

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We reserve the right to make technical changes. This can lead to differences between your product and how it is presented in brochures and other media.

---

## Table of Contents

1. Figures . . . . .	2
2. Preface / Manufacturer . . . . .	22
3. Labels / Model Identification . . . . .	23
4. General Remarks / Safety Instructions . . . . .	24
5. Intended Purpose . . . . .	25
5.1 Intended Use / Application Areas . . . . .	25
5.2 Indication / Patient Target Group / Intended Users. . . . .	25
5.3 Risks Associated with Use and Contraindications . . . . .	25
6. Product Description . . . . .	25
6.1 Scope of Delivery . . . . .	25
6.2 Standard Equipment . . . . .	25
6.3 Putting into Service. . . . .	25
6.4 Teaching the User . . . . .	26
7. Adjustment Options / Features . . . . .	26
8. Accessories . . . . .	28
9. Technical Data . . . . .	31
10. Material Specifications . . . . .	31
11. Cleaning, Maintenance, and Repair. . . . .	32
11.1 Cleaning, Care Instructions, and Disinfection on a Daily Basis . . . . .	32
11.2 Maintenance and inspection by the distributor / Maintenance Schedule. . . . .	32
11.3 Spare Parts . . . . .	33
11.4 Storage . . . . .	33
11.5 Disposal . . . . .	33
12. EU Declaration of Conformity . . . . .	33
13. Warranty . . . . .	34
14. Period of Use . . . . .	34
15. Service Life . . . . .	34
16. Reuse. . . . .	34
17. Distributor . . . . .	35

### 3. Labels / Model Identification

Label	Meaning
<p> <b>Thomas Hilfen für Körperbehinderte GmbH &amp; Co. Medico KG</b>                  Walkmühlenstraße 1 - 27432 Bremervörde - Germany  <b>Swifty Reha-Buggy</b>                  Reha-Buggys  <b>6806/5000</b>                  9800068065000                  MD CE 2021-04-22                  UDI-DI (01)04034089082202                  UDI-PI (21)1931150                  SN 1931150             </p>	<p><b>Barcode Label</b></p> <p>Position on the Product (Fig. 1)</p> <ul style="list-style-type: none"> <li>- Manufacturer</li> <li>- Product Name</li> <li>- Product Group</li> <li>- Item Number short / long</li> <li>- MD = Medical Device</li> <li>- Manufacture Date</li> <li>- Serial Number</li> </ul>
<p> <b>Thomashilfen</b>                  Walkmühlenstraße 1                  27432 Bremervörde                  Germany  <hr/> <b>Swifty</b>  <hr/>                 max. Nutzergewicht /                  max. user weight:  <b>35 kg / 77.2 lbs</b>  <hr/>                 CE  </p>	<p><b>Type plate</b></p> <p>Position on the Product (Fig. 2)</p> <ul style="list-style-type: none"> <li>- Manufacturer</li> <li>- Product Name</li> <li>- Max. user weight</li> <li>- CE marking</li> <li>- Follow Instructions for Use</li> </ul>
	<p><b>Securement Points</b></p> <p>Position on the Product (Fig. 3)</p> <p>Labelling of the 4 securement points on the product where the tiedown straps of the disability vehicle are to be secured.</p>

## 4. Remarks

### 4.1 General Remarks

- Read the Instructions for Use carefully before first use. If you have difficulty reading, please have them read to you. If the copy of the Instructions for Use is missing, you can download one from [www.thomashilfen.de](http://www.thomashilfen.de). The contents of the digital Instructions for Use can be enlarged.
- Check the packaging for any transport damage and to be sure the product is in flawless condition.
- Changes to the product are only permitted within the scope of the intended adjustment options.
- Cleaning and daily care can be provided by relatives and caregivers. Maintenance and inspection are done by the distributor.
- Clean the frame of the product as needed.
- Always wipe the product dry if it gets wet. Never put away wet after use.
- All of the screws to the product have to be warmed prior to removal because they are fastened with a thread locker. If the product is to be put back together again, all of the screws have to be secured with a thread locker.
- All labels have to remain on the product and should not be tampered with.

### 4.2 Safety Instructions

- Check at least every 3 months that the settings on the product still work for the user's current body measurements. If need be, have a distributor / orthopaedic technician adjust the settings.
- Do regular (weekly) checks to make sure that the setting options on the product work.
- Be careful of the moving parts when adjusting or folding the product.
- Only lift up the product by its firmly welded or screwed parts.
- Before each use check that all the nuts, bolts, and screws are tight.
- Make sure that both Allen screws are on the footrest.
- Please do not overload the product and note the maximum user weight and the maximum load capacity (see technical data).
- Never leave the user unattended with the product.
- The product should only be used by one user at a time. No other individuals should be placed in or on the product.
- All users have to be familiar with how the product works. This way in an emergency the user can be removed quickly from the product.
- Do not leave the user unattended while they are getting into or out of the product. If there is too much pressure put on the footrest, there is a risk of slipping or tipping.
- Always secure the user in the product with a restraining belt (standard equipment or accessory).
- Do not hang heavy bags or other objects on the product because this increases the risk of tipping. Use the basket (accessory) under the seat unit.
- Always make sure the brake / parking brake is working before using the product. If the brake isn't working right, the product shouldn't be used.
- The parking brake always has to be engaged when putting the user into or taking the user out of the product.
- When stopping briefly or for a longer time, use the brake so the product doesn't accidentally roll away.
- Wet tyres can have a negative impact on braking.
- If you are not using the product, don't leave the brake engaged. Leaving the parking brake constantly engaged can damage the tyres.
- Intense sunlight and heat can lead to high surface temperatures on the product. As a result, there is a risk of injury when touching the product. Only use the product once it has cooled down enough. In general, try to protect the product from direct sunlight.
- Wearing reflective clothing, particularly when it is dark out, makes you and your child more visible for other motorists – use this advantage.
- Please note the safety standards which apply to your vehicle as outlined in the vehicle's Instructions for Use to ensure the safety of the user.
- If there is a serious and unexpected accident with the product, please inform our company immediately and the responsible authorities.

## 5. Intended Purpose

### 5.1 Intended Use / Application Areas

The Swifty rehab stroller is a class 1 medical device.

The product can be used at a walking pace indoors and outdoors. The rehab stroller is a reliable mobility device that is designed for regular use. Easy adjustment options make transportation safe and secure over short distances. In addition, the product has a wide range of accessories for customised adjustments.

The rehab stroller is approved for transporting children in vehicles / disability vehicles in accordance with ISO 7176-19 and ANSI/RESNA WC/Vol. 1 – Section 19 .

Any use that goes above and beyond this does not comply with the intended use.

### 5.2 Indication / Patient Target Group / Intended Users

The product is suitable for transporting children with significant to total mobility impairment and functional injury to the lower extremities. Furthermore, the rehab stroller is for children with structural and / or functional limitations affecting the head and torso.

### 5.3 Risks Associated with Use and Contraindications

If used properly, risks can be eliminated during use. There are no known contraindications.

## 6. Product Description

### 6.1 Scope of Delivery

Your product is delivered with the standard equipment and comes preassembled. Any accessories ordered along with the product are included and have to be assembled subsequently. A copy of the Instructions for Use is enclosed with the product.

The Allen key which comes with the product is needed to adjust and tighten the footrest.

### 6.2 Standard Equipment

foldable chassis with integrated seat unit; seat and back pads; seat side guide pads; 5-point harness; fire retardant fabrics; breathable upholstery; hip-angle adjustable backrest; continuously adjustable seat depth; swivel front wheels with wheel lock; noflat EVA-wheels; height adjustable and tiltable footrest; foot brake; height adjustable push bar; crash eyes; bracket for sun and rain cover.

### 6.3 Putting into Service

- Carefully remove the transport protection and the packaging.
- Keep the packaging material and small parts out of the reach of children to prevent swallowing or choking.
- The product is delivered folded. Note the remarks for folding found in the section on adjustment options.

### 6.4 Teaching the User

The user is taught how to use this device by trained staff who will reference these Instructions for Use.

## 7. Adjustment Options / Features

The settings / adjustments must always be made by trained / instructed professionals (such as medical product consultants of specialist medical dealers).

### 7.1 Folding

Note: Be careful not to drop the rehab stroller onto the floor during folding to avoid bending and damaging the frame. Make sure the plastic plug of the folding mechanism is in the correct position (higher end of the plug is pointing to the rear of the rehab stroller). (Fig. 4)

Caution: The rehab stroller must not be folded up while the sun and rain cover is mounted.

### 7.2 Folding open

Hold the buggy on the top and on the side using the push bar and pull it upwards with a swinging movement (the locking slide audibly engages). Loosen the clamping lever at the back and adjust the desired back angle (see the section on backrest adjustment). (Fig. 5) Adjust the height-adjustable push bar to the desired position by pressing the side pushbuttons (Fig. 6).

### 7.3 Folding closed (smallest folded size)

Release the clamping lever on the back and push the backrest bar upwards in the rail. Pull up the locking slides on the push bar (right and left). (Fig. 4) and hold onto the safety bow mounted on the right-hand side. Using your upper body, push the push bar forward and fold the buggy closed in a forward direction (Fig. 7). To fold completely, take hold of the backrest bar and push it upwards again in the rail. (Fig. 8) Then tighten the clamping lever on the backrest again. (Fig. 5) Fold down the push bar on the backrest.

Now the rehab stroller can be securely gripped and transported using the tubes of the frame.

### 7.4 Parking brake function

To lock the brake, place your foot on the brake bow and push it down. Release the brake by placing your foot under the brake bow and pulling it upwards. (Fig. 8)

### 7.5 Locking function for the front wheels

Tightening: Turn the front wheel so that it is straight and let the locking latch bolt snap in (it clicks audibly).

Releasing: Pull out the locking latch bolt and lock it with a 90° rotation. (Fig. 9)

### 7.6 Backrest adjustment

Loosen the clamping lever on the back, adjust the desired hip angle by sliding the back bow on the back rail, and re-tighten the lever. (Fig. 5)

### 7.7 Seat depth

Loosen the two star grips beneath the seat and pull out the seat section towards you. The seat upholstery does not need to be removed to adjust the seat depth. Once you have made the adjustment re-tighten the star grips. (Fig. 15)

Tip: To make setting the seat depth easier, grasp the front edge of the seat surface with one hand while pressing your other hand against the bottom centre of the backrest. (Fig. 11)

### 7.8 Lower leg length

Unscrew the Allen keys (3 mm) on the right and left rear of the footrest tube guides. Move the footrest to the desired position by sliding it on the frame tube and tighten the screws again. (Fig. 12)

### 7.9 Footrest

The footrest can be folded upwards to make it easier for children to get into the buggy. (Fig. 13)

### 7.10 Seat and back pads

The seat and back pads consist of a single unit. It is pulled over the backrest in such a manner that the red logo is at the front and the lower part of the pad comes to rest on the seat. The rear of the pad is divided into two parts and is closed with a hook and loop fastener. Guide the two halves of the pad's back under the rehab stroller's rear bow (Fig. 15). Pull the pad into place and close the hook and loop fastener. (Fig. 16). Finally, close the hook and loop fastener at the lower end of the back (Fig. 17).

### 7.11 Lateral seat guide padding

The lateral seat guide padding covers the seat's metal side pieces and protects the user. (Fig. 18)

### 7.12 5-point belt

The 5-point belt is a simple positioning aid for the upper body, especially in the shoulder and pelvic area. (Fig. 19) The shoulder and lap belts are attached to the rear of the back plate. The belt length can also be adjusted there. The belt length can also be adjusted at the front when the child/user is sitting in the rehab buggy. To do this, pull the D-rings at the front to shorten the belt.

#### 7.12.1 Opening the belt buckle

To open the 5-point belt, press the red push button, open the clip fastener and release the harness (child safety lock).

#### 7.12.2 Adjustment of the shoulder strap height in the padding

Open the hook and loop fastener on the back pad. Open the upper folding buckles and pull the belts out from the front. Undo the back pad to the desired height using the stitched seam. Pull the straps through the resulting slots and secure them again in the folding buckles. Close the pad again. (Figs. 20 + 21)

### 7.12.3 Remove the 5-point belt

Open the Velcro fastener on the back padding at the back and open the folding buckles. Undo the knurled screw on the crotch strap at the front. Now pull out the 5-point belt from the front. (Fig. 22)

### 7.13 Sun and rain cover holders

The rehab stroller is equipped with holders for a sun and rain cover as standard. (Fig. 23) The sun and rain cover itself is available as an accessory.

### 7.14 Transport

The rehab stroller can be folded up for transport in a vehicle trunk. Heed the instructions for folding the rehab stroller.

### 7.15 Use as a seat in a motor vehicle

The rehab stroller has successfully passed the crash test according to the international ISO 7176-19 standard as well as ANSI/RESNA WC/Vol. 1 – Section 19.

It can therefore be used as a seat for children weighing 12 kg or more in a motor Vehicle.

However, Thomashilfen highly recommends that the transport of children with a body mass of less than 22 kg transfer to a Thomas Hilfen für Koerperbehinderte GmbH & Co. Medico KG Child Restraint System (sometimes referred to as an infant safety seat, a child restraint system, a child car seat, or ambiguously as car seats) in compliance with UNECE Regulation 44/04 / R129/03 to provide the best possible protection for the child.

If transfer to a Child Restraint System is not possible, due to the needs of the occupant e. g. a need for continual postural support, occupant discomfort, anxiety or a dependency of the medical device, then the decision to carry the occupant in their wheelchair or buggy should involve an individual risk assessment process. This risk assessment process should involve a therapist / competent risk assessor who would be able to support reasons not to transfer the occupant to a Child Restraint System.

Important: If the rehab stroller is to be used as a seat in a motor vehicle, the following instructions are to be heeded.

The rehab stroller has been designed for transport in the forward direction. Whenever possible, the user should be placed on one of the vehicle seats and the installed restraint system should be used. Use only chassis / passenger restraint systems that are in accordance with ISO 10542 to secure the rehab stroller to the floor of the vehicle.

The 4 attachment points on the rehab stroller are indicated by the following symbol:

#### 7.15.1 Preparing the rehab stroller for transport:

1. Remove the following parts (if attached); abduction block, tray, sun and rain cover, sun umbrella, grip rail, oxygen cylinder and the contents of the basket.
2. Use the installed rails to correctly position the rehab stroller in the forward direction in the vehicle. Actuate the parking brake.
3. Make sure that the back angle adjuster is fixed.

#### 7.15.2 Fastening the rehab stroller in a motor vehicle:

1. The rehab stroller must be safely secured in the vehicle using a 4-point belt system (according to ISO 10542-2, SAE 2249 or DIN 75078/2). (The manufacturers of these belt systems are, for example, Unwin Safety Systems und Q'Straint.)
2. The driver and / or the assistant should be familiar with the use of these belt / restraint systems.
3. The two front belts (snap hook) must be attached to the left and right steel rings above the front wheel fork. (Fig. 24)
4. The two rear belts (snap hook) must be attached to the two safety tie-down kits on the rear frame. (Fig. 25)

#### 7.15.3 Applying the restraint system:

1. The crash test was carried out with the H-belt. This H-belt should be used under the belt of the restraint system.
2. The user must be secured with an approved restraint system. Thomashilfen recommends using an automatic belt or a 3-point restraint system, in which the pelvis belt and the shoulder belt can be separated from each other. This is necessary to be able to properly guide the restraint system through the frame of the rehab stroller. (Manufacturers of such systems include Unwin Safety Systems and AMF-Brunns.)

3. The driver and / or the assistant should be familiar with the use of these belt / restraint systems.
4. The user's pelvis belt must be positioned flat on the user's pelvis without contacting the abdomen. The pelvis belt should extend between the side guide and the user on the right and left sides. (Fig. 26)
5. The pelvis belt must not be separated from the user by parts of the rehab stroller (sides, wheels) (Fig. 27)

Note: The belt must not be used when it is twisted.

6. The angle of the pelvis belt should be between 30° and 75°. (Fig. 28)
7. The pelvis belt should be as tight as possible, but should also be comfortable for the user.
8. The upper body belt of the restraint system must be fastened to the vehicle wall at a height so that it is ensured that the belt is located in the middle of the user's shoulder and passes under the push handle. (Fig. 27)

Note: Only a shoulder belt that is fastened to the side wall of the vehicle satisfies the current requirements for safely restraining the user.

## 8. Accessories

Remarks: Installation information is found in the Instructions for Use, which are enclosed with the accessory. If the copy of the Instructions for Use is missing, you can download one from [www.thomashilfen.de](http://www.thomashilfen.de). The assembly / disassembly of the accessories is done with standard tools.

### 8.1 Mounting the belts

Cut the backrest upholstery in the quilted seam depending on the desired mounting height.

Pass the two upper belts through the holes in the upholstery and insert them into the back plate. (Fig. 21).

Open the hook and loop of the back upholstery. Insert the belts into the cam buckles and adjust the desired belt length (Fig. 23).

The two bottom belt ends can be attached with cam buckles to the back plate (Fig. 23 / Circle). Guide the belts left and right next to the back upholstery to the corresponding cam buckle and fix them. Close the back upholstery.

Tip: The belt length can also be adjusted from the front. Pull the D-ring to shorten the belt.

### 8.2 Abduction block (Fig. 29)

The abduction block helps spread the legs at the hip joint and position the thighs. It goes at the knees and prevents the legs from being crossed.

### 8.3 Grip rail (Fig. 30)

The grip rail is affixed by the user and provides extra stability when sitting.

### 8.4 Tray (Fig. 31)

The tray consists of a firm vent tray which is attached in front of the user. On the tray various therapeutic activities can be done.

### 8.5 Lateral trunk supports, rigid (Fig. 32)

The rigid lateral trunk supports are affixed in the position needed on the back section and support the user's trunk on the left and right sides. As a result, control of the trunk is improved and supported. The user can still move their arms.

### 8.6 Lateral trunk supports, flexible (Fig. 33)

The lateral trunk supports are affixed in the position needed on the back plate and support the user to the left and right of the trunk. As a result, control of the trunk is improved and supported. The user can still move their arms. Both trunk supports can be bent slightly forward and are connected with a belt. In this way the user has extra support at the front.

### 8.7 Frame padding (Fig. 34)

The frame padding is put along the lateral side posts of the push bar. They go a long way to protect fidgety users from injuring themselves on the rolling chassis.

### 8.8 Seat minimizer set (Fig. 35)

The seat minimizer set reduces the seat depth, seat width, and the space for the lower leg thereby extending the use period of the seat. It consists of four parts: Head pillow, back pad, pelvic pad, footrest pad

**8.9 Back pad (Fig. 36)**

The back pad with its lateral padding provides additional upper body support for slim users. It is put in the lower area of the back plate on the seat cushion and reduces the lateral width.

**8.10 Pelvic pad (Fig. 37)**

The pelvic pad with its lateral padding provides extra thigh support for slim users. It is affixed to the padding of the seating plate and reduces the seat width.

**8.11 2-point pelvic belt (Fig. 38)**

The 2-point pelvic belt is a simple positioning aid to stabilise the user's pelvic area. It goes horizontally across the user's groin.

**8.12 Butterfly / chest harness (Fig. 39)**

The butterfly / chest harness is a padded positioning aid for the upper body, especially for the shoulders and pelvis. The "flexible" model is made of very soft material to prevent pressure marks.

**8.13 Seat waistcoat (Fig. 40)**

The seat waistcoat is a padded positioning aid for the upper body offering extra support for the user due to its large surface. It can be opened easily at the front.

**8.14 Soft pelvic harness (Fig. 41)**

The soft pelvic harness is a padded positioning aid for the pelvic area. It also supports the positioning of the thighs and prevents the legs from being crossed.

**8.15 Head pillow (Fig. 42)**

The head pillow gives the user simple lateral head support. The height can be adjusted.

**8.16 Foot straps (Fig. 43)**

Both foot straps position the user's feet directly on the foot plate of the footrest. The feet are no longer able to slip.

**8.17 Footrest pad (Fig. 44)**

The footrest pad is a solid pad which is affixed directly to the footplate. It also shortens the leg space if the product's shortest setting is still too long for the user.

**8.18 Sleeping bag / woven fur (Fig. 45)**

The sleeping bag extends from the seat to the footrest. The padded model protects the user in cool weather. The woven fur model protects the user in cold weather.

**8.19 Rain cover, transparent, with leg blanket (Fig. 46)**

The clear rain cover with leg blanket protects the user and the product in wind and rain. The user has a clear view of the surroundings.

**8.20 Sun / rain cover (Fig. 47)**

The sun / rain cover fully protects the user from direct sun rays, glare, wind, and overstimulation from sights and sounds in the environment. The caregiver can, if needed, watch the user through a flap.

**8.21 Sun / rain cover with leg blanket (Fig. 48)**

The sun / rain cover fully protects the user fully from direct sun rays, glare, wind, and overstimulation from sights and sounds in the environment. The caregiver can, if needed, watch the user through a flap. The leg blanket also protects the user's legs and feet.

**8.22 Rain shield (Fig. 49)**

The transparent rain shield fully protects the user and the product in wet and rainy conditions. The rain shield is a transparent tarp that can only be used in combination with a sun and rain cover.

**8.23 Umbrella (Fig. 50)**

The umbrella is affixed to the left or right of the rolling chassis. It protects the user from direct sun rays (incl. UV protection) and glare.

**8.24 Tip assist (Fig. 51)**

The tip assist is fastened on the left or right of the back axle. It makes it easier for caregivers to manoeuvre over low obstacles such as kerbs and thresholds.

**8.25 Basket (Fig. 52)**

The basket is positioned under the rolling chassis. It is used to carry important parts needed when on the go (please note the maximum load capacity).

**8.26 Pneumatic wheels (Fig. 53)**

The pneumatic wheels reduce jarring on uneven ground. Pneumatic wheels can be used instead of standard wheels.

**8.27 Oxygen tank holder (Fig. 54)**

The oxygen tank holder is designed to hold oxygen cylinders and can be attached to the right or left of the frame.

**9. Technical Data****Swifty**

Seat depth (with seat minimizer (accessory))	22 (17 cm) - 28,5 cm / 8.7 (6.7") - 11.2"
Seat width (with seat minimizer (accessory))	34 cm (23 cm) / 13.4" (9.1")
Back height	62 cm / 24.4"
min. shoulder height (with seat minimizer)	37 cm / 14.6" (26 cm / 10.2")
Lower leg length (with seat minimizer (accessory))	16 (12 cm) - 33 cm / 6.3 (4.7") - 13"
Back angle	90° to 115°
Foot angle	90°
Seat tilt fix	+15°
Footrest size (W x L)	31 x 18 cm / 12.2 x 7.1"
Overall dimensions (W x H x L)	61 x 102 x 98 cm / 24.0 x 40.2 x 38.6"
Dimensions when folded (W x H x L)	61 x 38 x 73 cm / 24.0 x 15.0 x 28.7"
Push bar height	80 - 119 cm / 31.5 - 46.9"
Wheel size, swivel wheels (front)	7,5"
Wheel size (rear)	10"
max. carrying capacity basket (accessory)	5 kg / 11.1 lbs
Weight	12.4 kg / 27.3 lbs
max. user weight	35 kg / 77.2 lbs

**10. Material Specifications**

Frame: powder-coated aluminium

Padding: 100 % Polyester

The material used for the padding is "flame retardant" and was tested in accordance with DIN EN 1021-1.

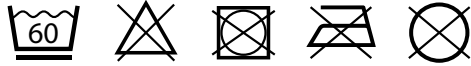
The material and components underwent standard testing to determine the extent of their resistance to inflammation and were assessed as being very unlikely to catch fire.

## 11. Cleaning, Maintenance, and Repair

### 11.1 Cleaning, Care Instructions, and Disinfection on a Daily Basis

#### 11.1.1 Padding

The padding can be easily removed in a few simple steps and washed separately. The padding consists of a fabric cover and sometimes fillers made of foam. Check if the padding has a zipper or a hook and latch fastener. If it does, open the padding and remove the foam filler before washing the cover. Please also note the following care instructions.

We would like to point out that even high-quality materials can fade as a result of long and intense exposure to sunlight or frequent washing.

When changing users, a “padding set for reuse” is offered by the manufacturer.

#### 11.1.2 Hook and Loop Fasteners

To keep the hook and loop fasteners working well, clean them now and then with a brush. Always close or cover the hook and loop fasteners before washing.

#### 11.1.3 Plastic Parts

The plastic parts on the product can be cleaned and disinfected with standard soap (neutral soap).

#### 11.1.4 Metal Parts

The metal parts on the product can be cleaned and disinfected with standard soap (neutral soap).

#### 11.1.5 Disinfection

Clean the padding, plastic parts, and metal parts before disinfecting.

Damp wipe all the plastic and metal parts on the product with disinfectant. Only water-based products should be used. When disinfecting note the directions from the manufacturer in question.

The padding and the covers have to be disinfected before reusing. To do so use the right detergent in keeping with the guidelines of the Association of Applied Hygiene VAH (Verbund für Angewandte Hygiene e.V.) or use a “padding set for reuse” offered by the manufacturer.

## 11.2 Maintenance and inspection by the distributor

### 11.2.1 Preparation

A copy of the Instructions for Use has to be enclosed with the product. If one is not available, request a copy from the distributor. Familiarise yourself with how the product works. If you are not familiar with the product, read the Instructions for Use before testing it. Clean the product before starting to test it. Please note the care instructions in the Instructions for Use and product-specific test instructions. If you have further questions, please contact the staff member you dealt with or Thomashilfen directly.

Service and repairs to the product should only be done by qualified staff. It is the user’s responsibility to ensure that the prescribed inspections / maintenance are done regularly and in a timely fashion. If damage is apparent, the user has to actively inform and have the damage fixed immediately by qualified staff.

### 11.2.2 Maintenance Schedule – Annual Inspection

	Area	Function	Damage	Replace
1	Seat	👁 / 🔄	👁 / 🔄	
2	Footrest	👁 / 🔄	👁 / 🔄	
3	Rolling chassis	👁 / 🔄	👁 / 🔄	
4	Push bar	👁 / 🔄	👁 / 🔄	
5	Angle adjustments	👁 / 🔄	👁 / 🔄	
6	Padding		👁	🔄
7	Belts	👁 / 🔄	👁 / 🔄	
8	Plastic parts	👁 / 🔄	👁 / 🔄	
9	Screws and nuts	👁 / 🔄	👁 / 🔄	
10	Tyres	👁 / 🔄	👁 / 🔄	
11	Brake	👁 / 🔄	👁 / 🔄	
12	Labels		👁 / 🔄	
<b>Accessory</b>				
13	Padding		👁	🔄
14	Belts	👁 / 🔄	👁 / 🔄	
15	Adjustment options	👁 / 🔄	👁 / 🔄	
16	Plastic parts	👁 / 🔄	👁 / 🔄	
17	Metal parts	👁 / 🔄	👁 / 🔄	
18	Screws and nuts	👁 / 🔄	👁 / 🔄	
<b>done by:</b>			<b>on:</b>	
👁 = Inspection   🔄 = Reuse   ✓ = Test completed				

### 11.3 Spare Parts

Only original spare parts should be used for repairs. A list of the spare parts is available at [www.thomashilfen.de](http://www.thomashilfen.de). To receive the right spare parts or for possible technical recalls, we normally require the serial number / UDI-PI and the exact UDI-DI.

Removal / installation of spare parts should only be done by qualified staff! In the event of returns, please make sure the product is securely packaged for transportation.

### 11.4 Storage

Store the product in a safe place when it is not being used. Do not lay or put any heavy objects on the product while it is being stored. Protect the product from weather like rain, intense sunlight, and frost. If the product is wet, wipe the metal and plastic parts dry and let the padding dry before storing the product.

### 11.5 Disposal

If the product is no longer usable due to its condition or it has reached the end of its service life, it can be recycled at the local waste disposal company. Please refer to the local guidelines for recycling and disposal.

## 12. EU Declaration of Conformity

We hereby declare as the party solely responsible that this product and any versions and accessories comply with the basic regulations. The current EU Declaration of Conformity can be downloaded at [www.thomashilfen.de](http://www.thomashilfen.de) or requested from us directly.

### 13. Warranty

Thomashilfen offers you the two-year warranty as required by law for this product. The warranty period starts when the item is handed over. The warranty includes all claims which impact function. Not included is damage which is the result of improper use (e.g. overloading) and natural wear and tear.

### 14. Period of Use

The product has a period of use of up to 6 years if used correctly and serviced regularly by a professional as per the information from the manufacturer. The product can be used beyond this point if it is in safe and technically sound condition. Time spent in storage at the distributor does not count towards the period of use.

The theoretical period of use does not equate to a guaranteed service life. It depends on the intensity of use and always undergoes an individual assessment by the distributor.

The period of use does not apply to wear parts like padding, covers, tyres, brake pads, etc.

### 15. Service Life

The product has a service life of 8 years or 2 reuses (3 users) if the requirements stipulated for the period of use are met. The service life consists of the period of use and the storage times between reuses.

### 16. Reuse

The product is suitable to be reused. Before being passed on to the next user the product always has to be carefully cleaned and disinfected. The condition has to be checked by trained staff for any wear and damage. Any issues have to be remedied to ensure the continued safe use of the product. A copy of the Instructions for Use is a component of the product and has to be passed on to the new user.

### 17. Distributor

#### Astris PME Pty Ltd

Young & Canberra Office:  
Astris PME Pty Ltd  
5 McVeigh St  
Young NSW 2594

Sydney Office:  
Astris PME Pty Ltd  
30 Prime Dr  
Seven Hills NSW 2147

Melbourne Office:  
Astris PME Pty Ltd  
9 Katherine Dr  
Ravenhall VIC 3023

Phone: +61 1300 131 884  
info@astris-pme.com.au  
www.astris-pme.com.au

#### Lighthouse Medical

#3 - Golden drive  
Coquitlam, British Columbia  
V3K6R2  
Customer Service: 886.609.6827  
orders@lighthousemedicaltd.com  
www.lighthousemedicaltd.com

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##### Tendercare Ltd.
Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG