



QTY	#	DESCRIPTION
1	1	"L" Frame
1	2	Lock Collar
4	3,5,8,9	Lock Knob
1	4	Cross Brace
1	6	Basket
1	7	Velcro Strip

## assembly instructions

1. Hold "L" frame (1) in an up-right position.
2. Slide lock collar (2) over "L" frame.
3. Insert lock knob (3) into lock collar and turn clockwise until snug. Do not over-tighten.
4. Slide cross brace (4) into lock collar.
5. Insert lock knob (5) into cross brace and turn clockwise until snug. Do not over-tighten.
6. Hang basket (6) on cross brace.
7. Secure basket with Velcro® strip (7).
8. Slide bottom of "L" frame (1) into back of seat post on scooter.
9. Insert lock knob (8, 9) on both sides of "L" frame and turn clockwise until tight.
10. Secure all lock knobs to ensure tight fit.