


The Caregivers' Cushioned Transfer Belt



The ErgoBelt Plus™ is the only cushioned transfer belt designed by caregivers for caregivers. The ErgoBelt Plus™ is ideal for assisting a patient from a sitting position to a standing position or for walking exercises. Containing thick additional padding for added comfort to the patient, the belt promotes proper ergonomics to help reduce injury to the caregiver. The sturdy nylon material and construction with reinforced handle straps makes the **ErgoSafe™** ErgoBelt Plus™ the perfect solution for manual patient handling.

Fitting The ErgoBelt Plus™

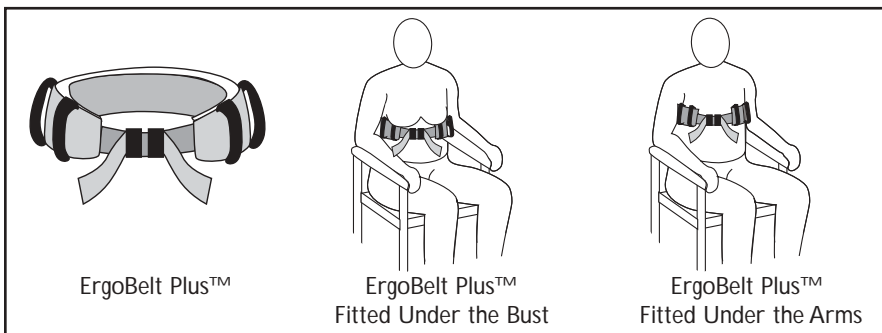

It is important to always explain to the patient your plan to transfer and how the belt will assist. Fasten the buckle to fit snugly around the patient's torso. The belt can be used around the upper body, just under the arms or around the waist. Ensure that the patient's garments are not causing discomfort where the belt is applied and be aware that clothing may ride up if the belt is not properly fitted.

Important Prior To Usage

This brochure is only a guide, as alternative methods may be used for patient handling. We strongly recommend that your facility administrator or risk manager review and approve all protocols used for safe patient handling. Prior to using any equipment, be sure to check for signs of wear to ensure proper functionality. When using the ErgoBelt Plus™ for the first time, practice with a fellow caregiver until you have a level of comfort with the product.

Cleaning Instructions


The ErgoBelt Plus™ can be cleaned with a disinfectant. It can also be hand or machine washed up to 80°C. Do not tumble dry or wring. Air dry only. Inspect frequently for signs of wear. Withdraw from use if any faults are apparent and replace immediately.

Correct Fitting Of Buckle

For proper buckling, be sure to first check that the straps are not twisted. Slide the buckle clip (A) into the buckle lock (B) until securely fastened. To remove the belt, simply press the sides of the buckle clip (A) together and slide outward.

Both parts Of The Buckle A" Slightly Curved And Should Only Be Fitted Together One Way

Smooth Top (Printing is underneath)

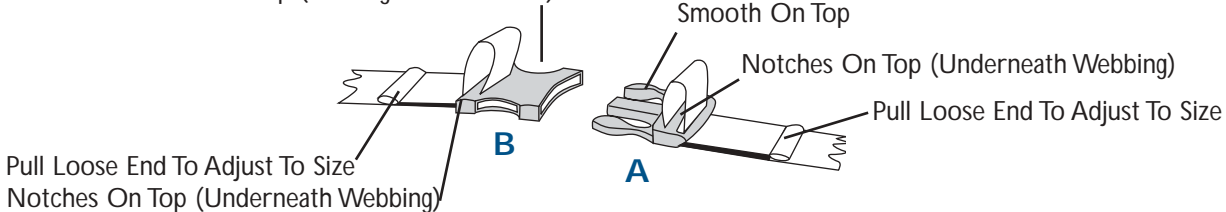

Buckle should secure without force and undo easily by squeezing the two side prongs in the middle of the buckle.

Visit us online at www.ergosafeusa.com to see our full line of ergonomically designed products.

Item No ES-B300