

FULL BACK SLING




FITTING INSTRUCTIONS
TECHNICAL SPECIFICATIONS
WASHING INSTRUCTIONS



USER GUIDE FOR THE FULL BACK SLING

The Full Back Sling is an easy fit, contoured sling designed to fit 85-90% of clients. It is designed for use as the traditional Quickfit Sling but offers greater comfort and negates the need for side suspenders. It has integrated, boned head support and leg padding as standard. The sling has five sizes - extra large, large medium, small and paediatric.

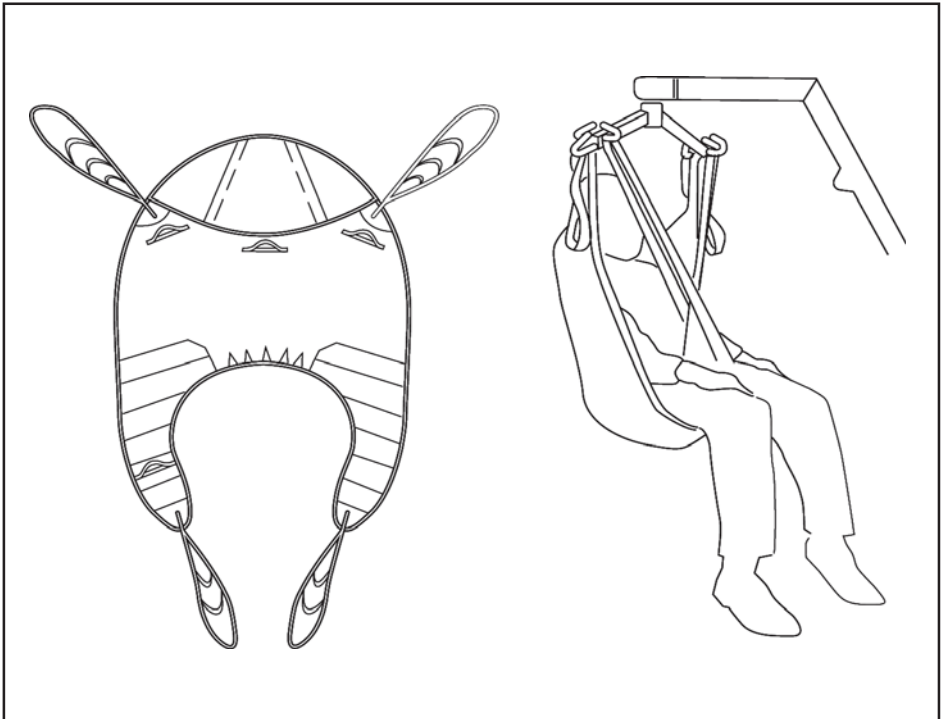


ADDITIONAL SLINGS ARE AVAILABLE TO SUIT INDIVIDUAL NEEDS.
YOU ARE ADVISED THAT YOU SHOULD ALWAYS SEEK THE ADVICE OF AN
AUTHORISED DEALER BEFORE PURCHASING.


WE WILL BE PLEASED TO ADVISE YOU AS TO YOUR NEAREST AUTHORISED DEALER.




HOW TO FIT FROM SEATED POSITION

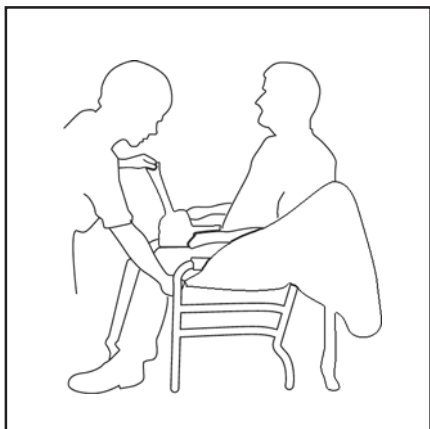

Raise the client's leg and feed the padded leg piece under and up between the legs. Ensure that the sling is not twisted or creased under the thigh. Repeat this procedure for the other leg.

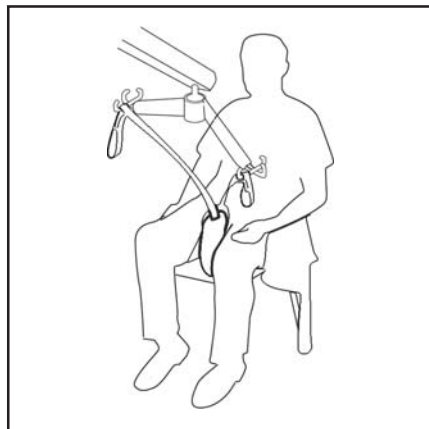

Feed the right leg support through the strap on the left leg support. Cross over leg straps, pass one leg strap through the other and attach to hoist on front hooks.

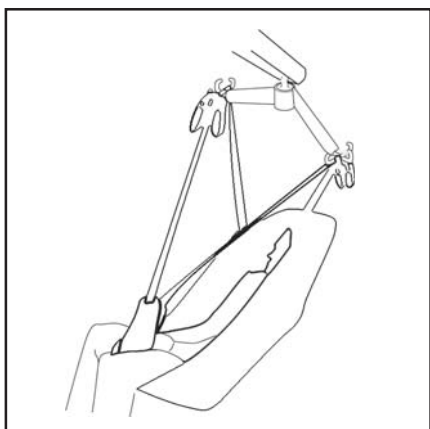


Raise lift to convenient height and attach the shoulder straps as shown. You may then raise the client to the required height. When seating the client the hand grip on the back of the sling may be used to ensure a good seated posture.

GOLDEN TIPS


- 1. THE CLIENT'S POSTURE CAN BE ALTERED BY ADJUSTING BOTH SHOULDER STRAPS.**
- 2. IF IN DOUBT, ATTACH TO COLOURED LOOPS.**
- 3. KEEP SHOULDER STRAPS AT EQUAL LENGTH.**

HOW TO FIT SLING FROM LYING POSITION

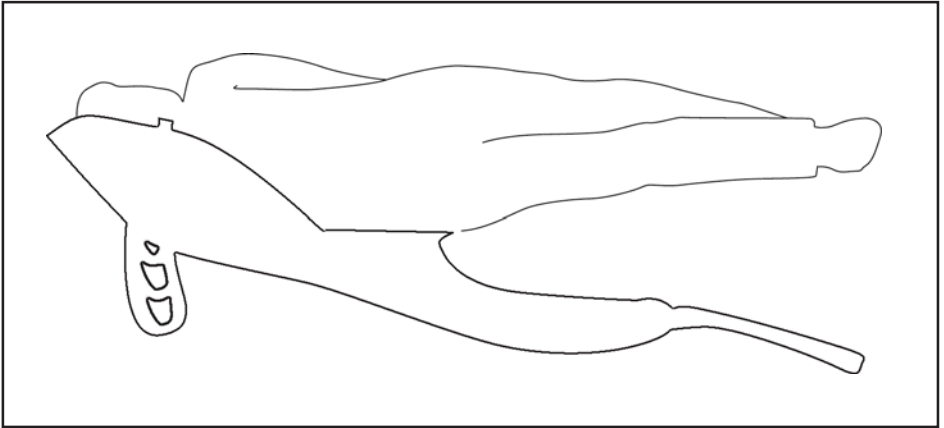

Draw sheet roll the client onto the sling, ensuring that the top of the commode aperture is at the base of the spine.

Bring the leg support straps up and between the client's legs and proceed as from the seated position, attaching to the shortest possible loops. N.B. If you wish to place the client onto a high bed it may be necessary to lower the patient onto an intermediate surface and adjust the strap length.

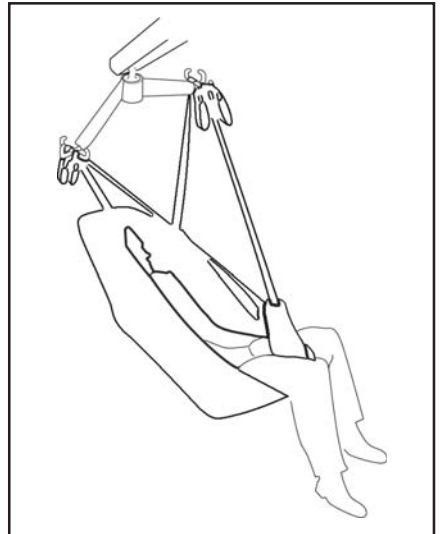


FULL BACK SLING TECHNICAL SPECIFICATIONS

STANDARD	POLYESTER BS 5827
CERTIFICATE OF CONFORMITY	QUAD 151 ISSUE 1
SAFE WORKING LOAD	460lbs / 210kg

SIZING

Each sling is marked with the following colour coded symbols, four of which will be crossed. Your sling is the uncrossed size, i.e. in this example Small (Red).

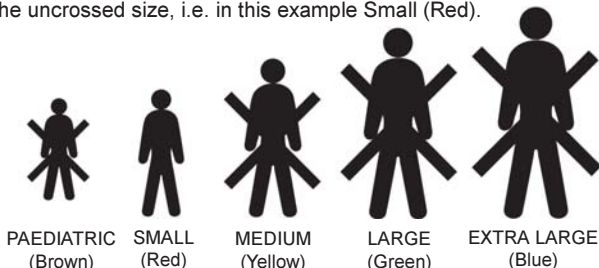

WASHING INSTRUCTIONS

Each sling is marked with the following washing instructions:

DO NOT WASH WITH BLEACH

Bleach will damage material. Slings can suffer damage during washing and drying. Check sling before each use. Bleached, torn, cut, frayed or broken slings are unsafe and could result in serious injury or death to patient. Destroy and discard worn slings. Do not alter slings. Machine wash warm. Cool tumble dry, air dry or dry at very low temperature.



		US	UK	
ORDER CODE	PAEDIATRIC (BROWN)	NA	SL	1067 (Polyester)
	SMALL (RED)	NA	SL	1068 (Polyester)
	MEDIUM (YELLOW)	NA	SL	1069 (Polyester)
	LARGE (GREEN)	NA	SL	1070 (Polyester)
	EXTRA LARGE (BLUE)	NA	SL	1071 (Polyester)



Sunrise Medical Inc.
 7477 East Dry Creek Parkway
 Longmont CO 80503. USA
 Tel: (303) 218 4600
 (800) 333 4000
www.sunrisemedical.com



Sunrise Medical Limited
 High Street Wollaston
 West Midlands DY8 4PS England
 Tel: 01384 44 66 88
 Fax: 01384 44 66 99
www.sunrisemedical.com