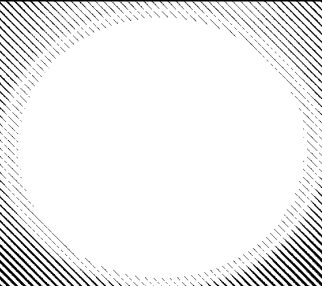
GUARDIAN®

Guardian® Toilet Safety Frame
User Instructions & Warranty
Model 30300

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www.sunrisemedical.com

IMPORTANT SAFEGUARDS

Read all instructions before using. Important information is highlighted by these terms:

WARNING—Important safety information for hazards that might cause serious injury.

CAUTION— Information for preventing damage to the product.

NOTE— Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

Please follow these instructions carefully whether commode is used bedside or over the toilet. Commode's height is adjustable for use in an elevated position. Back of commode is removable for use over the toilet.

⚠ WARNING

Please read carefully before use. Failure to comply may result in severe injury.

⚠ WARNING

Do not exceed the 250 lb. maximum weight capacity of this safety frame.

⚠ WARNING

Ensure all detent buttons are properly engaged in position prior to use.

FAILURE TO FOLLOW THESE INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

SAVE THESE INSTRUCTIONS.

ASSEMBLY INSTRUCTIONS

Before beginning assembly, identify all parts. If you are missing any parts, contact your retailer.

1. Remove existing toilet seat / lid assembly.
2. Place toilet safety frame bracket between toilet fixture and toilet seat / lid assembly with rounded part of bracket facing back of toilet (**Figure 1**).
3. Replace toilet seat / lid and reinsert mounting bolts through bracket and toilet fixture. Replace and tighten nuts.
4. Insert side frames into bracket. Depress detent button and slide side frame into desired position (**Figure 2**).
5. Adjust footpieces to desired height. Make sure both footpieces are fully engaged and adjusted at equal heights.
6. Make sure mounting hardware is tightened and all detent buttons are fully engaged before use.

Figure 1

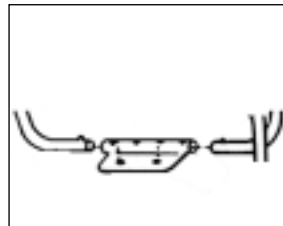
Figure 2

CLEANING

Your safety frame can be easily cleaned with regular household cleaning products. DO NOT use abrasive powders or scouring pads.

Grip Cleaning Instructions

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

NOTE- Tips on this product contain dry natural rubber.

LIFETIME LIMITED WARRANTY

Your Guardian product is warranted to be free of defects in materials and workmanship. This warranty is limited to the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to non-durable components, such as but not limited to rubber accessories and grips which are subject to normal wear and tear and replacement. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

In the event of a defect covered by this warranty, we will at our option repair or replace the product. For warranty service please contact the retailer where the product was purchased. If you have a question about your Guardian brand product or this warranty, please call us toll-free at 800.333.4000.

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Sunrise Medical Inc.