



Hitch N Ride RAMP UP CARRIER

MODELS: HNRU501

⚠ WARNING

READ ALL INSTRUCTIONS BEFORE USING THE RAMP UP SCOOTER CARRIER

NOTE: Failure to read instructions before use could result in injury or death.

Instructions for Installation and Use:



FIG. 1

CONTENTS: SEE FIGURE 1

- 1- SCOOTER CARRIER RAMP ASSEMBLY
- 2- SUPPORT BRACES
- 2- ANGLED RAMP SUPPORTS
- 1- 2" RECEIVER HITCH BAR
- 2- 5/16" X 3" QUICK PINS
- 4- 5/16" EYE BOLTS
- 2- 5/16" X 5" BOLTS
- 4- 5/16" X 2 1/2" BOLTS
- 12- 5/16" USS FLAT WASHERS (LARGER)
- 8- 5/16" SAE FLAT WASHERS (SMALLER)
- 10- 5/16" LOCKNUTS
- 2- SCOOTER SECURING STRAPS

STEP 1-SEE FIGURE 2

ASSEMBLE ANGLED RAMP SUPPORTS TO SCOOTER CARRIER ASSEMBLY USING 4- 5/16" X 2 1/2" BOLTS, 8-5/16" SAE FLAT WASHERS AND 4-5/16" LOCKNUTS.



FIG. 2



FIG. 3

STEP 2-SEE FIGURE 3

RAISE FOLDING RAMP AND PIN INTO UPRIGHT POSITION TO ANGLED RAMP SUPPORTS USING 2-5/16" X 3" QUICK PINS.

STEP 3-SEE FIGURE 4

STAND SCOOTER CARRIER ASSEMBLY ON ITS SIDE FOR INSTALLATION OF RECEIVER BAR AND SUPPORT BRACES.



FIG. 4

STEP 4-SEE FIGURE 5

ATTACH 2" RECEIVER BAR AND SUPPORT BRACES TO SCOOTER CARRIER USING 2-5/16" X 5" BOLTS, 4- 5/16" USS FLAT WASHERS AND 2-5/16" LOCKNUTS.



FIG. 5



FIG. 6

STEP 5-SEE FIGURE 6

ATTACH END OF SUPPORT BARS TO SCOOTER CARRIER USING 4-5/16" EYE BOLTS, 8-5/16" USS FLAT WASHERS, AND 4-5/16" LOCKNUTS.

****NOTE**** WHEN PROPERLY INSTALLED THE SUPPORT BARS WILL HAVE AN ARCH IN THE CENTER WHERE ATTACHED TO THE 2" RECEIVER BAR.

STEP 6-SEE FIGURES 7 & 7A

INSERT THE RECEIVER BAR OF THE SCOOTER CARRIER INTO RECEIVER HITCH OF VEHICLE AND SECURE WITH RECEIVER HITCH PIN.



FIG. 7



FIG. 7A



PROPERLY INSTALLED SCOOTER CARRIER

STEP 7-SEE FIGURE 8

TO LOAD SCOOTER ONTO CARRIER, REMOVE 2-5/16" X 3" QUICK PINS AND EXTEND RAMP TO GROUND.



FIG. 8



STEP 8-SEE FIGURE 9

LOAD SCOOTER ONTO RAMP.
****NOTE**** DO NOT RIDE SCOOTER ONTO CARRIER. IT MUST BE WALKED ONTO THE CARRIER.

STEP 9-SEE FIGURE 10

ONCE SCOOTER IS LOADED ONTO CARRIER, RAISE RAMP AND RE-PIN INTO UPRIGHT POSITION USING 2-5/16" X 3" QUICK PINS.



FIG. 10

STEP 10-SEE FIGURE 11

SECURE SCOOTER TO CARRIER USING 2 SECURING STRAPS PROVIDED.

SEE FIGURE 12

****NOTE**** WHEN SCOOTER IS NOT LOADED ONTO CARRIER YOU CAN FOLD THE RAMP FLAT AND PIN INTO PLACE USING 2-5/16" X 3" QUICK PINS.



FIG. 12

⚠ WARNING

NEVER EXCEED THE 500 LB. WEIGHT CAPACITY OF THE RAMP UP SCOOTER CARRIER.

⚠ WARNING

RAMP UP SCOOTER CARRIER SHOULD ALWAYS BE IN THE FOLDED AND SECURED POSITION WHILE VEHICLE IS IN MOTION. NEVER ALLOW ANYONE TO RIDE ON THE RAMP UP SCOOTER CARRIER WHILE VEHICLE IS IN MOTION.

For more information about this or other products, contact us at:

Great Day, Inc.

604 Kimbrough Dr. * Tallulah, LA 71282 * (866) 649-1918 * www.greatdayinc.com