Jay GS Cushion

User Instruction Manual & Warranty

Jay GS Kussen

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Brugervejledning & Garanti

THE JAY® GS CUSHION

The Jay GS Growth and Positioning Cushion is designed to provide a high degree of comfortable postural support and to meet the changing needs of growing individuals.

To ensure proper use of the cushion, please read these instructions and save them for future reference. It is recommended that the fitting of the Jay GS Cushion take place under the supervision of a doctor or therapist.

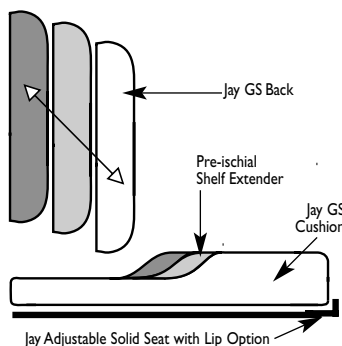
Caution: Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury. Always assess the need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair.

Fitting the GS Cushion

Place the Cushion on the seat of the wheelchair. (A solid base of support, like the Jay Adjustable Solid Seat or Solid Seat Insert is recommended.) If using an adjustable depth back on the wheelchair, like the Jay GS Back, position the back of the Cushion even with the front of the upright back posts. If using a fixed depth back, like a Jay® J2 Back or Jay Active™ Back, notch the rear corners of the base to establish seat depth to within 1" (2.5 cm) from the back of the knees (see score lines on base). This will allow the back of the Cushion to go between the posts. Place the individual on the Cushion allowing the buttocks to sit in the rear, carved-out portion and the legs to rest comfortably on the raised front portion. To achieve the lowest average sitting pressures possible, utilize the GS Pressure Relief Pad. Additionally, the footrests should be adjusted low enough to allow the legs to rest firmly on the front of the Cushion. The Cushion should come within 0 - 1" (0-2.5 cm) of the back of the knee. Assess the client. Are the hips level? Are the legs in good position? Is the client comfortable? If the answer to any of these questions is "no," see the Accessories section in this booklet.

How the Jay GS Cushion Grows

The Jay GS Cushion grows by maintaining the positional relationship between the user and the cushion.

The Jay GS Cushion shown with other components of the Growth and Positioning System.

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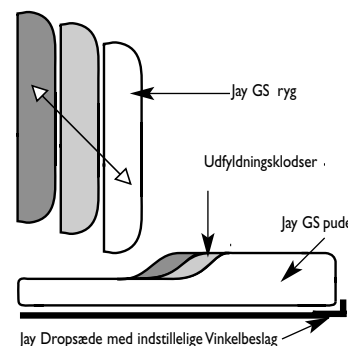
As the client grows, the area under the buttocks remains roughly the same, while the area under the legs grows backward. Using the Pre-ischial Shelf Extender (see diagram on bottom of previous page) will increase leg support and position the anti-thrust shelf where needed.

Growing the Cushion with a depth-adjustable back in the wheelchair.

Begin by seating the client forward on the Cushion (with the Back adjusted forward of the upright posts.) As the client grows, adjust the Back Support backwards toward the posts. This allows the user to be seated further back in the well and maintain proper seat depth. Pre-ischial Shelf Extenders can be added to position the anti-thrust shelf where needed and optimize leg support. (The ischial tuberosities, or seat bones, should be located approximately 1" to 2" (2.5 to 5 cm) behind the anti-thrust shelf.) One Shelf Extender is included with the cushion. (Others can be ordered as needed.) The Cushion "grows" by 1" (2.5 cm) every time a Shelf Extender is added.* When the Back has been adjusted as far back as possible, the Cushion can be moved forward for additional growth.

The front edge of the GS Cushion can extend up to 2" (5 cm) unsupported beyond the front of the seat base without affecting performance. The use of a Jay Adjustable Solid Seat with Lip Option is highly recommended when "growing" the GS Cushion in this way. It allows you to easily relocate the cushion in its desired position if you remove it from the wheelchair.

Growing the Cushion with a fixed back in the wheelchair.
Notch the rear corners of the base enough to allow the Cushion to slide between the upright posts of the wheelchair and to establish proper seat depth (approximately one to two finger widths between the back of knee and front of cushion). As the client grows, adjust the Cushion forward (always allowing the buttocks to sit in the carved-out portion of the cushion). Remember to maintain the desired distance between the back of the knee and the front of the Cushion. When the Cushion is adjusted forward by 1" (2.5 cm) or more*, add a Pre-ischial Shelf Extender to the cushion (included). This will maintain the anti-thrust properties of the GS Cushion.

The front edge of the GS Cushion can extend up to 2" unsupported beyond the front of the seat base without affecting performance. The use of a Jay Adjustable Solid Seat with lip option is highly recommended when "growing" the GS Cushion in this way. It allows you to easily relocate the cushion in its desired position if you remove it from the wheelchair. Growth in width of the Cushion is accomplished through the use of accessories. Lateral Pelvic Supports (hip guides) can be adjusted inward 1" (2.5 cm) on either side and then moved out later for growth at the rear of the Cushion. Lateral Thigh Supports (abductors) can be adjusted inward 1" on either side and then moved out later for growth at the front of the Cushion.

* If using the GS Pressure Relief Pad, use of only one Shelf Extender is recommended.

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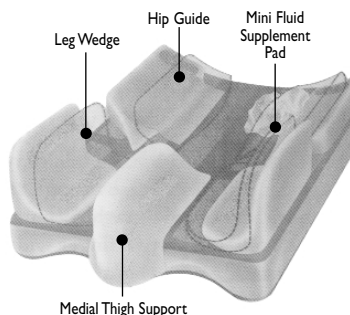
Note: The Jay GS Cushion may be used with different backs depending on the level of disability. For moderately involved clients, try the Jay J2 Back. For more severely involved clients, try the Jay Modular Back. Growth in the seating system is possible with all of the above products when combined with the GS Cushion.

Accessories

The GS Cushion interfaces with a wide variety of Jay components including:

- Pressure Relief Pad (not shown) for greatly improved pressure management.
 - Lateral Pelvic Supports (hip guides) for additional positioning or growth in the width of the Cushion.
 - Lateral Thigh Supports (abductors) to prevent excessive abduction or spreading of the legs or to provide width adjustment for the Cushion.
 - Mini Fluid Supplement Pads to compensate for a non-fixed lean or for added pressure relief and comfort.
 - A variety of Medial Thigh Supports (abductors) for increased leg separation.
 - A 1/2" (1 cm) Well Build-up (not shown) to decrease hip flexion for individuals with limited range of motion.
- The GS Cushion with a standard fluid pad or Pressure Relief Pad will accommodate most accessories. An upsized cover is recommended when larger accessories are used. Fitting of accessories should take place under the supervision of a doctor or a therapist.

Check for bottoming out

The standard Jay GS Cushion has less fluid than the Jay® J2 Cushion because it is designed primarily for posture, comfort, and control of abnormal tone. If the client is at high risk of skin breakdown, utilize the Pressure Relief Pad.

CAUTION: Failure to check the Pressure Relief Pad for bottoming out can result in skin redness or skin breakdown.

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To evaluate for bottoming out (Pressure Relief Pad only)

Place the client on the cushion without the cover for 2 minutes. Lift the client off the cushion, trying not to disturb the fluid. Examine the points where the ischials (seatbones) and coccyx (tailbone) were positioned. If there is 1/2" (1 cm) or less of fluid between the bottom of the depression and the base, extra fluid is needed. Fluid supplement pads for this purpose are available through your local supplier. For 10" (25 cm), 12" (30 cm), and 14" (36 cm) GS Cushions, utilize Mini Fluid Supplement Pads. For 16" (40 cm) and 18" (46 cm) GS Cushions, utilize Standard Fluid Supplement Pads. To use Fluid Supplement Pads, place them along the side of the well, under the Pressure Relief Pad. Replace fluid pad over them and re-evaluate.

Caution: The clinical indicator for tissue breakdown is skin redness. Periodically check the skin for redness. If skin redness occurs, discontinue the use of the Cushion and consult your doctor or therapist immediately.

Easy cleaning and assembly

A wheelchair cushion is a very important part of the life of the user. Keeping the cushion in good condition may ensure years of use. Proper cleanliness will also help extend the life of the Cushion.

Monthly cleaning and inspection is recommended. Check the cover for tears or excessive wear. Check the base to ensure any accessories are still in the right position. Inspect the fluid pad for punctures and fluid consistency. If the fluid ever feels firmer in one area, simply knead the fluid back to its original consistency and/or call Sunrise Medical.

Both the fluid pad and the foam base should be washed with warm water and soap. An incontinence-resistant cover will help reduce the need for cleaning. Do not steam autoclave. The removable cover can also be washed. Machine wash and dry on moderate heat. Do not dry clean.

To reassemble after cleaning

1. Replace any accessories on foam base (if used). Attach fluid pad to foam base along one side, on Velcro® attachment strip. Secure pad at center along Velcro® strip. Then attach the second side, same as first. The split in the front of the fluid pad should be attached evenly to the center Velcro strip. If using the 3" (7.5 cm) Medial Thigh Support (leg abductor), the fluid pad should be attached to the sides of the accessory.
2. Check for bottoming out as described previously.
3. Pull the front of the cover over the Cushion first, then stretch the back corners down around the base. Examine the top of the cover to ensure proper fit. Cover should not hammock between accessories.

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Use Jay covers only

The use of an improper cover can adversely affect the performance of the Jay GS Cushion. Use only Jay-designed covers in conjunction with the Jay GS Cushion. (Do not use additional covers over the Jay GS Cover.) If the client is not incontinent, we recommend the optional perspiration-reducing AirExchange™ cover for use in warm conditions. Incontinence-resistant covers are provided standard so that the cushion does not have to be washed after accidental spills.

Note: Even a snug fitting cover will leave extra "tucks" of fabric under the client's buttocks. This extra material will ensure total fit of the cover without tension against the buttocks.

Base modifications

The Jay GS Base can be easily modified at the factory or by your authorized supplier to accommodate specialized seating needs. These include: special sizes, leg length discrepancies, rail cutouts for dropped or angled seats, back corner cutouts to recess between the upright posts of the wheelchair, or specialized needs as requested.

Caution

The standard Jay GS Cushion is designed primarily to provide a high degree of postural control and comfort. If client is at risk for skin breakdown and pressure relief is a concern, the GS Pressure Relief Pad should be used.

Persons at high risk of skin breakdown should be thoroughly evaluated by a therapist prior to purchasing the GS Cushion. The Jay GS Cushion is no substitute for good skin care, including proper diet and cleanliness. In addition, any cushion should be tried for a few hours while a therapist or doctor inspects the skin to ensure that red pressure spots are not developing.

If skin develops redness, discontinue the use of the cushion and consult with a doctor or therapist. In addition, avoid sharp objects or exposure to excessive heat or open flame. If the cushion is left outside overnight at temperatures below 40° F (5° C), allow the cushion to warm to room temperature before using. Sitting on excessively hot or cold surfaces may cause skin damage.

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ENGLISH**24 MONTH LIMITED WARRANTY**

Each Jay GS Cushion is carefully inspected and tested to provide peak performance. Every Jay GS Cushion is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover. Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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This product is manufactured to comply with the 'Medical Device Directive' 93/42/EEC.

Manufactured under one or more of the following U.S. patents, plus U.S. and foreign patents pending.
U.S. Patent # 4,588,229; 4,660,238; 4,726,624; 4,761,843; 4,842,330; 5,018,790; 5,352,023; 5,362,543; 5,369,829; 5,390,384; 5,395,162; 5,457,833; 5,490,299; 5,397,517; 5,255,404, and other foreign patents.