

C O M P A C T P O R T A B L E S C O O T E R



Seat Color
Black Vinyl

Body Colors
Blue

Red

LiteRider^{GOLDEN}
Mini
GL109D

The popular LiteRider scooter now comes in a compact portable size! The new LiteRider Mini scooter features the same tubular frame design, with a new front basket design and new seat. The LiteRider Mini can be disassembled wirelessly and easily to be transported in virtually any size vehicle.



	LiteRider Mini
Model	GL109D
Medicare Code	K0800
Weight Capacity	250 lbs
Drive Wheels	Rear
Maximum Speed	4 mph
Operating Range up to ¹	8 miles
Ground Clearance under Deck	2.75"
Turning Radius	36.5"
Type Batteries	2-12AH
Colors	Red, Blue
Freewheel Mode	Yes
Electronic Speed Control	Yes
Electro-Mechanical Brakes	Yes
Charger	Off Board
Controller	50 Amp Dynamic R Series
Dimensions:	
Length	37.75"
Width	19.75"
Height (ground to top of back of seat)	32" - 34"
Height (ground to top of back of headrest)	N/A
Ground to top of seat	20" - 22"
Ground to top of deck	4.5"
Deck to Top of Seat	15.5" - 17.5"
Number of Seat Height Positions	3
Size of Increments	1"
Front Axle to Rear Axle	27.5"
Weight of Unit: (assembled)	101 lbs
Front Half	37 lbs
Rear Half	22 lbs
Seat w/Arms	24 lbs
Battery pack weight	16 lbs
Tires:	
Front	8"
Rear	8"
Casters	N/A
Rear Anti-Tip Wheels	2"
Standard Seat:	Stadium Style
Slide Seat	N/A
Back Height - no headrest	14"
Back Height - with headrest	N/A
Width x Depth	17" X 16"
Color	Black Vinyl
Optional Seats:	Stadium Style
Back Height - no headrest	16"
Back Height with headrest	N/A
Width X Depth	18" X 16"
Color	Black Vinyl
Warranty:	
Frame	Lifetime
Drive Train	2 Years
Electronics ²	2 Years
Batteries	1 Year

¹. Battery range tested at 200 lbs., and will vary due to rider weight, drive surface and terrain.

². Electronics warranty excludes batteries.

Enjoy quick and wireless disassembly!



Pick up the seat



Pull out the battery pack



Remove the baskets from under the seat and from the tiller (not shown)



To separate the rear transaxle from the frame, squeeze the release lever located on the underside of the frame



While squeezing the release lever, pull up on the rear frame



Disassembles into six pieces for transport and storage! No wires to connect or disconnect!



Red



Blue

Literature is current at time of printing. Golden Technologies reserves the right to make changes to the product or literature at any time.