

Luggie® QUICK START GUIDE

This Guide is designed to help you become quickly acquainted with your Personal Transportation (PT) by giving you basic start-up information. The owner's manual will elaborate in more detail on how your PT works, including fold, battery charging, and care and maintenance. PLEASE READ YOUR OWNER'S MANUAL THOROUGHLY.

1 REMOVING THE PT FROM THE CATON


1. Remove the packing material that surrounds the PT.
2. Pull the scooter out of the carton or suitcase carefully.

CAUTION! Do Not lift the PT by the plastic shrouds. Damage to your PT may result.



2 UNFOLD LUGGIE SCOOTER

1. Pull unfolding ring and carefully use two hands to make certain your Luggie that are flattening on the ground.
2. Pull-backward to release the seat-lock. Lift up the seat with another hand simultaneously.
3. Fasten security lock in the joint of front and rear chassis on two sides.
4. Release the tiller locking clamp.
5. Release the trigger to raise the tiller until the handlebars are at comfortable position, then fasten the tiller locking clamp.

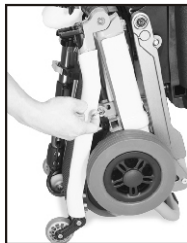


WARNING! Attempting to adjust the tiller while your PT is in motion could result in personal injury and/or damage to your PT.

3 DRIVING YOUR PT

1. Install and fasten the battery.
2. Press the switch button.
3. Press the throttle control levers to go forward or backward.

NOTE: ALWAYS switch button off before GET ON/OFF the PT. In addition to safety, this will help save battery power.

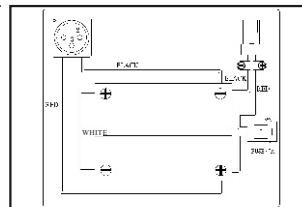

4 CHARGING THE BATTERIES

IMPORTANT

CHARGE THE BATTERIES COMPLETELY BEFORE DRIVING THE PT FOR THE FIRST TIME-AND EVERY TIME THEREAFTER. NEVER CHARGE THE BATTERIES FOR MORE THAN 24 CONSECUTIVE HOURS.

1. Open the charge socket cover.
2. Plug one end of the charger into the charge socket and the other into a standard wall outlet.
3. The charger has two lights: "red" light on-indicating power to the charger, and "orange" light on-indicating charging is in process. Charging is complete when the "orange" light turns to "green". 【see also the charging section of your user's manual】



5 OPERATING THE MANUAL FREEWHEEL LEVER

1. Before engaging or disengaging the freewheel mode, press the switch button off.
2. Place the PT on a dry, level surface.
3. Stand in the rear of the PT when engaging or disengaging the freewheel mode.
4. To position the drive mode, firmly pull the manual freewheel lever backward (located at the rear on the right).
5. To position the freewheel mode, firmly push the manual freewheel lever forward.



WARNING! Sitting on or placing the PT on an incline while in the freewheel mode could result in personal injury and/or damage to the PT.