

Foldable Toilet Safety Rail

Garde-corps pliable pour toilette

Barandilla de seguridad plegable para inodoro



User Instructions and Warranty

Mode d'emploi et garantie

Instrucciones para el
usuario y garantía

Weight Capacity
Capacité de poids de
Capacidad de peso de
300 lb (136 kg)



REF G5-200RX1

REF G5-200X4

Not made with Natural Rubber Latex.

Non fabriqué à partir de latex de caoutchouc naturel.

No está hecho con látex de caucho natural.

General Warnings

To ensure the proper assembly and operation of this product, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- **DO NOT** assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- **DO NOT** use if components are missing or are damaged. Contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.
- **DO NOT** exceed weight capacity of 300 lb (136 kg).
- **DO NOT** make any adjustments while using the product.
- **DO NOT** stand on the product.
- **DO NOT** attempt to use the arms to support full body weight.
- **DO NOT** use product if wobbly, unstable, or not level.
- **DO NOT** use product on uneven surfaces.
- **DO NOT** excessively twist or turn while using the safety rails as this may cause a loss of balance and/or damage to the product.
- **WARNING:** Ensure weight is evenly distributed between the arm supports when using the toilet rail.


Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

- All push buttons are fully protruded and are properly engaged.
- There are no loose, broken or missing components.
- All parts are secure.
- Ensure both rails are at the same width adjustment position before use.
- Both legs touch the floor simultaneously at all times.
- User evenly distributes weight between the arm supports when transferring in and out of the safety rails.
- The tips are not worn, broken or missing. Replace any missing or damaged tips before use.
- Ensure all legs are at the same height adjustment level before use.

Assembly

If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, ensuring all parts listed are included:

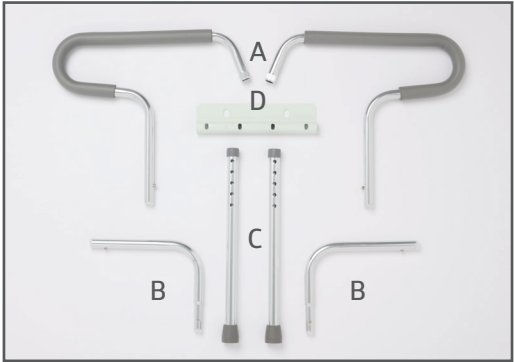

PART ID	PART NAME	QUANTITY
A	Handrail Tube	2
B	L-shaped Bars	2
C	Footpiece	2
D	Toilet Safety Frame Bracket	1

1. Remove existing toilet seat assembly, including bolts and nuts.
2. Place toilet safety frame bracket between toilet fixture and seat assembly with rounded part of bracket closest to the back of toilet (Figure 1).
3. Replace toilet seat assembly and reinsert the original mounting bolts through front holes of the bracket and toilet fixture (Figure 2). Tighten the nuts.



Figure 1



Figure 2