
# TILT ON-THE-GO

zippie

First of its kind. A truly transportable tilt wheelchair, specially designed for families on the move.

# TRULY TRANSPORTABLE TILT

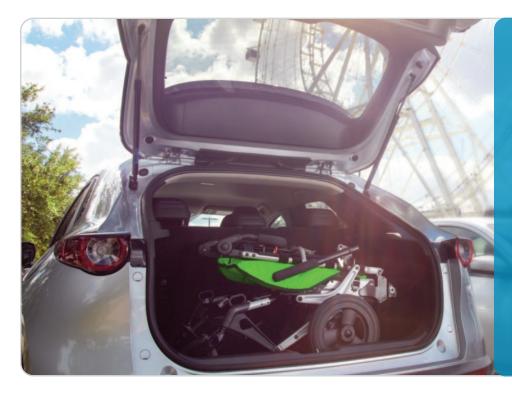


The patent-pending one-step fold of the ZIPPIE Sphynx is easy and intuitive, transforming the Sphynx into an ultra-compact package perfect for families on the move.







### MORE GO MADE EASIER

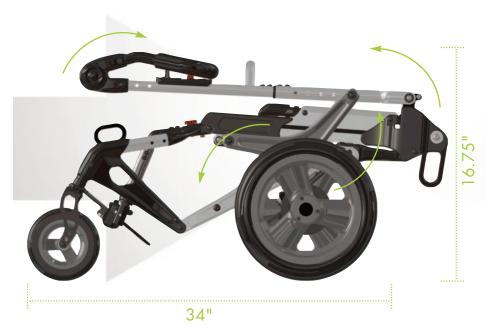

# The most compact transport dimensions in its class!

#### Seriously Compact. Highly Transportable.

- Transport dimensions (34" x 25.5" x 16.75") are small enough to fit within a standard compact car's trunk space, even within some suitcases.
- Weighing only 30 lbs. (13.6 kgs.) the Sphynx can be easily lifted in and out of vehicles for ultimate portability.
- Crash tested and approved for occupied transit in a motor vehicle (compliant with ANSI/RESNA WC-19), with a maximum user weight of 250 lbs.



#### 1. Adjustable Push Handle

to achieve the perfect height for different caregivers.

2. Stylish Upholstery and ability to bundle with JAY<sup>®</sup> and WHITMYER<sup>®</sup> seating and positioning options to meet each child's unique needs.

3. Adjustable Tilt setting of 10°, 20°, 30° with standard included recline of 85° to 100° for optimal positioning.

4. Depth Adjustable Seat

provides built-in growth of up to 5". Keep up with growing kiddos without the need for spare parts.

#### 5. Adaptable Leg Support

accommodates growth, changes in lower leg length, or asymmetrical leg lengths.

#### 6. Crash Tested

Approved for occupied transit (compliant with ANSI/RESNA WC-19)



## NUMEROUS BENEFITS of TILT and RECLINE

**TILT** allows the client to change their position and shift weight from the pelvis to the back, decreasing the strain of gravity.

For children with limited pelvic, thoracic or head control, this change in positioning may help reduce the potential of developing postural deviations later in life.

**RECLINE** opens up the seat to back angle which allows a change in position or posture.

Changing seating angles via tilt and/or recline may assist with feeding, digestion, respiratory function and visual orientation.

Recline can be quickly adjusted 85° to 100° whenever needed.

The static tilt setting can be adjusted to 10°, 20° or 30° to best accommodate the child's needs.





### JAY<sup>®</sup> and WHITMYER<sup>®</sup> SEATING and POSITIONING OPTIONS


A wide range of adaptable seating and positioning options can be added for more complex support to meet diverse user needs.





Sphynx with JAY<sup>®</sup> Seating

#### JAY<sup>®</sup> and Whitmyer<sup>®</sup> Options

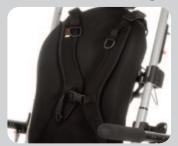

JAY<sup>®</sup> Center Opening Dynamic Harness



JAY® Lateral Thoracic Supports



JAY® Foot Supports



WHITMYER® Head Supports

### **OPTIONS** & ACCESSORIES



Upper Extremity Tray



Flip-Back Armrests



Utility Hook



Cup Holder



Curb Tipper



Rain Cape



Underseat Basket



Canopy



Headrest Extension

# **FUNCTION and STYLE!**

Canopy shown is optional.



Green

Black

#### ZIPPIE<sup>®</sup> SPHYNX<sup>™</sup> SPECIFICATIONS

HCPCS Code: E1232, E1234, E1161 Tilt: Static, Adjustable to 10°, 20°, 30° Back Support Angle Adjustment: 85°, 90°, 95°, 100° Transit Option: YES (ANSI/RESNA WC-19) User Weight Capacity: 250 lbs

Chair Weight: 30 lbs Seat Widths: 10", 12", 14", 16", 18" Seat Depths: 10" to 19" (Includes 5" depth growth, standard) Transport Dimensions: Folded 17" H x 26" W x 34" L

Scan to visit the Zippie Sphynx product page





