TurnTable

SystemRoMedic™

Manual - English

Manual - Svenska

Manual - Norsk

Manual - Dansk

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Manual - Español

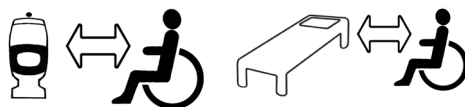
REF

5082

5083

5084

5085

SWL: 135-250 kg/300-550 lbs

Table of contents

Always read the manual	3
English	4
Svenska	5
Norsk	6
Dansk	7
Suomi	8
Deutsch	9
Nederlands	10
Français	11
Italiano	12
Español	13
Description	14

**EN Always read the manual**

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The TurnTable can be used for seated transfers between wheelchair and bed, toilet or chair/armchair. For increased security, the TurnTable can be used together with FlexiBelt/EasyBelt. TurnTable is available in three sizes with handles or without handle for tight spaces.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.

Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Special requirements and conditions

To transfer with TurnTable, the patient must be able to:

- support weight while standing
- stand up and maintain balance (with support)
- understand instructions

Placement and use TurnTable

Place the TurnTable on the floor beside the patient. Secure the TurnTable in place, before raising the patient, by placing the ball of your foot on the TurnTable and your heel on the floor. Steer the TurnTable by pushing with the front of the foot and braking with the heel. When the patient is about to sit down, secure the TurnTable in place as described above.

When transferring from bed to wheelchair, place the wheelchair close to the bed, so as to minimize transfer distance.

When transferring to the toilet, place the wheelchair close to the toilet, so as to minimize transfer distance. Pull the patient's pants up/down when the patient has risen slightly from the seat. Preferably, two caregivers should assist.

Care of the product:

Clean TurnTable with pressurized air or water. If necessary, disinfect with 70% alcohol. Teflon spray or a similar product can be used to lubricate between the plates so that they glide easily against each other.

CE Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

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SystemRoMedic™

For 25 years we have applied ourselves wholeheartedly to developing smart and easy-to-use assistive devices for easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

Transfer: products for moving patients between locations.

Positioning: products for repositioning in the same location.

Support: products that provide mobility support.

Lifting: products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Our mission, to help people, has always been, and will continue to be, the driving force of innovation. We love easy transfers.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com

handicare

Handicare AB
Veddestav. 15, Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se

Internet: www.handicare.com