

User Guide – Wall Grab Bars



INSTALLATION INSTRUCTIONS

Grab bars must be securely attached to the wall or backing to provide proper safety performance. **Nova Ortho-Med, Inc. shall have no responsibility for failures resulting from improper installations.** To properly install grab bar, follow the instructions listed below.

1. Position grab bar on wall and mark screw entry locations
2. Drill ¼" holes & insert plastic anchors
3. Position grab bar back on wall and insert the provided screws
4. Screw until tightened

CAUTION: GRAB BARS MUST BE INSTALLED PROPERLY TO PROVIDE SAFETY SUPPORT. TOGGLE BOLTS ARE SUGGESTED FOR PLASTER OR DRYWALL. LEAD ANCHORS FOR MASONRY INSTALLATIONS.

SAFETY CARE INSTRUCTIONS

- Grip grab bar securely before entering or exiting bathtub
- Use extreme caution on wet surfaces
- Persons using wall grab bar should be assisted during bathtub activities
- Wall grab bars can be cleaned with soap and water

LIFETIME LIMITED WARRANTY

Nova Ortho-Med, Inc. warrants this product to be free from defects in material or workmanship for a period of lifetime from date of purchase. If within the specified warranty period, this product is proven to be defective, it will be repaired or replaced at NOVA O.M.'s option. This warranty does not cover shipping costs incurred in replacement or repair of this product. NOVA O.M. stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This Lifetime Limited Warranty represents our confidence in the materials and workmanship of our products. Do not return any products to our office without prior consent. This warranty does not cover problems due to user negligence or misuse of the product. This warranty shall not extend to non-durable components, including but not limited to rubber tips, handgrips, casters, and other parts subject to normal "wear and tear".

THIS FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA O.M. SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.