PVI Ramps

Portable Ramp Instructions

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WARNING!
READ AND UNDERSTAND INSTRUCTIONS
BEFORE USING RAMP.

P.V.I. Ramps are designed for wheelchairs and electric scooters. P.V.I. ramps are rated for a 600lb capacity. This rating is based on double axle, evenly distributed loads. For your safety a caregiver or aid should be present at all times when using any ramp.

Proper setup is essential for safe loading and unloading. Make sure that the hook (Top End) is pressed firmly against the step or platform before using. Failure to do this may result in ramp slipping off step while occupied. When possible use steel pins provided to lock ramp in place. Although P.V.I. portable ramps are lightweight, safe lifting practices should be used when lifting. For folding and unfolding instructions, refer to the diagram found next to the handle on the ramp.

Slope discussion: P.V.I. strongly recommends that users never exceed a 3:12 ratio. Example: if your ramp is 6' long the maximum height of step or platform should not exceed 18". This degree of incline should be for unoccupied chairs or scooters only. Occupied chairs and scooters should never exceed a 2:12 ratio and be aided by a qualified assistant. Example: If your ramp is 6' long the maximum height of step or platform should not exceed 12".

Thank you for purchasing your new P.V.I. portable ramp. Prairie View Industries is committed to manufacturing the highest quality access ramps on the market today. Although other companies have copied our design, they have not achieved the quality standard that we guarantee with our products. Your safety is important to us; please read and follow all the information found in this manual and on your ramp.

1. **ALWAYS** have a qualified assistant present when using this or any other portable ramp.
2. **NEVER** exceed a slope greater than 2" on 12" with an occupied chair or scooter.
3. **ALWAYS** make sure that top of ramp is secured on step or landing before using. It may be necessary to anchor top of ramp to landing surface with steel pins provided.
4. **ALWAYS** follow manufacturer recommendations for chair or scooter.
5. **ALWAYS** use your lap belt.
6. **SLOPE RECOMMENDATIONS:**

1:12 SLOPE (4.8deg)

1. The ADA recommended slope for long (up to 30') home and commercial ramps.
2. Works for most strong unassisted manual chair users. although P.V.I. always recommends assistance.

2:12 SLOPE (9.5deg)

Acceptable grade for portable ramp use by occupied chairs and scooters with a qualified assistant.

3:12 SLOPE (14.5deg)

For loading UNOCCUPIED chairs and scooters.

RAMP RISE IN INCHES

A diagram illustrating a ramp. A horizontal line represents the ground. A vertical line represents the rise. A diagonal line represents the ramp length. The diagram shows a stepped profile of the ramp, indicating that the rise is the sum of several smaller rises. The label 'RAMP LENGTH' is placed above the diagonal line.

RAMP LENGTH

RAMP LENGTH

RAMP RISE IN INCHES


	2'	3'	4'	5'	6'	7'	8'	10'	12'
3"	7.2°	4.8°	3.6°	2.9°	2.4°	2.0°	1.8°	1.4°	1.2°
4"	9.6°	6.4°	4.8°	3.8°	3.2°	2.7°	2.4°	1.9°	1.6°
5"	12.0°	8.0°	6.0°	4.8°	4.0°	3.7°	3.0°	2.4°	2.0°
6"	14.5°	9.6°	7.2°	5.7°	4.8°	4.1°	3.6°	2.9°	2.4°
7"	17.0°	11.2°	8.4°	6.7°	5.6°	4.8°	4.2°	3.3°	2.8°
8"		12.8°	9.6°	7.7°	6.4°	5.5°	4.8°	3.8°	3.2°
9"		14.5°	10.8°	8.6°	7.2°	6.2°	5.4°	4.3°	3.6°
10"		16.1°	12.0°	9.6°	8.0°	6.8°	6.0°	4.8°	4.0°
12"			14.5°	11.5°	9.6°	8.2°	7.2°	5.7°	4.8°
14"				13.5°	11.2°	9.6°	8.4°	6.7°	5.6°
16"					12.8°	11.0°	9.6°	7.7°	6.3°
18"					14.5°	12.4°	10.8°	8.6°	7.1°
20"						13.8°	12.0°	9.6°	7.9°
22"							13.2°	10.6°	8.7°
24"							14.5°	11.5°	9.5°
26"								12.5°	10.2°
28"								13.5°	11.0°
30"								14.5°	11.8°
32"									12.5°
34"									13.3°
36"									14.5°

RAMP INCLINE IN DEGREES


Instructions For Unfolding P.V.I. Multifold Ramp

- Step 1. Position the ramp approximately the ramp's length out from step, van or platform. Stand ramp up on end with the open ends down and the handle side toward you. Reach down and lift first half of the ramp toward the step or landing area and set it down on top of support. Make sure hook is pressed firmly against step.
- Step 2. Open top half of ramp.
- Step 3. Unfold top half of ramp towards you and once again make sure that ramp hooks are pressed firmly against landing area.


Step 1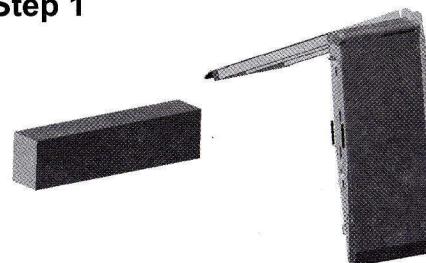

Step 2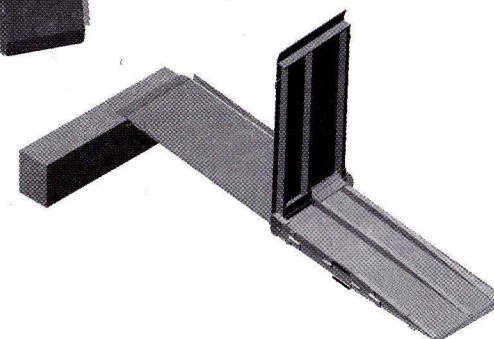

Step 3

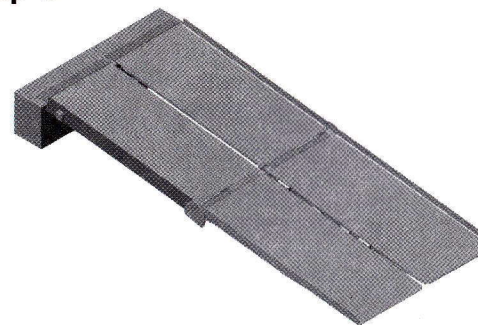PRAIRIE VIEW INDUSTRIES, INC.
LIMITED LIFETIME WARRANTY

1. PRAIRIE VIEW INDUSTRIES, INC. warrants to the original end purchaser of the product, so long as such purchaser continues to own the product, that the product is free from defects in material and workmanship under normal use and service. This warranty does not apply to any product that has been subject to misuse, abuse, neglect, alteration, accident, usage not in accordance with product instructions, acts of God, or improper installation, or that has been used for rental purposes or repaired by someone other than PRAIRIE VIEW INDUSTRIES, INC. This warranty does not cover normal deterioration of the product due to wear and exposure.
 2. This warranty is limited to repairing or replacing, at the option of PRAIRIE VIEW INDUSTRIES, INC., any product that is returned to PRAIRIE VIEW INDUSTRIES, INC. and is reasonably determined by PRAIRIE VIEW INDUSTRIES, INC. to be defective. The repair or replacement of a defective product under this warranty will be made by PRAIRIE VIEW INDUSTRIES, INC. without charge for parts or labor. This excludes shipping or delivery charges to and from PRAIRIE VIEW INDUSTRIES, INC.'s place of business. If the product has been discontinued, PRAIRIE VIEW INDUSTRIES, INC. may replace the product with a new product of comparable value and function. PRAIRIE VIEW INDUSTRIES, INC. also reserves the right to refund the purchase price as its exclusive warranty remedy.
 3. Any claim alleging that the product fails to conform to this warranty may be made only by the original end purchaser and only while such purchaser owns the product. A defective product that is covered by this warranty must be returned, at the purchaser's expense, along with proof of date of original purchase (such as receipt, check or other document PRAIRIE VIEW INDUSTRIES, INC. deems acceptable that shows the date of purchase and the identity of the product and purchaser), along with a description of the alleged defect, to PRAIRIE VIEW INDUSTRIES, INC., 2620 Industrial Drive, Fairbury, Nebraska 68352.
 4. EXCEPT AS SET FORTH HEREIN, PRAIRIE VIEW INDUSTRIES, INC. MAKES NO WARRANTIES, EXPRESSED OR IMPLIED, AND PRAIRIE VIEW INDUSTRIES, INC. DISCLAIMS AND NEGATES ALL OTHER WARRANTIES, INCLUDING, WITHOUT LIMITATION, IMPLIED WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, CONFORMITY TO MODEL SAMPLES, OR ANY WARRANTIES OR INDEMNITIES AGAINST INTELLECTUAL PROPERTY INFRINGEMENT.
- SOME STATES DO NOT ALLOW LIMITATIONS ON IMPLIED WARRANTIES, SO THESE LIMITATIONS MAY NOT APPLY TO YOU. IN NO EVENT WILL PRAIRIE VIEW INDUSTRIES, INC. BE LIABLE FOR ANY INDIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES OR FOR ANY LIABILITY IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT.
5. Before using the product, the purchaser must first determine the suitability of the product for its intended use, and the purchaser assumes all risk and liability whatsoever in connection therewith.
 6. No person, agent, distributor, dealer or company is authorized to change, amend or modify the terms of this warranty.
 7. This warranty gives the purchaser specific legal rights, and the purchaser may also have other rights that vary from state to state.
 8. The purchaser may not assign the purchaser's rights or obligations under this warranty without the prior written consent of PRAIRIE VIEW INDUSTRIES, INC.

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Prairie View Industries, Inc.