



R82 Scallop

Providing stability without preventing movement!

 **etac**[®]

R82 Scallop

The Scallop is designed to aid in floor sitting and to add extra support whilst sitting on a chair. Used by infants, children and youngsters with mild to moderate disabilities. This lightweight, portable, and versatile product is the perfect aid to enable participation in everyday activities with comfort, support and ease for the whole family.

Providing stability without preventing movement!

The Scallop allows users to move while being seated and supported. It provides stability without preventing movement. Whether you are going for a picnic in the park, learning in school, granny's house for tea, the Scallop is the perfect product to bring along.

The Scallop can be used in two different ways, as a floor sitter or attached to a chair. Whichever way you choose to use your Scallop the smooth rocking and hug like feeling make for a calm and peaceful seating experience.

Giving you a little extra!

The R82 Scallop is suitable for children with GMFCS level 1-3, children who require a little extra help while floor sitting or sitting on a chair. This product is also great for children with learning or behavioural disorders as it provides tactile stimulation which helps to reassure the child and aims to enable them to relax and concentrate while using the product.



Features

The Scallop comes in 4 sizes and can cater infants and youngsters. There are two playful colour combinations to choose from, Red & Blue or Green & Grey, both cushions can be removed from their shells and machine-washed at 60 degrees. Weighing less than 2kg your R82 Scallop is a convenient and lightweight seat that does not compromise on comfort.



Simple

The Scallop is designed to be simple with no fuss. It is made up of a flexible shell that helps to reinforce balance. It is held together by Velcro.




Portable

Weighing less than 2kg, the Scallop is easy to travel with. You can carry the product using the two handles provided.



Versatile

The Scallop is not only used for long sitting. It can also be attached to a wide variety of chairs to add extra support while sitting around a table to enable participation in everyday activities.

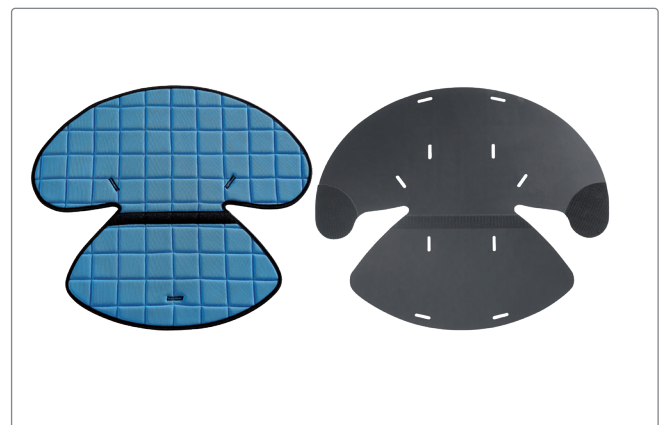

Removable cushion

The Scallop cushions come in two reversible colours and can be easily removed from the shells and machine-washed at 60 degrees.



Meet Evie

We had the great pleasure of meeting Evie Hopkins from Swansea, Wales, where the Scallop originally came from. Evie had been using the Scallop for 3 weeks and during this time she has used the Scallop in a variety of places and situations.

Participation & independence

"The introduction of the Scallop to facilitate Evie's inclusion has been life changing for Evie. She is now able to actively participate in many activities which weren't previously possible. She sits in the scallop on a normal chair at the table and plays board games and colours etc with her brothers and sisters. Evie loves playing FIFA with her brothers and for the first time ever, she sits on the floor with confidence. Evie is able to concentrate on what she is doing rather than worrying and panicking about whether she is able to maintain position."

"Evie has used the Scallop in school and it is particularly useful during circle time and music. Previously, Evie would be sitting in her wheelchair or specialist seat (which looks like a baby's highchair) and would be on a higher level than her peers, who would sit on the carpet. Now Evie is sat on the floor with her peers, playing instruments and talking which makes her feel fully included. The school reported that Evie "Is more engaged."

Recently, Evie has been invited to two birthday parties, one at the cinema and one more traditional, at her friend's home. Evie used the Scallop in both. The more limited room in her friend's house was not an issue (previously, Evie wouldn't have attended) since she sat at the table with her peers in her Scallop. Evie has been invited to cinema parties before and whilst she was able to attend; the seating for people with disabilities is in different areas, she would be sat away from the majority of her friends. Evie was able to sit in her Scallop in amongst her friends."

"Evie reports that she "Likes sitting up on her own!" (Without us physically supporting her)... "Because I can do it myself and don't need you to do it for me!"

Evie's favourite thing about the Scallop is "Sitting on the floor playing FIFA with Joseph and Ethan". Evie shares, "I like taking my Scallop to school because I can sit close to my friends!" Karen, Evies mother.



[Watch the video](#)



Keeping it simple


Standard

Carry straps, seat pit, 3-point belt for size 1 and 2, 2-point belt for size 3 or 4.




[Find out more](#)


Scallop provides the child with a sense of safety and security through tactile input of the seat on the body. It can be used on the floor or strapped to a chair to enhance social interaction and communication with peers.

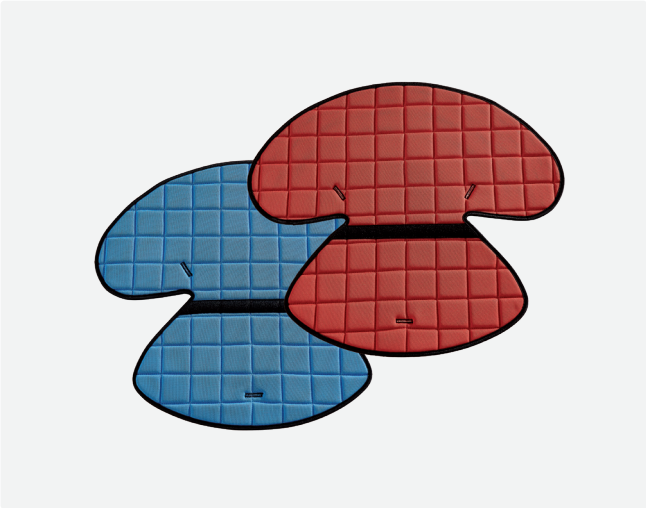

Red/Blue reversible cushion

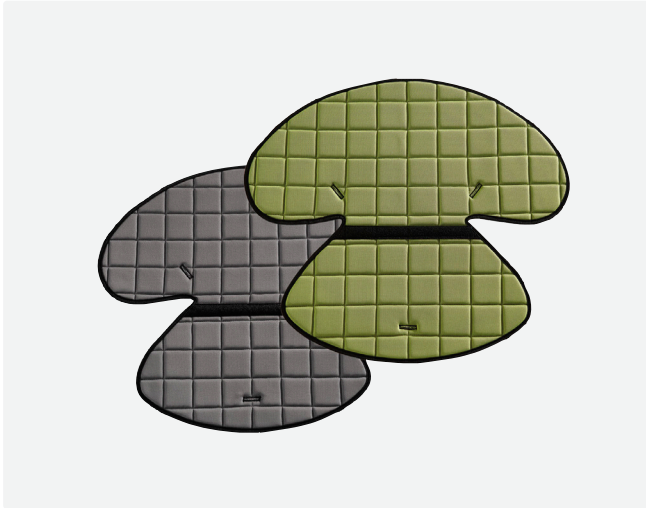

Grey/Green reversible cushion

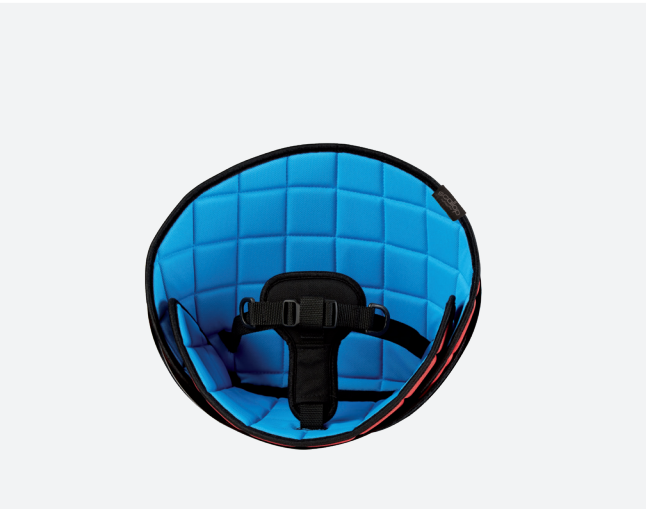
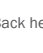
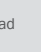



Standard Scallop size 1 or 2. Available in red/blue or grey/green reversible cushion



Standard Scallop size 3 or 4. Available in red/blue or grey/green reversible cushion

Measurements

		1	2	3	4
	Hip width	150–210 mm	210–240 mm	240–270 mm	270–380 mm
	Back height	330–380 mm	380–420 mm	420–470 mm	470 + mm
	Max. load	15 kg	22 kg	29 kg	70 kg
	Total mass	0,60 kg	0,78 kg	1kg	1,44 kg



Shell



cushion, reversable colour



3 point belt for size 1 and 2



3 point belt for size 3 and 4



2 Attachment straps



2 Carry handles



seat pit



Etac is a world-leading developer and provider of ergonomic assistive devices and patient handling equipment. Our heart lies in the solutions that optimise quality of life for the individual, their family and caregivers.

For the latest news and continuously updated product information – visit: www.etac.com

etac

Etac
PO Box 284
Botany NSW 1455
Australia
Tel +61 (02) 8213 6666
Fax +61 (02) 8213 6655
www.etac-au.com
au.sales@etac.com

etac[®]
Creating Possibilities