

# PRIDE MOBILITY PRODUCTS CORPORATION

## INSTALLING THE SCOOTER REAR VIEW MIRROR

### LGTRMIR1000—MIRROR, REAR VIEW, COMPLETE

If you require any assistance with these instructions, please call **Pride Technical Service** at **1-877-800-1248**.  
**Canada providers**, phone **1-888-570-1113**.


#### Tools needed for this installation:

- utility knife

#### Installation instructions:

1. Assemble the mirror as shown in the package instructions, using the longer of the two wedges included in the kit.
2. If the handlebar has a closed end, insert a utility knife blade into the end of the handlebar grip and cut around the inside of the handlebar.
3. Push the rear view mirror assembly into the handlebar of the scooter. See figure 1.

The rear view mirror is now ready for use.

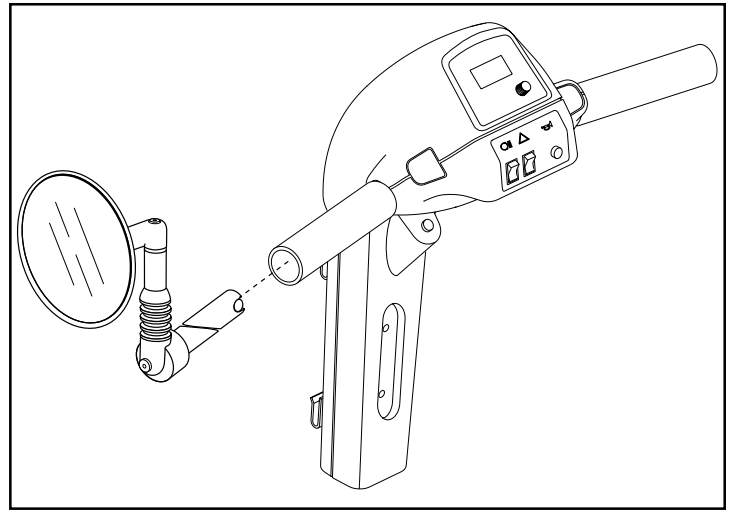

**Figure 1. Rear View Mirror Installation**