

Do Not Discard These Instructions

To ensure your safety and adequate installation of your Removable Side Rail, you **MUST** read the warnings, manufacturer disclaimers, and follow these instructions.

Warnings

1. After removal of the packaging, immediately dispose of all plastic packaging to avoid risk of Suffocation.
2. Various sizes, thickness and density of a mattress, or functions of your adjustable base may result in higher risk of entrapment occurrence between the your Mattress and Removable Side Rail.
If you have questions, we recommend you consult the dealer where you purchased the Side Rail before you start to use it.
3. Damages to your adjustable base may occur if the Removable Side Rail is installed on a pivot point of your adjustable base. If this is not clear, please consult the dealer where you purchased your Side Rail for a technical assistance.
4. The Removable Side Rail is designed to prevent users from the risks of rolling out of the bed and aid users in sitting up in the bed and is not designed as a push or pull handle when moving the bed.
Excessive weights or forces applied to the Side Rail may lead to deformation of the Side Rail or even damages to your Adjustable Base.

Warranty


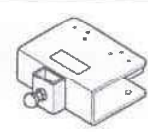

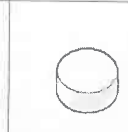

Your New Removable Side Rail has a 3 Year Limited Warranty, subject to the terms set forth above. The Purchaser must show original proof of purchase for the Warranty to be valid.
This Warranty is void and does not apply to any damaged occurred during transit or caused by misuse of user. The decision for repair or replacement of defective product under this warranty is the sole option and discretion of the manufacturer.

For further assistance, please contact the dealer whom you purchased your Removable Side Rail.

Fabric Care Instruction for the Side Rail Cover:


For spot cleaning, wipe an area with a light damp sponge or vacuum with a soft brush attachment to remove particles. Keep at a minimum of 30cm (12 inches) away from direct heat sources. For a deeper cleaning, blot liquid spills with a clean dry cloth. Wipe with a clean cloth dampened with warm water.

Part List

				
A. Side Rail*1	B. Side Rail Pedestal*1	C. Knob*2	D. Plastic Cap*2	E. Self-Tapping Screw*4


Step 1

Before you start to install the Side Rail, fully raise all motions of your Adjustable Base, including back rest, leg rest, lumbar support and head tilt. You may install your Side Rail Pedestal on the head section, mid section or foot section, making sure it is not installed on a pivot point of your Adjustable Base.
Please refer to the following drawings which show the Side Rail Pedestal(B) is only installed on a flat part of the deck of your Adjustable Base and not on a pivot point.

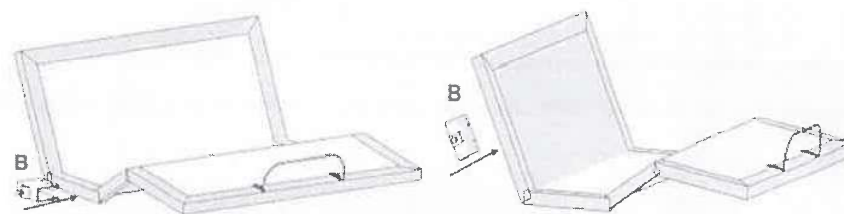


Step 2

Insert the Plastic Cap (D) into the space between Side Rail Pedestal (B) and the backside of your adjustable base deck. Securely tighten the Knob (C) through the Side Rail Pedestal (B).

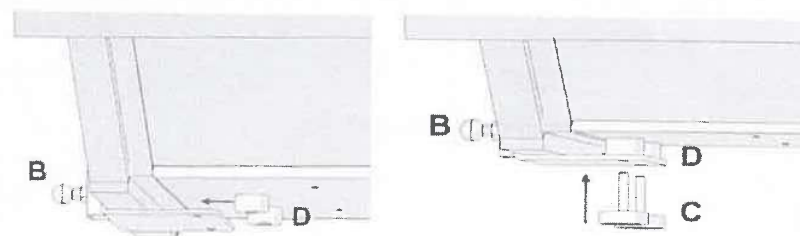

Step 3

Tighten the Self-Tapping Screws (E) through the Side Rail Pedestal (B) into the deck of your Adjustable Base. (An Electronic Screw Driver is recommended). This process is very important in helping you to firmly fix the Side Rail Pedestal to the deck(B).

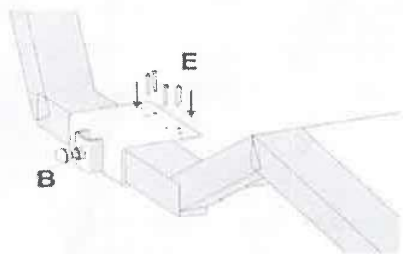


Step 4

There is a security lock on the Side Rail Pedestal (B) to secure the installation of your Side Rail (A).

Pull out the plug on the Side Rail Pedestal (B) and insert your Side Rail (A). Release the plug when you adjust your Side Rail (A) to the desired height position.

Repeat the steps whenever you need to adjust the height position of your Side Rail (A).

There are three options for the height adjustment of your Side Rail. Find the appropriate height to meet your individual needs.

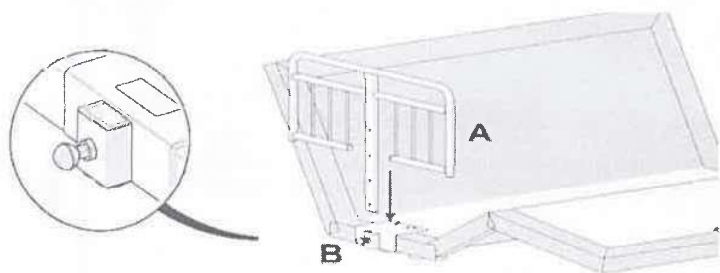

Step 5

Now the installation of your Removable Side Rail is complete.

User's Manual for Removable Side Rail

