

New Rifton Stander Prone | Supine | Multi-Position

Size 3 S430

The size 3 Stander shown in supine configuration. It can also be ordered in prone or multi-position configuration.

Size 3 features:

- Gas spring assisted height adjustment
- Prone upper trunk support
- Additional higher sockets for trunk strap on the Prone upper trunk support
- Size 3 trunk laterals, head laterals, and thigh medial/laterals



Prone upper trunk support is uniquely designed to comfortably support all body types, including females.



0°-30° independent hip abduction is standard on all sizes and configurations.



Tilt adjustment: the padded body support tilts 90° from horizontal to vertical.



Height adjustment: the column allows 14 inches of height difference, adjusted with a gas spring assisted hand crank.



Advanced leg support option accommodates hip and knee flexion contractures.



New Stander features and benefits

- **Base frame:** The two locking, 8" side wheels located near the user's center of gravity, paired with the two casters to the front and back, provide excellent maneuverability and stability.
- **Body support:** The body support is a large padded area supporting the user's pelvis and trunk and is the base for several important straps and lateral supports.
- **Caregiver handle:** The caregiver handle allows easy maneuvering, even while the user is in the product. It can be quickly adjusted up or down to be positioned at a convenient height or out of the way below the the multi-position tray.
- **Height adjust:** Height adjustment helps make transfers easy and allows positioning the user's feet near the floor to be at eye level with their peers.
- **Independent hip abduction:** Abduction while standing and weight bearing is critical for hip health. Rifton's new standers come with 0°-30° of abduction for each lower extremity.
- **Tilt adjustment:** The padded body support tilts 90° from horizontal to vertical. Adjust the angle by squeezing the double safety trigger on the caregiver handle which releases the locking gas spring. The continuous adjustment locks in any position when the trigger is released.
- **Adjustable sandals:** Sandals provide secure foot support with a heel cup (adjustable in length) and two straps over the foot, each of which closes with a strong, magnetic buckle. The sandals adjust forward and back and rotate using the knob underneath. When transitioning between prone and supine, the sandals rotate 180°.
- **Room for growth:** With options to adjust overall height plus individual components, this stander accommodates growth and a wide range of sizes.
- **Advanced leg supports:** The optional advanced leg supports allow hip and knee flexion to accommodate contractures.

Prone configuration:

- **Comes with:** frame, body support, trunk laterals, pelvic harness, trunk strap (standard or trunk rotation control), leg supports, sandals
- **Optional:** prone knee cuffs, tray, additional trunk laterals, medial/lateral thigh supports, advanced leg supports, upper trunk support

Supine configuration:

- **Comes with:** frame, body support, trunk laterals, trunk strap (standard or trunk rotation control), knee, hip (standard or hip rotation control), leg supports, sandals
- **Optional:** head blocks, butterfly harness, tray, additional trunk laterals, medial/lateral thigh supports, advanced leg supports, knee straps, split knee supports

Multi-position configuration: the same stander can be converted to either prone or supine.

- **Comes with:** frame, body support, trunk laterals, knee straps, pelvic harness, hip strap (standard or hip rotation control), trunk strap (standard or trunk rotation control), leg supports, sandals
- **Optional:** head blocks, butterfly harness, prone knee cuffs, tray, additional trunk laterals, medial/lateral thigh supports, advanced leg supports, split knee supports, upper trunk support

