EasyTRANSFER

WAVE

Gebogenes Rutschbrett **Curved Transfer Board**

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01. Purpose and Indication

Rolko-EasyTRANSFER WAVE is a transfer board that enables a patient to change his or her seating position, also when the mobility capability is severely impaired. Changing the seating position, for example, between a wheelchair a bed or into another seating can be done with the transfer board also without the support of an assistant, provided that the patient has sufficient strength. Even in the case of immobility, with the support of assistants or caregivers in the home and care area, the repositioning with the transfer board is possible.

02. Contraindication

Our transfer board is not suitable for all applications that deviate from the intended use and/or may lead to danger. The product is not suitable to overcome seat gaps longer than 20 cm. It must not be loaded with more than 130 kg.

03. Safety Instructions

- Please mind the instructions listed in this manual before using the product.
- A damaged / defective transfer board must never be used and must be replaced immediately.
- Always ensure that all support surfaces (e.g. wheelchair) are secured against slipping / rolling away.
- Do not try to overcome seat gaps longer than 20 cm.
- Use proper clothing, that allows a better sliding. The usage in contact with naked skin is to avoid.
- Keep the transfer board out of reach of sunlight, heat and fire.
- Danger of slipping or tipping: Do not leave the transfer board lying on the floor!
- Do not use the board when it is damp, wet or dirty!
- Take note of the the correct handling of the product in the following pages.
- Ensure that loose clothing does not hinder the transfer.
- Make sure that a possible height difference between the support surfaces does not impair your safety during the use. The transfer board must always rest securely on both support surfaces.
- We recommend to carry out the transfer under supervision of a helper.
- If serious incidents occur in connection with this product, the incidents must be reported to both Rolko Kohlgrüber GmbH and the competent authority of the EU member state in which the user or patient is established.

04. Use (images on the next page)

Before using the transfer board for the first time please check it for any damages that may have occurred during the transport. The photos below are not true to scale and do not necessarily reflect the exact delivery scope. The example below demonstrates the transfer from a bed to a wheelchair. However, the use of the board is universally possible, e.g. between a wheelchair and a car or a toilet and back.

1. Place the transfer board between the bed and the wheelchair and ensure that both support surfaces are secured against slipping/rolling away. Also make sure that the wheelchair is parallel to the bed (fig. A).
2. The distance between the bed and the wheelchair must not exceed 20 cm (fig. B). If available and possible, remove the arm and foot rests of the wheelchair (fig. C).
3. Now hold the transfer board with one hand (fig. D) and bend your upper body slightly and carefully towards the wheelchair (fig. E). Before each movement, check the stability of the board on both surfaces.
4. We recommend the transfer to be carried out under the supervision of a helper.
5. To remove the board, raise your buttocks and pull the board (fig. F).

05. Care, Cleaning and Disinfecting

Wash off the transfer board with a damp cloth, if needed with the addition of a mild cleaner. Also if necessary, disinfectants suitable for plastic can be used. Always allow the board to dry completely. Clean and disinfect the board before each re-use.

06. Maintenance and Service

Carefully inspect the board before each use for optical defects. Make sure that the material is undamaged and the plate is in perfect condition. Ensure that a safe functional capability of the board by carefully bending it: the material must react elastically but be firm and stable. The surface must be smooth and undamaged so that gliding is possible without any problems. In case of doubt compare the material properties with the ones of a new product. If you notice a deformation of the board, surface damages, cracks, discolouration, wear-out or other defects, dispose the product immediately. A defective or damaged transfer board must never be used.

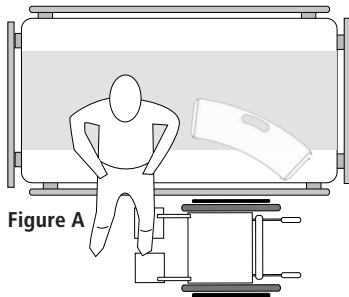
Figure A

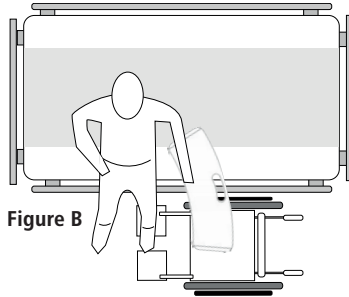
Figure B

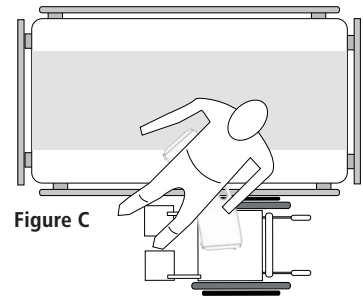
Figure C

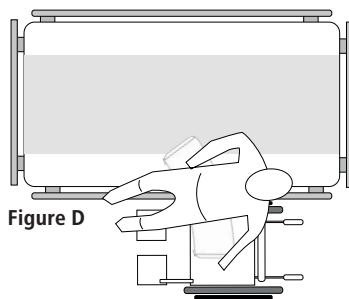
Figure D

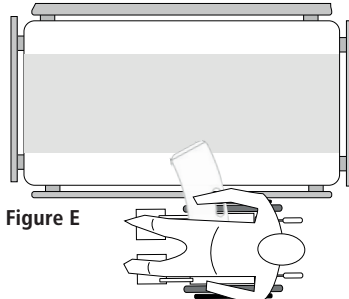
Figure E

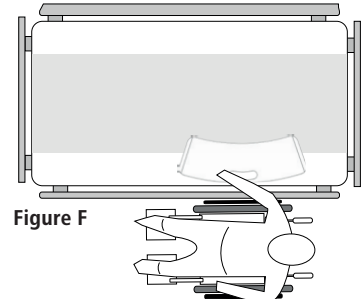
Figure F

07. Re-Use

Rolko-EasyTRANSFER WAVE is suitable for re-use. Clean and disinfect the transfer board before each re-use. Please note that the exposure to UV radiation, chemical substances, mechanical stress, natural aging as well as careless handling may cause a premature wear on the board, which is a fact that not directly influenced by the manufacturer. For this reason, a replacement of the product is recommended every 10 years. This manual is a part of the transfer board and must be passed on when given to another person.

08. Disposal

The transfer board is to be disposed of safely with household waste.

09. Technical Data

Item no.:	RBK-KV	RBK-KV-2
Length:	670 mm	750 mm
Width:	215 mm	218 mm
Material:	PP-H / TPE-S	PP-H / TPE-S
Material thickness:	10 mm	10 mm
Weight:	1,2 kg	1,4 kg
Load capacity:	max. 130 kg	max. 130 kg
Delivery unit:	1 piece	1 piece

Manufacturer

Follow the instructions

Production date

Item no.

Batch

Medical device

Unique device identification

As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN ISO 21856. This user manual and the corresponding declaration of conformity are available for download at www.rolko.com for you.

Operating Instructions: Transfer Board WAVE / RBK-KV / RBK-KV-2 / 2023-12-28 / Rev. 12
Basis-UDI-DI: 426067605RBEASYTRANSFERS2

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved. The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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01. Purpose and Indication

Rolko-EasyTRANSFER makes the transfer of patients from a wheelchair to a bed, toilet, car etc. easier. With the transfer board a person can move from one seat to another without having to get up. In this way, patients can overcome small gaps, with the support of a helper or completely independently, thus the transfer board is also a support aid in the care and in the home area.

02. Contraindication

Our transfer board is not suitable for all applications that deviate from the intended use and/or may lead to danger. The product is not suitable to overcome seat gaps longer than 20 cm. The product must not be loaded with more than 130 kg.

03. Safety Instructions

- Please mind the instructions listed in this manual before using the product.
- A damaged / defective transfer board must never be used and must be replaced immediately.
- Always ensure that all support surfaces (e.g. wheelchair) are secured against slipping / rolling away.
- Do not try to overcome seat gaps longer than 20 cm.
- Use proper clothing, that allows a better sliding. The usage in contact with naked skin is to avoid.
- Keep the transfer board out of reach of sunlight, heat and fire.
- Danger of slipping or tipping: Do not leave the transfer board lying on the floor!
- Do not use the board when it is damp, wet or dirty!
- Ensure that loose clothing does not hinder the transfer.
- Make sure that a possible height difference between the support surfaces does not impair your safety during the use. The transfer board must always rest securely on both support surfaces.
- We recommend to carry out the transfer under supervision of a helper.
- This manual is a part of the transfer board and must be passed on when given to another person.
- If serious incidents occur in connection with this product, the incidents must be reported to both Rolko Kohlgrüber GmbH and the competent authority of the EU member state in which the user or patient is established.

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- Place the transfer board between the bed and the wheelchair and ensure that both support surfaces are secured against slipping/rolling away. Also make sure that the wheelchair is next to the bed at an angle of approx. 40 ° (fig. A).
- The distance between the bed and the wheelchair must not exceed 20 cm (fig. B). Place the cut-out of the board on the wheelchair wheel (fig. C). If available and possible, remove the arm and foot rests of the wheelchair.
- Now hold the transfer board with one hand (fig. D) and bend your upper body slightly and carefully towards the wheelchair (fig. E). Before each movement, check the stability of the board on both surfaces.
- We recommend the transfer to be carried out under the supervision of a helper.
- To remove the board, raise your buttocks and pull the board (fig. F).

05. Care, Cleaning and Disinfecting

Wash off the transfer board with a damp cloth, if needed with the addition of a mild cleaner. Also if necessary, disinfectants suitable for plastics may be used. Always allow the board to dry completely.

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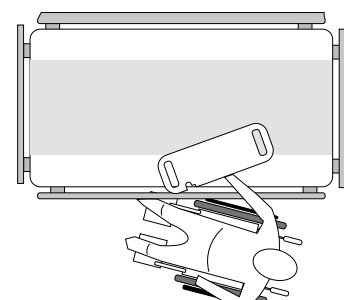
Fig. F

07. Re-Use


Rolko-EasyTRANSFER is suitable for re-use. Clean and disinfect the transfer board before each re-use. Please note that the exposure to UV radiation, chemical substances, mechanical stress, natural aging as well as careless handling may cause a premature wear on the board, which is a fact that not directly influenced by the manufacturer. For this reason, a replacement of the product is recommended every 10 years.

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09. Technical Data

Item no.: RBK-700-01
Length: 700 mm
Width: 230 mm
Thickness: 12 mm
Material: PE, natural colour
Load capacity: max. 130 kg
Delivery unit: 1 piece

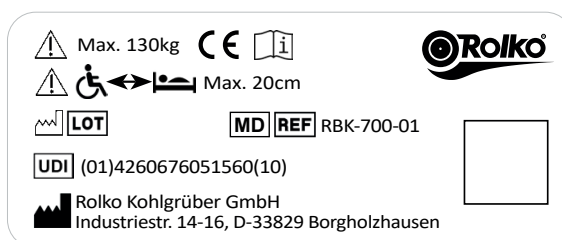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**© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen**
CH-REP. CH-Importeur: INNOVAhelp GmbH, Thunstr. 5a, CH-3110 Münsingen