
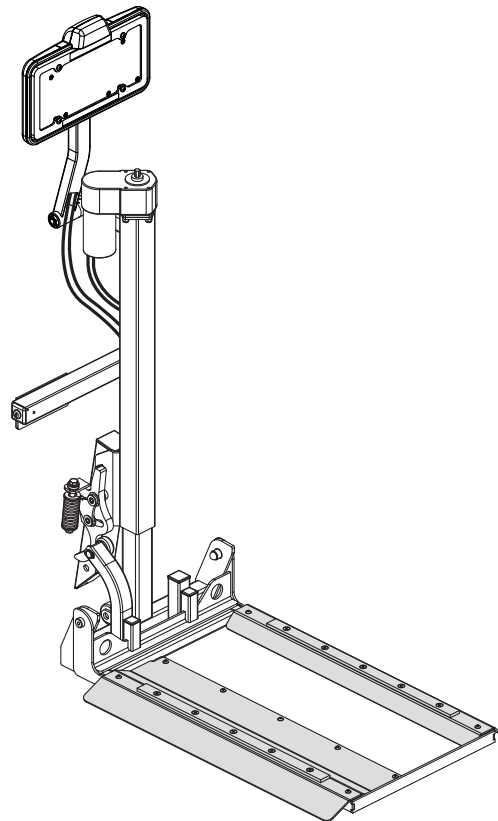
Outlander Series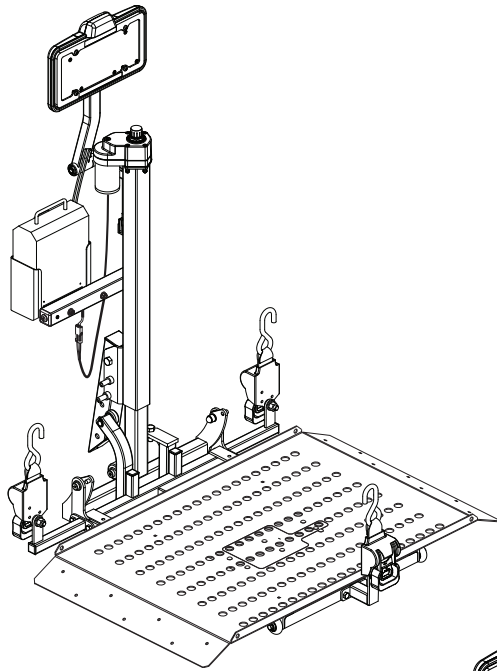

OWNER'S MANUAL

Including Models/Incluant des modèles:
**Outlander, Outlander DE, Outlander LP, and
Outlander XL**

Pride®
Lifts & Ramps
www.pridemobility.com

SAFETY GUIDELINES

WARNING! An authorized Pride Provider or qualified technician must perform the initial setup of this lift and must perform all of the instructions in this manual.

The symbols below are used throughout this owner's manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.

WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.

MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.

PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and red slash.

Please fill out the following information for quick reference:

Pride Provider: _____

Address: _____

Phone Number: _____

Purchase Date: _____ Serial Number: _____

***NOTE:** This owner's manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.*

***NOTE:** The Outlander Series Lift System and its components are not made with natural rubber latex. Consult with the manufacturer regarding any after-market accessories.*

CONTENTS

I. INTRODUCTION	4
II. SAFETY	5
III. INSTALLATION	10
IV. OPERATION	15
V. BATTERY CHARGING	18
VI. CARE AND MAINTENANCE	20
VII. TROUBLESHOOTING	22
VIII.WARRANTY	23
APPENDIX I - SPECIFICATIONS	24

I. INTRODUCTION

Welcome to Pride Mobility Products (Pride). Congratulations on the purchase of your new lift system. The lift system design combines the most advanced state-of-the-art components with modern, attractive styling. We are certain that the design features and trouble-free operation will add convenience to your daily living and ensure complete satisfaction.

At Pride, your safety is important to us. **Please read and follow all instructions in this manual before operating your lift system for the first time.** These instructions were produced for your benefit. Your understanding of these instructions is essential for the safe operation of your new lift system.

Pride is not liable for damage to property or personal injury arising out of the unsafe use of a lift system. Pride is also not liable for any property damage or personal injury arising out of the failure of any person and/or user to follow the instructions and recommendations set forth in this manual or any other instructions or recommendations contained in other lift system related literature issued by Pride or contained on the lift system itself.

INTERNET AND PRIVATE PURCHASES

If you purchased your product over the Internet or from a previous owner and you have any questions about the safe use and/or maintenance of the product, please visit our web site **www.pridemobility.com** or contact your authorized Pride Provider.

PURCHASER'S AGREEMENT

By accepting delivery of this product, you promise that you will not change, alter, or modify this product or remove or render inoperable or unsafe any guards, shields, or other safety features of this product; fail, refuse, or neglect to install any retrofit kits from time to time provided by Pride to enhance or preserve the safe use of this product.

SHIPPING AND DELIVERY

Before using your lift system, make sure your delivery is complete as some components may be individually packaged. If you do not receive a complete delivery, please contact your authorized Pride Provider immediately. Where damage has occurred during transport, either to the packaging or content, please contact the delivery company responsible.

INFORMATION EXCHANGE

We want to hear your questions, comments, and suggestions about this manual. We would also like to hear about the safety and reliability of your new lift system and about the service you received from your authorized Pride Provider.

Please notify us of any change of address, so we can keep you apprised of important information about safety, new products, and new options that can increase your ability to use and enjoy your lift system. Please feel free to contact us at the address below:

Pride Mobility Products Corporation
Attn.: Customer Care Department
182 Susquehanna Ave.
Exeter, PA 18643-2694
info@pridemobility.com

Pride Mobility Products Company
380 Vansickle Road
St. Catharines, Ontario L2R 6P7

NOTE: If you ever lose or misplace your copy of this manual, contact us and we will be glad to send you a new one immediately.

II. SAFETY

PRODUCT SAFETY SYMBOLS

The symbols below are used on the lift system to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand them completely.

Do not sit on mobility device while activating lift or during transport.

Do not open the swing-away hitch adapter with a mobility device on the lift.

Crush Hazard—Pinch/crush points created during assembly and operation. Keep hands clear.

Crush Hazard—Moving lift can crush. Keep feet clear during operation.

Read and follow the information in the owner's manual.

Maximum lifting capacity.

Do not use as a step.

Lift system intended for use with unoccupied mobility device. Do not use product to lift people.

II. SAFETY

Removal of grounding prong can create electrical hazard. If necessary, properly install an approved 3-pronged adaptor to an electrical outlet having 2-pronged plug access.


Do not connect an extension cord to the charger cord.

Do not allow unsupervised children to play near the lift system while the batteries are charging.

Avoid exposure to rain, snow, ice, salt, or standing water whenever possible. Maintain and store in a clean and dry condition.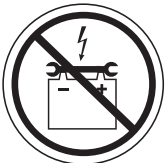

Keep tools and other metal objects away from battery terminals. Contact with tools can cause electrical shock.

Battery posts, terminals, and related accessories contain lead and lead compounds. Wear goggles and gloves when handling batteries and wash hands after handling.

Explosive conditions exist!

II. SAFETY


Disposal and recycling - Contact your authorized Pride Provider for information on proper disposal of your Pride product and its packaging.

Use only AGM or Gel-Cell batteries to reduce the risk of leakage or explosive conditions.

Corrosive chemicals contained in battery.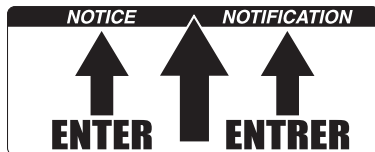

NOTICE! Enter platform where noted.

II. SAFETY

LIFTING CAPABILITIES

The lift system is an electromechanical device designed to lift various types of mobility devices for the purpose of vehicle transport. The Outlander is designed for use with Class II, Class III, or Class IV hitches. Check the hitch capabilities of your vehicle prior to use. Under no circumstances should the lift system be made to exceed this weight limit. Subjecting the lift system to the strain of hoisting more than it is designed to lift may cause it to fail, resulting in damage to the mobility device and/or injury to the lift operator. Refer to the mobility device owner's manual for information on the overall weight of the mobility device before lifting.

WARNING! Adding accessories, oversize batteries, or a different seat will increase the weight of your mobility device. Verify with your authorized Pride Provider that the total weight of your mobility device, including additions, does not exceed the maximum hitch allowance.

INSTALLATION

The lift system is designed to be mounted to a vehicle hitch system. Before lift installation, ensure that the hitch shows no signs of rust or wear and that the front and rear vehicle suspension is acceptable. If any vehicle alterations are necessary, make sure that you fully understand the instructions for drilling or cutting before making any alterations.

WARNING! Before drilling or cutting into the vehicle, make absolutely certain through visual inspection that there are no obstructions in the path of the drill bit, such as the fuel tank, exhaust pipes, or electrical wires.

LIFTING NON-PRIDE PRODUCTS

The lift system is an extremely versatile device, which users may employ to lift items other than Pride products. Pride has no control over such use, nor can Pride anticipate every possible use to which a lift system may be put. Lifting non-Pride products with the lift system is done at the operator's own risk, and Pride accepts no liability for damage or injury resulting from such use.

PRELIFT INSPECTION

Before operating the lift system, inspect your surroundings to ensure that the lift system has a clear path of operation. Remove any possible obstructions before operating the lift and do not allow children to operate or play near the lift.

WARNING! A frayed or worn lifting strap can snap, resulting in damage to the mobility product being lifted and injury to the lift operator.

OPERATOR POSITIONING

The operator of the lift should stand a safe distance from the unit being lifted/lowered to ensure that his/her feet are never positioned under the raised lift platform. The operator should also keep his/her hands clear of the lift system to avoid the potential for injury.

TRANSPORT VEHICLE POSITIONING

Be sure your vehicle is parked on flat, level ground before attempting to load, lift, or remove a mobility device with the lift system. Make sure the vehicle emergency park brake is engaged before loading or unloading a mobility device.

WARNING! Attempting to lift a mobility product when a vehicle is not on level ground will cause the mobility product to swing toward or away from the vehicle, making it difficult to get the mobility product into the vehicle.

II. SAFETY

HITCH CLASS INFORMATION

Hitches come in a variety of sizes and are designated by class. The classes range from Class I to Class IV. A heavier load requires a larger class, which has a higher *tongue weight*. Tongue weight is the amount of weight (downward force) that the load puts on the hitch. See figure 1.

The combined weight of your lift and mobility device gives you the **minimum** tongue weight for your application. Exceeding the tongue weight of a hitch creates an unsafe driving situation and can damage the hitch and/or the vehicle. See figures 2 and 3.

WARNING! Adding accessories, oversize batteries, or a different seat will increase the weight of your mobility device. Verify with your authorized Pride Provider that the total weight of your mobility device, including additions, does not exceed the maximum weight allowance.

WARNING! Never exceed the maximum lifting capacity of a lift regardless of the class of hitch being used. Doing so may cause lift and/or vehicle damage.

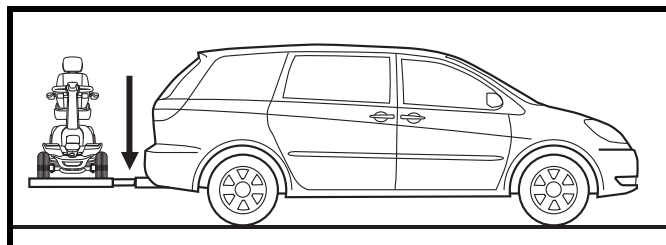
Figure 1. Tongue Weight Pressure Point

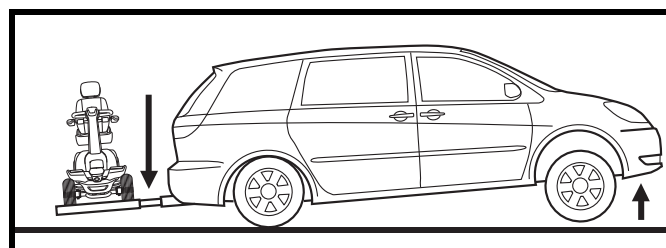
Figure 2. Unsafe Driving Situation

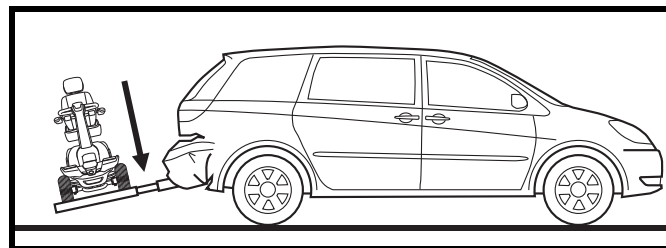
Figure 3. Vehicle Damage

Hitch Classes

- A Class I hitch has a maximum tongue weight of 200 lbs. (90.71 kg).
- A Class II hitch has a maximum tongue weight of 350 lbs. (158.75 kg).
- A Class III hitch has a maximum tongue weight of 500 lbs. (226.79 kg).
- A Class IV hitch has a maximum tongue weight of 1200 lbs. (544.31 kg).

WARNING! Before installing the lift and applicable hitch systems, check your vehicle owner's manual for hitch maximum tongue weight capacity. Do not exceed the maximum tongue weight capacity of your vehicle.

III. INSTALLATION

LIFT SYSTEM INSTALLATION

The Outlander is designed to be easily installed into any vehicle equipped with a Class II, Class III or Class IV hitch receiver. The hitch receiver should be installed no higher than 21 in. (53.34 cm) from the bottom of the hitch tube to the ground.

WARNING! The lift system should be installed by an authorized Pride Provider or service technician only.

Follow these steps to install the lock-down arm:

1. Lay the lift system flat on the floor.

NOTE: If your lift system is equipped with a permanent threaded stud, removal of the pivot tube and pivot bolt is not necessary for lock-down arm installation. See figure 4.

2. Apply power to the lift until there is enough clearance to remove the pivot bolt. See figure 4.

NOTE: If the pivot bolt does not remove easily, apply power to the lift and make a slight adjustment to the angle of the pivot tube. Repeat as necessary.

3. Remove the pivot bolt. Keep hardware for reassembly.
4. Remove the pivot tube. See figure 4.
5. Insert the lift arm securement screw through the countersunk hole of the U-bracket (from the inside out).
6. Position the lock-down arm onto the screw and insert the pivot bolt through the U-bracket and lock-down arm. The pivot bolt is used for alignment purposes at this time.
7. Secure the nut to the securement screw and install the cap. Remove the pivot bolt.
8. Position the pivot tube onto the U-bracket and reinsert the pivot bolt through the U-bracket, pivot tube, and lock-down arm.
9. Install the nut back onto the pivot bolt and install the cap. Do not overtighten the nut. Ensure the pivot tube pivots freely without binding.
10. Attach the supplied warning label to the lift platform. See figure 5.
11. Adjust the tension bolt on the H-frame that actuates the lock-down arm clevis pin. This adjustment should compress the spring on the lock-down arm pivot foot to ensure proper securement of the scooter to the lift.

WARNING! Improper adjustment of the tension bolt may affect the securement of the scooter to the lift!

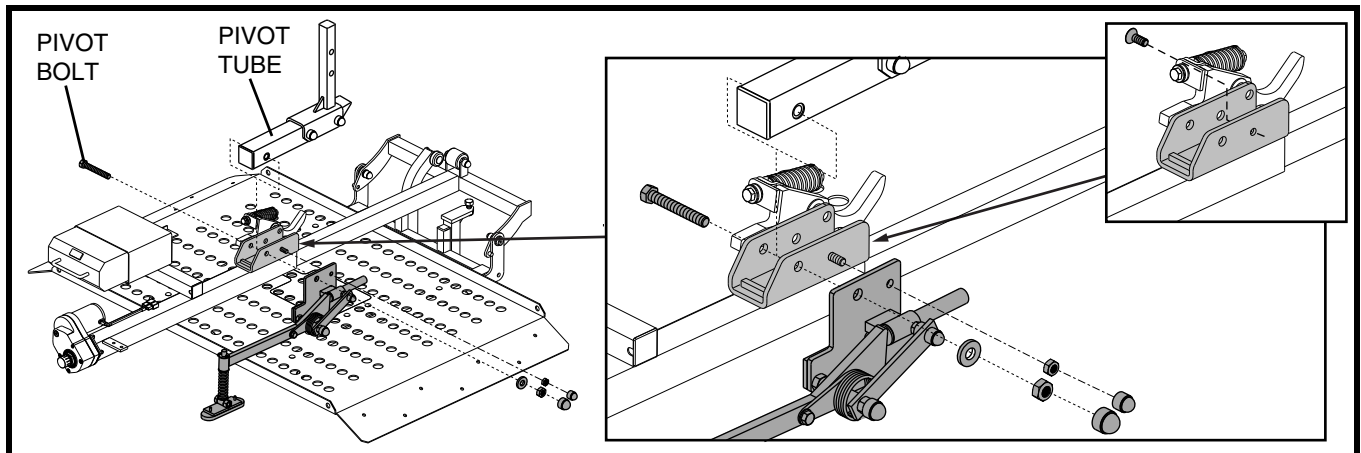
Figure 4. Lock-down Arm Installation

III. INSTALLATION

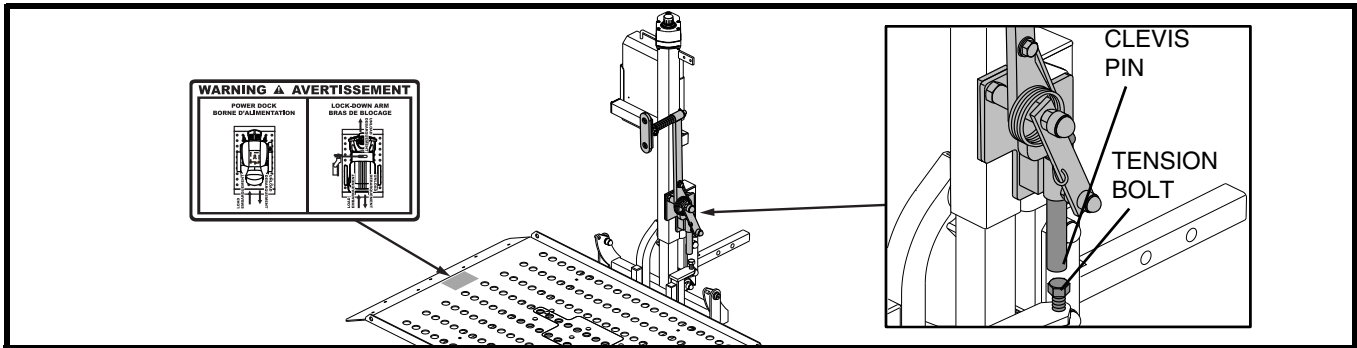
Figure 5. Tension Bolt and Clevis Pin Adjustment

Follow these steps to install the lift system:

1. Remove the preassembled mounting hardware from the hitch tube.
2. Mount the hitch tube to the lift frame using the previously removed mounting hardware. Install the hardware in the direction indicated in **figure 6**.
3. Slide the hitch tube into the vehicle hitch receiver, aligning the mounting hole closest to the lift with the mounting hole in the hitch receiver. See **figure 7**.

WARNING! Lift system components are heavy. Ask for assistance if necessary when attaching the lift frame to the hitch receiver. Use proper lifting techniques and avoid lifting beyond your capacity.

WARNING! Avoid pinch points! Do not hold the lift frame by the pivot points when mounting the lift frame to the hitch receiver.

4. Secure the hitch tube with the supplied bolt and nut, inserting the bolt from the threaded side of the hitch tube. See **figure 7**.

WARNING! Threading the bolt from the wrong side or allowing the bolt to turn while tightening the nut will dislodge the threaded insert in the hitch tube.

WARNING! Do not attach a Class II hitch tube to a Class III hitch receiver.

NOTE: Make sure the hitch bolt is tightened firmly before securing the nut to the hitch assembly to minimize the chance that the lift will tilt during transport.

NOTE: Make sure the hitch tube and lift platform are parallel to the ground before operating the lift system. An angled hitch tube may cause unreliable lift operation and/or damage to the lift system.

5. Route the lift system wiring harness through the vehicle. Refer to “Wiring Harness Installation.”

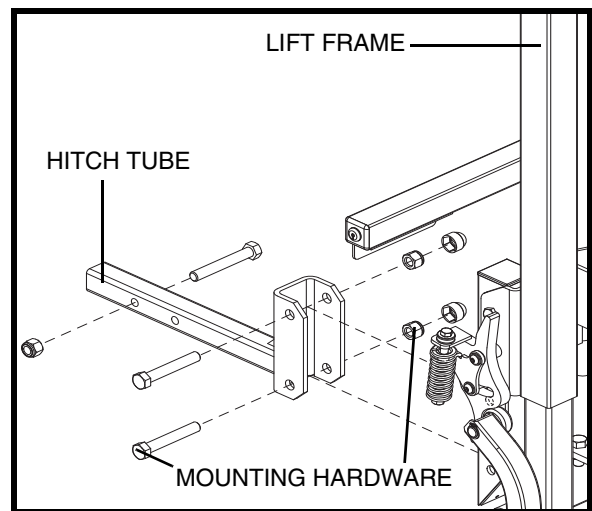
Figure 6. Hitch Tube Assembly

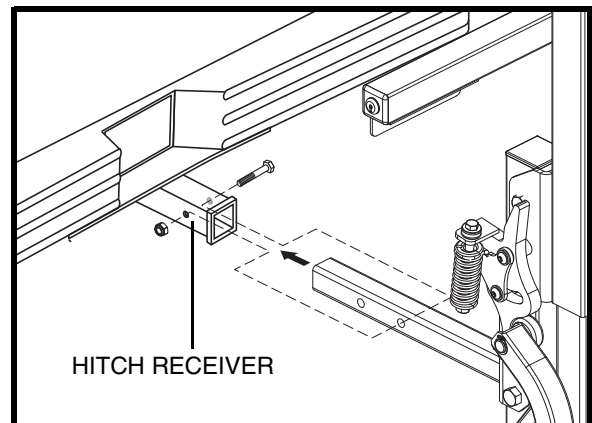
Figure 7. Lift Frame Vehicle Mounting

III. INSTALLATION

WARNING! Route the wiring harness through the vehicle rather than under the vehicle to avoid coming in contact with sharp edges, extreme temperatures, moving parts, and roadway debris including road salt and other highly corrosive materials. Power shorts may occur if wires come in contact with hot exhaust parts or sharp edges.

WARNING! Never attach the wiring harness to a secondary power source. Do not attempt to use trailer wiring to power the lift system. The wiring harness must be connected directly to the vehicle battery.

NOTE: If your lift is equipped with an onboard battery, refer to “Onboard Battery Installation.”

6. Install the wheel chocks to the lift platform. Refer to “Wheel Chock Installation.”

Wiring Harness Installation

The wiring harness is approximately 25 ft. (7.62 meters) long and will accommodate most vehicles.

NOTE: You may wish to perform a practice run before installing the wiring harness. Route a piece of light rope (equivalent to the gauge of the wiring harness) along the anticipated path observing contact points, potential rubbing/chafing points, and any sharp edges. Remove sharp edges with a fine grade file, then treat the steel with a rust inhibitor or metal sealant.

Follow these steps to route the lift system wiring harness:

1. Route the lift system wiring harness (starting at the lift) through the interior of the vehicle until you reach the vehicle battery. **See figure 8.** You will typically find pass-through points at the rear lighting assemblies, which will enable you to route the cable through the trunk and under the back seat.
2. Conceal the harness behind or under the interior panels of the vehicle (there should be existing holes). Be certain that the harness is protected with a rubber grommet when passing it through the metal panels and into the engine compartment.
3. Inside the engine compartment, secure the harness to the firewall and the inner fender with plastic wire ties. Use care not to cause abrasions to the wiring harness. It is important to secure the wiring harness at various points along its run of the vehicle.

WARNING! Do not cut or shorten the wiring harness. If the harness is too long, coil the excess wire and secure it with plastic wire ties.

4. Connect the red wire to the positive (+) terminal and the black (grounding) wire to the negative (-) terminal of the vehicle battery.

WARNING! Before operating the lift, inspect the wiring harness for proper routing and grounding. Improper routing can cause damage to the harness, and improper grounding can cause damage to the electrical system.

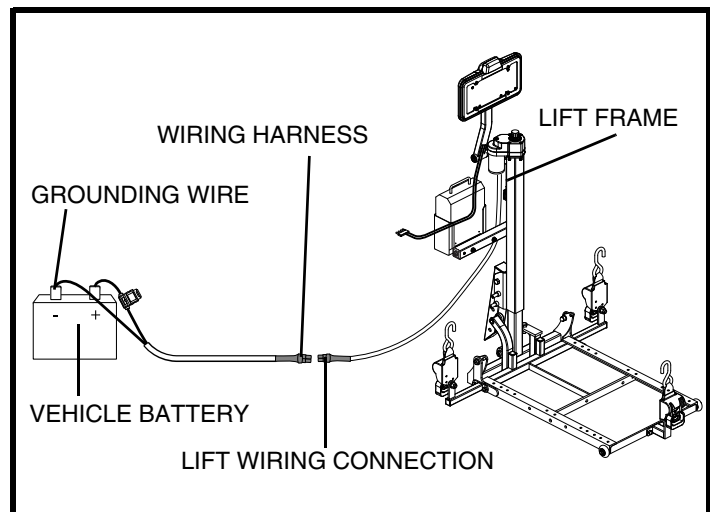
Figure 8. Wiring Harness Installation

III. INSTALLATION

License Plate Holder

The license plate holder should be rotated to the upright position during installation. **See figure 9.**

1. Pull the pivot weldment out to disengage keyway. You may need to loosen the pivot nut with a 3/4-in. socket wrench. **See figure 9A.**
2. Rotate the license plate holder to its upright position. Once in the upright position, the license plate holder will automatically lock into place. **See figure 9.**
3. Ensure the keyway is engaged then tighten the pivot nut if previously loosened. **See figure 9A.**
4. Remove the license plate from your vehicle and attach it to the license plate holder using the supplied fasteners.
5. Insert the lighting harness into the vehicle's trailer hitch harness.

Onboard Battery Installation

The optional onboard battery is designed for easy installation as it does not require any wires to be routed through or under the vehicle.

Follow these steps to install the onboard battery:

1. Align the mounting holes in the battery case bracket with those in the accessory bracket located under the key switch tube. **See figure 10.**
2. Secure the battery case to the accessory bracket with the supplied hardware. **See figure 10.**
3. Insert the onboard battery into the battery case, making sure that the connector on the bottom of the battery aligns and fully connects with the mating connector inside the battery case.
4. Plug the battery cord on the bottom of the battery case into the mating connector running from the motor. **See figure 10.**

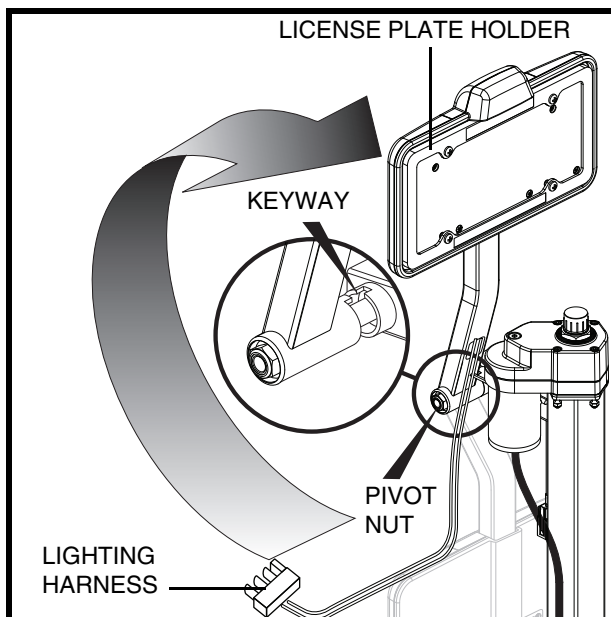
Figure 9. License Plate Holder

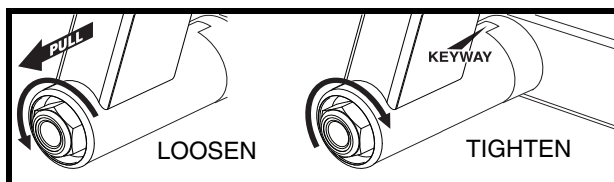
Figure 9A. Pivot Weldment

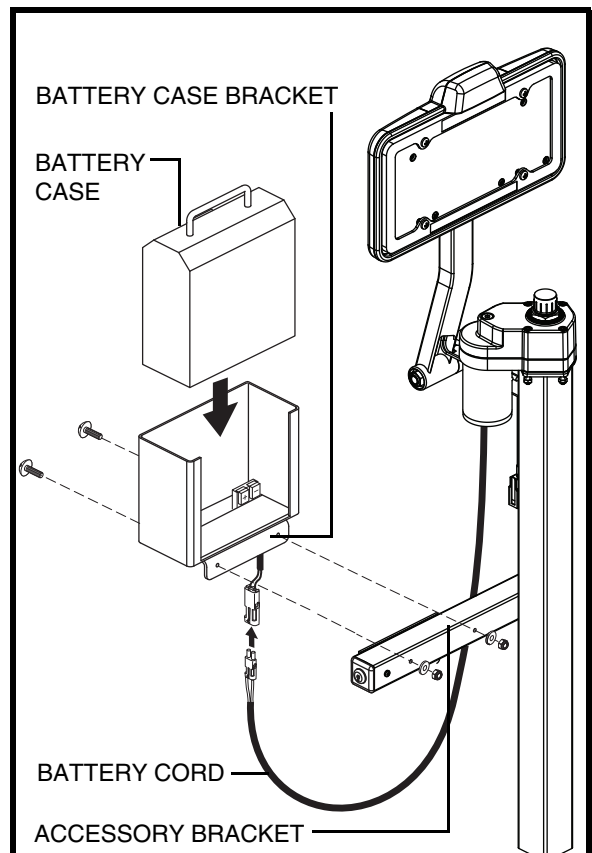
Figure 10. Onboard Battery Installation

III. INSTALLATION

Wheel Chock Installation

Your lift is supplied with four adjustable wheel chocks, that when installed, aid in centering your mobility device during loading.

Follow these steps to install the wheel chocks:

1. Drive your mobility device onto the lift platform, positioning it as near to the center as possible.
2. Position the wheel chocks on the platform.
Scooters: Place a wheel chock in front of the front wheel(s) and behind the rear wheels or behind the front wheels and in front of the rear wheels.
Power chairs: Place a wheel chock in front of and behind the drive wheels.
3. Take note of the adjustment holes the wheel chocks align with on the platform and remove the chocks.
4. Remove the mobility device from the lift platform.
5. Raise the platform high enough to be able to install hardware from beneath it.
6. Position the wheel chock bottom on the underside of the platform, aligning it with the wheel chock top. **See figure 11.**
7. Secure the wheel chock sections to the platform using the supplied hardware. **See figure 11.**
8. Repeat these steps to install the remaining wheel chocks to the lift platform.

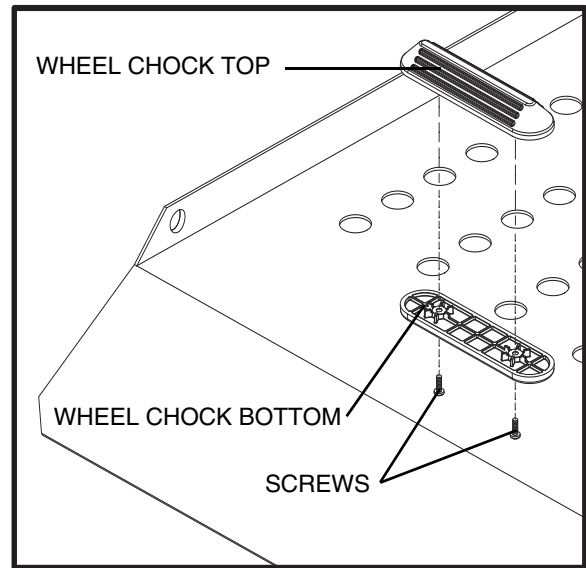
Figure 11. Wheel Chock Installation

NOTE: The Outlander LP does not require wheel chock installation. The platform's design supports the mobility device wheels so that wheel chocks are not required. See figure 12.

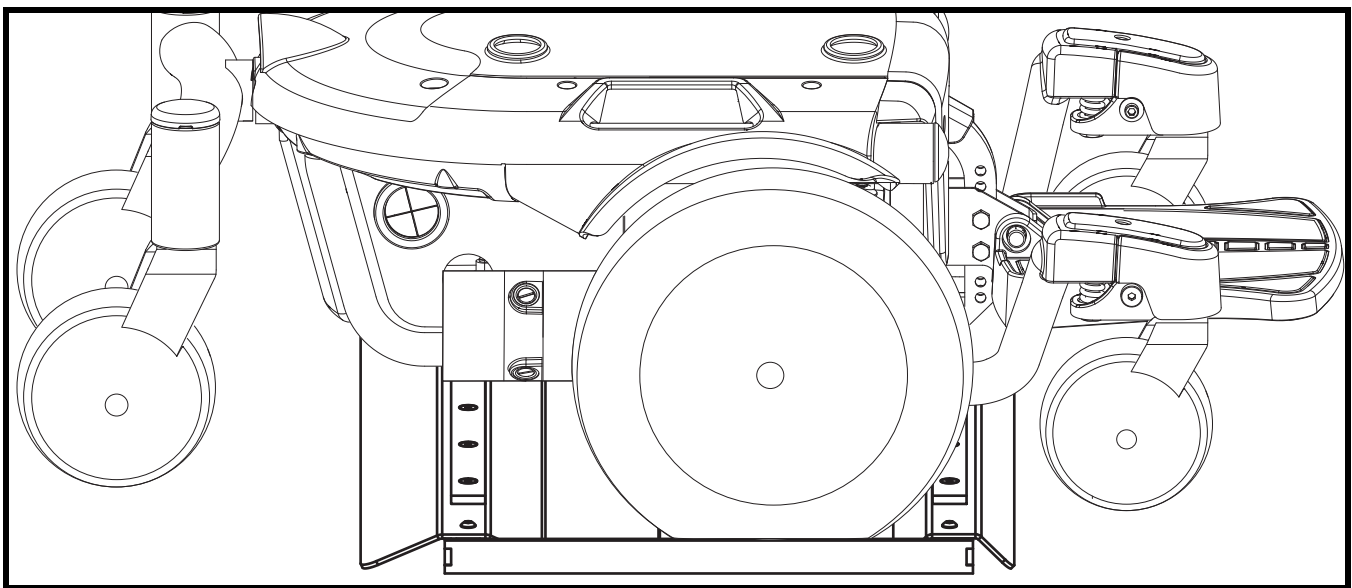
Figure 12. Wheel Placement on Outlander LP Platform

IV. OPERATION

LIFT SYSTEM OPERATION

WARNING! The lift system is intended for the transport of mobility devices only. The mobility device must be unoccupied before operating the lift.

WARNING! Do not exceed the load limits of your vehicle or hitch system as specified by the vehicle manufacturer.

WARNING! Use extreme caution when driving with the lift installed on your vehicle. Be aware that the lift extends away from the back of the vehicle and extra clearance is needed when backing up, turning, parking, or driving through driveway entrances/exits. Drive slowly when negotiating pot holes, road bumps, and railroad tracks.

Follow these steps to operate the lift system:

1. Insert the key into the key switch.
2. Turn and hold the key clockwise to lower the lift platform. See figure 13. Release the key as soon as the lift platform touches the ground.

Loading and Securing Your Mobility Device

1. Set the speed control of the mobility device to the slowest setting and carefully drive your mobility device onto the lift platform, being sure to center the mobility device on the platform to the best of your ability.

NOTE: You may wish to load the mobility device while standing next to it using the manual freewheel feature. Refer to the mobility device owner's manual for more information on manual freewheel mode.

WARNING! If the lift is equipped with an automatic lock-down arm, load the scooter onto the lift platform from the driver's side of the vehicle to ensure proper contact of the automatic lock-down arm with the scooter floorboard. Loading the scooter from the passenger side will cause the automatic lock-down arm to interfere with the scooter seat when the platform is raised.

WARNING! If the lift is equipped with the power dock option, the power chair must be loaded and unloaded from the driver's side of the vehicle. Loading the power chair from the passenger side will not allow for proper securement of the device to the power dock adaptor.

2. Shut down the power to the mobility device, remove the key if applicable, and make sure the unit is in drive mode (drive motors engaged). Refer to the mobility device owner's manual for more information on drive mode.

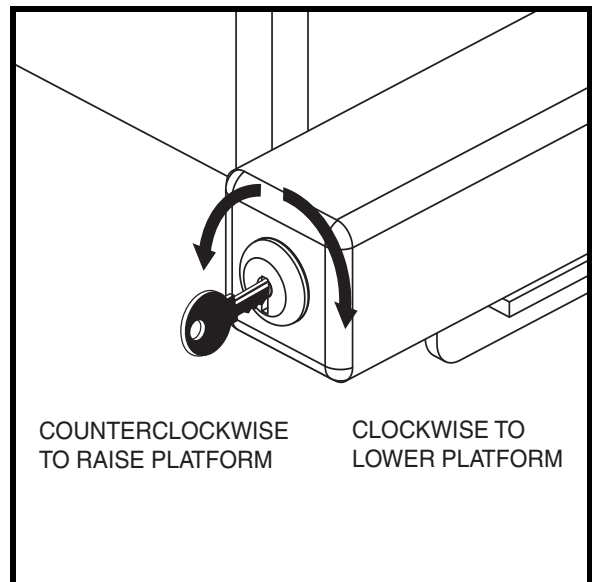
Figure 13. Key Switch Operation

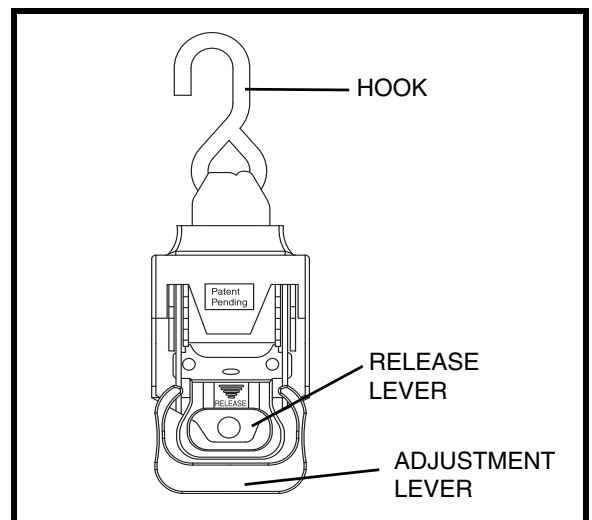
Figure 14. Retractable Tie-down Strap

IV. OPERATION

WARNING! Never leave a mobility device in manual freewheel mode during lift system transport, as the mobility device could roll.

3. Secure the mobility device to the platform using either the retractable tie-down straps (see figure 14) for power chairs or the automatic lock-down arm for scooters.
- Squeeze together the release and adjustment levers on the retractable tie-down strap and simultaneously bring both levers forward to a 90-degree position until they “click” into place. See figures 14 and 15. This will unlock the tie-down strap.
 - Pull the strap out and secure it to the chair restraint tab. See figure 16.
 - Once each strap is secured in the proper position, squeeze together the release and adjustment levers and simultaneously bring both levers rearward to their original positions. See figures 14 and 17. This will lock the tie-down strap.
 - To tighten the strap, rotate only the adjustment lever forward and rearward in a ratcheting motion until the tie-down strap reaches an adequate level of resistance for mobility device securement.

WARNING! Never attempt to secure the tie-down straps over the seatback or armrests of a mobility device.

WARNING! For lift systems equipped with the automatic lock-down arm, do not attempt to adjust the pressure of the arm. This feature has been configured by your authorized Pride Provider to meet your needs and requires no further adjustment.

WARNING! If your scooter is equipped with pneumatic drive tires, make sure the tires are inflated to the recommended pressure labeled on the tire sidewall. This will ensure that the scooter is at the correct deck height for proper lock-down arm positioning.

NOTE: Inspect the tie-down straps before each use to ensure there are no signs of fraying or wear. Ensure that the mobility device is secured properly before transport. Pride is not liable for damage or loss resulting from improper securing of a mobility device.

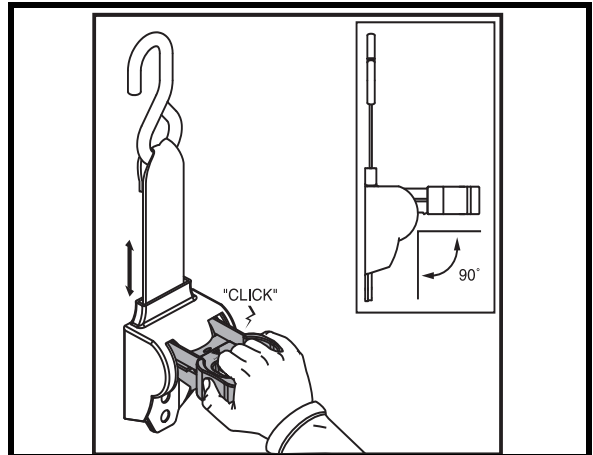
Figure 15. Unlocking the Retractable Tie-down Strap

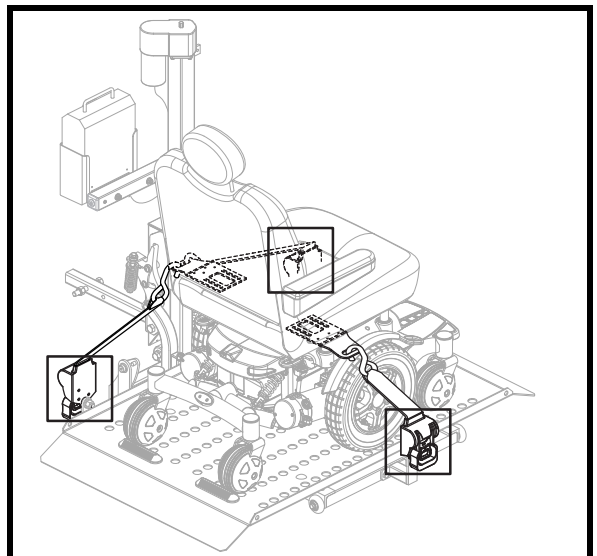
Figure 16. Retractable Tie-down Strap Securement

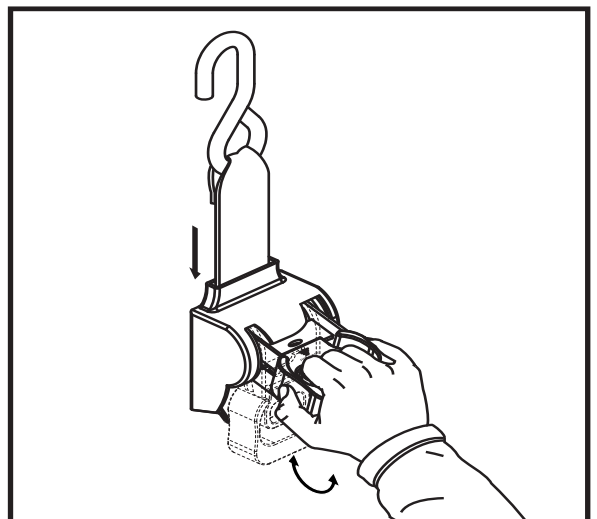
Figure 17. Locking the Retractable Tie-down Strap

IV. OPERATION

4. Raise the lift platform by turning and holding the key in the counterclockwise direction. See figure 13. Continue to raise the lift until the motor stops, then release the key.

NOTE: As the lift platform rises, the automatic lock-down arm activates to secure the scooter to the lift platform. You may need to adjust the scooter's tiller position to prevent interference with the smooth operation of the automatic lock-down arm.

NOTE: The motor will stop automatically at the top of its stroke and will emit a clicking sound. Release the key upon hearing this sound.

NOTE: Pay attention to the path of the lift platform during operation. Make sure the lift does not rub against or interfere with the vehicle in any way. If you notice any contact between the lift platform and the vehicle, stop lift operation immediately and contact your authorized Pride Provider for assistance.

NOTE: The Outlander LP is the only model that provides a hold-down bar to secure power chair products during transport as a standard feature. See figure 18.

Follow these steps to remove a mobility device from the lift system:

1. Lower the lift platform to the ground.
2. Unfasten the tie-down straps or allow the lock-down arm to automatically return to the vertical position.
 - Unlock and loosen the tie-down straps.
 - Rotate the **adjustment lever only** on each strap forward and rearward in a ratcheting motion until each strap reaches its original furled position.
3. Unload the mobility device from the lift platform.

WARNING! Do not attempt to manually pull the lift platform down from the stowed position. Doing so will result in product damage and will void the product warranty.

4. Raise the lift platform for storage, noting that it will automatically fold into a stowed position.

MANUAL LIFT SYSTEM OPERATION

The lift system is equipped with a manual crank that serves as backup in the event of a power failure. Using the supplied tool, rotate the manual crank (located on top of the motor housing) clockwise or counterclockwise to move the platform up or down. See figure 19.

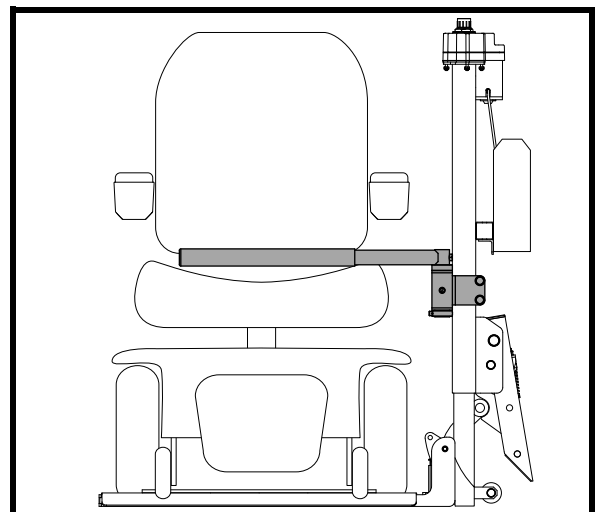
Figure 18. Power Chair Hold-down Bar

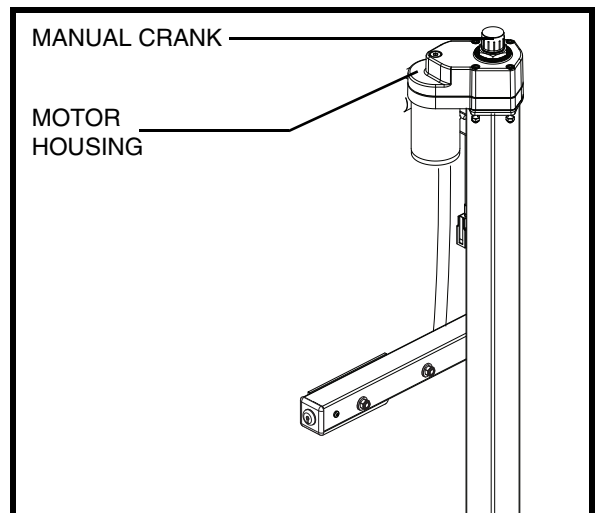
Figure 19. Manual Crank Operation

V. BATTERY CHARGING

BATTERY CHARGING

Follow these steps to charge the onboard battery:

1. Remove the onboard battery from the battery case and place it near a standard electrical outlet.
2. Plug the charger cord into the mating socket located on the bottom of the battery case, then connect the charger cord to a standard electrical outlet. See figure 20.

NOTE: It is recommended that you replace the battery every 3–5 years. Store the battery in a moisture-free environment when not in use.

MANDATORY! Read the battery charging instructions in this manual and in the manual supplied with the battery charger before charging the battery.

PROHIBITED! Never use an extension cord to plug in your charger cord. Plug the charger cord directly into a properly wired standard electrical outlet.

PROHIBITED! Removal of the grounding prong can create an electrical hazard. If necessary, properly install an approved 3-pronged adaptor to an electrical outlet having 2-pronged plug access.

PROHIBITED! Do not allow unsupervised children to play near the lift system or battery charger while the battery is charging.

PROHIBITED! Do not expose the battery charger to rain or other sources of moisture unless it has been tested for outdoor use. Refer to the manual supplied with the battery charger for more information.

WARNING! Explosive gases may be generated while charging the battery. Keep the battery and the battery charger away from sources of ignition such as flames or sparks and provide adequate ventilation when charging the battery.

WARNING! Do not attempt to charge the battery and operate the lift system at the same time.

WARNING! Do not attempt to wire the lift system directly to the vehicle battery if the lift system is equipped with an onboard battery.

WARNING! You must recharge the lift system battery with the supplied battery charger. Do not use an automotive-type battery charger.

WARNING! If the battery is exposed to adverse or extreme weather conditions, then it must be allowed to adjust to the difference in environmental conditions before being charged.

WARNING! Inspect the battery charger, wiring, and connectors for damage before each use. Contact your authorized Pride Provider if damage is found.

WARNING! Do not attempt to open the battery charger case. If the battery charger does not appear to be working correctly, contact your authorized Pride Provider.

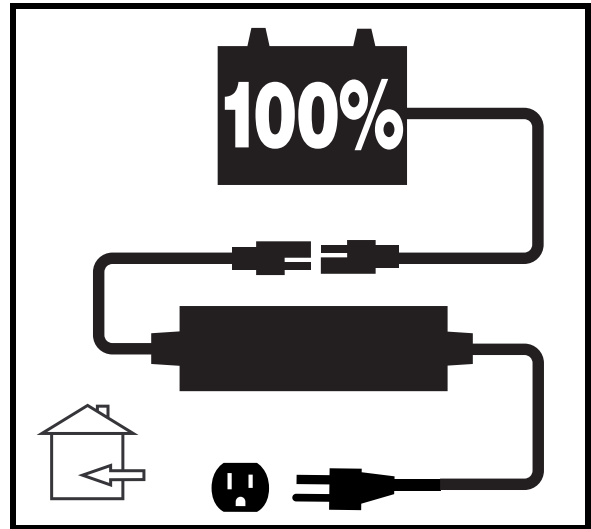
Figure 20. Battery Charging

V. BATTERY CHARGING

BATTERY REPLACEMENT

WARNING! Battery posts, terminals, and related accessories contain lead and lead compounds. Wear goggles and gloves and wash hands after handling.

MANDATORY! Use only AGM or Gel-Cell batteries to reduce the risk of leakage or explosive conditions.

WARNING! Corrosive chemicals contained in battery.

WARNING! Keep tools and other metal objects away from battery terminals. Contact with tools can cause electrical shock.

WARNING! Connect the battery harnesses in the proper manner. **RED (+)** cables must be connected to positive (+) battery terminals/posts. **BLACK (-)** cables must be connected to negative (-) battery terminals/posts. **REPLACE** cables immediately if damaged. Protective caps must be installed over all battery terminals.

NOTE: If you encounter a damaged or cracked battery, immediately enclose it in a plastic bag and contact your local waste disposal agency or your authorized Pride Provider for instructions on disposal and battery recycling, which is our recommended course of action.

Follow these steps to replace the onboard battery (see figure 21):

1. Unfasten the battery securement strap located at the bottom of the battery box.
2. Remove the hardware from the battery securement bracket and remove the bracket.
3. Remove the hardware from the battery connector.
4. Slide the battery out of the battery box to expose the battery harness.
5. Disconnect the battery harnesses from the battery terminals.
6. Connect the battery harnesses to the new battery.
7. Slide the new battery into the battery box.
8. Reinstall the hardware to secure the battery connector and tighten.
9. Reinstall the battery securement bracket and tighten the hardware.
10. Refasten the battery securement strap.

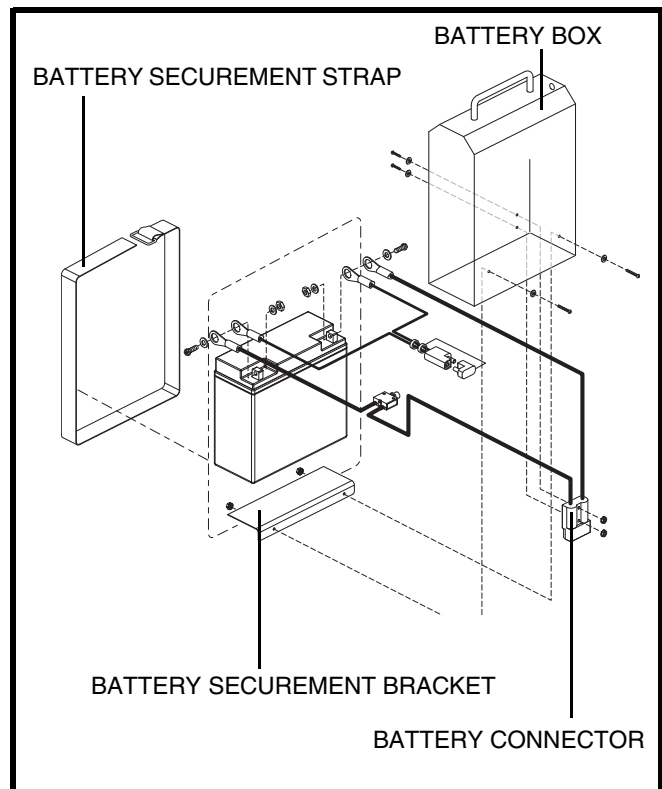
Figure 21. Battery Replacement

VI. CARE AND MAINTENANCE

CARE AND MAINTENANCE

Your lift system requires a minimal amount of care and maintenance. If you do not feel confident in your ability to perform the maintenance listed below, you may schedule inspection and maintenance at your authorized Pride Provider. The following areas require periodic inspection and/or care and maintenance.

Daily Maintenance Schedule

This lift system may come equipped with any of the securement devices listed below. Inspect these devices for loose or damaged parts on a daily basis. Have your authorized Pride Provider repair or replace any problem parts before using the lift system.

- Retractable Tie-down straps
- Lock-down arms
- Power dock

90-Day Maintenance Schedule

- Clean and lubricate the T-bar inner telescoping tube. Use all-weather lithium grease only. **See figure 22.**
- Lubricate all pivot points on the lift. Use all weather lithium grease only.
- Inspect and clean the power-supply connections as necessary.
- Inspect all hardware and tighten or replace as necessary.
- Visually inspect lift system for loose or damaged parts. Have your authorized Pride Provider repair or replace any problem parts before using the lift system.

Two-Year Maintenance Schedule

If your lift is equipped with a power dock, replace the securment bolt in the securement bracket located on your mobility device.

Five-Year Maintenance Schedule

Inspect the “Spring-Activated Mechanical Switch” and lubricate or adjust spring tension as necessary. **See figure 23.**

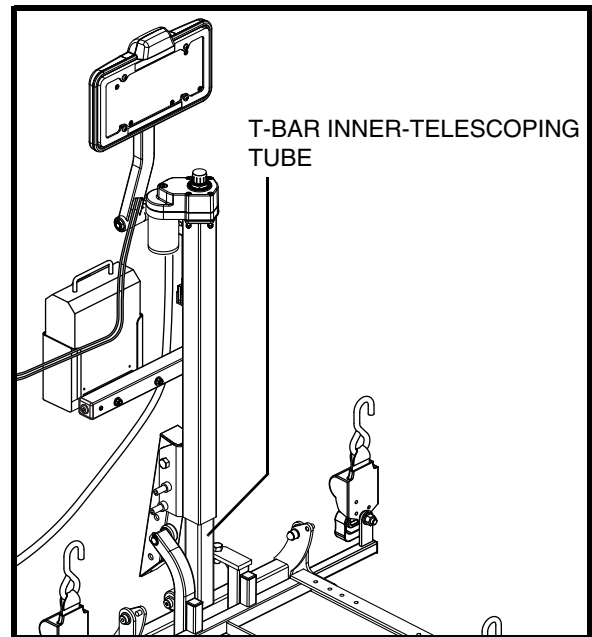
Figure 22. Lift Frame Maintenance

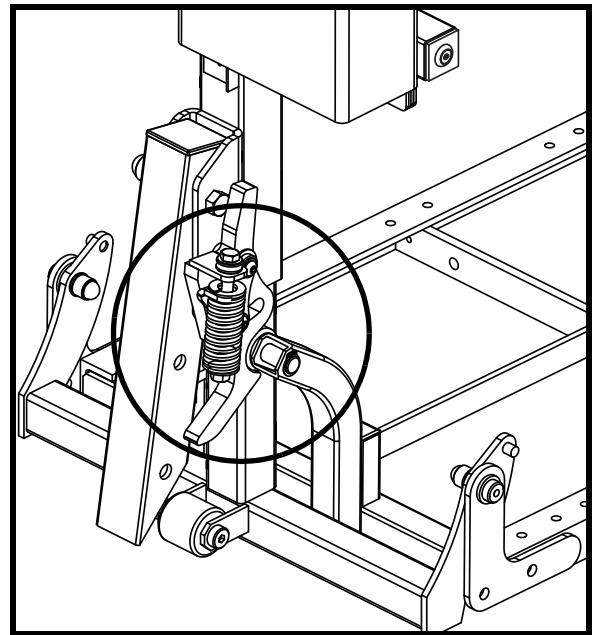
Figure 23. Spring-activated Mechanical Switch

VI. CARE AND MAINTENANCE

General Cleaning Guidelines

The lift system can be hand washed along with the vehicle with soap and water. However, automatic drive-through car washes should be avoided as they may interfere with and cause damage to the lift.

Wiring Harness

The wiring harness should be checked periodically for corrosion and/or decay. Contact your authorized Pride Provider if the wiring harness needs to be replaced.

Lift Platform/Metal Components

- Check the lift platform for wear or damage. Contact your authorized Pride Provider for repair or replacement.
- The lift platform surface may become slippery if exposed to moisture (ice, snow, rain, mist/spray conditions, etc.). Use caution when loading/unloading a mobility device in these conditions and wipe off any excess moisture from the lift platform.
- Regularly inspect the metal lift components for scratches. Treat any problem areas with a rust inhibitor or metal sealant.

If your lift system is exhibiting wear that you feel is questionable in any way, contact your authorized Pride Provider immediately. Do not attempt to repair, adjust, or modify the manufacturer settings and materials.

Lift Removal

- Raise platform to stowed position.
- Disconnect power source.
- Lubricate hitch mount bolt with WD-40 prior to removal.

VII. TROUBLESHOOTING

TROUBLESHOOTING

Any electromechanical device occasionally requires some troubleshooting. However, most of the problems that may arise can usually be solved with a bit of thought and common sense. Many of these problems occur because the battery is not fully charged or because the battery is worn down and can no longer hold a charge.

What if the motor on my lift system will not operate?

- Make sure the onboard battery or vehicle battery is fully charged.
- Make sure all connections to the motor and battery are secure.
- Check the wiring harness for signs of wear or decay. Replace the harness if necessary.

What if the lift system operates slowly up or down?

- Make sure the onboard battery or vehicle battery is fully charged.
- Make sure all connections to the motor and battery are secure.
- Check the wiring harness for signs of wear or decay. Replace the harness if necessary.

What if the lift platform does not fold into the stowed position?

Check to make sure there are no obstructions in the T-bar pivot points.

What if the lift platform does not lower completely to the ground?

- Verify that the hitch receiver is at the proper height. The distance from the bottom of the hitch to the ground should not exceed 21 in. (53.34 cm).
- Check to make sure that the area under the lift platform is free from obstructions.

What if the lift bottoms out when driving over bumps or driveway entrances?

Verify that the hitch receiver is at the proper height. The distance from the bottom of the hitch to the ground should be 21 in. (53.34 cm). Installing the riser tube adaptor will provide an additional 4 inches (10.16 cm) of rise to the lift, keeping the lift from bottoming out.

VIII. WARRANTY

THREE-YEAR TRANSFERABLE LIMITED WARRANTY

For three (3) years from the date of purchase, Pride Mobility Products will repair or replace at our option, free of charge, any mechanical or electrical component found upon examination by an authorized representative of Pride to be defective in material and/or workmanship.

This warranty does not extend to those items, which may require replacement due to normal wear and tear.

- Labor, service calls, shipping, and other charges incurred for repair of the product, unless specifically authorized by Pride Mobility Products IN ADVANCE, are excluded.

Exclusions also include components with damage caused by:

- Contamination
- Abuse, misuse, accident, or negligence
- Battery fluid spillage or leakage
- Commercial use, or use other than normal
- Improper operation, maintenance, or storage
- Repairs and/or modifications made to any part without specific consent from Pride Mobility Products
- Circumstances beyond the control of Pride

Pride cannot be held responsible for any defects in the motor vehicle in which the lift system is installed, defects in the lift system and/or securement kit caused by defects in any part of the motor vehicle, or damage to the motor vehicle caused by installation or operation of the lift system.

ONE-YEAR WARRANTY

The battery is covered by a separate one (1) year warranty provided by the battery manufacturer. The batteries are not warranted by Pride Mobility Products.

SERVICE CHECKS AND WARRANTY SERVICE

An authorized Pride Provider must perform warranty service. Do not return faulty parts to Pride Mobility Products without prior written authorization. All transportation costs and shipping damage incurred while submitting parts for repair or replacement is the responsibility of the purchaser.

Failure to follow the instructions, warnings, and notes in the owner's manual and those located on your Pride lift product can result in personal injury or product damage and will void Pride's product warranty.

There is no other express warranty.

IMPLIED WARRANTIES

Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to one (1) year from the date of purchase and to the extent permitted by law. Any and all implied warranties are excluded. This is the exclusive remedy. Liabilities for consequential damages under any and all warranties are excluded.

Some states do not allow limitations on how long an implied warranty lasts or do not allow the exclusion or limitation of incidental or consequential damages. The above limitation or exclusion may not apply to you.

This warranty gives you specific rights, and you may also have other rights, which vary from state to state.

Please fill out and return the product registration card to Pride Mobility Products. This will aid Pride in providing the best possible technical and customer service.

APPENDIX I - SPECIFICATIONS

SPECIFICATIONS	
Lift System Dimensions ¹	Overall Height: 50 in. (127 cm) Overall Width: Standard models: 49.25-56.75 in. (125.1-144.14 cm) from hitch with platform down 20 in. (50.80 cm) from hitch with platform-folded LP model: 28 in. (71.12 cm) Platform Width: Standard models: 28 in. (71.12 cm) LP model: 19.125 in. (48.58 cm)
Weight Capacity	350 lbs. (158.75 kg)
Maximum Mobility Device Width	27 in. (68.58 cm)
Maximum Hitch-to-Ground Clearance	21 in. (53.34 cm)
Motor	Fully sealed, 12-volt DC
Onboard Battery (Optional)	12-volt, 18 Ah
Charger	110-volt, off-board
Distance from Hitch Tube to Front	35 in. (88.9 cm)
Lift System Weight ¹	Outlander: 92 lbs. (41.73 kg) Outlander DE: 92 lbs. (41.73 kg) Outlander XL: 92 lbs. (41.73 kg) Outlander LP (10-inch): 76 lbs. (34.47 kg) Outlander LP (14-inch): 75 lbs. (34 kg)

¹ Due to manufacturing tolerances and continual product improvement, this specification can be subject to a variance of (+ or -) 3%.

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Copyright © 2013