

LAY ON SIDE, PULL OUT



****DO NOT TILT THE BOX UPSIDE DOWN AND TRY TO USE GRAVITY TO REMOVE THE CHAIR, IT WILL RESULT IN DAMAGE TO THE CHAIR****

- Open the top of the box as received and remove the foam lid inside the package
- To remove the power chair, orientate the box such that when it is laid down on the ground the padded bar is at the top and the footplate is at the bottom
- Grab the top bar and begin to pull your Travel Buggy out horizontally parallel to the ground

UNFOLD & SECURE BACK LOCK



- Open the VISTA like a lawn chair by pulling the top bar and seat away from each other
- Continue pressing down on the seat until the chair is on its 4 wheels
- **IMPORTANT!** Secure the safety latch at the rear of the chair to lock the power chair in its unfolded position

CONNECT THE BATTERY



- **IMPORTANT!** Connect the battery to the motors by pressing the red connectors into each other



INSERT THE JOYSTICK



- Locate the joystick controller. Slide the joystick controller into the arm rest tube (image 1). The length of the joystick controller can be adjusted for comfort before or after secured
- Press up on quick release latch to secure the joystick in place (image 2)

CONTROLLER

Speed Control

- **INCREASE SPEED**
Press the  button to increase the speed
- **DECREASE SPEED**
Press the  button to decrease the speed

5-Speed Control Lights

- SPEED CONTROL LIMITS**
- 1-Light On 0.5 MPH
 - 2-Lights On 1 MPH
 - 3-Lights On 2 MPH
 - 4-Lights On 3 MPH
 - All 5-Lights On 4 MPH



Power ON/OFF

- **POWER ON/OFF**
Press the GREEN button to turn ON/OFF the power chair

Battery Indicator Lights

There are a total of 5 battery power indicator lights: 3 Green, 1 Yellow and 1 Red. From right to left the lights will disappear as the battery drains indicating the amount of charge remaining. Recharge at any point before the last Red light disappears, indicating a fully depleted battery

GETTING IN AND OUT



- Before sitting in the Power Chair, be sure the power button is OFF and the rear brake levers are in the UP (locked) position
- If you prefer a front entry, we recommend flipping up the foot plate
- DO NOT step on the footplate when entering or exiting VISTA Power Chair as it may cause the power chair to move in an unsafe manner
- To enter on the side pull on the arm release button as shown in image 3. Raise arm rest back and away. When seated, lower arm rest back into position and ensure it is locked

- VISTA Power Chair has an easy to use 360 degree joystick controller featuring multiple speeds and a battery life indicator
- Take some time in a safe, controlled environment to learn the operation of the controller, the speed and the maneuverability of the model
- Be sure to adjust the speed depending on your environment and while turning
- To slow down, slowly bring the joystick back toward the initial position.
- To stop, just bring the joystick back to the initial position
- Use extra care when you drive the power chair in reverse

FREE WHEEL MODE



- To put your VISTA Power Chair in Free Wheel Mode, come to a full stop and turn OFF the power on the controller
- Push DOWN on the two red levers located on each rear wheel as shown