

# TWO YEAR LIMITED WARRANTY

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of original purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

*Wenzel<sup>ite</sup>Re/hab*



**PORT WASHINGTON, NY 11050**

© 2002 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China



*Wenzel<sup>ite</sup>Re/hab*

## **Otter Bath Chair**

**Instructions & Warranty**

Item #'s

OT 1000, OT 2000, OT 3000

**[www.drivemedical.com](http://www.drivemedical.com)**

---

## Recommended Use:

---

The Otter Bath Chair provides support during bathing in the bathtub. The seat, back and legs are fully adjustable. All adjustments are tool-free.

---

## Assembly:

---

Your Otter Bath Chair comes in one carton completely assembled. This carton contains: Bath chair, head cushion strap, 2 positioning straps and a set of leg straps.

If your shipment is incomplete, please call a Drive authorized dealer.

**Optional Accessory:** Tub Stand,  
Item # OT 8010

---

## Maintenance And Care:

---


1. Inspect the chair periodically for cracks, breaks, loose or missing parts or malfunctions. Check the fabric for tears or worn areas. If a product needs service or replacement parts, contact an authorized Drive dealer.
2. Wash chair and fabric with soap and water or 10% bleach. Do not use pine oil cleaners. Fabric and straps may be removed and machine-washed. Engage hook and loop before washing. Fabric cover may be washed on cool setting and drip-dried. Straps may be washed in warm water and dried on cool setting. Do not iron.

---

## Adjustments: (SEE FIGURE 1)

---

1. Angle of seat is adjusted using both triggers (B). Pull triggers, rotate seat to desired position, and release trigger. Angle of back is adjusted using both triggers (C).
2. Head cushion strap height can be adjusted by opening strap in the back and positioning it at desired height. Securely engage hook and loop of strap once positioned. Slide head cushions (A) along the strap to adjust width.
3. Straps (D) can be adjusted as needed by inserting in any of the loops along the entire unit.
4. To adjust legs (E), pull crossbar (F) outward, rotate legs & push in to engage.

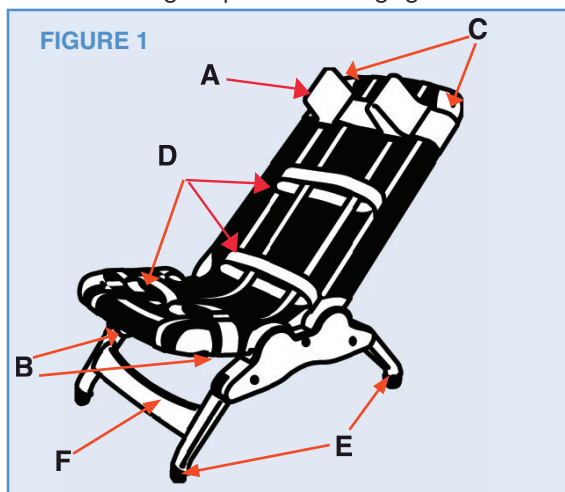

---

## Caution:

---

Adult supervision is required at all times.

The straps and supports must be carefully adjusted for the comfort and security of the child.

Child must be strapped in at all times.