Operation Manual

Comfort Unite Back Support Comfort Relief Back Support Comfort Embrace Back Support Visco Back Support

permobil

Contact Information

 The Comfort Companies, LLC
1960 South Calhoun Road
New Berlin, Wisconsin 53151 USA

Phone:
U.S.A.: 1-800-736-0925
Outside the USA: +1-618-277-9150
Fax: +1-406-522-8563

Customer Support:
orders.comfort@permobil.com
permobil.com

WARNING

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in serious injury.

CAUTION

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in minor to moderate injury or damage to the equipment or other property.

Incident Reporting

In the event of a serious incident or injury, contact Customer Support.

Contents

Intended Use	2
Parts Detail and Specifications	
Important Safety Information	3
Comfort Relief: BOA System	4
Wings for Comfort Unite 1	5
Wings for Comfort Embrace	6
Chest Guide Brackets for Comfort Embrace	
Cover Removal and Installation	7-9
Foam Removal and Installation	9
Moldable Stays	10
Air Insert Inflation, Adjustment, Removal and Installation	11
Cleaning and Disinfecting	12-13
Troubleshooting	13
Maintenance	14
Storage, Transport, Disposal and Recycling	
Warranty	
Español	15
Français	29
Português	43

Symbols Glossary

 Manufacturer	 Date of manufacture	 Consult instructions for use
 Quantity	 Model number	 Medical Device
 Batch code	 Size (width x length x depth)	 Top of product
 Weight limit	 Fits	 Meets flammability standards
 Cleaning instructions	 Disinfecting instructions	 Unique Device Identifier

Please Note

Supplier: These instructions must be given to the user of this product.
Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.

Products may be covered by one or more U.S. and foreign patents and trademarks, including COMFORT-TEK®, and STRETCH-AIR®.

BOA® is a registered trademark of Boa Technology Inc.

GlideWear® is a registered trademark of Tamarack® Habilitation Technologies, Inc.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

The Comfort Companies, LLC is a part of Permobil.

© 2025 Permobil

REV 2025-01-27

The following products are featured in this manual:

Comfort Unite 1 Back Support
Comfort Unite 2 Back Support
Comfort Unite 3 Back Support
Comfort Unite 1 LTS Back Support
Comfort Unite 1 Wing Back Support
Comfort Embrace Back Support
Comfort Embrace LTS Back Support
Comfort Relief Back Support
Comfort Relief LTS Back Support
Visco Back Support

Intended Use

The Comfort Unite Back Supports are back support devices that are intended to provide trunk support and positioning for wheelchair users. The Comfort Unite has a weight limit of up to 450 lbs (204 kg) and must be properly sized for the individual.

The Comfort Relief Back Supports are back support devices that are intended to provide trunk support and positioning for wheelchair users. The back supports utilize an adjustable cable system to allow for individualized contouring of the back depth. The Comfort Relief has a weight limit of 300 lbs (136 kg) and must be properly sized for the individual.

The Comfort Embrace Back Supports are planar back support devices that are intended to provide trunk support and positioning for wheelchair users. The Comfort Embrace has a weight limit of up to 450 lbs (204 kg) and must be properly sized for the individual.

The Visco Back Support (Visco Back) is designed to provide trunk and pelvic support to the individual.

The back supports are considered replacement back supports for wheelchairs and weight limits are based on back support width:

Width [in. (cm)]	Weight limit [lbs. (kg)]
0 - 18 (0 - 46)	250 (113)
20 (51)	300 (136)
22 (56)	350 (159)
24 (61)	400 (181)
26 - 28 (66 - 71)	450 (204)

A clinician who is experienced in seating and positioning should determine whether the product is appropriate for the individual's particular seating needs. The product's intended use statement alone should not be relied upon to make that determination.

The medical products featured in this document are intended to be only one part of an overall care regimen that includes all seating and mobility equipment and therapeutic interventions. A clinician who is experienced in seating and positioning should identify this care regimen after evaluating the individual's physical needs and overall medical condition.

A clinician should also evaluate for visual, reading, and cognitive impairment to determine the need for caregiver assistance or the need for other assistive technology, such as large-print instructions, to ensure proper product use.

Parts Detail and Specifications

Components included: back support, back support cover, including any foam components; air inserts (certain models only); moldable stays (certain models only); operation manual

  **IMPORTANT!** This manual includes instructions for the back shell and cover components only. Read and follow the instructions provided with the mounting hardware to ensure correct and safe installation and use of this product on the wheelchair.

To order replacement parts, contact the product provider or Customer Support.

Materials:

Covers (features and materials will vary): COMFORT-TEK Cover: two-way stretch, fluid-resistant front and breathable, two-way stretch back, with a zipper. Diamond Spacer Cover: breathable, two-way stretch front and back, with a zipper. GLIDEWEAR with Diamond Spacer: shear-reduction front and breathable, two-way stretch front and back, with a zipper. Refer to the cover care label for material content.

Back shell, wings, and hardware: aluminum


Foam: polyurethane and polyethylene

Moldable stays: aluminum

Air Inserts: polyether polyurethane front; urethane-coated nylon back

Expected product lifetime: 5 years

 **PSIS WARNING:** Make sure that the product is positioned so that the bottom portion makes contact with the Posterior Superior Iliac Spine (PSIS). If the product is NOT positioned to make contact with the PSIS, pelvic and trunk control may be reduced.

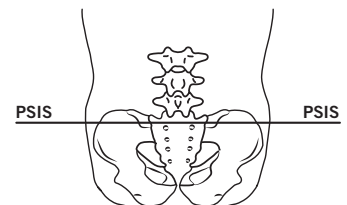Important Safety Information

WARNINGS:

- DO NOT use the product without first reading and understanding all of the provided instructions. If unable to perform any tasks in this manual, seek assistance by contacting a clinician, equipment provider, distributor, or Customer Support.
- Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- The product should be installed by an equipment provider or clinician.
- The product must be sized correctly to the individual and the wheelchair.
- Skin/soft tissue breakdown can occur due to a number of factors, which vary by individual. Check skin frequently, at least once a day. Redness, bruising, or darker areas (when compared to normal skin) may indicate superficial or deep tissue injury and needs to be addressed. If there is any discoloration to skin/soft tissue, STOP USE immediately. If the discoloration does not disappear within 30 minutes after disuse, immediately consult a healthcare professional.
- DO NOT modify the product or use non-compatible products, materials, accessories or components, which may result in a non-secure installation of the back support, bodily injury, and product damage or failure.
- If a headrest is necessary, follow the headrest manufacturer's instructions and safety information. Before installing another manufacturer's headrest, consult with a clinician or service provider to ensure compatibility.
- Altering the product may affect its performance, result in product failure, and void the warranty.
- Before using the first time, and then frequently, confirm the tightness of attaching hardware, and check for wear and breakage. DO NOT continue to use if any part cannot be tightened or is broken. Periodically, the product should be thoroughly inspected by the equipment provider.
- As with any addition to a wheelchair, the product may affect the wheelchair's stability. Before using the product, consult with a clinician or the equipment provider to determine whether the wheelchair should be modified.
- Be sure to assess the need for additional safety features for the wheelchair, such as anti-tip bars or other available options, to ensure stability during use.
- Check to make sure that the product does not interfere with any other wheelchair components. Interference could jeopardize effectiveness and user safety and cause damage.
- DO NOT use the back support without both the cover and the foam.

WARNINGS:

- Unless otherwise noted in this instruction manual, DO NOT remove, install or adjust the cover, foam, wings or any other components while the user is seated in the wheelchair.
- DO NOT lean over the top of the back support. Doing so may cause a change in the center of gravity, and the wheelchair could tip over.
- DO NOT lift or push a wheelchair using any part of an back support or hardware because it may become detached from the wheelchair.
- When using with power recline: after installing the product on a power wheelchair, fully articulate any power recline features to ensure there is no possible equipment interference. Do so slowly while watching closely.
- DO NOT expose the product to high heat, open flames or hot ashes. Testing or certification claims, including for flammability, may no longer apply to this device when it is combined with other products or materials. Check testing and certification claims for all products used in combination with this device.
- Metal components may become hot if exposed to high temperatures or cold if exposed to low temperatures. The cover assembly surface will acclimate to the surrounding temperature. Take appropriate precautions, especially when the product will contact unprotected skin.
- Keep small hardware components out of the reach of children and any individuals who may have a tendency to place inedible objects in their mouths. Swallowing or inhalation may lead to serious injury or death. IMMEDIATELY seek emergency medical assistance.

WARNINGS - Motor Vehicle Transportation:

- When possible, DO NOT use during transport. Repeated or excessive force (braking or crash) could potentially change adjustment or cause extra wear, shortening the life of the product. DO NOT use the equipment if it has been in a crash.
- Whenever possible, transfer out of the wheelchair installed with the back support and into a manufacturer-installed vehicle seat, and use the vehicle's crash-tested occupant restraint system.
- To determine whether motor vehicle third-party testing has been performed with the specific hardware-back support combination, contact the equipment manufacturer.

CAUTIONS:



- Protect the product from blunt force shocks that may cause damage and/or breakage.
- Removal of the back support may be needed before folding the wheelchair.

Comfort Relief: BOA System

Adjustment

⚠ WARNING: Before using the Comfort Relief, adjust the BOA system, as needed. Regularly check the BOA system, over time, and adjust, if needed. If the back support is used by a different individual or in a different wheelchair, repeat adjustment.

1. The user should be seated in their usual position.
2. To open the BOA reels: Push down on one side of the reel while pulling up firmly on the other side of the reel. There will be an audible sound when the reel pops open.
3. Once popped open, the tension will automatically be released. To release the tension further, pull on the reel.
4. Allow the user to immerse into the back support.
5. Make adjustments in the following order:
FIRST: Adjust the lowest cable, which should be located at the pelvic region.
SECOND: Adjust the second-lowest cable, which should be located at the lumbar region.
THIRD: Adjust the top cable(s) as needed.
6. Push to engage the BOA reel. Rotate, as shown, to adjust the tension. There will be a clicking sound as the reel is rotated. The cables will maintain an even tension as they conform to the user's shape.
7. If further adjustment is needed, repeat the instructions on this page.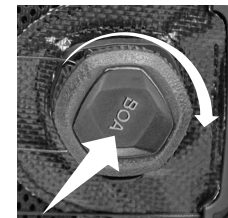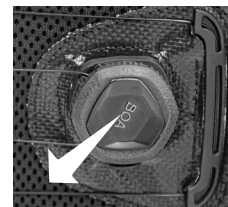


Replacing the Straps


⚠ WARNING: DO NOT replace the straps while the user is in the wheelchair.

Note: A single strap must be replaced with a single strap, and a double strap must be replaced with a double strap.

1. To remove a strap: Release the hook and loop fastener strips, and pull out the fastener strips from the slots on each side of the back support.
2. To install a replacement strap: Feed the hook and loop fastener strips through the slots in each side of the back support. Make sure the straps and cables are not tangled and the BOA reel is facing outward as shown in the image of the Comfort Relief on this page. Secure the hook and loop fastener strips.
3. Refer to the "Adjustment" instructions above to adjust the strap.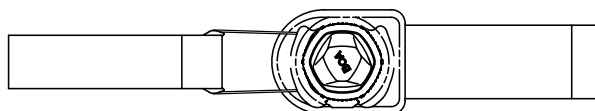

Single Strap

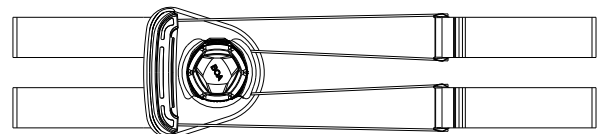Double Strap

Wings for Comfort Unite 1

⚠ WARNINGS:

- Adjustment requires removing the protective cover and foam. Also, the back support and components may shift suddenly during adjustment.
- The wings should be adjusted before use and re-adjusted as needed, over time.

Parts included with two wings:

- M6 x 1.0 x 30 button head cap screw (x4)
- M6 x .058 in. washer (x4)
- M6 x 1.0 double nut (x2)
- 5mm hex key

To Remove the Wings

1. Remove the cover to access the wing fasteners. See "Cover Removal and Installation".
2. Support the wing while loosening the screws, using the 5 mm hex key.
3. Remove the wings.

To Attach and Adjust the Wings

1. The user should transfer from the wheelchair.
2. Remove the cover. See "Cover Removal and Installation".
3. Attach each wing to the back shell, using the provided hardware and 5mm hex key. Tighten the screws until secure.

Order of assembly:

Back side: 2 screws [B], 2 safety washers [C], 2 washers [D], back shell [F]

Front side: wing [A] and double nut [E]

4. Adjust as needed: Use the 5 mm hex key to loosen the screws and adjust for rotation or medial-lateral positioning. [Fig. 1] Re-tighten the screws until secure.
5. If necessary, the user may transfer into the wheelchair briefly to confirm correct placement/position of the wings. After confirming, the user should transfer out of the wheelchair for cover installation.
6. Reinstall the cover. See "Cover Removal and Installation".
7. Confirm that all attachment hardware is secure. The user may transfer to the wheelchair.

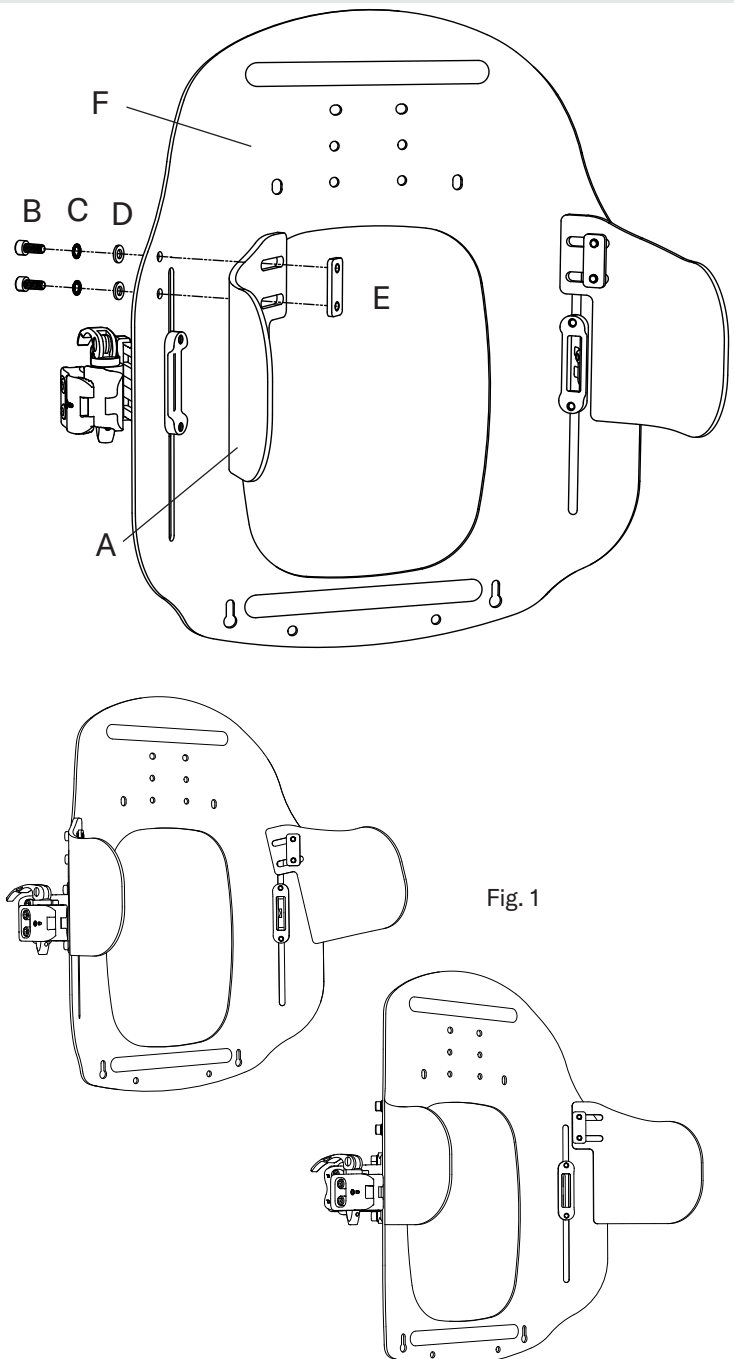
Fig. 1

Wings for Comfort Embrace

Parts included with two wings: M6 x 1.0 x 30 socket head cap screw (x4); M6 safety washer (x4); M6 x 3 mm washer (x4); M6 x 1.0 double nut (x2); 5 mm hex key

⚠ WARNINGS:


- The wings should be adjusted before use and re-adjusted as needed, over time.
- Seat depth may need to be adjusted when installing the wings as a loss of seat depth may occur.

Options

Interior Tracks: Most (but not all) Comfort Embrace back shells include interior tracks in addition to the exterior tracks. The interior tracks allow for a greater range of adjustment or offset positioning, as needed.

Flipped Orientation: The wings may be used in a flipped orientation, as shown.

Additional Wings: For three-point lateral support, additional wings may be used. If additional wings are needed, contact Customer Support.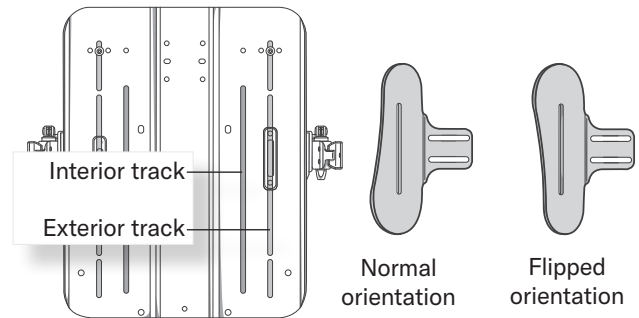

Remove the Cover

Remove the cover to access the wing fasteners. See “Cover Removal and Installation”.

Attach the Wings

1. Determine placement and orientation of the wings.
2. Align the slots in the wing with a slot in the Comfort Embrace and attach, using the provided hardware and 5 mm hex key.


Order of assembly*

Back side: 2 screws [B], 2 safety washers [C], 2 washers [D], Comfort Embrace back shell [F]

Front side: wing [A] and double nut [E]

*Compass Power Mount Hardware:

When using Compass Power Mount Hardware, wings are attached to the front side of the shell with Button Head Cap Screws (provided in Compass Power Mount Hardware Kit Bag).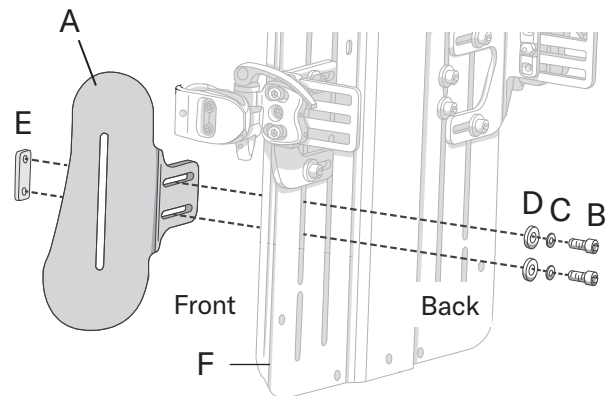

Compass 4 Hardware, back w/ Socket Head Screws

Adjust the Wings

1. Use the 5 mm hex key to loosen the screws to adjust for height or medial-lateral positioning or to rotate the wings.
2. Re-tighten the screws until secure.

Reinstall the Cover

After adjustment is complete, reinstall the cover. See “Cover Removal and Installation”.


Chest Guide Brackets for Comfort Embrace

Feed the straps through the chest guide brackets (if using). If needed, the brackets can be adjusted vertically, using the 4 mm hex key. To access the chest guide brackets, remove the cover. See “Cover Removal and Installation”.

Order of assembly:

Front side: screw [G], washer [H], Comfort Embrace back shell [F]

Back side: chest guide bracket [J]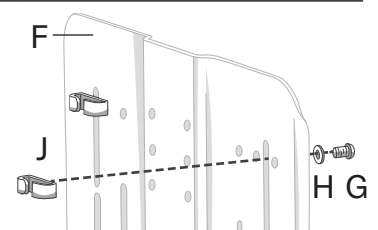


Cover Removal and Installation

Comfort Unite Back Support

To remove the cover:


1. Fold back the bottom cover pocket to expose the stud straps.
2. Pull the stud straps down and out to release from the holes on each side of the bottom of the back shell. If the back shell has wings, pull on the side pockets to release the cover from the wings.
3. Lift the cover up from the top of the back shell and remove cover.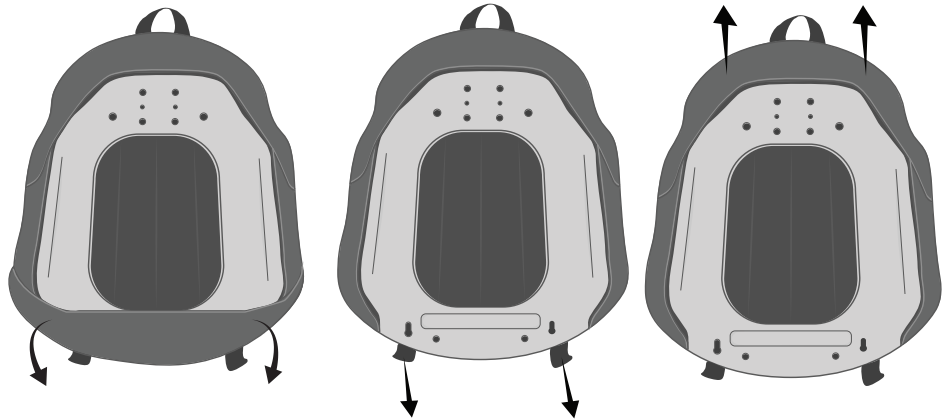

To install the cover:

Note: If the cover has Moldable Stays, removing them may make it easier to install the cover; they can be reinserted after cover installation. Refer to the "Moldable Stays" instructions in this manual.


1. Slide the top cover pocket over the top of the back shell. If the back shell has wings, insert the wings into the side pockets on the cover.
2. Pull the stud straps down and insert into the holes and slide up into place on the bottom of the back shell.
3. Pull the bottom cover pocket up and over the bottom of the back shell.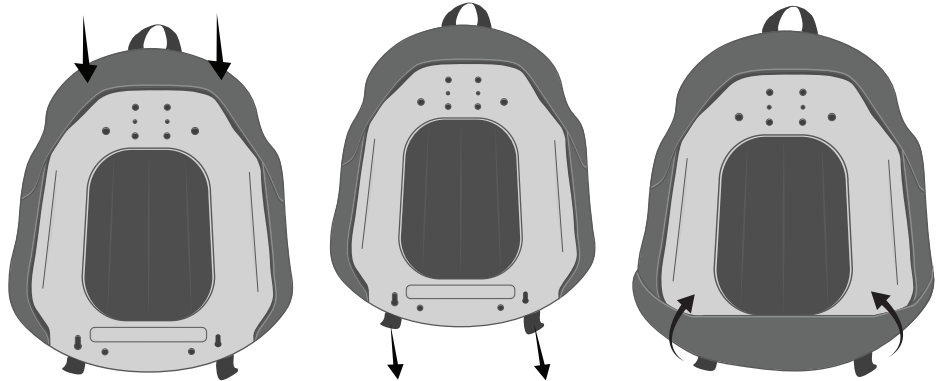

Comfort Relief Back Support


To remove the cover:

1. Locate the stud straps attached to the bottom of the back shell and slide up to release from the holes on the bottom of back shell. Then remove the bottom cover pocket from the bottom of the back shell.
2. Reach under the top cover pocket and pull the stud straps down and out to release from the holes on the back shell. Lift the cover up from the top of the back shell to fully remove the cover.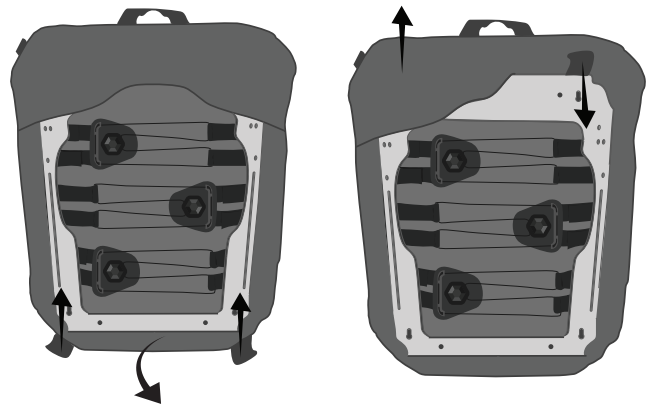

To install the cover:

1. Slide the top cover pocket over the top of the back shell. Lift the top cover pocket to locate the stud straps. Pull each stud strap down to insert into the holes on the back shell and then slide up to secure the top of the cover.
2. Pull the bottom cover pocket over the bottom of the back shell. Locate the stud straps in the bottom cover pocket and insert each one into the holes on the back shell and then slide down to secure the bottom of the cover.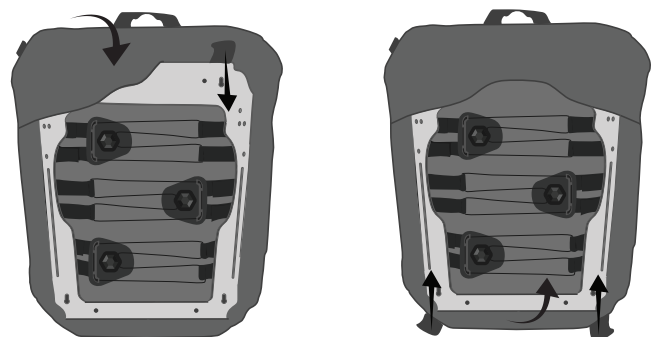


Cover Removal and Installation, continued

Comfort Embrace Back Support


To remove the cover with Back Cane Hardware:

1. Pull the elastic bands on the hardware cover flap from around the edges of the bottom of the back shell.
2. At the bottom of the cover, pull the stud straps down and out to release from the holes on the bottom of the back shell.
3. Lift the cover up from the top of the back shell to fully remove the cover.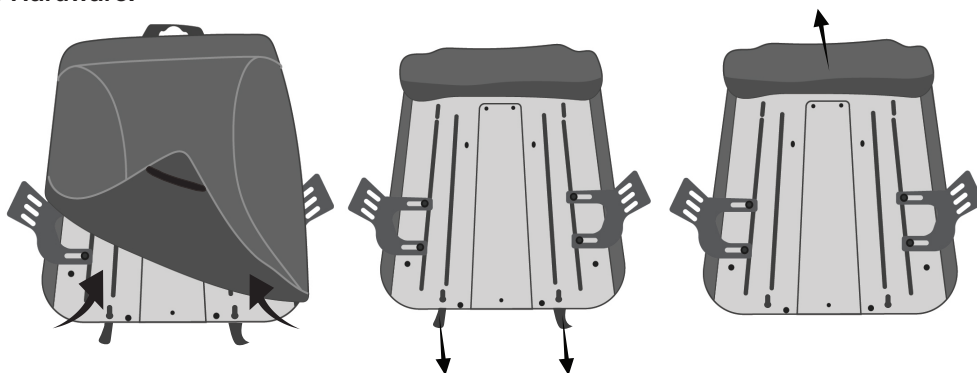


To remove the cover with Compass Power Mount Hardware or Direct Mount:

1. At the bottom of the cover, pull the stud straps down and out to release through the holes on the back shell.
2. Lift the cover up from the top of the back shell to fully remove the cover.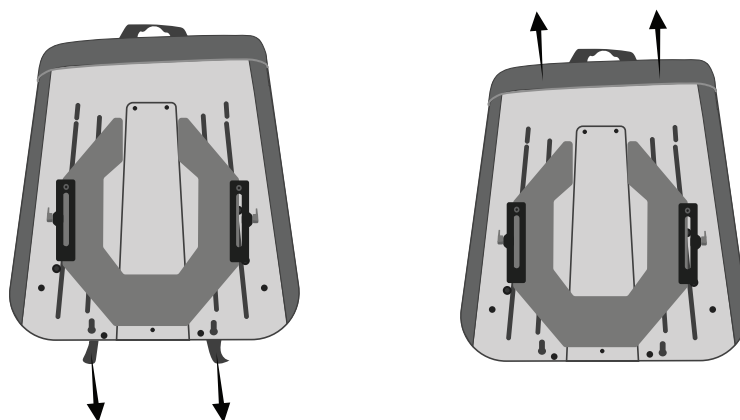

To install the cover with Back Cane Hardware:

1. Lift the hardware cover flap up and slide the inner top cover pocket over the top of the back shell.
2. At the bottom of the cover, pull each stud strap down and insert into the opening on the bottom of the back shell and slide up into place.
3. Take the elastic bands on the hardware cover flap and place around the edges of the bottom of the back shell to secure in place.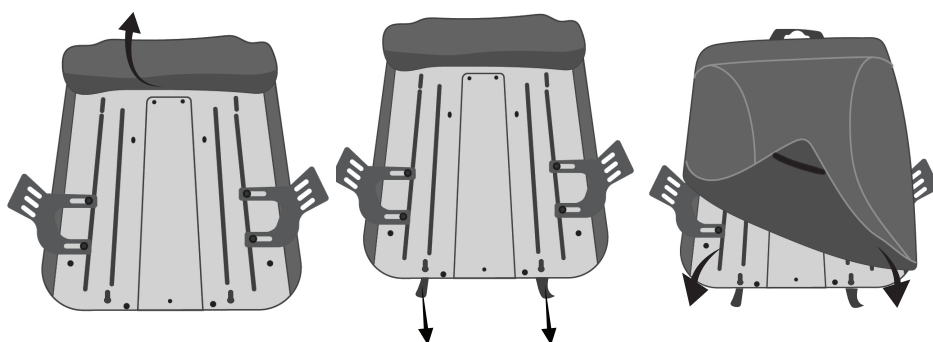


To install the cover with Compass Power Mount Hardware or Direct Mount:

1. Leave the hardware cover flap down and slide the outer cover pocket over the top of the back shell.

Note: The bottom portion of the hardware cover flap may need to be folded up to access the stud straps.

2. At the bottom of the cover, pull each stud strap down and insert into the opening on the bottom of the back shell and slide up into place.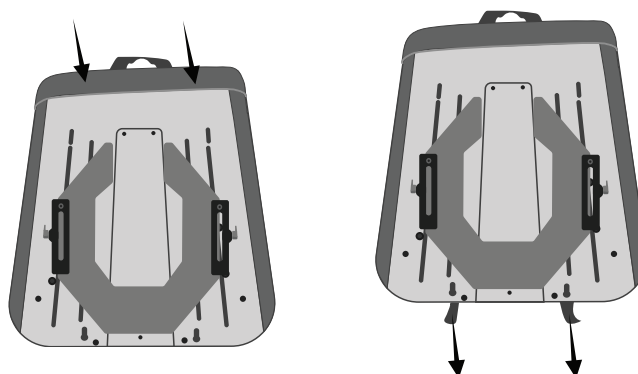


Cover Removal and Installation, continued

Visco Back Support


To remove the cover:

1. Release the hook and loop fastener strips, and then pull the straps from the rings.
2. Remove the top flap and bottom flap from the back shell, and remove the cover.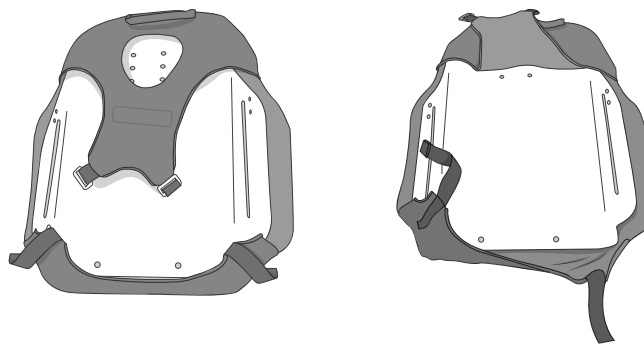

To install the cover:

1. Place the bottom of the back shell into the bottom cover flap, and pull the top cover flap over the top of the back shell.
2. Place the straps through the rings and secure the straps to the lower part of the cover with the hook / loop fastener strips.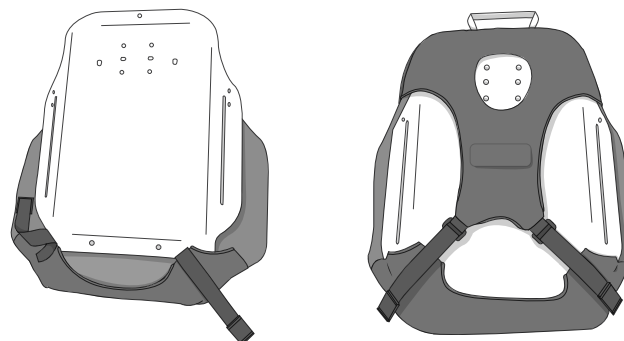


Foam Removal and Installation

To remove the foam:

1. Open the zipper located at the top of the cover.

Note for lateral air inserts only: Remove the lateral air insert valve hoses from the elastic guides located on both sides inside the cover.


2. Remove the foam.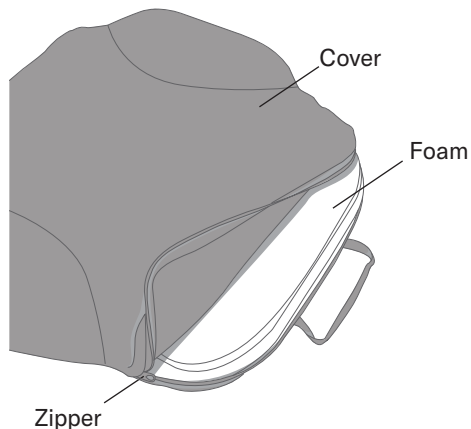

To install the foam:

1. Hold the edges of the foam together and insert into the cover.
2. Once the foam is completely inside the cover, close the zipper located at the top of the cover.

Note for lateral air inserts only: Insert the lateral air insert valve hoses into the elastic guides located on both sides inside the cover.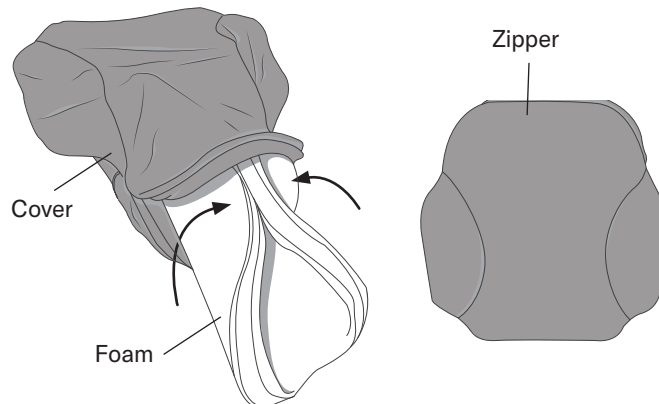

Moldable Stays

Note: Two Moldable Stays are provided with all models of Comfort Unite and Comfort Embrace. Using the Moldable Stays is optional. If they will not be used, they may be removed from the cover and stored in a container that protects them from moisture, contaminants and damage.

WARNINGS:

- Be careful with the metal edges of the Moldable Stays during installation; use the covered ends when handling the stays.
- The Moldable Stays may be adjusted before use. They may need to be adjusted again, over time.
- The Moldable Stays **MUST** be fully inserted in the pockets. If the Moldable Stays do not reach the bottoms of the pockets, they may fail to provide the intended support.
- If a clinician determines that it is not safe to remove and replace the Moldable Stays while the user remains seated in the wheelchair, the user should transfer from the wheelchair while the Moldable Stays are being removed and installed.

To adjust the Moldable Stays:

1. With the user seated in their usual position in the wheelchair, the clinician should evaluate the user's positioning needs and determine any needed adjustment of the Moldable Stays.

2. Remove, shape, and replace the Moldable Stays as described below.

IMPORTANT! The user should transfer from the wheelchair if it is determined unsafe during adjustment of the Moldable Stays. Refer to the warning above.

3. Open the zipper located at the top of the cover. Remove the two Moldable Stays from the pockets. (Fig. 1)
4. Using hands and/or a firm surface, shape the Moldable Stays.

Note: Fig. 2 shows the possible ways to shape the Moldable Stays. Two (2) Moldable Stays are provided with each Comfort Unite or Comfort Embrace.

5. Place the Moldable Stays in the pockets, as prescribed. Make sure to fully insert the Moldable Stays.
6. Close the zipper.

If the user has transferred from the wheelchair, they may transfer back and into their usual seated position.

7. Confirm that the adjustment is correct. If further adjustment is needed, repeat the instructions on this page.

Fig. 1: Remove the stays for shaping

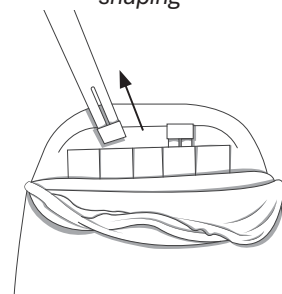
Fig. 2: Options for shaping

Air Insert Inflation, Adjustment, Removal and Installation

⚠ WARNINGS:

- Check inflation frequently, at least once a day!
- The air insert(s) must be used with the air cells facing the individual.
- Make sure the air insert valves are securely positioned within the elastic guides on both sides inside the cover. Incorrect positioning may cause the hose to kink, preventing the air insert from inflating or deflating.
- The individual's comfort level may be reduced if an air insert is not positioned correctly or is under-inflated or over-inflated.
- If an air insert does not appear to be holding air, see "Troubleshooting". If still not holding air, contact an equipment provider or Customer Support.
- Changes in altitude may require adjustment to an air insert. Check the inflation level of any air inserts when changing altitude.

⚠ CAUTIONS:



- DO NOT use the valve hose as a handle to carry or pull an air insert.
- DO NOT use an air insert as a water flotation device (e.g., a Life Preserver).
- If an air insert has been in temperatures lower than 32° F (0° C) and exhibits unusual stiffness, allow the cushion to warm to room temperature. Repeat adjustment instructions before using.
- Keep air cells away from sharp objects.
- DO NOT use an air insert as a seat cushion.
- DO NOT use a pump or repair kit other than those approved for use with an air insert.

Inflation and Adjustment

Notes: Before adjusting the air inserts, make sure the cover with the foam pad and air inserts is correctly installed on the back shell. Make sure that the back support is correctly installed and adjusted on the wheelchair.

Follow these instructions to inflate the air insert(s).

1. Open the zipper at the top of the cover. Locate the inflation valve(s) and hose(s) and gently pull out of the cover.
2. Slide the hand pump nozzle over the inflation valve(s). Open the inflation valve(s), at least one full rotation. Inflate the air insert(s) until it feels firm.
3. Close the inflation valve(s) until it comes to a stop. Remove the hand pump. Make sure the inflation valve(s) is closed.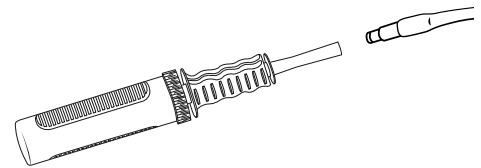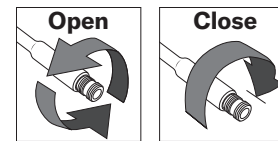


⚠ **WARNING:** Safely transfer the individual into the wheelchair.

4. If desired, adjust the air insert:
 - a. The individual may be seated in the wheelchair in their normal position.
 - b. Open the inflation valve and remove air until the individual is comfortable. Close the inflation valve.
5. With the cover installed on the back shell (See "Cover Removal and Installation"), tuck the valve hoses in the top of the cover and close the zipper.


Removal

1. Remove foam from the cover (See "Foam Removal and Installation"). Untuck the edges of the air insert(s) from the edges of the foam cut out.
2. Slide the valve hose through the hole in the foam pad to completely remove the air insert.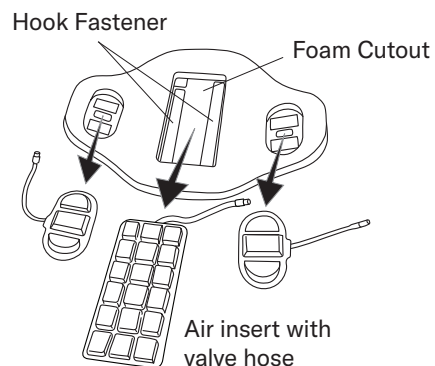

Installation

1. Slide the valve hose through the hole in the foam pad. The air cells should face out.
2. Tuck the edges of the air insert(s) under all of the edges of the foam cutout. Press to secure the air insert(s) to the fastener strip in the cutout area.
3. Place the foam pad and the air insert(s) into the cover. (See "Foam Removal and Installation").

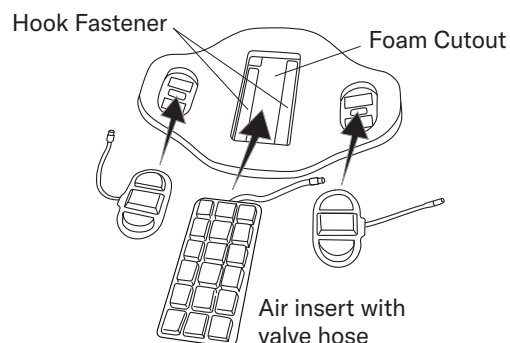Cleaning and Disinfecting

⚠️ WARNINGS:

- Follow all manufacturer bleach container and germicidal disinfectant safety guidelines.
- Cleaning and disinfecting are separate processes. Cleaning must precede disinfection. Before use by a different individual: clean, disinfect, and check product for proper functioning. Clean the product regularly and when it becomes soiled.

Recommendation: When washing by hand, use room-temperature water.

Metal Components

⚠️ **CAUTION:** Before cleaning or disinfecting the back shell, remove the cover.

Cleaning: Hand wash with a sponge, using soap and water. Rinse with clean water. Wipe dry with a clean cloth and then allow to air dry completely before use.

Disinfecting: Wipe gently with a cloth dampened with a household disinfectant. Rinse with clean water. Wipe dry with a clean cloth and then allow to air dry completely before use.

Cover Components

⚠️ CAUTIONS:

- Frequent heat drying and bleaching will cause fabric to break down. Air drying and cleaning without bleach are recommended whenever possible to help extend the life of the cover.
- DO NOT steam clean or autoclave. DO NOT iron.
- Remove the cover from the foam before cleaning or disinfecting.
- Turn the cover inside out or, if possible, remove any hook and loop fasteners or other materials that may snag the fabric during cleaning / disinfecting.


Surface cleaning COMFORT-TEK fabric only: Spray with common household or commercial antibacterial cleansers or disinfectants. Wipe with a damp, water-only cloth after disinfecting. Let dry completely before use.

Cleaning all fabrics: When possible, hand wash in warm water with mild detergent, rinse thoroughly with clean water, or machine wash warm using common laundry detergent. Tumble dry low or air dry. Let dry completely before use.

Disinfecting all fabrics: Machine wash in hot water, using common laundry detergent. Or hand wash, using 1 part household liquid bleach per 8 parts water. Keep the product wet with the bleach solutions for 10 minutes, then rinse thoroughly. Tumble dry low or air dry. Let dry completely before use.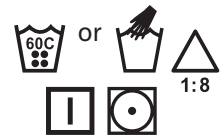

Foam Components

⚠️ **WARNING: Soiled foam should be replaced.** DO NOT use contaminated foam with multiple users. Disinfectants are NOT effective on porous surfaces such as foam. Replacement foam components may be purchased separately.

⚠️ CAUTIONS:

- DO NOT machine wash or machine dry. DO NOT submerge the foam in any liquid, including water.
- Exposing the foam to light for an extended period of time will cause discoloration but will not affect the functional features of the foam.

Cleaning the foam: Remove the cover. Wipe lightly with a damp cloth. DO NOT use soap. Let dry completely before replacing the cover.

Care Symbols

Wipe with a damp cloth.	Hand wash.	Dry flat.	Do not tumble dry.
Bleach (1 part bleach: 8 parts water).	Tumble dry low.	Line dry.	Do not machine wash.
Machine wash warm, gentle.	Machine wash hot, normal.	Bleach (1 part bleach : 9 parts water).	Dry flat in the shade.

Cleaning and Disinfecting, continued

Air Inserts

Note: Remove the air inserts from the cover and foam pad prior to cleaning and disinfecting.

CAUTIONS:


- Make sure the inflation valves are closed. DO NOT allow water or cleaning solution to enter the air insert.
- DO NOT machine wash or dry
- DO NOT expose to direct sunlight.
- DO NOT autoclave.
- Rinse thoroughly. Detergent residue may cause air cells to stick together.

To Clean: Close valves. Place in a large sink. Hand wash, using mild liquid hand soap, hand dishwashing liquid, laundry detergent, or a multipurpose detergent (follow product label instructions). Gently scrub all cushion surfaces, using a soft, plastic bristle brush, a sponge, or a cloth. Rinse with clean water. Air dry.

To Disinfect: Follow the cleaning instructions, using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes. Rinse thoroughly with clean water. Air dry.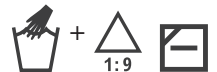

Note: Deflated air cells may be easier to thoroughly clean. To deflate the air cells: with the inflation valve open, roll up and/or compress the cushion, and then completely close the inflation valve.

Troubleshooting

For additional assistance, contact the equipment provider, distributor, or Customer Support.

The Moldable Stays do not provide the intended support.	- Check to make sure the Moldable Stays are fully inserted into the pockets of the cover and that the bottoms of the stays rest against the back shell. - Check to make sure the Moldable Stays are the correct length and provide the intended support. Contact Customer Support for assistance.
Not able to install the cover.	- Check to make sure that the cover is completely on the back shell. - If the cover has Moldable Stays, removing them may make it easier to install the cover; they can be reinserted after cover installation. (See "Moldable Stays").
BOA cables do not maintain their tension.	Check that the BOA reels are engaged. A clicking sound should be heard when rotating the BOA dials to tighten the cables.
The air insert is not holding air.	Confirm that the inflation valve is completely closed. Inspect for inflation valve and hose damage. Look for holes in the air insert. If very small holes or no holes are visible, follow the instructions in the Repair Kit provided with the product.
Not able to inflate or deflate an air insert.	Confirm that the inflation valve is open. Inspect the inflation valve and hose for damage.
The air insert does not feel comfortable.	- Add or remove air to adjust to your comfort level (see "Air Insert Inflation, Adjustment, Removal and Installation"). - Make sure air cells face the front of the wheelchair. - Make sure the cover is properly positioned (see "Cover Removal and Installation").
The air insert slides inside the cover.	Make sure the air insert is properly positioned and the fabric back is secured to the fastener strip in the cutout area of the foam pad (see "Air Insert Inflation, Adjustment, Removal and Installation"). The perimeter of the air insert should be tucked in between the foam layers around the edge of the cutout area.
A component is damaged or missing.	See "Warranty" or contact Customer Support for warranty or replacement.

Maintenance

⚠ WARNING: DO NOT continue to use if any part cannot be tightened or is broken. Periodically, check the back shell, cover and all components for damage and wear, and replace if necessary. Some wear with regular use is expected.

Frequently: Confirm the tightness of attaching hardware and re-tighten loose components following the instructions in this manual.

Storage, Transport, Disposal and Recycling

Storage or Transport: Clean and disinfect the product. Place in a container that protects the product from moisture, contaminants and damage. After storing or transporting, inspect the product to ensure it was not damaged (see "Troubleshooting"), and follow instructions to prepare the product for use.

Disposal: The components of the products in this manual are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/regional regulations. At the end of its useful life, treat as healthcare waste and dispose of according to local healthcare-waste guidelines. Incineration must be performed by a qualified, licensed waste management facility.

Recycling: Contact a local recycling agency to determine recycling options for the product.

Warranty

This product is covered by a 5 year warranty against manufacturer defects with the exception of product covers that are eligible for warranty up to 6 months.

If a manufacturing defect should occur, discontinue use immediately. Wear from regular use, misuse, damage by an accident or damage from "an act of Nature" (e.g., flood, tornado, earthquake and/or fire) is not considered a manufacturing defect. Replacement parts are available by contacting a supplier, distributor, or retailer.

The following will void the warranty: modifications made in attempt to interface with other products (where compatibility is limited, as described in product information); alterations made to the product; or defects caused by irregular use.

Warranty Replacement Procedure

Warranty claims should be initiated by the original purchaser, who should contact the Durable Medical Equipment (DME) supplier, distributor or retailer from whom the product was purchased, if they are available, or any authorized Permobil DME supplier, distributor or retailer.

Whenever possible, the DME supplier, distributor or retailer with a warranty issue should contact Customer Support and provide the original purchase order number, sales order number, or invoice number.

Before a warranty replacement order or quote may be processed, Customer Support must issue a Return Merchandise Authorization (RMA).

Información de contacto

 The Comfort Companies, LLC
1960 South Calhoun Road
New Berlin, Wisconsin 53151, EE. UU.

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