GUARDIAN®

Guardian® Drop Arm Commodes

User Instructions & Warranty

Model 98204- Padded Drop-Arm Commode

Model 98202- Non-Padded Drop-Arm Commode

Includes pail, lid and splash guard.

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www.sunrisemedical.com

IMPORTANT SAFEGUARDS

Read all instructions before using. Important information is highlighted by these terms:

WARNING—Important safety information for hazards that might cause serious injury.

CAUTION— Information for preventing damage to the product.

NOTE— Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

Please follow these instructions carefully whether commode is used bedside or over the toilet. Commode's height is adjustable for use in an elevated position. Back of commode is removable for use over the toilet. Cantilevered arm drops for left-to-right leg movement.

WARNING

Please read carefully before use. Failure to comply may result in severe injury.

WARNING

Ensure all detent buttons are properly engaged in position prior to use.

WARNING

Do not exceed the maximum weight capacity of this commode. Listed below please find the model you own to ensure that you are within the weight capacity.

| <u>Model No.</u> | <u>Weight Capacity</u> |
|------------------|------------------------|
| 98202 | 300 lbs. (136 kg.) |
| 98204 | 350 lbs. (159 kg.) |

FAILURE TO FOLLOW THE ABOVE INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

SAVE THESE INSTRUCTIONS.

INSTALLING BACK

1. Depress detent buttons on back piece and insert into commode to the desired position.
2. Ensure that the detent buttons are fully engaged and the back does not move.

ADJUSTMENT INSTRUCTIONS

Height

1. Depress detent button on leg piece and move up or down to the desired position.
2. Ensure that the detent button is fully engaged and leg does not move.
3. Repeat steps 1 and 2 for the remaining three legs.
4. Make sure the commode sits level and all of the legs are at the same height.

Arms

To Drop Arm:

1. Press the push button located at the sides of the seat.
2. Flip arm down until it is completely out of the way.

To Raise Arm:

1. Pull up on arm until the button engages in the locked position.
2. Make sure arm is completely engaged prior to use of armrest

OVER THE TOILET USE

Use only as instructed by your healthcare professional.

1. Remove back by pushing detent buttons and pulling back away from commode.
2. Lift seat of commode and remove bucket.
3. Replace with splash guard.
4. Lift lid of toilet. Position commode over toilet with splash guard in bowl and lower commode seat.
5. Check to ensure that commode is level and stable before sitting.

CLEANING

Your commode can be easily cleaned with regular household cleaning products.

DO NOT use abrasive powders or scouring pads.

Grip Cleaning Instructions

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

NOTE– Tips on this product contain dry natural rubber.

LIFETIME LIMITED WARRANTY

Your Guardian product is warranted to be free of defects in materials and workmanship.

This warranty is limited to the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to non-durable components, such as but not limited to rubber accessories and grips which are subject to normal wear and tear and replacement. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

In the event of a defect covered by this warranty, we will at our option repair or replace the product. For warranty service please contact the retailer where the product was purchased. If you have a question about your Guardian brand product or this warranty, please call us toll-free at 800.333.4000.

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Sunrise Medical Inc.