## LIFETIME LIMITED WARRANTY

Your Guardian® product is warranted to be free of defects in materials and workmanship. This warranty is limited to the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to non-durable components, such as but not limited to rubber accessories and grips which are subject to normal wear and tear and replacement. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

In the event of a defect covered by this warranty, we will at our option repair or replace the product. For warranty service please contact the retailer where the product was purchased. If you have a question about your Guardian® brand product or this warranty, please call us toll-free at 800.333.4000.

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©2002 Sunrise Medical 3.02  
000-6253 Rev.A

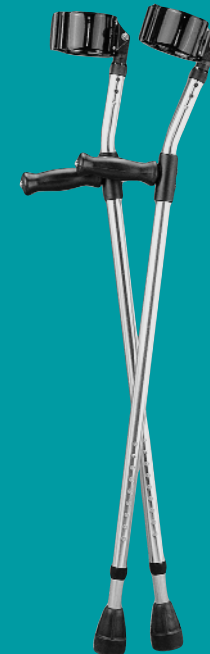
# GUARDIAN®

*The Choice of Healthcare Professionals*

**FOREARM CRUTCHES  
USER INSTRUCTIONS  
& WARRANTY**

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## **WARNING**

For safety, these precautions must be taken when using this product.

### Before each use make sure:

- You are correctly fitted.
- All spring buttons are engaged and all screws are tightened.
- Tips are in good condition. Replace tips immediately when worn.
- Hand grips are attached securely and do not slip.
- Nothing is loose or uneven and there are no rattles or cracks in the metal, plastic, etc. or any other irregularities.
- Product is used only with the manufacturer's parts and accessories—have seller install.
- User does not have any handicaps, disabilities or is on medication, etc. that a doctor, therapist, and/or seller would deem to interfere with the safe usage of the product.

Do not exceed the maximum weight capacity of your crutch. Listed below please find the model you own to ensure that you are within the weight capacity. Failure to comply may result in injury.

Model #	Weight Capacity
05160	300 lbs. (136 kg.)
05161	300 lbs. (136 kg.)
05162	300 lbs. (136 kg.)
05175BLK	300 lbs. (136 kg.)
05176BLK	300 lbs. (136 kg.)
05163	200 lbs. (90 kg.)
05144X	175 lbs. (79 kg.)
05144	150 lbs. (68 kg.)

Use only as instructed by your healthcare provider.

Always use extreme care when using a forearm crutch. Avoid all potential safety hazards including slippery, uneven or soft surfaces and objects in path.

Failure to follow the above instructions may result in serious injury.

### Proper Height

When using forearm crutches, ensure that both crutches are adjusted to an equal height and that all spring lock detent buttons are properly seated.

- Adjust so that while the user stands as upright as possible, the handgrips are at the level of the user's hip joint, or break of the wrist. The user's elbows should bend 30°, and the wrists should be straight as the user grasps the hand grips.
- For cuff height, adjust so that the cuff is 1" to 2" below the bend on the elbow.

### Proper Usage

- User's weight should be carried on the handgrips with the wrists straight.
- User should consult physician or therapist to be shown proper walking gait.
- Do not subject crutch to sudden impact or jolt.

### Grip Cleaning Instructions

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

**Tips on this product contain dry natural rubber.**

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