**SAVE THESE INSTRUCTIONS**

*NOTE: Check ALL parts for shipping damage. In case of shipping damage, DO NOT use. contact Carrier/Dealer for further instruction.*

**SAFETY SUMMARY**

To ensure the safe installation, operation and use of the Essential Cushion, these instructions **MUST** be followed:

**WARNING**

**DO NOT** use this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions, and Instructions**, contact distributor before attempting to use this equipment, otherwise, injury or damage may result.

Check all parts for shipping damage and test before use. In case of damage, **DO NOT** use. Contact the distributor for further instructions.

The Essential Cushion **IS NOT** designed for use in the treatment of pressure sores or by individuals at risk of developing pressure sores.

Your therapist and physician should be consulted if you have any questions regarding weight relief, self-examination of the skin, or individual limitations and needs.

The information contained in this document is subject to change without notice.

Make sure the hook and loop fastening straps are securely attached between the seating surface and cushion before using.

**INSTALLATION WARNING**

After **ANY** adjustments, repair or service and before use, make sure all attaching hardware is tightened securely - otherwise injury or damage may occur.

**INTRODUCTION**

The Essential Cushion is designed to provide both pelvic stability and comfort for the user. The cover is made from a water repellent fabric to protect the foam.

---

## **ATTACHING THE CUSHION TO THE SEATING SURFACE (FIGURE 1)**

The cushion attaches to the wheelchair seating surface using **hook** and **loop** fastening straps.

### **CUSHION COVER**

The bottom of the cushion cover is equipped with two (2) strips of the **loop** portion of the fastening strap.

Attached to the loop portion are two(2) strips of double-sided hook.

### **WHEELCHAIR SEATING SURFACE**

1. Determine whether the wheelchair seating surface has hook or loop attachment strips.
2. If the wheelchair has loop attachment strips, verify that the double sided hook strips are securely attached to the loop attachment strips on the bottom of the cushion.


*NOTE: The zipper on the cushion should face rearward.*

3. Align the front edge of the cushion with the front edge of the seating surface.
4. Secure the cushion to the seating surface, making sure that the double sided hook is securely attached to the strips of loop on the cushion cover and the chair.

### **WARNING**

**Make sure the hook and loop fastening straps are securely attached between the seating surface and cushion before using. If the cushion is not secured to the wheelchair seating surface, exercise caution when transferring in and out of the wheelchair.**

5. If the wheelchair has hook attachment strips, remove the double sided hook strips from the cushion.
6. Align the front edge of the cushion with the front edge of the seating surface.
7. Secure the cushion to the seating surface, making sure that the strips of loop on the cushion cover are securely attached to the strips of hook on the chair.
8. If the wheelchair seating surface has neither hook or loop attachment strips, remove the double sided hook from the bottom of the cover.
9. Align the front edge of the cushion with the front edge of the seating surface.
10. Place the cushion on the wheelchair seating surface.

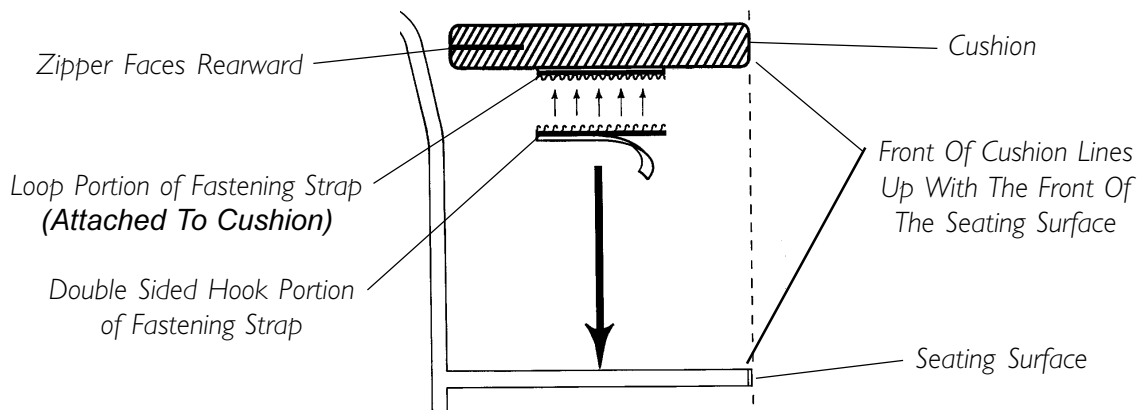**FIGURE I - ATTACHING THE CUSHION TO THE SEATING SURFACE**

## **USE OF THE ESSENTIAL CUSHION**

If the user finds the cushion uncomfortable or finds it difficult to maintain a stable position, the following checks should be made:

1. If the user weighs more than **250 lbs. (113.4 kg)**, verify that he or she is not compressing the foam such that it is not providing support and pressure relief.
2. Verify that the cushion is oriented properly. The back of the molded foam is marked with the word, "**BACK**". When the cover is properly installed, the zipper will be at the back.
3. If the cushion is old, it may not be capable of providing support and should be replaced.

## **CLEANING**

### **CAUTION**

The cover is designed to protect the foam against a user's incontinence and to provide fire retardancy so the cushion must not be used without its cover. If the cover is torn, it must be replaced.

## **COVER**

### **CAUTION**

Machine wash cold. Gentle cycle. Mild detergent. **NO** fabric softeners or bleach. Tumble dry low or air dry.

## **CUSHION**

### **CAUTION**

Wipe down surfaces with a slightly dampened cloth. **DO NOT** submerge cushion in water. If cushion foam becomes contaminated because of incontinence, it should be replaced because the foam is absorbent and cannot be cleaned.

## **UNITED STATES LIMITED WARRANTY**

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW**

**APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for the period of one (1) year of use by original purchaser. All covers are warranted for 90 days only. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. **C.O.D.** shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. THE MANUFACTURER SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

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© 2003 Invacare Corporation
Invacare is a registered trademark of Invacare Corporation.
Yes, you can. and Essential are trademarks of Invacare Corporation.
Invacare Corporation [www.invacare.com](http://www.invacare.com)