**Assembly, Installation and Operating
Instructions - English
Instructions pour l'assemblage, l'installation et
l'utilisation - French Canadian
Instrucciones de Ensamble, Instalación y
Operación - Domestic Spanish**

**GRAB BARS
BARRES MURALES
AGARRADORES**

DEALER: THIS MANUAL MUST BE GIVEN TO THE USER OF THE WHEELCHAIR.

USER: BEFORE USING THIS WHEELCHAIR, READ THIS MANUAL AND SAVE FOR FUTURE REFERENCE.

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Yes, you can.™

SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. In case of shipping damage, DO NOT use. Contact Carrier/Dealer for further instructions.

SAFETY SUMMARY

To ensure the safe setup and installation of the Grab Bars, these instructions **MUST** be followed:

GENERAL WARNINGS

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these Warnings, Cautions and Instructions, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

After **ANY** adjustments, repair or service and **BEFORE** use, make sure all attaching hardware is tightened securely.

Grab Bars add security for bath and commode areas when attached as recommended. To provide optimum safety and function, location, size, and angle of bar(s) **MUST** be determined by a qualified professional.

Use **EXTREME** caution on wet tub or floor surfaces.

Users with limited physical capabilities should be supervised or assisted when using Grab Bars.

The Grab Bars can provide support and increases stability for an individual weighing up to 250 lbs. (114 kg.). Grab Bars are **NOT** designed to support the total weight of an individual. Use the bar for assistance **ONLY**.

INSTALLATION WARNINGS

For proper installation, Grab Bars **MUST** be aligned so the mounting brackets are positioned over wall studs. At least two (2) of the three (3) screws on each end of the grab bar **MUST** go through the wallboard or tile and secure into the wall studs for solid support.

Be sure Grab Bars are correctly and securely installed. **NEVER** use Grab Bars if any looseness is noted.

MODEL

	WHITE VINYL COATED	CHROME	STAINLESS STEEL
GRIPPING DIAMETER	1-Inch	1-Inch	1-1/2-Inches
WALL CLEARANCE	3-3/4-Inches	3-3/4-Inches	1-1/2-Inches
12-inch	312	512	2712
16-inch	316	516	2716
18-inch	318	518	2718
24-inch	324	524	2724
32-inch	332	532	2732
36-inch	N/A	N/A	2736
42-inch	N/A	N/A	2742

Tools Required:
3/8-inch Power Drill 1/8-inch Drill Bit # 2 Phillips Screwdriver

INSTALLING GRAB BARS (FIGURE 1)

NOTE: Refer to the GENERAL WARNINGS and INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

WARNING

Before installation, locate the wall studs behind the wall (a stud finder can be obtained from a hardware store). Standard measurement of wall studs is 16-inches between centers (FIGURE 1). As older homes do not always fall in line with current codes, locating the wall studs is imperative. Grab Bars can be installed vertically, horizontally or at an angle, but the screw holes of the grab bar **MUST** be aligned so at least two (2) of the three (3) mounting screws are secured into the wall studs.

1. Remove the grab bar from the packaged container.
2. Determine placement for the grab bar.
3. Place grab bar on wall at desired position.
4. With a pencil, mark the hole positions on one side.
5. Repeat STEP 4 for the other side of the grab bar.
6. Place grab bar aside.

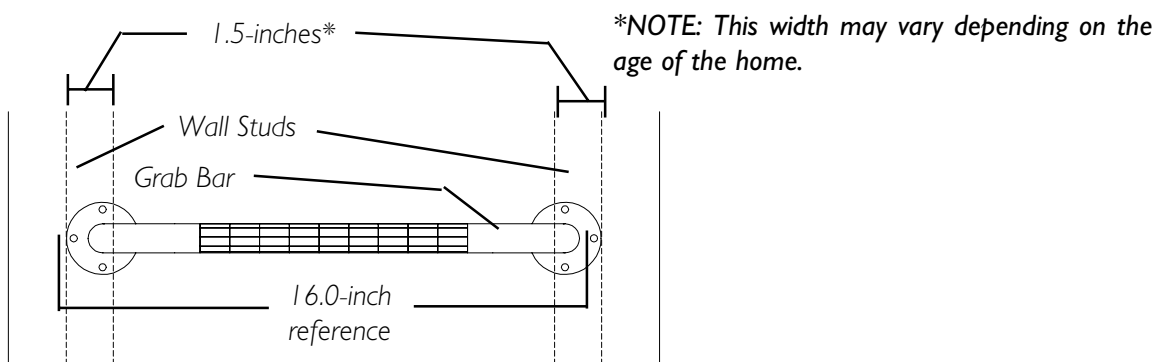
WARNING

When drilling into the wall, exercise care to avoid any electrical wiring or plumbing that may be located behind the wall. As older homes do not always fall in line with current housing codes and requirements, know where internal wall wiring is located so that no wires will interfere with your installation. Damaged electrical wiring can cause electrical shock and fire.

Proper installation is extremely important. Special fasteners may be required if wood is not in desired location. If in doubt, installation should be done by a qualified professional.

NOTE: If installing grab bar over ceramic tile, a carbide drill bit will be required for proper drilling.

7. Use the 3/8-inch power drill and 1/8-inch drill bit to drill all holes into the wall and wall stud(s) at the marked positions.
8. Place grab bar on wall aligning mounting holes with the holes in the wall.
9. Insert screws into one side of the grab bar **(HAND TIGHTEN ONLY!)**.
10. Repeat STEP 9 for opposite side of the grab bar.
11. Securely tighten all mounting screws.

NOTE: Grab bar shown is White Vinyl Coated or Chrome.

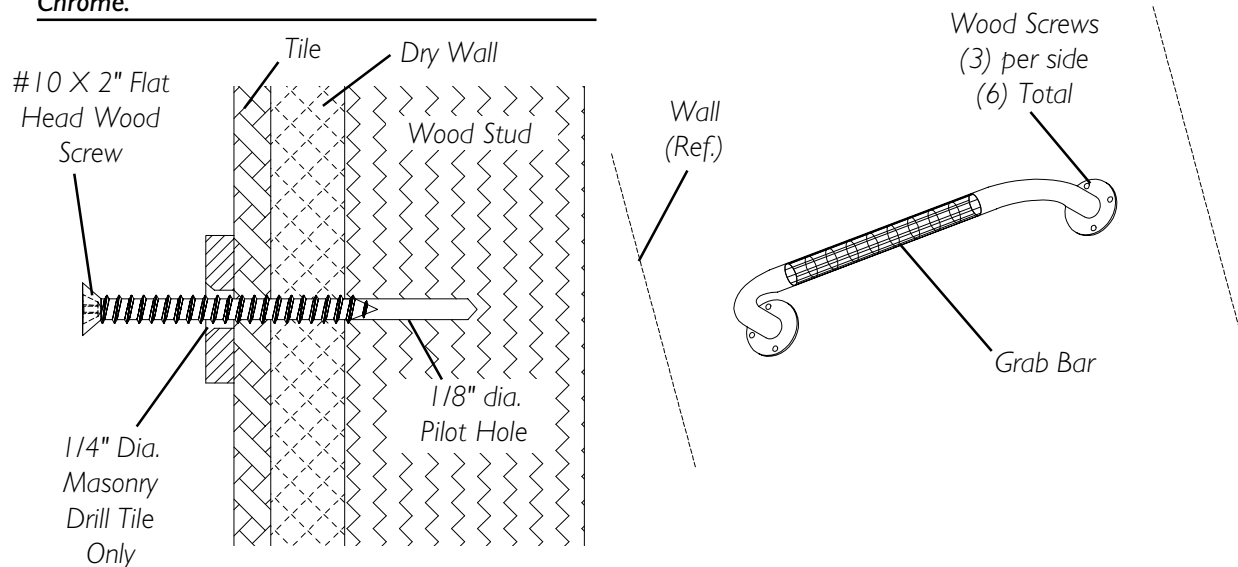
FIGURE I - INSTALLING GRAB BARS

USING GRAB BARS

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Grasp the grab bar when exiting or entering the bath tub.

CARE AND MAINTENANCE

1. Use of mild soap and water or a non-abrasive cleaner is recommended for cleaning.
2. Periodically inspect the grab bar for damage or looseness, etc. **DO NOT** use if any damage exists. Contact your authorized Invacare dealer.

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for the lifetime of use by the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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Translate---To ensure the safe setup and installation of the Grab Bars, these instructions **MUST** be followed:

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This image shows a full page of white paper with horizontal black ruling lines. The word "NOTES" is printed at the top center in a bold, black, sans-serif font. Below the title, there are approximately 28 evenly spaced horizontal lines extending across the width of the page, providing space for writing.

Yes, you can.™

Invacare Corporation

www.invacare.com

USA

One Invacare Way
Elyria, Ohio USA
44036-2125
800-333-6900

Canada

570 Matheson Blvd E Unit 8
Mississauga Ontario
L4Z4G4 Canada
800-668-5324

Invacare is a registered trademark of
Invacare Corporation.

Yes, you can. is a trademark of
Invacare Corporation.

© 2002 Invacare Corporation
Part No. 1089632 Rev B 08/02