DELUXE HAND SHOWER - MODEL NO. 828
HAND SHOWER - MODEL NO. 826
Assembly, Installation and Operating Instructions

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HAND SHOWER - MODEL NO. 826
DELUXE HAND SHOWER - MODEL NO. 828

Assembly, Installation, and Operating Instructions

SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact Carrier/Dealer for further instruction.

SAFETY SUMMARY

The following recommendations are made for the safe installation and use of the Hand Shower:

WARNING

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these Warnings, Cautions and Instructions, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

| DESCRIPTION | QUANTITY | |
|--------------------|----------|-----|
| | 826 | 828 |
| Deluxe Hand Shower | - | 1 |
| Hand Shower | 1 | - |
| Shower Hose | 1 | 1 |
| Gasket | 2 | 1 |
| Wall Bracket | 1 | 1 |
| Double-Faced Tape | 1 | 1 |
| Anchors | 2 | 2 |
| Screws | 2 | 2 |

INSTALLING THE WALL BRACKET

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

NOTE: The wall bracket can be mounted with either the screws or double-face tape provided.

SCREWS - If installing the wall bracket into ceramic tile, masonry or paneled drywall, use the screws and plastic anchors provided. If installing the wall bracket into a pre-fab bath enclosure of fiberglass or metal it may be necessary to use toggle bolts or molliers.

DOUBLE-FACE TAPE - Use double face tape only on hard surfaces such as tile, fiberglass, etc. DO NOT use on any painted surfaces or wall paper.

TOOLS NEEDED: Electric Drill, 1/4-inch masonry drill bit, screwdriver, adjustable wrench, pliers and steel wool.

Screw Installation (FIGURE 1)

1. Determine type of surface that wall bracket will be mounted.
2. Position wall bracket on wall approximately 4-inches below and either to the left or right of the shower head.
3. While holding the wall bracket at the desired position, mark the mounting hole locations.
4. Remove the wall bracket and drill 1/4-inch holes.
5. Insert anchors into mounting holes.
6. Position the wall bracket over the mounting holes and secure to the wall with screws provided.

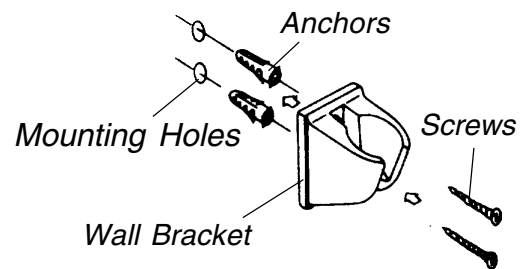
FIGURE 1 - SCREW INSTALLATION

Double-Face Tape Installation (FIGURE 2)

1. Peel off one (1) side of protective cover on double-faced tape. Place the exposed sticky side onto the wall bracket.
2. Position wall bracket on wall approximately 4-inches below and either to the left or right of the shower head.
3. Clean shower wall surface with a dry cloth.
4. Peel off remaining protective cover on double-faced tape.
5. Press wall bracket onto the shower wall.

NOTE: Let the wall bracket sit for 24-hours before using.

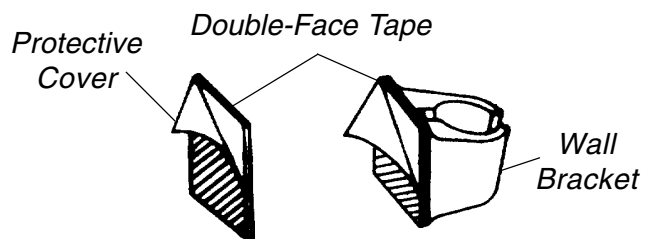
FIGURE 2 - DOUBLE-FACE TAPE INSTALLATION

INSTALLING HAND SHOWER (FIGURE 3)

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

CAUTION

When installing the shower hose onto the shower arm, diverter valve, tub spout or hand shower, tighten by hand only.

NOTE: Refer to the installation instructions supplied with the diverter valve or tub spout before performing the following procedure.

1. Perform (1) of the following:

INSTALLING ONTO SHOWER ARM (DETAIL "A") -


- A. Remove existing shower head from shower arm.
- B. Clean the threads on shower arm with steel wool.
- C. Install pipe thread sealing tape onto the shower arm.

INSTALLING ONTO DIVERTER VALVE (DETAIL "B") OR TUB SPOUT (DETAIL "C") -


- A. If necessary, remove existing shower accessory from diverter valve.

2. Place gasket into end of shower hose.
3. Make sure the gasket is in place and install the shower hose onto the shower arm (DETAIL "A") or diverter valve (DETAILS "B" & "C"). Hand tighten only.
4. **MODEL 826 ONLY** - Perform the following steps:
 - A. Place the other gasket into other end of shower hose (DETAIL "D").
 - B. Make sure the gasket is in place and install the shower hose onto the hand shower (DETAIL "D"). Hand tighten.


DETAIL "A" INSTALLING ONTO SHOWER ARM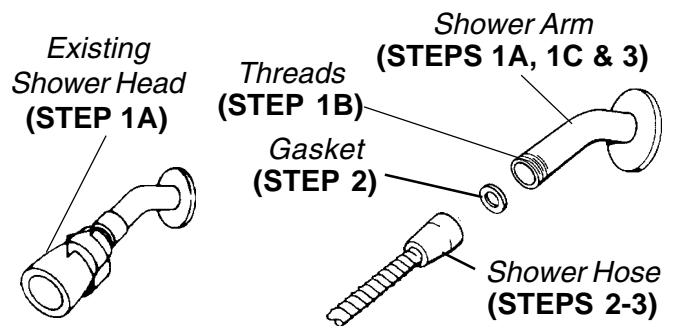


DETAIL "B" INSTALLING ONTO DIVERTER VALVE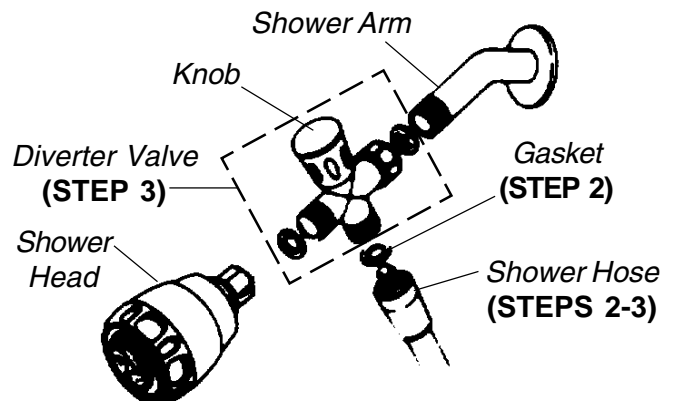

DETAIL "C" INSTALLING ONTO TUB SPOUT

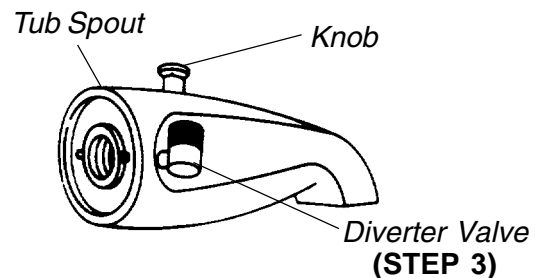DETAIL "D" COMPLETE INSTALLATION

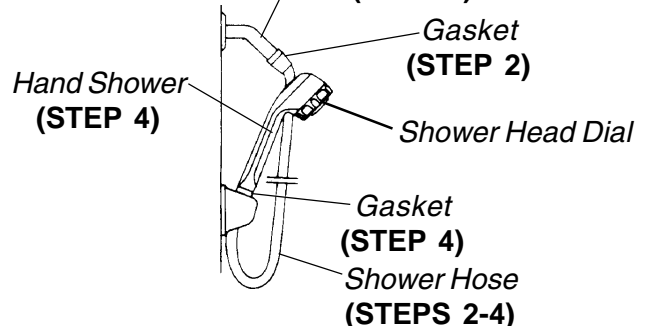NOTE: Hand shower shown is installed onto shower arm.

**FIGURE 3 - INSTALLING/USING
HAND SHOWER**

USING HAND SHOWER (FIGURE 3)

NOTE: MODEL 828 - The shower head allows for the possibility of three (3) different water flows: Fine/Firm, Pulsating Massage or Combination.

NOTE: MODEL 826 - The shower head allows for the possibility of two (2) different water flows: Regular and Pulsating.

Shower Arm

1. Turn the shower head dial until the desired water flow is achieved.

Diverter Valve or Tub Spout

1. Pull the knob on the diverter valve or tub spout to use the hand shower.
2. Turn the shower head dial until the desired water flow is achieved.

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for one (1) year for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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