

Jay J2 Deep Contour




SADMERC Code: E2615

1 Adjustable lateral support shims gently hug the body for increased trunk stability.


2 Adjustable posterior/lateral pelvic support shims help stabilize the pelvis laterally and contour the foam.

3 Adjustable posterior pelvic support contours the foam and positions the pelvis. Long, short or offset hook options for optimum fit to wheelchair.

Includes a deep aluminum back shell, a deep contour foam back with an AirExchange™ cover, two lateral support wedges, two posterior pelvic supports, and universal mounting hardware.

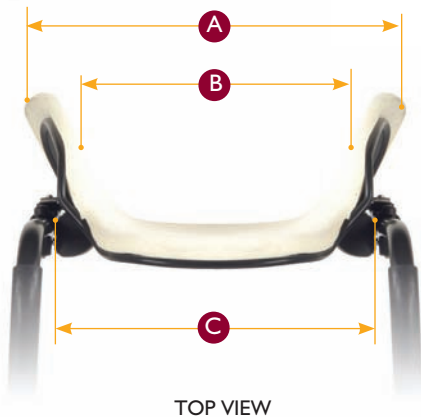

TOP VIEW

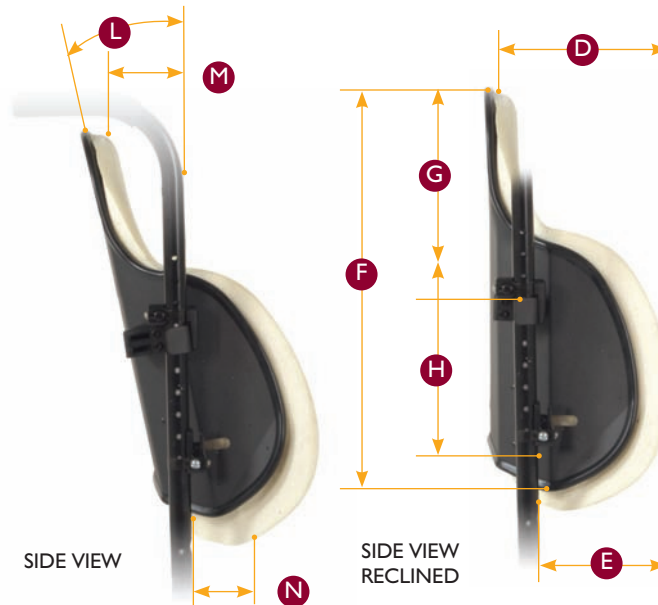

SIDE VIEW

SIDE VIEW RECLINED

Effects to seat depth with relation to recline (standard hooks).

L	M	N
-3°	1.00	1.00
0°	1.55	.86
3°	2.10	.76
6°	2.65	.55
9°	3.20	.40
12°	3.75	.29

Effects to seat depth with relation to recline (long hooks).

L	M	N
7°	3.50	.38
10°	4.13	.33
13°	4.75	.28
16°	5.38	.23
19°	6.00	.18
22°	6.63	.13

CLINICAL

- Designed for the client who has moderate postural needs.
- Suitable for the client who needs a lightweight, easy-to-use back support that provides both pelvic and trunk positioning.
- Suitable for the client who has changing postural needs over time.
- Provides excellent posterior pelvis stabilization and posterior thoracic support, facilitating thoracic extension.
- Addresses the needs of the client with skin integrity issues on the spinous process.
- Facilitates efficient wheelchair propulsion.
- Back angle is adjustable to facilitate visual orientation.

TECHNICAL

- Lightweight, contoured aluminum shell, contoured foam insert and optional posterior– lateral pelvic supports provide exceptional pelvic and trunk support.
- Angle adjustability of up to 15°.
- Optional spinal fluid pad insert provides pressure and shear reduction.
- Height adjustable.
- Shape and size minimizes interference with upper extremity functional activities.
- Extended and offset hardware kits are available to allow for a custom fit.
- A personalized fit is easy with positioning accessories that attach with Velcro®.
- Easy installation and simple fittings.
- Does not compromise seat depth.

THORACIC

LATERAL

- Provides moderate to more aggressive lateral support.
- Most suitable for the person with decreased lateral trunk stability.

- Deep lateral walls provide excellent lateral trunk stability.
- Foam wedges provide opportunity for a more customized fit.
- Posterior lateral pelvic positioning can be customized through the use of positioning accessories.

PART #	OUTSIDE WIDTH OF SHELL	WIDTH OF SUPPORT SURFACE	ATTACHING WIDTH FOR CHAIRS	UPPER CONTOUR DEPTH OF BACK	LOWER CONTOUR DEPTH OF BACK	HEIGHT OF BACK SHELL	UPPER ATTACHMENT POINT	LOWER ATTACHMENT POINT
	A	B	C	D	E	F	G	H
2314	14.50	9.00	11.75-13.0	6.00	6.25	17.25	8.00	6
2315	15.50	10.25	12.75-14.0	5.50	6.00	18.50	9.00	6
2316	16.50	11.75	11.50-13.5	6.13	7.00	18.50	9.13	6
2317	17.25	12.00	14.75-16.0	7.00	6.50	18.25	9.00	6
2318	18.25	13.00	15.75-17.0	7.00	6.50	18.00	9.00	6
2319	19.00	14.00	16.75-18.0	7.00	6.50	18.00	9.00	6
2320	20.00	15.25	17.75-19.0	7.00	6.75	18.00	9.00	6